BOSERENITY
Smart Healthcare Solutions

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TABLE OF CONTENTS

SYSTEM PRESENTATION	2
Intended Use	2
Intended Benefits	2
Material Provided	3
Precautions for Use	3
Contraindications	4
Safety Instructions	4
NEURONAUTE DAILY USE	5
Every Morning	5
BEFORE WEARING THE DEVICE	6
Prerequisite	6
CHARGING THE BATTERY	7
WEARING THE DEVICE	8
Installing the Device	8
Verifying the Installation	8
Reporting an Event	9
Recorder LED Status	9
Battery LEDs Status	10
RISKS LINKED TO USING THE DEVICE	11
TECHNICAL SPECIFICATIONS	13
Operating Conditions	13
Technical Specifications	13
Manufacturer Information	14
Contact	14
Recycling the Neuronaute Parts	14
Document History	14
SYMBOLS	15
MEDICAL DEVICE CLASSIFICATION	16
ABBREVIATIONS	16

SYSTEM PRESENTATION

The main **Neuronaute Plus** objective is to help the healthcare professionals to analyze potential neurological disorders using the EEG records.

Intended Use

Neuronaute Plus is a connected medical device composed of a device with its extender modules, a holding band, a battery and a battery charger.

Neuronaute Plus can be used in combination with compatible cap or electrodes, a Cloud platform and a mobile application.

Neuronaute Plus enables the acquisition, recording, storage and transmission of physiological signals in order to analyse potential neurological disorders.

Neuronaute Plus is intended to be used on patients of all ages.

Neuronaute Plus is intended to be used by a care giver for patients under 15 years old.

Before using **Neuronaute Plus**, the patient must be trained by a healthcare professional.

Neuronaute Plus is intended to be used in the following environments:


- Hospital / Hospital ambulatory setting / Healthcare establishment
- Medical research facility
- At the patient's home

Intended Benefits

The benefits of the medical device are:

- It enables to increase the number of acts on a same patient and to record more patients.
- Distant diagnostic.
- Data is stored on a non-volatile device, preventing data loss in case of sudden record interruption

Material Provided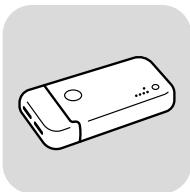


The **Neuronaute Plus** enables the EEG signal acquisition of physiological signal.

It connects to a cap / set of electrodes.

It receives, stores and transmits EEG physiological signals through Wi-Fi and Bluetooth.

A button located on the device enables the patient to notify events to the healthcare professional.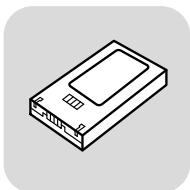


The **battery** is a removable rechargeable electronic device.

It has a maximum autonomy of 24h.

It connects to the **Neuronaute Plus**.

Two batteries and a charging station provided.

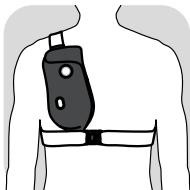The **holding band** is an optional accessory provided with the device.

The holding band enables to wear the device over the clothes.

The holding band is intended for users who have an underbust measurement of at least 70cm.

Precautions for Use

First installation
with a healthcare
professional.

Contraindications

User strongly contagious.

Sensory processing disorder having an effect on skin sensitivity.

Open wound in an area either covered or surrounded by the device.

Cardiorespiratory disorders which may be increased by compression of the thorax.

Behavioral problems making the patient excessively agitated or aggressive.

Motor or mental disorders that is incompatible with using the device.

Use of the data gathered by the device nor any of its components to establish a brain death diagnosis.

Allergy to one of the components of the textile: polyamide, polyester, elastane, any synthetic material.

Safety Instructions

Do not smoke near the device.

Never use a damaged device.

Avoid intense activity while using the device.

Do not use a battery that shows autonomy issues.

Do not use the system simultaneously with any other medical device that has not been validated along with the system.

If the system is painful to wear, remove the device and contact a healthcare professional.

NEURONAUTE DAILY USE

Every Morning

1

Without disconnecting any cable, remove the **Neuronaute Plus** from the holding band.

See "Wearing the Device", page 8

2

Clip a charged battery to the **Neuronaute Plus**.

See "Charging the battery", page 7

3

Check that the **Neuronaute Plus** recording LED is green.

See "Recorder LED Status", page 9

4

Install the **Neuronaute Plus** back in the holding band.

Put the removed battery on charge.

See "Charging the battery", page 7

See "Verifying the Installation", page 8

BEFORE WEARING THE DEVICE

Prerequisite

Before installing the **Neuronaute Plus**, check that:

- ✓ The precautions for use, safety instructions and contraindications are observed.
- ✓ The device is clean.
- ✓ There is no damage or missing part on the device.
- ✓ The device is dry.
- ✓ The battery is fully charged.

CHARGING THE BATTERY


The battery has a 24h autonomy and must be changed every day.

1

2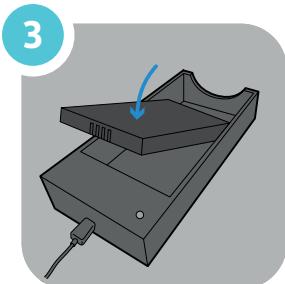


3


Without disconnecting any cable, remove the device from the holding band.

Remove the battery cover located on the rear of the device and remove the battery from the device.


Put the battery on charge. After a few seconds, the charger LED flashes orange to indicate that the charge starts.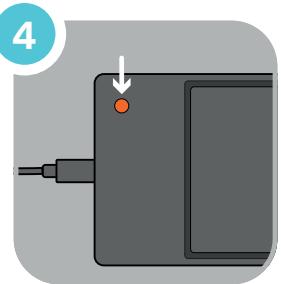

4

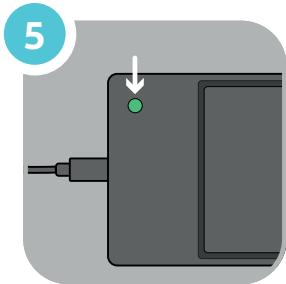5

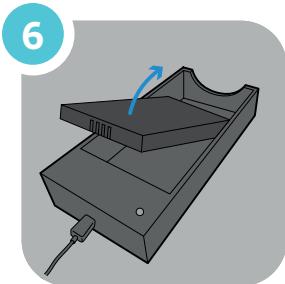6

The LED turns to steady orange to indicate that the charge is ongoing.

Once the battery is charged, the charger's LED turns to steady green.

Remove the battery from the charger.

WEARING THE DEVICE


Installing the Device

1

Adjust the holding band around the arm, over the clothes.

2

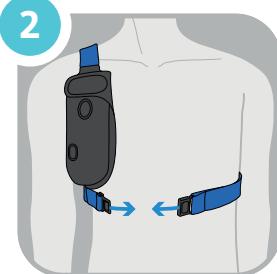Put the holding band around the chest.

3

Adjust the part of the holding band located on the back.

4

Put the device in the textile holding band.

5

Fasten the bands around the device using the Velcro tap.

Verifying the Installation

Once installed, the front of the device shall be similar to the opposite figure. The device must be worn over clothing.


i Note

Do not turn off the device.

i Note

Do not unplug the cap, electrodes or extender during the record.

Reporting an Event

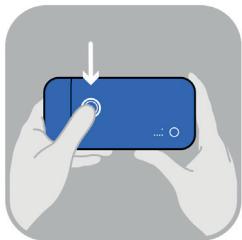Pressing the event button enables the patient to notify any feeling or relevant information to the healthcare professional (activity, environment, pain...). The notified event will be consulted by the healthcare professional during the analysis.

Recorder LED Status

The recorder LED gives information on the current record status.

LED DISPLAY	DESCRIPTION
Steady blue	Record ongoing (live streaming off)
Flashing blue (slow)	Awaiting configuration
Flashing blue (fast)	Record ongoing Data transfer ongoing
Steady green	Record ongoing (live streaming on)
Flashing yellow (fast)	End of record Data transfer ongoing
Flashing yellow (slow)	End of record Data transfer complete
Steady purple	Impedance being displayed
Red	Error. Contact the support

Battery LEDs Status

The battery LEDs give information on the current battery status.

LED DISPLAY	DESCRIPTION
4 steady LEDs - white	Battery level <75%
3 steady LEDs - white	Battery level between 50-75%
2 steady LEDs - white	Battery level between 25-50%
1 steady LED - white	Battery level <25%
1 steady LED - red	Battery level <10%
1 flashing LED - red	Battery level <5%
4 flashing LEDs - red	No battery, backup power source

i Note

Once the battery level drops under 10%, change the battery for a fully charged one.

RISKS LINKED TO USING THE DEVICE

Information on Residual Risks

- It is recommended to use the device away from any extreme source of heat and cigarettes. Device is at risk of igniting if a flame or a burning object goes near.
- The device will be damaged by water: there is a risk of erroneous measurements if clothing is considerably damp.
- Do not use the device in case of allergy to one of the components: polyamide, polyester, elastane, any synthetic material.

Safety Information

- The Neuronaute is an electrical device containing type BF applied parts in contact with the patient in accordance with the standard IEC 60601-1.
- Electromedical devices require special precautions regarding EMC (Electromagnetic Compatibility) and must be installed and operated in accordance with the EMC information provided in the Instructions for Use.
- The use of accessories, transducers and cables other than those specified (except for transducers and cables sold by the manufacturer of Neuronaute) such as replacement parts for internal components, may cause an increase of emissions or a reduction of immunity of the device.
- Do not use the system simultaneously with any other medical device that has not been validated along with.
- Do not use a modified, damaged or degraded device (packaging damaged, product damaged, corrosion...).
- In case of cable or electrode disconnection, unplug the cable or electrode from the recorder and the patient.

Risks Associated with Batteries

In order to limit any potential risk of fire caused by a battery short circuit, the following provisions have been taken:

- The battery is intrinsically safe since it was designed according to the IEC 62133 standards.
- The plastic case protects the device against shock.

Therefore it is possible to consider that the risk is equal to the risk generated by carrying a cellular phone or any other device that uses any rechargeable lithium battery.

Risks Associated with Radio frequency transceiver

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

This equipment generates uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation.

If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

Warning

This equipment must not be modified, altered, or changed in any way without signed written permission from BioSerenity. Unauthorized modification may void the equipment authorization from the FCC and will void the BioSerenity warranty.

TECHNICAL SPECIFICATIONS

Operating Conditions

The device is designed to be used under the following environmental conditions:

CHARACTERISTICS	ENVIRONMENTAL CONDITIONS
Usage temperature	Between +5 °C and +40 °C

Technical Specifications

COMPONENT	VALUE
Weight	Approx. 200 g (with batteries)
Dimensions	155 mm x 73 mm x 23 mm
Supply voltage	3.7 V / 1.1 Ah
Charger power supply	100-240 V, 50/60 Hz
Battery autonomy if fully charged	24 hours maximum
Battery lifetime	200 cycles / measures
Bluetooth (per module)	BLE
Wi-Fi (per module)	A/B/G/N 2.4 Ghz and 5 Ghz
CHARACTERISTIC	USE VALUE
Noise level of EEG channels	<6 µVp-p over 0.1-50 Hz
Sampling frequency	250 Hz or 500 Hz
Minimum bandwidth at -3dB	0.1-60 Hz
Input dynamic range	≤10% at ±150 mV
Common mode rejection ratio	≥-80 dB at 50 Hz and 60 Hz
Input power	<3.7 W
Accuracy of amplitude and rate of variation	≤±20 % of output nominal value or ±10 µV, according to the highest value at rate of variation of 12 mV/s
RF wireless communications information (e.g. FCC rules)	FCC part 15C Host Device FCC ID: 2A223-NNPREGV1 Contains FCC ID : 2AAQS-ISP1507 Contains FCC ID : 2A223-NNPV1

Manufacturer Information

BioSerenity SAS
ICM-iPEPS
47, bd de l'Hôpital
75013 PARIS - France

Contact

Internet

The information provided is available and updated on the website:
<https://www.bioserenity.com/>

Support

To contact the support, use the following address:

support.neuro@bioserenity.com

For any further question, contact a healthcare professional.

FAQ

The FAQ is accessible from the mobile application and at the following link:

<https://bioserenity.zendesk.com/hc/en-us>

For any further question, contact a healthcare professional.

Materiovigilance

Every serious incident related to the device must be notified to the supplier and to the competent authority from the user state.

Recycling the Neuronaute Parts

The device includes electrical wires and electronic components that cannot be discarded along with household waste.

Contact a healthcare professional to learn about the return procedures for any device component, in accordance with the legislation in force.

Document History

REVISION	DATE	DESCRIPTION
A	April 1, 2022	Initial creation
I	November 3, 2023	Update and modifications

SYMBOLS

SYMBOL	DESCRIPTION
	Manufacturer
	Date of manufacture
	Warning
	Serial number
	Catalog reference
	Do not use if package is damaged
	Keep away from sunlight
	Lower and upper limit of temperature
	Direct current
	Keep dry
	Upper and lower limit of humidity
	The product must be eliminated in a suitable salvage and recycling structure
	Atmospheric pressure limitation
	Type BF applied part
	Operating instructions
	Medical device
Rx Only	United States Federal law restricts medical devices to sale by or on the order of a licensed healthcare practitioner.
	Batch code
	Do not expose the device to a magnetic resonance environment
	On / Off
	For this country, the 5.15-5.35 GHz is restricted to indoor use.

SYMBOL	DESCRIPTION
	Marking on the exterior of the product or on parts of the device, including RF transmitters or involving RF (Radio Frequency) electromagnetic energy for diagnosis or treatment. DEVICES and EM SYSTEMS which include RF transmitters or which intentionally apply RF electromagnetic energy for the diagnosis or treatment must bear the symbol IEC 60417-5140 (2003-04) relating to non-ionizing radiation.

MEDICAL DEVICE CLASSIFICATION

The **Neuronaute Plus** components are classified as follows:

Europe

The **Neuronaute Plus** is a class IIa medical device.

USA

The **Neuronaute Plus** is a class II medical device.

ABBREVIATIONS

ABBREVIATION	TERM
BLE	Bluetooth Low Energy
BF	Body Floating
EEG	Electroencephalogram
EM	Electromagnetic
EMC	Electromagnetic Compatibility
EN	English
FAQ	Frequently Asked Questions
FR	French
FCC	Federal Communications Commission
ID	Identifier
IEC	International Electrotechnical Commission
LED	Light-emitting diode
RF	Radio Frequency

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BioSerenity – SAS au capital de 261427 euros