


Trihear

Clip Pro Pocket Amplifier



User Manual
Model: TH-L12M


Thank you for purchasing your new TH-L12M Clip Pro Pocket Amplifier. This user manual will provide you with all of the information you need to get started with your Clip Pro Pocket Amplifier. If you have any problems setting up or using your Clip Pro Pocket Amplifier, please contact us by email at support@trihear.com or visit the QR code to log in to our website: <https://www.trihear.com>

Package Contents

- 1 x Clip Pro Pocket Amplifier
- 1 x Type-C Cable
- 1 x 3.5mm AUX Cable
- 1 x Earphone
- 1 x User Manual

M size is equipped by default.

Product Overview

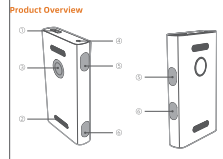


- ① Type-C
- ② Volume +
- ③ Volume -
- ④ Mode Switch
- ⑤ LED
- ⑥ AUX Port
- ⑦ Charging Port
- ⑧ LED
- ⑨ Power On/Off
- ⑩ Power On/Off

Product Specifications


- Power Input: 5V=1A
- Charging Port: Type-C
- Frequency response range (Hz): 200-4000
- Equivalent input noise (dB): ≤ 25
- Total harmonic distortion: $\leq 3\%$
- Maximum saturated sound pressure level OSPL90 (dB): 120±3
- High frequency average OSPL90 (dB): 116±2
- MHA Full of Gain (dB): ≤ 40
- Hearing amplifier working time: 24hrs
- TV Adapter working time: 2hrs

How to use

1. Hearing Amplifier Charging
Please charge the device with included Type-C Cable, the estimated charging time is 45 mins.
*Power adapter not included, please use regular 5V DC adapter (We recommend you full charge the device before first use)

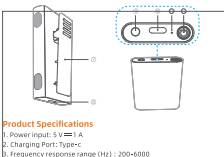


2. TV Adapter Charging
Please charge the device with included Type-C Cable, the estimated charging time is 2hrs.
*Power adapter not included, please use regular 5V DC adapter (We recommend you full charge the device before first use)

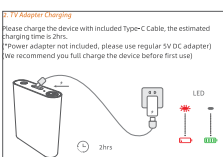


Note:
TV adapter cannot be used during charging.

3. Connect Earphones
Please insert the included Type-C earphones in the hearing amplifier.

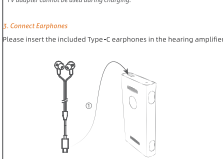


Warning

- Please choose suitable domes from S/M/L size (M size is equipped by default).
- Attach the domes to the earphones (no need if you choose M size).
- Put the earphones in the ears.

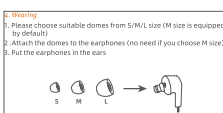



4. You could use either clip or lanyard when wearing

Use Clip

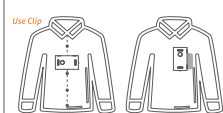


Use A Lanyard

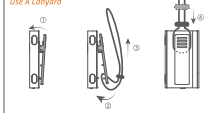


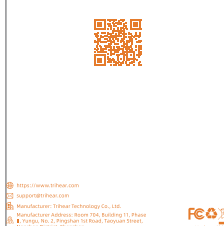

<https://www.trihear.com>
support@trihear.com
Manufacturer: Trihear Technology Co., Ltd.
Manufacture Address: Room 10, Building 11, Phase 1, Hengde No. 2, Program Road, Taidong Street, Hangzhou, Zhejiang, China




6. Hearing Amplifier Power On
Switch to 'on' to power on the device. Voice prompts indicate battery status.

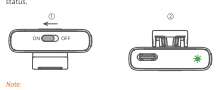

Note:
Please wear earphones before turning on the device.

7. Volume Adjustment for hearing amplifier
① The left and right channels can be adjusted separately, single click 1/3 volume to adjust volume until you feel comfortable.
② Each time you click, there will be 'bump tone', and you will hear voice prompts: 'maximum volume' or 'minimum volume' when reached maximum/minimum volume.



1 Times \leftarrow \rightarrow Maximum/Minimum volume

8. Mode switch

A. Hearing Mode
Once power on, the default mode is 'hearing mode', you can adjust the volume separately according to the guide above.


As for 'Hearing Mode' Clip Pro amplifier supports two sub-modes: 'Indoor mode' and 'Outdoor Mode'.

Indoor Mode: For more natural human voice while less noise suppression


Outdoor Mode: For better noise suppression

B. TV Mode

a. TV Adapter Connect to TV
Please connect the TV Adapter to TV with included 3.5mm AUX Cable.

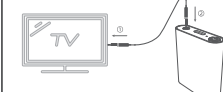

b. TV Adapter Power On
Long Press power button on the TV adapter for 3s to power on.



c. Long Press the hearing amplifier 'Q' button for 2s to switch to TV mode, you can hear voice prompts 'please open TV adapter', then wait the hearing amplifier to auto connect TV adapter, once connected you will hear voice prompts 'TV mode', now you can enjoy TV programs.

TV Adapter Mode:
After entering TV mode, it will try to connect to TV adapter automatically.

Long Press 3 Second

d. Volume Adjustment for watching TV
1) Adjust volume on hearing amplifier (you could press either volume control button on left or right side, the left and right volume will stay at same level).
2) Adjust TV volume with TV remote control.

e. You can switch back to hearing mode via long pressing hearing amplifier 'Q' button for 2s.

Note:
Kindly remember that the hearing mode is default after power on, and it will remember the volume after bedtime.

FAQ

Q: How long does it take to full charge the hearing amplifier?
A: It will full charge in 45mins.

Q: Could it be used with other Type-C earbuds or earphones?
A: No, you need to use included earphones and type-C cable (IMPORTANT: the earphones is customized for the hearing impaired), in case of no sound or bad performance problems.

Q: Hearing amplification is not good?
A: Please adjust volume by clicking 1/3 volume button to your suitable volume.

Q: I could hear my own voice loudly.
A: You could adjust hearing amplifier position to make the Mic far from your mouth.

Q: I couldn't hear other people talking in noisy environment.
A: Please make sure Mic faces towards to the people you want to hear.

Q: No sound from the receiver.
A: Please check if the Mic is blocked, or the Type-C port is blocked, make sure the switch is slipped to 'on'.

Q: Cannot connect hearing amplifier with TV adapter.
A: 1. Make sure your hearing amplifier and TV adapter both power on.
2. Long press the hearing amplifier 'Q' button for 2s to change into TV mode.
3. If still have any problems please feel free to contact us via: support@trihear.com

Q: Volume of TV is still not loud enough via the hearing amplifier is at maximum volume.
A: Turn up the TV volume using the TV Remote Control.

Q: Disconnected when Move TV adapter.
A: The connection distance is within ten meters, place the hearing amplifier to be connected close to the TV adapter.

FCC STATEMENT
This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions:
(1) This device may not cause harmful interference, and
(2) This device must accept any interference received, including interference that may cause undesired operation.
Warning: Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.
NOTE: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

Safety Instructions

- Avoid dropping.
- Do not disassemble.
- Do not submerge in water.
- Avoid extreme temperature.
- Do not use the device outdoors during thunder storm.
- Use original or certified USB cables.
- Do not use any corrosive cleaner/roll to clean.
- Avoid using excessive volume for long time.
- Avoid thermal radiation, stay off when near XMR, MRI, CT scanner, or other medical machines.
- Dispose old appliances according to local regulations.

Correct Disposal of This Product
(Waste Electricals Electronic Equipment)
This Marking shown on the product or its literature, indicate that it should not be disposed with other household wastes at the end of its working life.
To prevent possible harm to the environment or human health from uncontrolled waste disposal, please separate it from other types of wastes and recycle it responsibly to promote the sustainable reuse of material resources. Household user should contact either the retailer where they purchased this product, or their local government office, for details of where and how they can take this item for environmentally safe recycling.
Business users should contact their supplier and check the terms and conditions of the purchase contract. This product should not be mixed with other commercial wastes for disposal.

Legal Disclaimer
Statements regarding dietary supplements have not been evaluated by the FDA and are not intended to diagnose, treat, cure, or prevent any disease or health condition.

尺寸: 105x175mm

材质: 157g双铜 四色印刷 风琴折

注意: 黑色边框为裁切线 不印刷