

Operation manual of Bluetooth wireless hearing aid headset

Model: T13

FCC ID:2A2MX-T13

Packing list

Earphone: 1, charging cable: 1, silicone earplug: large, medium and small, each pair, instruction manual: 1, packing box: 1 set

Product specifications

Product Name: Bluetooth wireless hearing aid headset

Bluetooth distance: 10M

Bluetooth Name: T13

Input / output: 5V / 1A

Headset battery: 40mAh

Battery in charging chamber: 400mAh

Charging time: 2h

Operation guide

Hearing aids

1. Open the cover, take out the left and right earphones, put on both ears respectively, and the earphones will automatically turn on the hearing aid function
2. Click left headphone or right headphone to automatically switch 4 levels of hearing aid volume
3. Three consecutive clicks on the left or right headphones can turn off or turn on the hearing aid function

Playing music with Bluetooth

1. Open the cover of the charging chamber, take out the earphone, and the earphone will pair automatically

1. Turn on the Bluetooth of the mobile phone, search the pairing name T13, and connect the

headset

2. Open the mobile music player to listen to music
3. Put the earphone back into the charging bin and turn off the music playing function automatically
4. Long press the left headset for 2 seconds to switch to the previous song

Long press the right headset for 2 seconds to switch to the next song

5. Click on the left ear to decrease the volume by one space

Click on the right ear and add 1 volume

Answer the phone

1. When an incoming call rings, double-click the left or right earphone to answer the call
2. During a call, double-click the left or right headset to hang up
3. When an incoming call rings, press the left or right earphone for 2 seconds to reject the call
4. During the call, click the left headphone to decrease the volume and the right headphone to increase the volume

One key reset (restore factory settings)

Put the earphone into the charging chamber, open the charging chamber cover and press the round button of the charging chamber for 10 seconds to clear all pairing records and automatically restore to the factory settings.

NOTE: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.

- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help

Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions:

- (1) this device may not cause harmful interference, and
- (2) this device must accept any interference received, including interference that may cause undesired operation.