

## RECOMMENDATIONS OF CLEANING AND DISINFECTING

To ensure proper hygiene, the manufacturer recommends a thorough cleaning of BABY TYLER after each use as per instructions below:



The manikin must be cleaned by disinfecting the lips and the face, removing the face skin from the head after each training session.

For more thorough disinfection, immerse the face skin in a container with water of disinfectant, wash carefully and keep dry for proper functioning.



After each use, BABY TYLER mouthpiece must be disinfected and allowed to dry for correct functioning.



The BABY TYLER lung is semi-disposable and can be used to train group of people in a session without needing to change it.

If the chest stops rising during the practice session, check that the lung is not moist. If it is, replace it with a new one.

## REPLACEMENTS



Scan this QR CODE for extra multimedia resources!



### Federal Communications Commission Statement and Industry Canada Statements

TORSO MONITOR  
FCC ID: 2A25DCPRTY1  
FCC ID: 2A25DCPRTY2  
IC: 27592-CPRTY1  
IC: 27592-CPRTY2

Changes or modifications not expressly approved by Vicedo Marti S.L. could void the user's authority to operate the equipment.

This device complies with Part 15 of FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

This device contains licence-exempt transmitter(s)/receiver(s) that comply with Innovation, Science and Economic Development Canada's licence-exempt RSS(s). Operation is subject to the following two conditions: 1. This device may not cause interference. 2. This device must accept any interference, including interference that may cause undesired operation of the device.

L'émetteur/récepteur exempt de licence contenu dans le présent appareil est conforme aux CNR d'Innovation, Sciences et Développement économique Canada applicables aux appareils radio exempts de licence. L'exploitation est autorisée aux deux conditions suivantes : 1. L'appareil ne doit pas produire de brouillage; 2. L'appareil doit accepter tout brouillage radioélectrique subi, même si le brouillage est susceptible d'en compromettre le fonctionnement.

MADE  
SPAIN



WORLDPOINT ECC, Inc.  
1326 South Wolf Road, Wheeling, Illinois 60090, USA  
Phone: (888) 322-8350 | Intl. Phone: +1 (847) 465-3200  
worldpoint.com



# Baby Tyler®

## MANUAL



### BABY TYLER INCLUDES

1 BABY TYLER CPR Manikin, 1 monitor, user manual, 4 lungs, 2 mouthpieces, 2 batteries 9V and 1 transport bag.

### Baby Tyler® LIFELIKE INFANT MANIKIN

Its size, weight, and movements are ideal for performing correct infant CPR, adapting to training needs.




#### ➤ REALISTIC BREATHING SIMULATION

Realistically train with a pediatric resuscitator

The head tilt enables students to learn the correct breathing technique

#### ➤ CHOKING PRACTICE

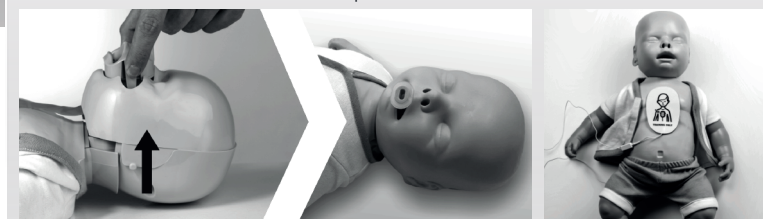

#### ➤ OROPHARYNGEAL AIRWAY

To practice with a oropharyngeal airway, remove the mouthpiece, replace the skin and measure, and insert the airway.

#### ➤ AED TRAINING

Pads do not leave residue on the skin



WIRELESS  
MONITOR



ELECTRONIC  
FEEDBACK



BLIND MODE



TEST REPORT

WorldPoint  
Products

# BABY TYLER

# KEY FEATURES & FUNCTIONS




## ASSEMBLY

1. Remove the chest skin.
2. Remove the face skin.



3. Move the lung below the jaw.
4. Connect the lung to the mouthpiece
5. Place the lung on the chest.

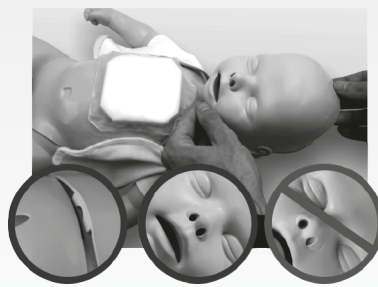


7. Place chest skin and attach to the studs.

## SETTING UP

Insert batteries (9V x 2) into the torso and the monitor (included in each BABY TYLER pack).

## PAIRING

Switch on the monitor  and press BABY TYLER's torso until you hear the double beep sound and the "TORSO" light turns green. BABY TYLER is now ready to use.

*When switching on the monitor, the default always activates on standard mode.*



## KEY FUNCTIONS


### COMPRESSION FEEDBACK

RATE: 100 <=> 120 bpm

DEPTH: 1/3 Chest Diameter



GREEN

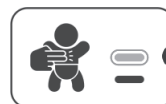

GREEN




RED



RED



WAIT 10 SECONDS FOR TEST REPORT

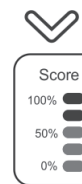

## ADDITIONAL FUNCTIONS

### STANDARD/BLIND MODE

Allows you to select standard or blind mode



STANDARD MODE






BLIND MODE



*Feedback is not visible during practice in blind mode. View the test report 10 seconds after the session has finished in both modes.*

## MONITOR RESET

If you are unable to pair the equipment with the steps provided, reset the monitor.

Monitor reset: Press the "Mode" button for 7 seconds approx. until the rate  and compression  icons flash once.

Make sure the monitor is  switched off, then switch it on and begin the pairing process.

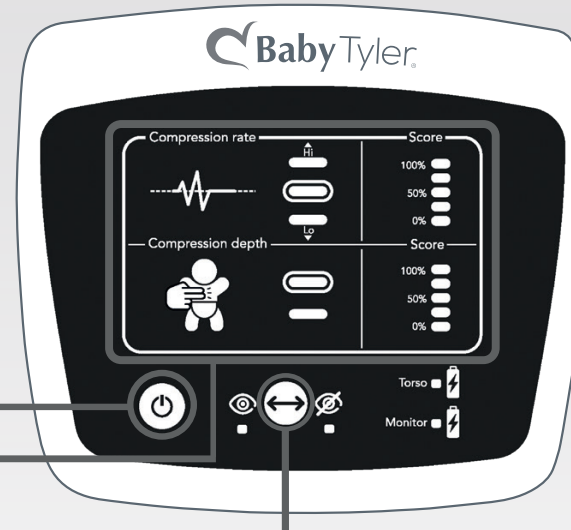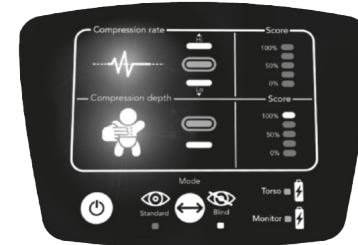

## BATTERY CAUTION

The batteries supplied with the BABY TYLER should be replaced when the red light appears on the monitor indicator.

Using uncharged batteries with either the monitor or the torso will impede proper functioning of BABY TYLER.

There are no interference implications using rechargeable batteries.

