

## RECOMMENDATIONS OF CLEANING AND DISINFECTING

To ensure proper hygiene, the manufacturer recommends thorough cleaning of CPR TAYLOR after each use as per instructions below:



The simulator must be disinfected at the end of each training session. Separate the face skin from the head to clean lips and face.



For better cleaning, put the face skin in a container with water or disinfectant, clean carefully and keep dry for proper functioning.



After each use, CPR TAYLOR Mouth and Airway System must be disinfected and allowed to dry for correct functioning.



CPR TAYLOR's Lung is semi-disposable, and can be used to train a group of people in a session without needing to change it.

If the chest stops rising during a practice session, first ensure the Mouth and Airway System is properly connected to the lung and the head is positioned correctly. Check that the lung is not moist. If the lung is moist, replace it with a new one.

## REPLACEMENTS



Scan this QR CODE for extra multimedia resources!



### Federal Communications Commission Statement and Industry Canada Statements

TORSO MONITOR FCC ID: 2A25DCPRTY1 IC: 27592-CPRTY1  
FCC ID: 2A25DCPRTY2 IC: 27592-CPRTY2

Changes or modifications not expressly approved by Vicedo Marti S.L. could void the user's authority to operate the equipment.

This device complies with Part 15 of FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

This device contains licence-exempt transmitter(s)/receiver(s) that comply with Innovation, Science and Economic Development Canada's licence-exempt RSS(s). Operation is subject to the following two conditions: 1. This device may not cause interference. 2. This device must accept any interference, including interference that may cause undesired operation of the device.

L'émetteur/récepteur exempt de licence contenu dans le présent appareil est conforme aux CNR d'Innovation, Sciences et Développement économique Canada applicables aux appareils radio exempts de licence. L'exploitation est autorisée aux deux conditions suivantes : 1. L'appareil ne doit pas produire de brouillage; 2. L'appareil doit accepter tout brouillage radioélectrique subi, même si le brouillage est susceptible d'en compromettre le fonctionnement.

MADE IN SPAIN



WORLDPOINT ECC, Inc.  
1326 South Wolf Road, Wheeling, Illinois 60090, USA  
Phone: (888) 322-8350 | Intl. Phone: +1 (847) 465-3200  
worldpoint.com



CPR Taylor®

## MANUAL



MEETS  
AHA/ERC  
ARC

### CPR TAYLOR INCLUDES

1 Manikin, 1 monitor, user manual, 5 lungs, 2 Mouth and Airway System, 2 batteries 9V, 2 head locking clips and 1 transport bag.




➤ TWO IN ONE SIMULATOR  
Adult and child CPR training with a single manikin



➤ LIFE-LIKE BREATHING SYSTEM  
Exclusive breathing system, including a unique functional nose



➤ JAW THRUST MANEUVER

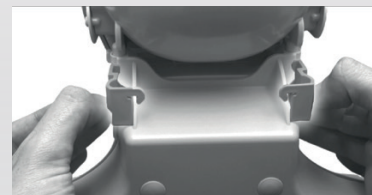

➤ HEAD LOCKING CLIPS



WIRELESS  
MONITOR



ELECTRONIC  
FEEDBACK



2 in 1  
ADULT & CHILD  
FUNCTION



BLIND MODE



TEST REPORT


WorldPoint  
Products

## CPR TAYLOR



## ASSEMBLY

1. Remove the chest skin.
2. Remove the face skin.
3. Make sure the head is correctly attached to the two buffers linking it to the torso.

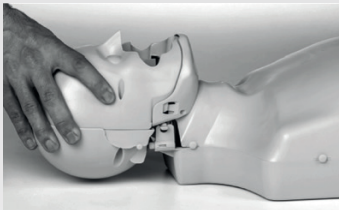


4. Move the lung below the jaw.
5. Connect the mouth and airway system to the lung.
6. Make sure the mouth and airway system is correctly attached to the head.



7. Please ensure:

- A. The lung is attached on the marked circle using the sticker.
- B. The lung rests loosely on the neck.

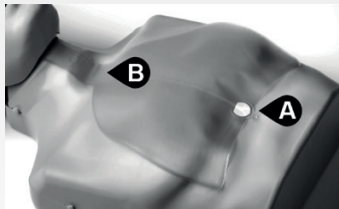

8. Fit the face skin, checking that the nose, mouth and upper tab are in the correct position.
9. Place chest skin and attach to the studs.



## KEY FEATURES & FUNCTIONS

### SETTING UP

Insert batteries (9V x 2) into the torso and the monitor (included in each CPR TAYLOR pack).

### PAIRING


Switch on the monitor and press CPR TAYLOR's torso until you hear the double beep sound and the "TORSO" light turns green. CPR TAYLOR is now ready to use. *When switching on the monitor, the default always activates on adult mode.*

### MONITOR RESET

If you are unable to pair the equipment with the steps provided, reset the monitor.

Monitor reset: Press the "Mode" button for 7 seconds approx. until the rate and compression icons flash once.

Make sure the monitor is switched off, then switch it on and begin the pairing process.

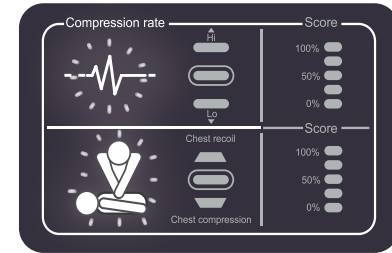




## KEY FUNCTIONS

### COMPRESSION FEEDBACK


#### RATE



#### DEPTH AND RECOIL

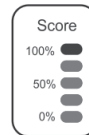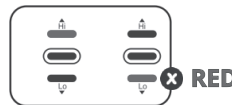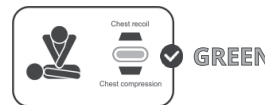

WAIT 10 SECONDS FOR TEST REPORT

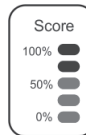

## ADDITIONAL FUNCTIONS

Allows you to select standard or blind mode

### STANDARD MODE



Adult Child

Tap the button until you get a steady light in your preferred option (Adult or Child).

### BLIND MODE



Adult Child

Tap the button until you get the flashing light in your preferred option (Adult or Child).

*Feedback is not visible during practice in blind mode. View the test report 10 seconds after the session has finished in both modes.*

### BATTERY CAUTION

The batteries supplied with CPR TAYLOR should be replaced when the red light appears on the monitor indicator.

Using uncharged batteries with either the monitor or the torso will impede proper functioning of CPR TAYLOR.

There are no interference implications using rechargeable batteries.

