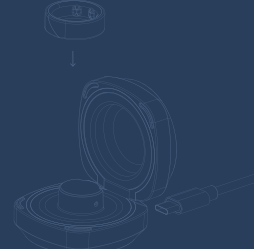
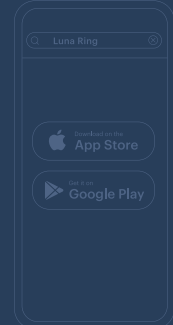
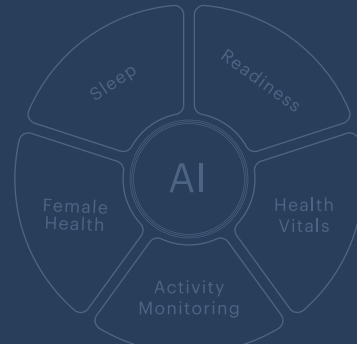
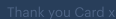


RN004主包装说明书,85*85mm,105g双铜+4C+双面印+过防刮花触感膜, 风琴折

 USER MANUAL LUNARING	INTRODUCTION  AI Coach Sleep Tracking 24/7 Heart Rate Monitor Stress Activity Tracking Women Care HRV SpO2 Temperature MEET THE LUNA RING Luna Ring is a sleek wellness device that helps the user track sleep, readiness, stress and multiple other health markers with advanced sensors. The Luna app provides personalized insights via AI led analytics. The Ring, built of fighter-jet-grade Titanium is made to last and can survive wide environmental conditions. Welcome to the future of Fitness.	SETUP INSTRUCTIONS  Connect the cable and place the ring on the charging dock for charging and pairing	SETUP INSTRUCTIONS  Download the Luna Ring app and create an account to pair your Luna Ring.	SETUP INSTRUCTIONS  • • • • Clockwise lighted ring indicates that the ring is charging — The whole circle lights up to indicate that the ring is fully charged	FEATURES  The Luna app monitors multiple health markers. The Luna AI sits at the center to advice you on best ways to improve your bio-markers.	SOFTWARE UPDATES & TROUBLESHOOTING OTA UPDATE SUPPORT Keep your Luna Ring up to date with the latest features and fixes. Easily update through the app with seamless Over-The-Air (OTA) support for improved performance and new capabilities. Troubleshooting Guide 1. Sleep not detecting Ensure the ring fits snugly. Unpair and reconnect via the app. If the issue persists, contact Luna Support through the app or website. 2. Battery Issues Fully charge the ring before use. Avoid extreme temperatures. If the issue persists, contact Luna Support through the app profile or Luna Support. 3. Connectivity Issues Keep the ring close to your phone, place it on charge, and reconnect. Switch your phone's Bluetooth off and on, and try again. Contact Luna Support if unresolved. 4. Step count Wear the ring on suggested fingers for precise tracking. Sync regularly with the Luna App. Contact Luna support via the app or website.	COMPLIANCE AND CERTIFICATION FACTORY RESET A factory reset unpairs the ring from your user account and restores default settings. Use the Luna app to perform this action. Reconnect the ring and set it up again for use. If you need further assistance, contact Luna Support through the app or Luna website. HARD RESET If your Luna Ring becomes unresponsive, place it on the charger and gently tap three times while holding the charger. This will factory reset the ring. Reconnect it using the Luna app and set it up again. For further assistance, contact Luna Support through the app or Luna website.	COMPLIANCE AND CERTIFICATION FCC REQUIREMENT Any changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment. This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation. Note: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses, and can radiate radio frequency energy, and if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures: -- Reorient or relocate the receiving antenna. -- Increase the separation between the equipment and receiver. -- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected. -- Consult the dealer or an experienced radio/TV technician for help.
--	--	--	--	--	---	---	---	---

DISPOSAL AND RECYCLING INFORMATION Electrical and electronic devices have become an integral part of our daily lives. By recycling, reusing materials, or finding other ways to utilize old devices, consumers can make a significant contribution to the protection of the environment. They should not be disposed of with domestic waste. Consumers are legally required to return these devices to designated public collection points or the point of sale at the end of their service lives. The specifics of this process are outlined in the national law of each respective country. By adhering to these practices, consumers can make a substantial contribution to the preservation and protection of the environment. TIPS FOR SAFE AND EASY RING REMOVAL Finger size can naturally undergo variations throughout the day, making it occasionally challenging to remove the ring from the finger. In case of any difficulty in taking the ring off, please follow the guidelines provided below: • Wet the finger with cold water and apply gentle soap. Slowly twist the ring until it comes off. If the finger is swollen, raise the hand to normalize the blood pressure and try removing the ring again. • In case of any significant discomfort and difficulty in removing the ring, seek immediate medical attention. MAINTENANCE INSTRUCTIONS • To clean the ring, use a gentle cloth or hand wash it using mild soap and water. • This ring is a water activity-compliant tech product that can be comfortably worn during showers, bathing, and swimming.	• Avoid wearing the ring during activities such as strength training, using heavy tools like shovels, or handling heavy objects made of metal, ceramics, or stone. It is also recommended to avoid wearing the ring alongside other rings or objects composed of metal, ceramics, stones, or diamonds. The ring is susceptible to scratches and has the potential to scratch softer metal jewelry or other objects in close proximity, such as gold, silver, or aluminum phone covers. Additionally, certain ceramic phone covers with a delicate coating may be prone to scratching. • Keep the ring out of reach of children as it is not designed for individuals under the age of 18. • Avoid exposing the ring to excessive heat. • Keep the ring away from sharp objects as it may puncture the ring or its battery. SAFETY INFORMATION • Kindly note that the Luna Ring and its associated services are not designed for the purpose of diagnosing, treating, curing, or preventing any diseases or medical conditions. The information and guidance provided are purely for informational purposes and cannot substitute the expertise of healthcare professionals or physicians. If you have any inquiries regarding a medical condition or if you intend to make any alterations to your sleep or activity based on information or guidance from our services, it is always advisable to consult a physician. Please refrain from disregarding or postponing seeking professional medical advice due to any information you have obtained from Luna services. • Luna Ring shall not be held responsible for any health issues that may arise from the information or guidance provided through Luna Ring and its services. If you decide to make any modifications to your sleep or activity based on the information obtained from our services, you acknowledge and accept that you do so entirely at your own risk. It is crucial	to be attentive to your body's reactions and responses. For instance, if you experience unexpected, recurring, or prolonged pain, fatigue, or discomfort as a result of changes made to your sleep or activity, we strongly recommend consulting a physician before continuing with such adjustments. It should be noted that the information and guidance provided by the Luna Ring and its services may not accurately apply if your physiological functions and responses significantly deviate from population averages due to medical conditions or rare natural variances. • Exercise caution to prevent the ring or any other worn product from getting entangled with stationary structures or heavy objects while in motion. If you notice any redness or skin irritation on your finger caused by wearing the ring, it is advisable to remove it promptly. If the symptoms persist for more than 2-3 days after discontinuing the use of our product, seek immediate consultation with a dermatologist. • Our product is not designed to be placed in the mouth under any circumstances. It is not a toy and should not be used by children. Please ensure that children are supervised while handling the product, as it could present a choking risk.	WARRANTY AND SUPPORT LUNA RING WARRANTY POLICY Luna Ring product and their accessories have a warranty period of 1 year for technical issues, and 3 months for accessories. However, it doesn't cover physical damages. Free door-to-door services are provided throughout the warranty period from the date of purchase. Proof of purchase in the form of an invoice (irrespective of the sales channel) is required to be presented for availing the warranty services. Luna Ring may choose not to provide services if the invoice is not presented. If the issue is not resolved over a call or an e-mail, we will arrange to pick up the device from your address, for repair or replacement. In case of non-availability of a reverse pick-up facility at your location, Luna Ring will request you to self-ship the product to our warehouse and share the tracking details with the Luna Ring customer service team. When a customer self-ships the product, the cost must be incurred by the customer. Luna Ring will not be responsible for any loss or damage during transit if you self-ship your product to us. Luna Ring will not cover any liability or claim pertaining to any kind of rashes or skin allergy due to usage of Luna Ring products including consequential or resulting liability, damage or loss to property or life, injuries, loss of any kind of personal data arising directly or indirectly out of any defects in our products. While Luna Ring will make every effort to carry out repair at the earliest, we would like to make it expressly clear that we are under no obligation to do so in a specified period of time. OUT OF WARRANTY If an out-of-warranty product reflects a technical issue and the customer wants to avail the services, the customer must incur a certain amount of charges. A replacement can be given on a chargeable basis if the product is not repairable.	This can be processed only if the specific product is in stock. Luna Ring Warranty is not applicable in the following cases: In case of any damage to the product, customer abuse, repair by unauthorized persons, misuse detected, product attacked or damaged by house pests, pets, rodents, accidental or incidental damage, spillage of any kind of liquid on the device, or device subjected to extreme temperatures. Wear and tear that occurs due to regular usage of the device will also not be covered under warranty. This warranty will automatically terminate on the expiry of the warranty period of 12 months (as determined by proof of purchase), even if the product is not in use during the warranty period for any reason. CUSTOMER SUPPORT For assistance with setting up or using your Luna Ring, Luna App, or other services, visit our support center at https://luna.freshdesk.com/support/home. If you need further help, our support team is here to assist you. Looking for, you can reach out to us at support@luna.freshdesk.in. REGISTER YOUR WARRANTY Your Luna Ring comes with a 1-year limited hardware warranty, automatically registered from the date of delivery. For more details contact Luna support for assistance.	LEGAL AND DISCLAIMERS INTELLECTUAL PROPERTY RIGHTS The product, including but not limited to its design, technology, algorithms, software, hardware, and any associated documentation, is the sole and exclusive intellectual property of Luna Ring. This includes all copyrights, trademarks, patents, trade secrets, and any other intellectual property rights pertaining to the product. Any unauthorized use, reproduction, distribution, modification, or disclosure of the product, in whole or in part, without the express written consent of Luna Ring, is strictly prohibited.	BOX CONTENTS Luna Ring x 1 USB Type-C Cable x 1 Charger x 1 Thank you Card x 1 User Manual x 1	SPECIFICATIONS MATERIAL • Durable titanium • PVD coating • Non-allergenic, non-metallic, seamless inner ring moulding • 5ATM water-resistant (up to 50m/164 ft) WEIGHT & DIMENSIONS • Width: 8mm • Thickness: 2.0mm • Weight: 3 to 5 grams (depending on the ring size) CONNECTIVITY • Bluetooth low-energy (BLE 5) • Automatic firmware updates via the Luna Ring app • Compatible with iOS 14 or later and Android 6 or later BATTERY & POWER • Up to 5 days of battery life • Full charge in 60 to 80 minutes (depending on the ring size) SENSORS • Green LEDs (optical heart rate sensor) • Red LED (blood oxygen sensor) • Photoplethysmography (PPG) sensors • Skin temperature sensor • Photodiodes • 3-axis accelerometer
--	---	--	--	---	--	--	--