

Operator: Shenzhen Deda Medical Technology Group Co., Ltd
Address: Building 39-1, Universiade Software Town, 8288 Longgang Avenue,
Henggang Street, Longgang District, Shenzhen
Tel:0755-89760988 Fax:0755-89760688 Email:service@dedakj-health.com
Website: www.dedamed.com Service hotline: 400-830-2119
Manufacturer Address:Deda Industrial Park, Jianshe Avenue,
Huaihua High-tech Zone, Hunan Province
MADE IN CHINA

DEDAKJ®



K-GE22e

Manual Instruction for Intelligent AI Eye Massager

In order to enjoy the product better, please read the instruction before the use.



DEDAKJ®

Come from German Create the Quality Life

DEDAKJ is the German brand that was exclusively agented by Deda Group. The R & D center is located in Munich, Germany. The R & D team is consisted of international industry leaders, professors, overseas doctors, masters and industrial senior engineers. DEDAKJ is operated around three health sectors of high-end medical devices, healthcare devices and health appliances. At present, the products cover the household health oxygen instrument, medical-level oxygen instrument, atomizer, blood-pressure meter, podiatry machine, eye protection instrument, cervical massager, massage chair, knee physiotherapy device, vision training instrument, electric toothbrush, tooth-washing device, water purifier, etc. DEDAKJ is committed to allowing people's life to be more high-quality, with inheritance of moral quality as its initial heart, fashion appearance and modern technology as its brand personality!

CATALOG

| | | | |
|-------------------------|----|------------------------------|----|
| Attentions | 01 | Product function | 08 |
| Questions and Solutions | 03 | Use method | 09 |
| Spec. and parameters | 04 | Fault repair | 13 |
| Product feature | 05 | Steps on battery dismantling | 14 |
| Product configuration | 06 | Reservation protection | 15 |
| Key operation | 07 | | |

Attentions

- In order to ensure that you use this product correctly and safely, please read the Instruction Manual carefully before the use.
- The company shall not bear any consequences arising from improper use or non-compliance with the instructions relevant to this product.
- This product will block your vision. Do not use this product while walking or driving.
- This product has the heating function, those who are insensitive to the heat must pay attention to using it.
- Do not use this product if there are discomfort symptoms such as skin damage, swelling inflammation, congestion on massage area.
- Do not use if those have operation, glaucoma, cataract, Retinal detachment, eye disease, high intraocular pressure or discomfort on massage area.
- After the use, the visual field is blurred at first, close your eyes for 1-3 minutes, and the it will slowly turn clear.
- Please charge the product with the USB cable.
- Please stop it if you feel uncomfortable during the use.

- Do not use in the wet environment such as bathroom.
- Please keep this product away from water, fire and corrosive environment.
Pregnant, menstrual women do not use massager.
- Please remove the contact lens, hair clip and hair accessories to avoid discomfort during massage before use.
- If it's necessary to clean, you should wipe with the disinfection cotton of neuter cleanness, do not use the clean water.
- This product is designed as the health-care product and is not applicable for diagnostic, therapeutic or other commercial use.
- Turn off the power and remove the battery before discarding the product.
- Batteries must be removed and discarded in accordance with local laws and regulations before the disposable.
- Please obey the local laws and regulations to dispose of the used batteries.
- Please use the product as stated in the instruction manual. We will not bear any loss or injury caused by improper use.

Common questions and solutions about eye massager

1. The sound is heavy in massaging?

Answer: The pressing and contracting of the air bag is a process of repeatedly charging and deflating the air bag by the pump. In the process of charging and deflating, the sound of rapid going in and out of the air bag is normal.

2. After massaging, your eyes were blurred?

Answer: When closing eyes to massage for a long time, eye muscles are relaxed, it needs to adapt to the light when opening eyes. If you start the hot compress, eye blood flow will speed up, becoming blind, this is a normal phenomenon and can be alleviated in a short time.

3. The massager doesn't fit the face perfectly?

Answer: The proper gap is to reserve space to inflate the air for the balloon. If the fit is too tight, the balloon will compress the eye after inflated, which will make eyeball uncomfortable.

4. How to clean the flannel liner?

Answer: It's advised to wipe thoroughly after spurt wet paper towel hind with the medical alcohol. Wipe clean and stay for 10-15 minutes. Use it till the alcohol is completely evaporated.

5. How long and how often should it be used?

Answer: it is advised to use 1-2 times a day and 15 minutes each time. Appropriate time and frequency can massage the eyes in full while also avoiding revised effect of the excessive long-term massage eyes acid.

Spec. and parameters

Name: Intelligent AI Eye Massager

Model: K-GE22e

Color: Morandi green

Rated input: 5V \Rightarrow 1A

Rated power: 5W

Weight: 335g

Size: L203xW78xH115

Batter capacity: 1200mAh

Timing: 15min

Implementation standard: EN55014 EN60335 FCC PART 15 B

Product feature

1. The 6-mode massage meets the eye massage with different fatigue, optional freely, care for your eyes.
2. 6-airbag massage fully massage each eye area.
3. Precise control of heating temperature, constant temperature hot compress, promote blood circulation to relieve eye fatigue.
4. Bluetooth music function. Connect to mobile phone Bluetooth, enjoy the massage and music physical and mental pleasure from the music.





5. Slide switch mode, one slight stroke, and it's easy to switch mode.
6. 180° foldable, sending storage bag, it's easy and convenient to carry.

Product configuration

Massager host X 1
Storage bag X 1
USB charging cable X 1
Manual instruction X 1
Qualification certificate X 1

Key operation

1. Speaker port
2. On-off key
3. Slide touch function mode to switch key
4. Charging interface/mark outlet
5. Retractable buckle

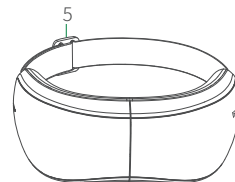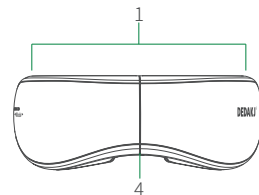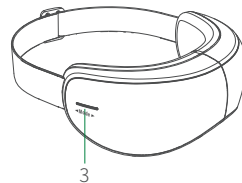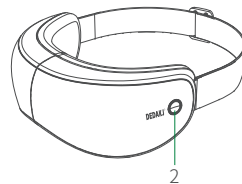

Product function

Please remove your contact lens, hair clip and hair accessories to avoid discomfort before use. Do not operate the product when your hands are wet.

Preparation before use:

Start: Long press switch for 2 seconds and it will start.

Shutdown: In the state of power on, long press the switch key for 2 seconds, or send voice command “please shutdown” and it will shut down.

Restore factory settings: long press “switch key” to connect USB interface for power, continue long press for 10s to restore the strength adjustment, volume adjustment and the factory default settings.

Warning sign

Power indication: When the power is lower, the red light of the switch key will flash; The red light will flash when the voice prompt that power is

lower and charging; The white light is always on when the power is full. In the process of charging, the product cannot work. Charging time is 2-3 hours.

Use method

Mode switching: Right hand slide to the right back (or left front) side, the function mode will switch

Mode 1: Standard eye massage mode (air pressure + hot compress + music + timing)

Open the air bag for massaging, hot compress, play the music, start the 15-minute for timing massage. This model is the standard massage model, which is suitable for daily eye massage.

Mode 2: Enhanced eye massage mode (air compress+ vibration + hot compress+ music +timing).

Switch to Mode 2, start the airbag massage, vibration massage, hot compress, play music, start 15 minutes timing massage. This model is suitable for long-time work, study, driving and other eye fatigue when the eye massage care.

Mode 3: hot compress eye massage mode (hot compress+ music +timing)

Switch to Mode 3, turn on the heat compress, play the music, and start the 15-minute timing massage. This model is suitable for light eye massage during the intermission of work, study, driving and so on.

Mode 4: Sleep eye massage mode (hot compress + sleeping music +timing)

Switch to Mode 4, turn on the heat compress, play sleep music, and start a 15-minute timing massage. This model is suitable for eye massage at the night rest, travel rest, relax before the night sleeping.

Mode 5: Massage eye massage mode (air pressure+ timing)

Switch to Mode 5, open the airbag massage and start the 15-minute timing massage. This model is suitable for light eye massage during the intermission of work, study, driving and so on.

Mode 6: Acupoint eye care mode (vibration +pressure +hot compress+ timing)

Switch to Mode 6 and start the acupoint massage function hot compress and start the 15-minute timing massage. This model is suitable for light eye massage during the intermission of work, study, driving and so on.

Strength adjustment: Press the switch key 2 times in a short way continuously, you will hear the voice prompt “you have entered the strength adjustment mode”. Right hand slides to switch the function mode toward the right (or the left front) side, massage strength changes from Gear1-5, the greater the number is, the greater the strength will be. Stop operation for 10 seconds and then exit strength adjustment mode.

Volume adjustment: press the switch key 3 times continuously in a short way, and you will hear the voice prompt “you entered the volume adjustment mode”. Right hand slides the function mode switch key toward the right (or left front) side, volume change from Gear 1-5, the greater the number is, the greater the volume will be. Stop operation for 10 seconds and then exit volume adjustment mode.

Bluetooth connection: Open your phone or computer Bluetooth to search for Bluetooth device. You can find Bluetooth device “Intelligent AI

Eye Massager”; Click to complete the pairing and you’ ll be able to play music via Bluetooth.

Fault repair

If the failure happens during operation, please perform the following checks. If the failure cannot be eliminated or the product is still not ideally operated, please cut off the power to professional and technical personnel for maintenance. Please do not disassemble this product!

| Reduce the strength | Possible causes | Basic solutions |
|-----------------------------------|----------------------------|--|
| Automatic shutdown after power-on | The lithium battery is low | If yes, please connect the power adapter to charge. If the charging is still failed, please replace another battery. |
| Can’ t turn the power on | | |
| Weak massage | | |

Steps on battery dismantling

Basic steps on the battery dismantling:

Cut off the external power supply or shut down, remove the product shell.

Cut off two power cords that connect the battery in turn.

Remove the battery and stick to the broken section of the power cord with the insulation tape.

Wasted batteries should be placed at a fixed collection station.

Please follow local environmental laws and guidelines to dispose of the battery and care for the environment.

- The battery must be removed from the appliance before it is discarded.
- The appliance must be powered off when removing the battery.
- Batteries should be disposed of safely.
- Please do not dismantle it by yourself.



Attention

Reservation protection

Reservation

If the storage term exceeds 3 months, the product should be checked and recharged. Do not fit to the floor, walls or roof in the storage. Indoor it should be well ventilated well, avoid the exposure to the strong sunlight and corrosive gas erosion.

Transport

The product packaging is equipped with simple anti-vibration facilities, and suitable for air, rail, road and ship transport, should avoid rain, snow splash, avoid, inversion and collision.

Power check

Check the power supply, if the power supply voltage is beyond the prescribed range, it shall not be turned on.

Maintenance

The measures against moisture, rodent and pollution shall be taken. Cleaning should be carried out in the state of shutdown. After the use, the instrument should be placed in a safe place. Do not place heavy matters on the instrument. When there is a breakdown, the professional and technical personnel should be informed for maintenance, non-professional personnel do not dismantle and assemble the product.

Disposal and recovery

This product should not be discarded with household wastes. When you decide to dispose of this product, please do so in accordance with local environmental laws and guidelines.

Warning:

Changes or modifications to this unit not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

NOTE:

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

FCC Statement

This device complies with part 15 of the FCC rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation. Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

NOTE: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

The device has been evaluated to meet general RF exposure requirement. The device can be used in portable exposure condition without restriction.

