
Femometer Lilac Kegel Exerciser II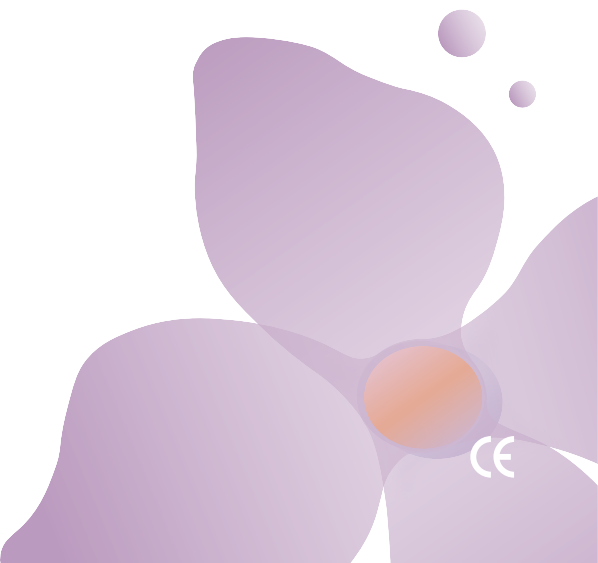

English ----- 01 - 12

German ----- 13-23

Italian ----- 24-34

French ----- 35-45

Spanish ----- 46-56

INTENDED USE

Designed to improve pelvic floor muscles after childbirth or after a decline in pelvic floor strength. Designed to help improve the contractive force of pelvic floor muscles, relieve the stress of urinary incontinence and reduce prolapse of vagina and uterus.

EN

WARNING

1. People who are pregnant should consult with a physician before using Lilac Kegel Exerciser.
2. Do not use Lilac Kegel Exerciser for the first 6 weeks after childbirth or pelvic surgery as it may interfere with healing.
3. Using Lilac Kegel Exerciser during active symptoms of any pelvic disease such as herpes, sexually transmitted diseases (STDs), vaginitis, or yeast infection may cause discomfort and worsen your symptoms.
4. Lilac Kegel Exerciser is for your personal use only and should not be shared with other users as it may spread disease.
5. Do not use Lilac Kegel Exerciser at the same time as a barrier contraceptive device (such as diaphragm or cervical cap) or a pessary as it may interfere with the proper use of these devices.
6. If you regularly use Lilac Kegel Exerciser to treat urinary incontinence and your symptoms do not improve, see your doctor.
7. Do not dismantle the unit. Only authorized personnel should carry out any repairs.
8. Handle the unit with care.

CONTRAINDICATIONS

Lilac Kegel Exerciser should not be used in the following situations unless you have consulted a physician:

During menstruation:

A place where there is a disease/disorder such as inflammation or infection in the vaginal area. When using contraceptives such as diaphragms, cervical caps, or pessaries.

During pregnancy:

In the first 6 weeks after childbirth.

In the first 6 weeks after an injury and/or operations in the vaginal or pelvic area.

Where you're using implants involved in artificial insemination.

If you have dilated veins in the vaginal area.

Areas where there are malignant tumors.

If you have a prolapsed uterus.

If you're suffering with hypertension of the pelvic floor.

BUTTON DESCRIPTION


Button Name	Operation	Function
Power Button	Long Press	Power On/ Off
Reset Button	Short Press	Restore Settings

SPECIFICATION

Product Name	Lilac Kegel Exerciser
Model	FM-LILAC-201
Maximum Noise Value	≤ 85dB
Input Voltage	5V
Rated Current	≤ 200mA
Material	Body-safe silicon/ ABS
Size	Length: 101.7mm. Diameter: 35mm
Weight	61g
Battery Capacity	200mAH Li-Ion Polymer
Vibration Frequency	7200 RPM
Operation Time	3 Hours
Pressure Sensor	10mmX3
Waterproof	IPX6
Classificaton	Type BF Equipment
Class of Pollution	Class 2
Working Conditions	Temperture: 5°C~40°C Humidity: ≤80% Air pressure: 86kPa~106kPa
Storage Conditions	Temperture: -20°C~55°C Humidity: ≤93% Air pressure: 70kPa~106kPa

FEATURES

This product has a built-in pressure sensor. When the device is under pressure, vibration feedback will be generated. This product can be combined with Kegel Women's fitness App for online pelvic floor muscle training. You can use the vibration feedback function to use the product offline.

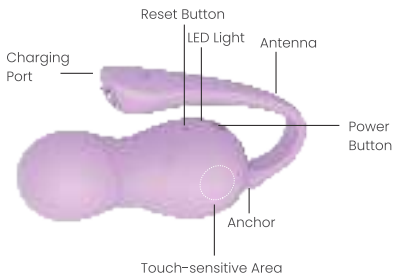TROUBLESHOOTING

Error condition	Problem	Solution
White light is flashing	Disconnect the app	Make sure your phone's bluetooth is turned on, then restart lilac.
Red light is flashing	Low battery	Recharge
When charging, the red light is off	Full battery	-

INDICATOR LIGHT

State	State Description
White Light flashing	The device is turned on but disconnected.
No light	The device is off. / The device is out of power. The device is on and connected . The device is disconnected but under pressure.
Red light on	The device is charging.
Red light flashing	The device is low battery.
Red light from on to off	The device is full of battery.

USAGE PROCEDURES

Step 1: Download the App

Go to the App Store or Google Play, search for "Kegel Exercises for Women". Or scan the QR code to download.

Step 2: Connect to the Device

1. Press the Power button on Lilac II. When the light is flashing, the device is on.


Open the app and scan below QR code for free courses which can guide you to do more professional and standardized kegel exercises.

2. When a connection message pops up on your phone, it means that the device is successfully connected. The device lights up and stops flashing.

Step 3: Feel confident with new pelvic floor strength

Insert the Lilac II into the body slowly with the indicator lights up, and start exercising (a water-soluble lubricant or condom is recommended).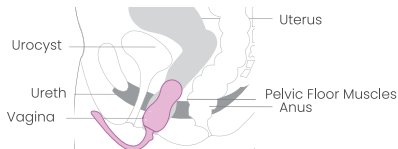

Note: Please leave at least 4 cm of the cord of Lilac II outside the body. This helps maintain a good connection.

Step 4: Power Off


Press the power button for 3 seconds to turn the device off. The indicator light will go off to indicate the product is now off.

Note: Lilac II will automatically turn off after disconnecting from the App for 5 mins.

NON-APP MODE

Turn on Lilac II and squeeze the pelvic floor muscles to activate the vibration only setting.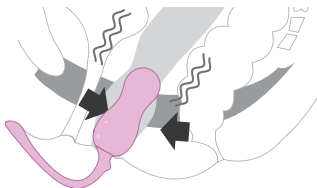

NON-BLUETOOTH MODE

Turn on Lilac II and squeeze the pelvic floor muscles to activate the vibration only setting.

After the device scans the code for the first time and syncs to the APP, you can choose the APP exercises in the non-Bluetooth mode. However pressure data is not available in non-Bluetooth mode.

BATTERY CHARGING

Step 1:

Align the charging cable with the magnetic suction head at the bottom of the device. The red light will be on when charging.

Note: Please make sure that the arrow of the charging head is facing upwards.

Step 2:

Connect USB to your smartphone power adapter or computer to charge.

**Step 3:**

The indicator light will be on if the device is charging normally. The light will turn off when charging is finished.

Note: When using it for the first time, please fully charge the device and keep it charged for an extra hour.

CLEANING AND STORAGE

Please clean Lilac II with water-soluble detergent or anti-bacterial soap and let it dry.

Keep Lilac II in a cool and ventilated place. Keep out of direct sunlight and out of reach of children.

The product has an expected 'use-life' of 5 years. Please recycle according to local guidelines if the product is not longer in use.

Note: Not recommended to use the device during menstruation.

Please consult a physician before use if you have any gynecological symptoms or discomfort.

FAQ

Q: My phone cannot connect to the App. Why?

A: First, please ensure that Femometer App has permission to access to Bluetooth. Second, check whether Bluetooth is turned on and whether your device is turned on (the indicator light will flash after turning on). If all the above conditions are met, please restart the device by pressing the power button for 3 seconds and then pressing the power button again.

Q: Why does the device keep disconnecting during use?

A: The cord of Lilac is the antenna. Please ensure that there is at least 4cm of cord outside your body. This ensures a better connection.

Q: I was tensing very hard but no change in the pressure is being shown. Why?

A: The device may be placed in the wrong position, resulting in the pelvic floor muscles not exerting force on the sensor. Please adjust the angle of Lilac to ensure the indicator light is facing upward and there is at least 4cm of cord outside your body.

FAULT DIAGNOSIS

1. In stand alone mode, turn on the product and press the pressure sensing area. If there is no vibration, please check whether the battery needs charging.

2. In the mobile phone control mode, turn on the product. If the connection cannot be established with the app, please check the phone and power supply.

PRECAUTIONS FOR THE SAFE USE OF THE BUILT-IN BATTERY

1. Please use the USB cable to charge the product battery.
2. If the battery liquid comes into contact with your skin, please wash with water and seek medical assistance.
3. Please dispose of used batteries according to local (national or regional) regulations.
4. Do not drop the battery from any height. Avoid damaging the battery with sharp objects.
5. Do not expose the battery to any metal objects (including coins). This can avoid short circuit, discharge, heat generation or leakage.
6. Do not try to disassemble the battery. This may damage the device.
7. Do not heat the battery or drop it into fire, this could cause an explosion.
8. Do not store in high temperatures and in strong, direct sunlight.
9. If overheating occurs while charging or using the product, please stop charging or using it immediately.

CONTAINS

Lilac Kegel Exerciser	X1
USB Cable	X1
User Manual	X1
Storage Bag	X1

WARRANTY TERMS & CONDITIONS

1.We offer a period of ONE (1) YEAR from the date of original purchase to prevent defects caused by workmanship or materials. If you find any defects during the warranty period and notify the after-sales customer service team, we will replace the product free of charge as appropriate.

2.The warranty covers working parts that affect the function of the product. It does NOT cover cosmetic deterioration caused by fair wear and tear or damage caused by accident, misuse or neglect. Any attempt to open or take apart the product (or its accessories) will void the warranty.

3.Claims under warranty must be supported by reasonable evidence that the date of the claim is within the warranty period. To validate your warranty, please keep your original purchase receipt together with these warranty conditions for the duration of the warranty period.

EN

INDEX OF SYMBOL			
	Type BF Equipment		Manufacturer
	The product must be in a disused separate collection and recycling, and may not be mixed with other waste and scrap		
	Read user manual before use		

FCC WARNING STATEMENT

Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment. This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- - Reorient or relocate the receiving antenna.
- - Increase the separation between the equipment and receiver.
- - Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- - Consult the dealer or an experienced radio/TV technician for help.

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and
(2) this device must accept any interference received, including interference that may cause undesired operation.

The device has been evaluated to meet general RF exposure requirement.

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Lollipop Technology(Hangzhou)Co., Ltd.
Lotus NL B.V.