

Wrist Electronic Blood Pressure Monitor

Model No. : FC-BP200



Manual Ver.: 1.0

Software Ver.: FC-BP200-V1.0

Issuing Date: 2022/05/09

- Please read this instruction manual carefully before using your unit.
- Please keep this instruction manual well for future use.
- Thanks for choosing the Digital Blood Pressure Monitor.
- The PATIENT is an intended OPERATOR.

Revision Record

Document No.	Description	Software REV	Date
FRN-FC-BP200-001	First established	FC-BP200-V1.0	2022/05/09

Contents

Contents.....	3
1. Before Using the Unit.....	4
1.1 Introduction.....	4
1.2 Important safety notices.....	5
1.3 Warning and safety notices.....	6
2. Operating Instruction.....	6
2.1 Know your unit.....	6
2.2 Battery Installation / Replacement.....	8
2.3 Settings.....	8
2.4 Proper Use of the wrist Cuff.....	9
2.5 Take a Measurement.....	12
2.6 Use the Memory function.....	13
2.7 About Blood Pressure.....	15
2.8 About Blood Pressure.....	16
3. Care and Maintenance.....	17
3.1 Care and Maintenance.....	17
3.2 Calibration and Service.....	18
3.3 Error Indicators.....	18
3.4 Troubleshooting.....	18
3.5 Technical Data.....	19
3.6 Warranty Information.....	24

1. Before Using the Unit

1.1 Introduction

■ Thank you for purchasing Wrist Electronic Blood Pressure Monitor.

■ The unit uses the oscillometric method of blood pressure measurement. It means the unit

detects the movement of your blood through your brachial artery, and converts your blood

pressure into a digital reading. The unit is simple to use because a stethoscope is not needed












while using an oscillometric monitor.





■ The unit stores automatically 2*120 sets of measurement values. You can read the stored

data conveniently by pressing the memory button.

■ The unit comes with the following components:

- Main Unit
- Cuff
- Instruction Manual printed in English

SYMBOLS USED IN THIS INSTRUCTION MANUAL	
	Indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.
	Indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury to the user or patient or damage to the equipment or other property.
	Please read this instruction manual thoroughly before using the unit. Please keep for future reference. For specific information about your own blood pressure, CONSULT YOUR DOCTOR.
	Transport package shall be kept away from rain.
	AUTHORISED REPRESENTATIVE IN THE EUROPEAN COMMUNITY
	Refer to instruction manual/booklet
	MANUFACTURER
	SERIAL NUMBER
	Type BF applied part
	The marking of electrical and electronics devices according to Directive 2002/96/EC. The device accessories and the packaging have to be disposed of waste correctly at the end of the usage. Please follow Local Ordinances or Regulations for disposal
	Device used within the Magnetic Resonance (MR) environment is prohibited.

	Date of manufacture
	Model number
	Medical device
	Country of manufacture

1.2 Important safety notices

To assure the correct use of the product basic safety measures should always be followed including the precautions listed as below:

- ☐ Please read all information in the instruction manual and any other literature included in the box before using the unit.
- ☐ Please contact your physician for specific information about your blood pressure. Self-diagnosis and treatment by measure results may be dangerous. Follow the instructions of your health care provider.
- ☐ Operate the unit only as intended. Don't use for any other purpose. The unit is intended to use in measuring blood pressure and pulse rate for adult only, not recommended using for neonatal baby at home or medical centre.
- ☐ Do not use a cellular phone near the unit. It may result in operational failure.
- ☐ Please avoid using in high radiant area in order to make your measuring data correctly.
- ☐ Do not disassemble or attempt to repair the unit or components. Do not use the equipment in places where there are flammable gas (such as anesthetic gas, oxygen or hydrogen) or flammable liquid (such as alcohol) .
- ☐ Do not use a mobile phone or other devices that emit electromagnetic fields near the unit. This may result in incorrect operation of the unit.
- ☐ Indicating that too frequent measurements can cause injury to the PATIENT due to blood flow interference;
- ☐ Please remove the batteries if the unit will not be used for Three months or More.
- ☐ Please do not put CUFF over a wound, as this can cause further injury;
- ☐ Regarding the effect of blood flow interference and resulting harmful injury to the PATIENT caused by continuous CUFF pressure due to connection tubing kinking;
- ☐ Do not install the wrong battery polarity.
- ☐ After the battery is exhausted, please replace four two batteries.
- ☐ If over three months do not use the device, please remove the battery as it may cause leakage, overheat, rupture and damage the blood pressure monitor body.

1.3 Warning and safety notices

Regarding the application of the CUFF and its pressurization on any limb where vascular access or therapy, or an arteriole-venous (A-V) shunt, is present because of temporary interference to blood flow and could result in injury to the PATIENT;

Regarding the application of the CUFF and its pressurization on the arm on the side of a mastectomy;

Pressurization of the CUFF can temporarily cause loss of function simultaneously used monitoring ME EQUIPMENT on the same limb;

Regarding the need to check (for example, by observation of the limb concerned) that operation of the AUTOMATED Blood Pressure Monitor does not result in prolonged impairment of the circulation of the blood of the PATIENT.

When the arm is oppressed by air pressure, please loosen CUFF or remove batteries.

Please do not touch the patient and battery output simultaneously when measurement.

Warning: Do not allow to use the lure connectors. If lure lock connectors be used in the construction of tubing, there is a risk that they might be inadvertently connected to vascular fluid systems and allowing air to be pumped into a blood vessel.

Warning: Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the Blood Pressure Monitor, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.

Warning: The Blood Pressure Monitor is placed in the position not touched by pets, pests or children.


Warning: The hazard of accidental ingestion due to the small parts that can be disassembled (such as batteries, etc.) after being touched by a child;

Place the product out of the reach of children, etc.; use it under the supervision of an adult, etc.;


2. Operating Instruction

2.1 Know your unit


Body

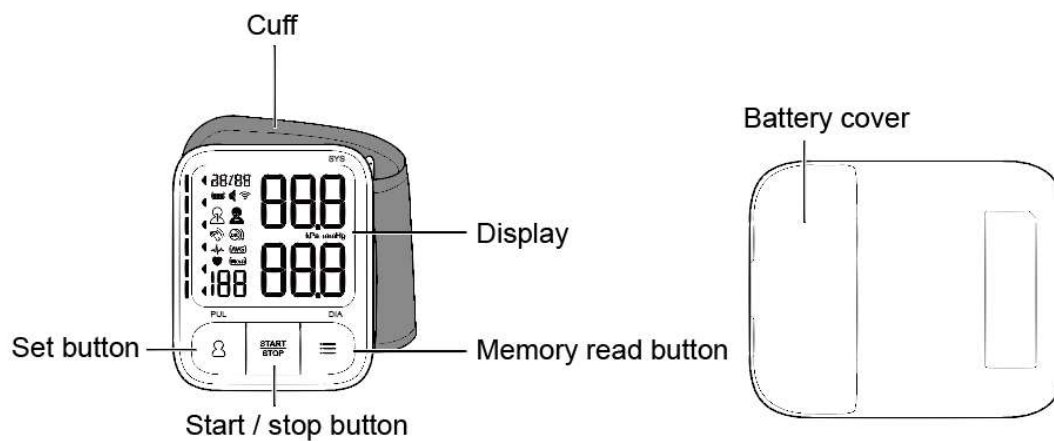

Display

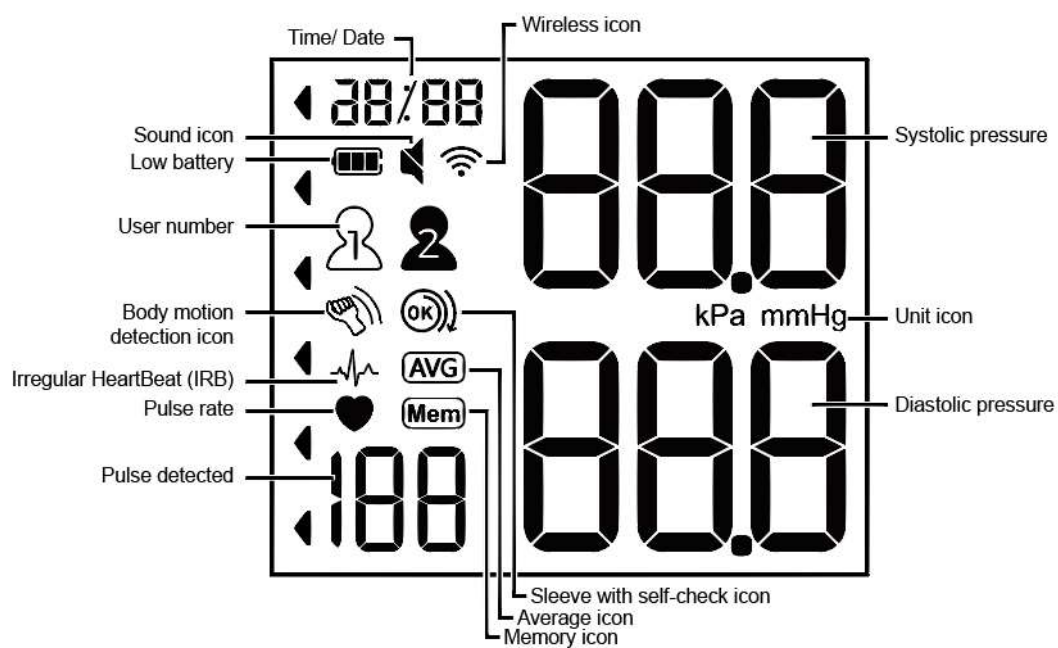


Cuff

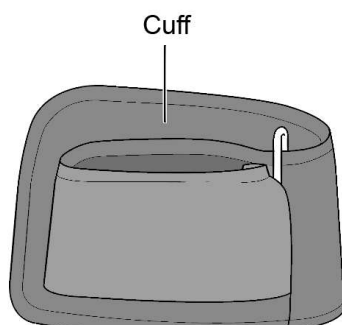

Power

2 AAA Alkaline Batteries

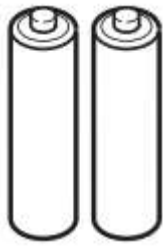

Do not use rechargeable batteries.

Instruction Manual

2 AAA Alkaline Batteries


- Please keep the instruction manual well after using.

2.2 Battery Installation / Replacement

2.2.1 Remove the battery covers from the battery compartment.

2.2.2 Install 2 “AAA” size batteries to match the + (positive) and - (negative) polarities with the polarities of the battery compartment as indicated.

2.2.3 Replace the battery cover

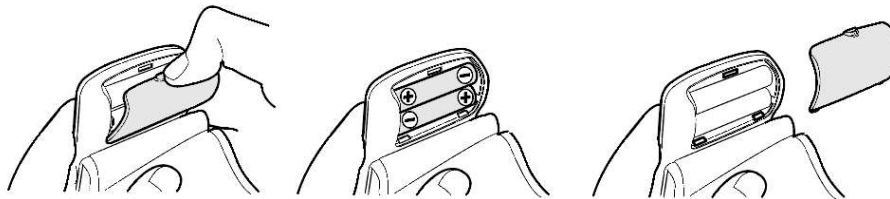


Caution!

- Replace batteries if the symbol of low battery appears on the display.
- Never leave any low battery in the battery compartment since they may leak and cause damage to the unit.

2.3 Settings

2.3.1 To set year

Long press the “SET” button when power off, you can see the year icon appears and flashes, then press the “MEM” to increase one year, long press the “MEM” button and the value quickly changes.

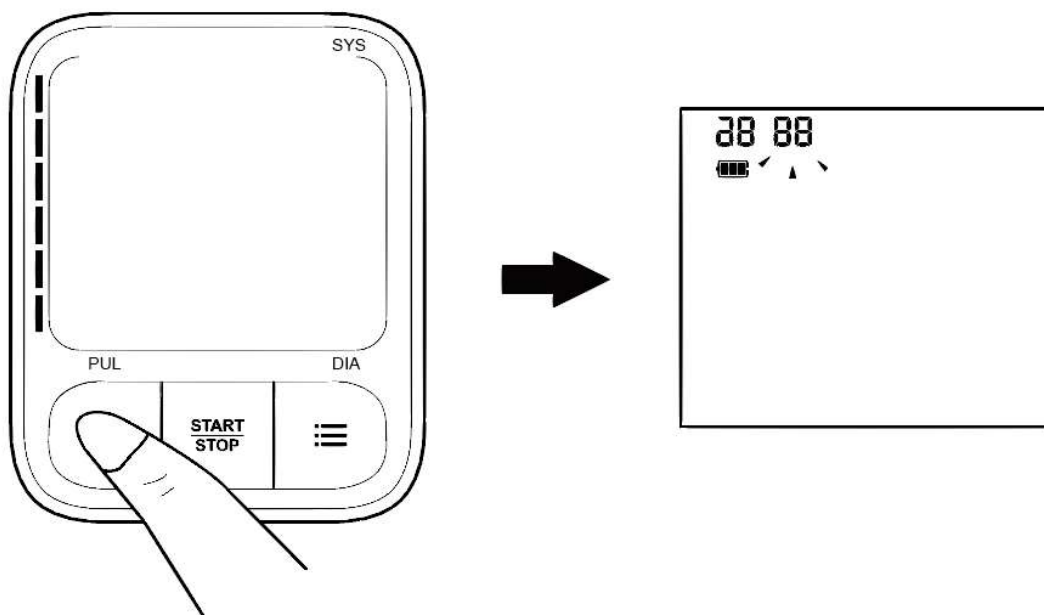


2.3.2 To set date and time


Based on first step and press "SET" button again. The mark of month and date appears and flashes, press the "MEM" button to change the month and press the "SET" button to continue with next step.

Then set the day, the hour and the minute by using the same way.


Month Setting

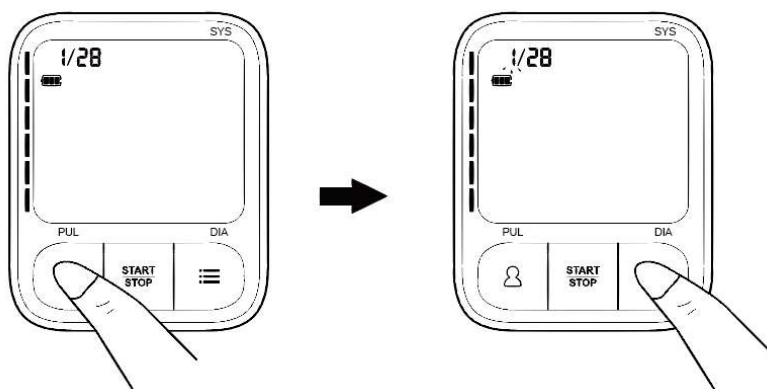


Day Setting

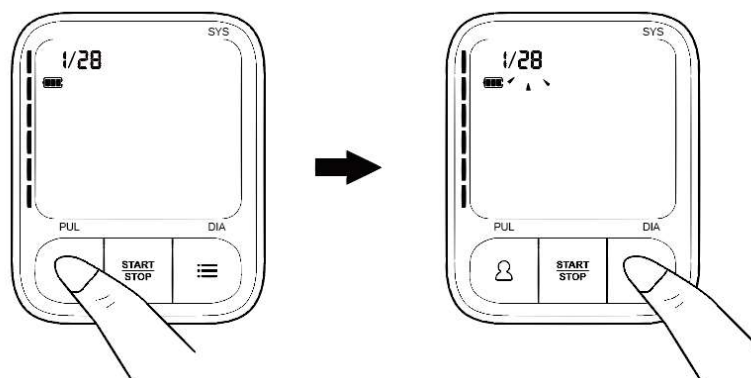

Hour Setting

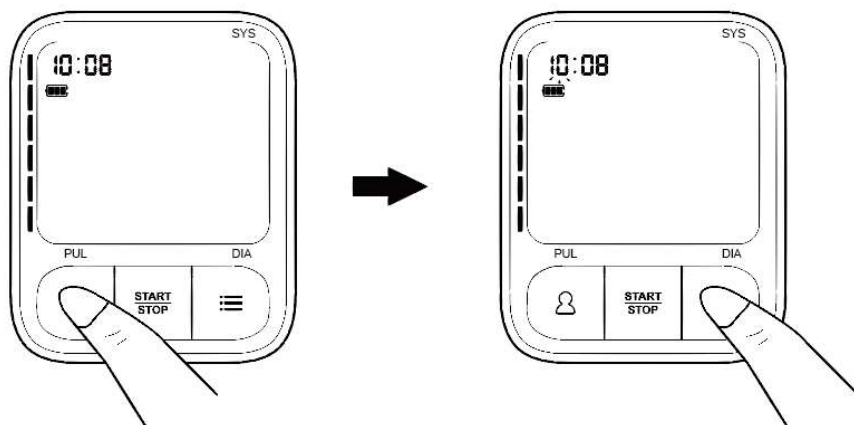

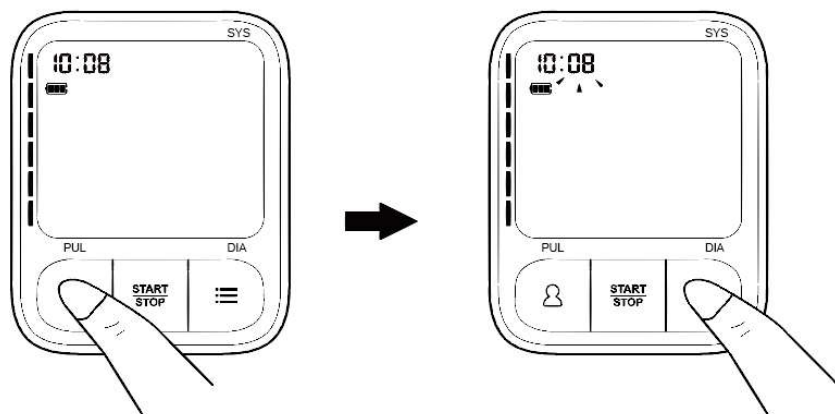

Minute Setting



2.3.3 To set unit


Based on the second step and press the “SET” button to change the previous unit(kPa or mmHg),

the mark “” appears and flashes, now the BPM is in mmHg status, and then press the “MEM”


button, to change it into “”, now the BPM is in kPa status.

Presss the “ start/stop ” button to confirm the current setting.

Unit Setting KPa & mmHg

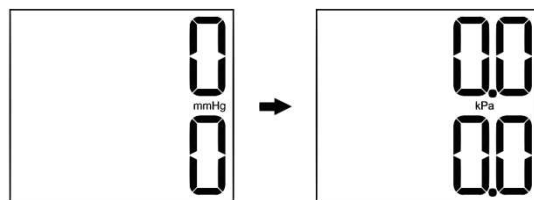


2.3.4 To set voice

Based on third step, press the “SET” button again, you can see the “” icon appears and flashes, then press the “MEM” button to adjust the sound on or off.


when the screen shows “”, Blood Pressure Monitor sound is on; when the screen shows”

SP
OFF ", Blood Pressure Monitor sound is off. Press "START/STOP" button to save the current setting and turn off.

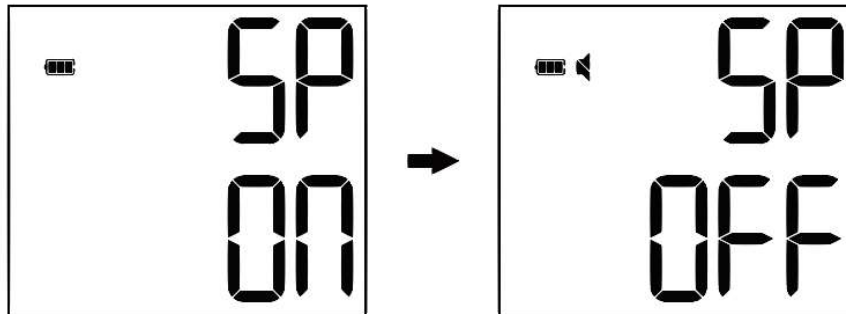

2.3.5 To set user


Press the "SET" button when power off, and then press the "SET" button again to switch User 1 or user 2.

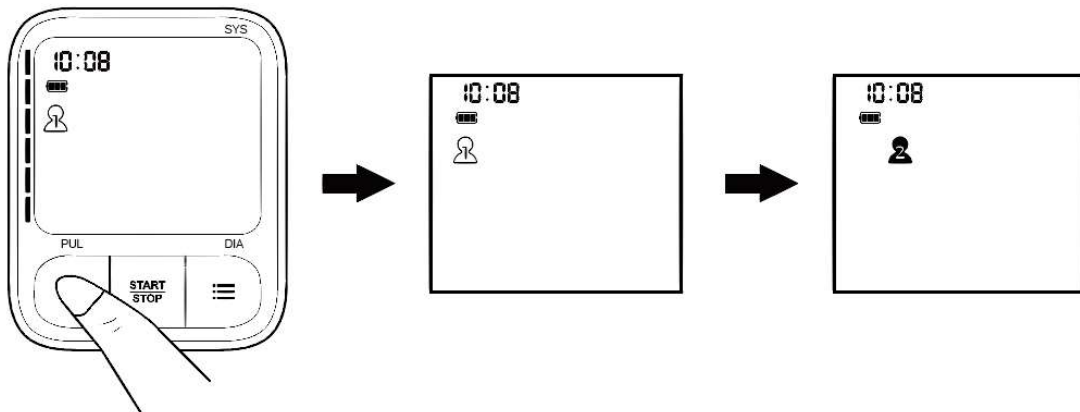


The operation of setting can be stopped at any time by pressing "START/STOP" button.

2.4 Use cuff correctly

1. With your palm up, wrap the cuff around your wrist.

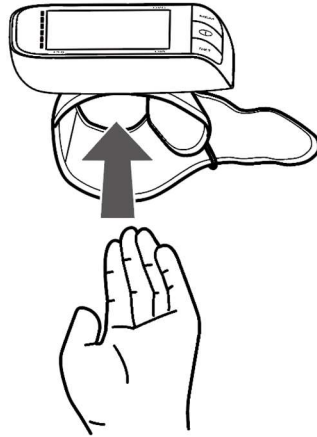


2. ※ Leave space between the wrist and palm
1cm 2cm (about one index finger)
The distance between the left and right width) to
avoid covering
Cover the ulnar bone (as shown).

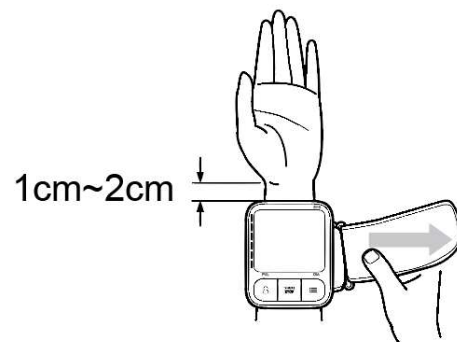

3. Hold the end of the cuff Lateral pull



4. Before the measurement, please rest for at least 15 minutes in a relaxed and quiet state. When measuring, place your elbows on the table, keep the height of the host and your heart consistent, palms up, and your body relaxed.

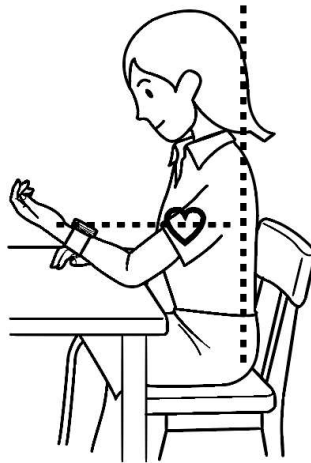


1. The device has the WHO warning strip “■” symbol display function, with the blood pressure value change, the higher blood pressure value “ ■” symbol will appear relatively in the higher position (refer to WHO definition of high blood pressure levels-reference table).
2. The device has the average of the last three display function (press the memory key for the first time to display the value that is the average of the last three measurements).
3. The device has a large screen display and digital legible.
4. The device has kPa and mmHg display switching and measurement functions.
5. The device has double memory lookup function and can store 120 set of measurements for each person to remember the normal status of your blood pressure.
6. Clock function: year, month, day, hour, minute setting.
7. Low power detection: detecting low power under any working state, LCD displays “■ ” symbol indicates low power.
8. Overpressure protection function: when the pressure is more than 300mmHg, the device will exhaust fast automatically.
9. Automatic shutdown function: no operation for 1 minute the device will shut down automatically.
10. Heartbeat prompting voice.
11. Measurement completed prompting voice.
12. Incorrect prompting function.


2.5 Take a Measurement

2.5.1 Press the “START/ STOP” button.


All display symbols appear on the screen. The cuff starts to inflate automatically.

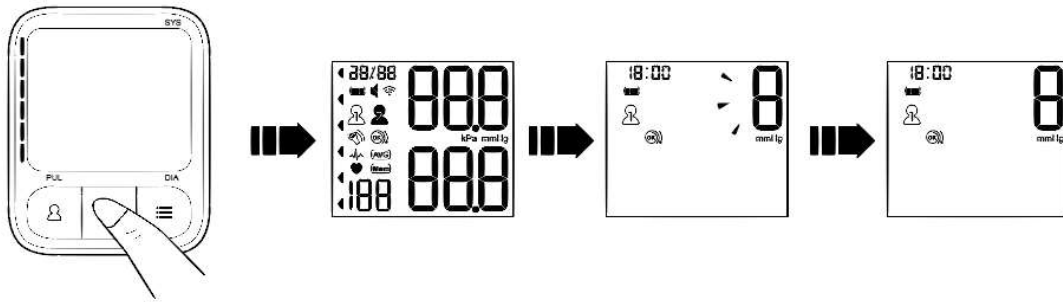


2.5.2 Measurement start. When pressurized to stop blood flow, the Blood Pressure Monitor stops pressure before automatically leaking air and automatically measuring blood pressure. The heartbeat symbol flashes once a pulse is detected. When the heart beat symbol appears and flashes, the Blood Pressure Monitor detects the pulse and begins to calculate the pulse automatically.

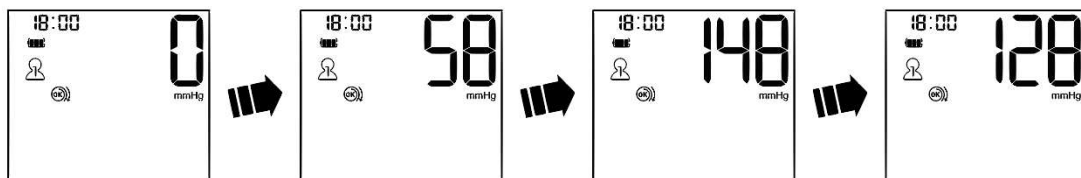

2.5.3 The blood pressure and pulse rate are displayed when the measurement completed. The cuff is deflated automatically, and all of the measurement results be stored in the memory with date and time.

The “” symbol will be displayed if irregular heartbeat is detected.

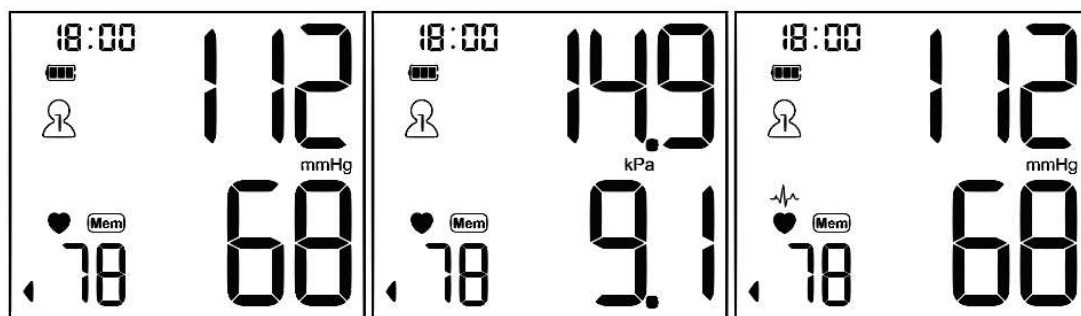

2.5.4 Press the “START/ STOP” button to turn off the unit.

The unit will automatically turn off after two minutes if no more operation.

NOTE: The inflation or measurement can be stopped by pressing the “START/ STOP” button at any time.

2.6 Use the Memory function


The unit stores the blood pressure and pulse rate in the memory after completing a

measurement each time. 2x120 sets of measurement values can be stored automatically. The earliest record will be deleted automatically to save the latest measurement value when more than 2x120 sets.

The unit also calculates an average reading based on the values of the latest 3 times measurement .

2.6.1 To enter the memory mode & to read the average value


Press the “MEM” button while the unit is off, the unit enters the memory mode and the average value of the latest 3 times measurement values of the current user will be displayed.

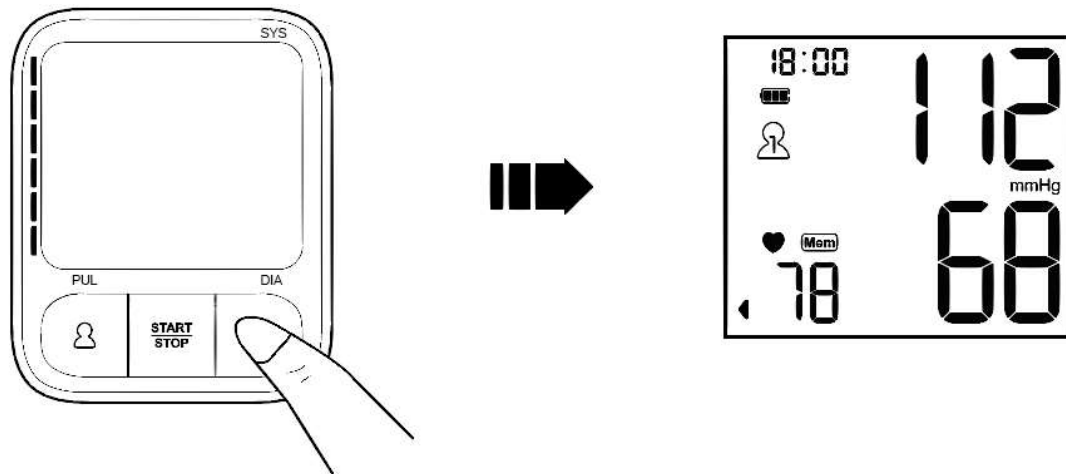

2.6.2 To read the measurement value

Press the “MEM” button when power off, the latest measurement values with the measuring month, date and time will be displayed.

Press the “MEM” button once more, the measurement values before the latest will be displayed, and so on.

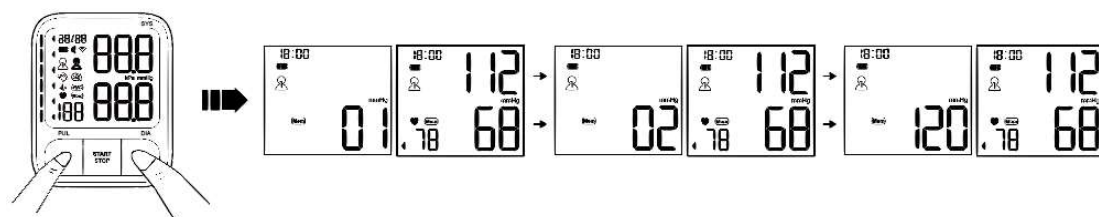


2.6.3 To delete the memory value

In the memory mode, long press the “MEM” button, the unit displays the following symbol to cancel all the memory records of the current user.

Please operate the delete memory function cautiously.

Press the “START/STOP” to cancel the memory records of the current user if you are sure all the data of corresponding user can be deleted, the unit will turn off after deleting.

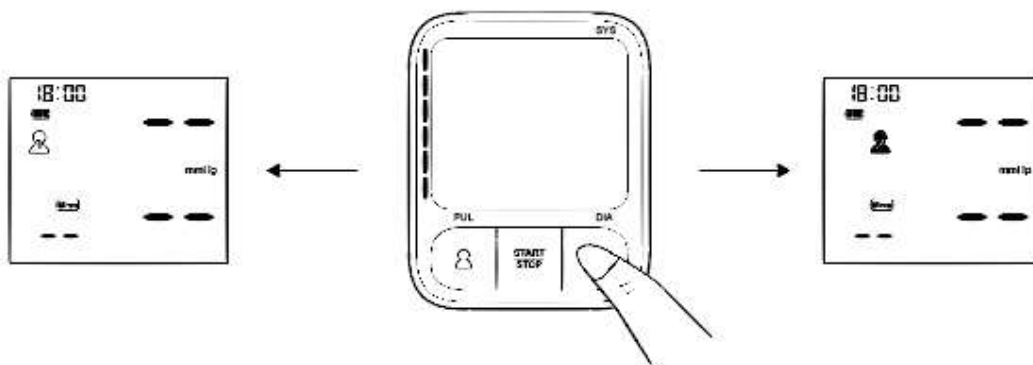


2.7 About Blood Pressure

2.7.1 Irregular Heartbeat Symbol IRB

When the unit detects an irregular rhythm two or more times during the measurement, the irregular heartbeat Symbol will appear on the display with the measurement values.

An irregular heartbeat rhythm is defined as a rhythm that is more than 25% slower or 25% faster from the average rhythm which detected while the monitor is measuring the systolic blood pressure and the diastolic blood pressure.

If the Irregular Heartbeat Symbol () displays with your measurement results, we recommend you consult your physician, and follow the doctor's directions.




2.7.2 Blood Circulation

The blood circulation is responsible for supplying the body with oxygen.

Blood pressure is the pressure exerted on the arteries.


The systolic blood pressure value (higher pressure or top value) represents the blood pressure produced by contraction of the heart muscle.

The diastolic blood pressure value (lower pressure or lower value) represents the blood pressure produced by relaxation of the heart muscle.

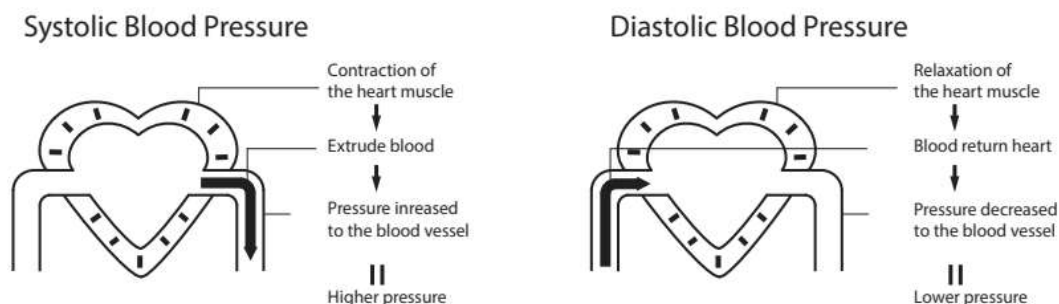

2.7.3 Health and Blood Pressure

The incidence of hypertension increases with age. In addition, if lack of exercise, excess body fat and high levels of cholesterol(LDL), would sticks to the inside of blood vessels, which reduces elasticity of these vessels. Hypertension accelerated arteriosclerosis which can lead to serious conditions such as stroke and myocardial infarction. For these reasons it is very important to know whether the blood pressure is within a healthy range. Blood pressure fluctuates from minute to minute, throughout the day. Therefore it is essential to take regular measurements to help you identify an average blood pressure.

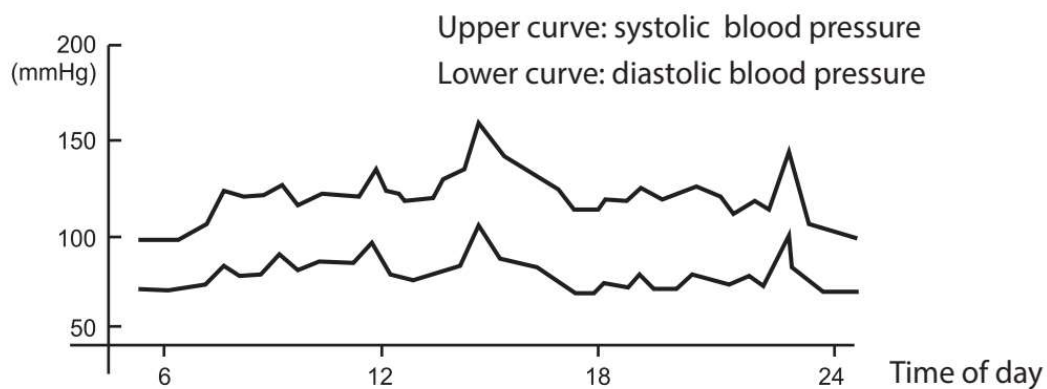


2.8 About Blood Pressure

2.8.1 Classification of Blood Pressure

After each measurement is completed, the LCD display will show your position automatically on the six segments of the bar indicator which corresponds to World Health Organization (WHO) Blood Pressure Indicator.

Reference Material: Journal of Hypertension 1999, Vol 17 No.2

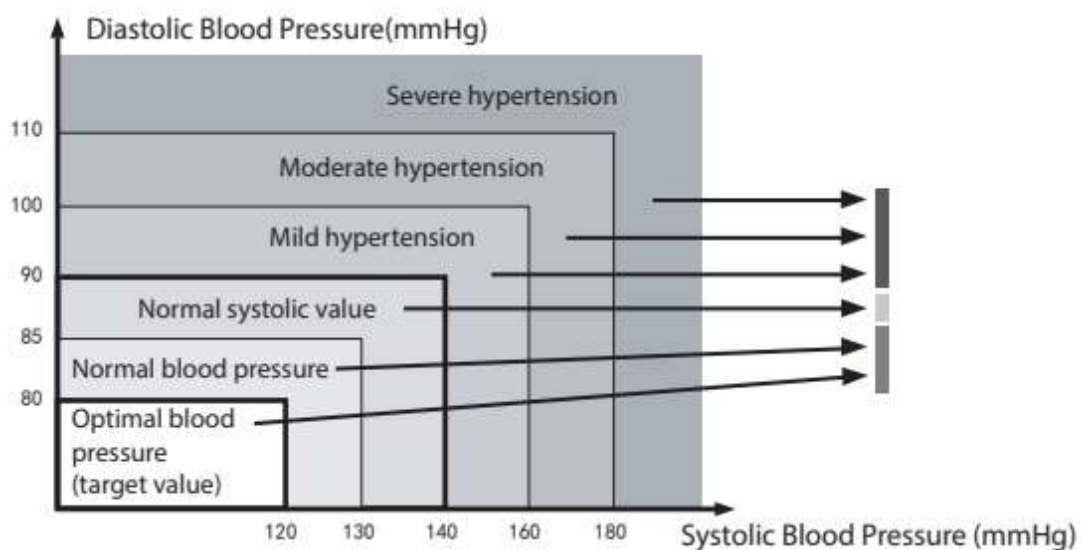

*Note!

When a person's systolic and diastolic pressures fall into different categories, the higher category should apply.

2.8.2 Symptoms of High Blood Pressure

High blood pressure can go unnoticed for a long time, since it doesn't cause noticeable symptoms. The following are possible causes of abnormally high blood pressure:

- Overweight;
- High cholesterol level;
- Smoking;
- Excessive alcohol consumption;
- Stress and emotional upset;
- Excessive consumption of salt;
- Lack of physical exercise;
- Genetic / hereditary predisposition;
- Underlying illnesses, such as kidney disorders or endocrine disturbance.

2.8.3 Treatment of High Blood Pressure

If your blood pressure reaches upper values of 140~160 mmHg and lower values of 90~95 mmHg in repeated measurements over several days, you should consult doctor for detailed medical examination. You can assist the treatment prescribed by your doctor in the following ways:

- Loose weight and lower your cholesterol level;
- Reduce the consumption of alcohol;
- Reduce the intake of salt;
- Stop smoking;
- Take regular exercise;
- Monitor your blood pressure.

3. Care and Maintenance

3.1 Cleaning/Disinfecting and Maintenance

To keep your blood pressure monitor in the best condition and protect the unit from damage, please follow the directions listed below:

3.1.1 Clean the blood pressure monitor frequently after use by a user. Do not use any abrasive or volatile cleaners. Never immerse the unit or any components in water.

3.1.2 Use a soft dry cloth towel to clean this blood pressure monitor, if it is very filthy you can wet the towel with water or neutral detergent, wring out it and wipe the monitor.

3.1.3 Disinfection the cuff with moistened 75% alcohol cotton wool.

3.1.4 Store the unit in a safe and dry location. Do not fold the cuff too tight. Do not subject the unit to extreme hot or cold temperatures/humidity and do not subject the unit under the direct sunlight.

3.1.5 Avoid subjecting the unit to strong shocks, such as dropping the unit on the floor.

3.1.6 Remove the batteries if the unit will not be used for three months or longer. Always replace all the batteries with new ones at the same time.

3.1.7 Use the unit consistent with the instruction provided in this manual. Use only authorized parts and accessories.

3.1.8 Do not carry out repairs of any kind yourself. If a defect occurs, Please contact the local distributor.


3.1.9 We will provide circuit diagrams, component part lists, descriptions, calibration instructions, or other information to assist service personnel who is trained by the manufacturer and has got the relevant qualification certificate.


3.2 Calibration and Service

The accuracy of this blood pressure monitor has been carefully tested and is designed for a long service life. It is generally recommended to have the unit inspected every two years to ensure correct functioning and accuracy. Please consult local authorized distributor or dealer.

3.3 Error Indicators

The following symbols will appear on the display when measuring abnormally.

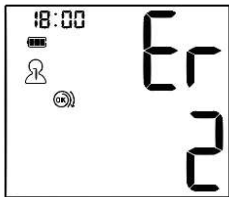Symbol	Cause	Correction
	The course of inflating appears error	Wrap the cuff correctly and tightly
		Inflate over again after ensuring
	When measurement fails	Do not move your arm and body and keep quiet
		Measure over again according to correct way
Err1:	Abnormal sensor vibration	
Err2:	Can't detect enough heartbeats or count blood pressure	
Err3:	Abnormal measurement results	
Err4:	The wristband is too loose or leaking (pressurized less than 30mmHg within 10 seconds)	
Err5:	Trachea is blocked	
Err6:	Large pressure fluctuations during measurement	
Err7:	The pressure exceeds the upper limit of 300mmHg	
Err8:	The calibration data is abnormal or not calibrated	

	When the batteries power is too low	Replace all of the worn batteries with new ones
---	-------------------------------------	---

3.4 Troubleshooting

Problem	Causes and Solutions
No power	Replace all of the worn batteries with new ones
No display appears on the display screen	Check the battery installation for proper placement of the batteries polarities
Measurement values appear too high or too low	Blood pressure varies constantly. Many factors may effect your blood pressure, including stress, time of day, how you wrap the cuff. Review the sections "Proper Way of Measurement" and "Take a Measurement"

3.5 Technical Data

3.5.1 Speci-cations

1	Model	FC-BP200
2	Measurement Method	Oscillographic measurement method
3	Display	LCD Digital Display
4	Wireless transmission	Bluetooth LE4.2 module
5	Measurement Range	Blood Pressure range: 0~299mmHg (0 kPa - 39.9 kPa) Pulse: 40 to 180 beats/min
6	Accuracy	Static Pressure: $\pm 3\text{mmHg}(\pm 0.4\text{ kPa})$ Pulse rate: Within $\pm 5\%$ of reading
7	Inflation	Automatic inflation by pump
8	Deflation	Automatic rapid deflation
9	Pressure Detection	Semiconductor pressure sensor
10	Memory	2 persons * 120 memories
11	Power supply	2 AAA alkaline batteries (not included), DC3V
12	Battery life	Approximately 300 measurements when using alkaline batteries at the room temperature of 22°C and by using three times a day and inflating to 170 mmHg
13	Storage Temperature	-4°F to 140°F (-20°C to 60°C), and Humidity: 10 to 95% RH Atmospheric pressure: 70 - 106kPa
14	Operating Temperature	50°F to 104°F (10°C to 40°C), and Humidity: 15 to 85% RH

15	Automatic Power-OFF	Within 2 Minutes
16	Weight of Main Unit	About 95.4 g (Not including batteries)
17	External Dimensions	80. 4×65. 5×24mm
18	Cuff	13.5-21.5cm
19	Electric Shock Protection	Internal power supply appliance type B
20	Accessorial Components	Cuff, Instruction Manual
21	Device Life	5 Years

To improve performance, these specifications are subject to change without notice.

The device, accessories and the packaging have to be disposed of waste correctly after the end of the usage, so that the risk of patient or user can be lowered to acceptable level.

3.5.2 Statement

- The unit is intended to be used to measure blood pressure (systolic and diastolic) and heart rate from the upper arm by using the oscillometric method.
- The unit is intended for use in only adult population, not applied to other populations such as neonatal baby.
- It can't be used while the wrist (arm) has bleeding or wound to avoid the blood flowing from the wound when pressurizing.
- The device's accessories and batteries have to be disposed of waste correctly at the end of the usage, please follow Local Ordinances or Regulations for disposal.
- Applied part: CUFF.

Protection Class: Internally powered equipment.

• Applied Part Type: Type BF.

Moisture Protection: IP21, continue operation.

Altitude<2000m;

Overvoltage:II;

Pollution degree:2

- The risk of patient and user can be lowered to acceptable level.
- The unit might not meet its performance specification if stored or used outside the following specified temperature, humidity and altitude ranges.
- The unit satisfies the requirements of IEC60601-1 Medical electrical equipment,
IEC 60601-1-2: Electromagnetic compatibility –Requirements and tests
IEC 80601-2-30: Non-invasive sphygmomanometers - Part 1: Requirements and test methods for non-automated measurement type EN1060-3: Non-invasive sphygmomanometers - Part 3: Supplementary requirements for electro-mechanical blood pressure measuring systems.

3.5.3 IEC 60601-1-2: 2014 ME EQUIPMENT and ME SYSTEMS identification, marking and documents for Class B product

The ME EQUIPMENT or ME SYSTEM is suitable for home healthcare environments and so on.

Warning: Don't near active HF surgical equipment and the RF shielded room of an ME system for magnetic resonance imaging, where the intensity of EM disturbances is high.

Warning: Use of this equipment adjacent to or stacked with other equipment should be avoided

because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.

Warning: Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.”

Warning: Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the Wrist Electronic Blood Pressure Monitor (M/N: FC-BP200), including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.

If any: a list of all cables and maximum lengths of cables (if applicable), transducers and other ACCESSORIES that are replaceable by the RESPONSIBLE ORGANIZATION and that are likely to affect compliance of the ME EQUIPMENT or ME SYSTEM with the requirements of Clause 7 (EMISSIONS) and Clause 8 (IMMUNITY). ACCESSORIES may be specified either generically (e.g. shielded cable, load impedance) or specifically (e.g. by MANUFACTURER and EQUIPMENT OR TYPE REFERENCE).

If any: the performance of the ME EQUIPMENT or ME SYSTEM that was determined to be ESSENTIAL PERFORMANCE and a description of what the OPERATOR can expect if the ESSENTIAL PERFORMANCE is lost or degraded due to EM DISTURBANCES (the defined term “ESSENTIAL PERFORMANCE” need not be used).

Technical description

1.all necessary instructions for maintaining BASIC SAFETY and ESSENTIAL PERFORMANCE with regard to electromagnetic disturbances for the expected service life.

2. Guidance and manufacturer’s declaration -electromagnetic emissions and Immunity

Table 1

Guidance and manufacturer’s declaration - electromagnetic emissions	
Emissions test	Compliance
RF emissions CISPR 11	Group 1
RF emissions CISPR 11	Class B
Harmonic emissions IEC 61000-3-2	Class A
Voltage fluctuations/ flicker emissions IEC 61000-3-3	Compliance

Table 2

Guidance and manufacturer’s declaration - electromagnetic Immunity		
Immunity Test	IEC 60601-1-2 Test level	Compliance level
Electrostatic discharge (ESD) IEC 61000-4-2	±8 kV contact ±2 kV, ±4 kV, ±8 kV, ±15 kV air	±8 kV contact ±2 kV, ±4 kV, ±8 kV, ±15 kV air
Electrical fast transient/burst IEC 61000-4-4	Power supply lines: ±2 kV 100 kHz repetition frequency	Power supply lines: ±2 kV 100 kHz repetition frequency
Surge IEC 61000-4-5	line(s) to line(s): ±1 kV.	line(s) to line(s): ±1 kV.
Voltage dips, short	0% 0.5 cycle At 0°, 45 °, 90 °,	0% 0.5 cycle At 0°, 45 °, 90 °,

interruptions and voltage variations on power supply input lines IEC 61000-4-11	135 °, 180 °, 225 °, 270 ° and 315 ° 0% 1 cycle And 70% 25/30 cycles Single phase: at 0 0% 300 cycle	135 °, 180 °, 225 °, 270 ° and 315 ° 0% 1 cycle And 70% 25/30 cycles Single phase: at 0 0% 300 cycle
Power frequency magnetic field IEC 61000-4-8	30 A/m 50Hz/60Hz	30 A/m 50Hz/60Hz
Conducted RF IEC61000-4-6	150KHz to 80MHz: 3Vrms 6Vrms (in ISM and amateur radio bands)80% Am at 1kHz	150KHz to 80MHz: 3Vrms 6Vrms (in ISM and amateur radio bands) 80% Am at 1kHz
Radiated RF IEC61000-4-3	10 V/m 80 MHz – 2,7 GHz 80 % AM at 1 kHz	10 V/m 80 MHz – 2,7 GHz 80 % AM at 1 kHz
NOTE UT is the a.c. mains voltage prior to application of the test level.		

Table 3

Guidance and manufacturer' s declaration - electromagnetic Immunity								
Radiated RF IEC61000-4-3 (Test specifications for ENCLOSURE PORT IMMUNITY to RF wireless communications equipment)	Test Frequency (MHz)	Band (MHz)	Service	Modulation	Maximum Power(W)	Distance (m)	IEC 60601-1-2 Test level (V/m)	Compliance level (V/m)
	385	380 – 390	TETRA 400	Pulse modulation 18 Hz	1,8	0,3	27	27
	450	430 – 470	GMRS 460, FRS 460	FM ± 5 kHz deviation 1 kHz sine	2	0,3	28	28
	710	704	LTE	Pulse modulation 217 Hz	0,2	0,3	9	9
	710	–	Band 13,17					
	780	787						
	810	800	GSM 800/ 900, TETRA 800, iDEN 820, CDMA 850, LTE	Pulse modulation 18 Hz	2	0,3	28	28
	870	–						
	930	960						

			Band 5					
	1720	170	GSM	Pulse	2	0,3	28	28
	1845	0 –	1800;	modulati				
	1970	199	CDMA	on				
		0	1900;	217 Hz				
			GSM					
			1900;					
			DECT;					
			LTE					
			Band					
			1, 3,4,					
			25;					
			UMTS					
	2450	240	Bluetoo	Pulse	2	0,3	28	28
		0 –	th,	modulati				
		257	WLAN,	on				
		0	802.11	217 Hz				
			b/g/n,					
			RFID					
			2450,					
			LTE					
			Band 7					
	5240	510	WLAN	Pulse	0,2	0,3	9	9
	5500	0 –	802.11	modulati				
	5785	580	a/n	on				
		0		217 Hz				

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions:

- (1) This device may not cause harmful interference, and
- (2) this device must accept any interference received, including interference that may cause undesired operation.

NOTE: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules.

These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that

to which the receiver is connected.

-Consult the dealer or an experienced radio/TV technician for help.

MODIFICATION: Any changes or modifications not expressly approved by the grantee of this device could void the user's authority to operate the device.

RF exposure information:

This device complies with FCC radiation exposure limits set forth for an uncontrolled environment.

State: 

Bluetooth connection and transmission function: a) Turn on the blood pressure monitor, turn on the Bluetooth, connect to the Bluetooth;b) Search for " FC-BP200 " on your bluetooth receiver; c) Connecting the "FC-BP200" until it is connected; d) Start data transmission,the data your measured will be transmitted to your bluetooth receiver after measurement is completed.

3.6 Warranty Information

☐ The unit, excluding the unit cuff, is guaranteed to be free of defects in workmanship and materials for a period of two years from the date listed on the purchase record, when used in accordance with the instructions provided with the unit. The unit cuff is warranted to be free from defects in materials and workmanship appearing within one year from the date listed on the purchase record when the unit is used in accordance with the instructions provided with the unit.

☐ This guarantee does not cover damage incurred by using without following in accordance with the instructions, accidental damage, or being tampered/serviced by unauthorized service agents.

☐ For repair under this warranty. Our authorized service agent must be advised of the fault within the period of the warranty. This warranty covers parts and labor only under normal operations.

A transportation fee or freight fee that may be incurred will be the owner's responsibility. Any defect resulting from natural causes, e.g., flood, hurricane etc., is not within this guarantee.

☐ Unit subjected to misuse, abuse, and neglect of these manual contents, non-instructional purposes, and unauthorized repair or modifications will be excluded from this warranty.

This guarantee specifically excludes expendables and consumables, for example batteries.

All warranty claims must be directed to the distributor responsible for the sale of the device.

The content of this warranty is subject to change without further notice.

☐ **WARNING:** No modification of this equipment is allowed.

Indication for Use

Wrist Electronic Blood Pressure Monitor is intended to measure the systolic and diastolic blood pressure and pulse rate of an adult individual by using a non-invasive technique in which

an inflatable cuff is wrapped around the wrist arm.

The devices' features include irregular pulse rhythm detection during measurement, and will display a alert signal with the reading when irregular heartbeat is detected.

Considerations for measuring

1.The correct using method

- (1) Elbow on the table; comfortably seated; legs uncrossed; back and arm supported;
- (2) Keep the center of the cuff and the heart or nipples at the same height.
- (3) Do not put the clothes into the cuff.
- (4) Palms up, body keep relaxed.
- (5) Sitting in a chair keep your foot flat on the ground.

A recommendation that the PATIENT relax as much as possible and not talk during the measurement PROCEDURE; a recommendation that 5 min elapses before the first reading is taken; need to avoid compression or restriction of the connection tubing.

2.Ideal environment for measuring blood pressure

- (1) Get up in the morning, when the mood is relaxed.
- (2) No consciousness to the toilet
- (3) Room temperature is about 20℃
- (4) Aquiet place and there was no noise around.

3.Measure blood pressure at the same time every day: Blood pressure is constantly changing and cannot judge the situation only by the result of once blood pressure measurement, according to a period of time, repeated measurements to determine blood pressure condition is more reliable


So please stick to measure blood pressure every day, select the mood in a day can keep the most stable state period such as early in the morning after getting up. Ideal method is to measure as far as possible at the same time every day.

Note: be sure to have more than 2-3-minute interval between two measurements.

Depending on your individual physiological characteristics, between the two measurements may need to rest for a longer time.

Please keep the wristband around the wrist before you start measuring.

Do not measure your blood pressure on bus.

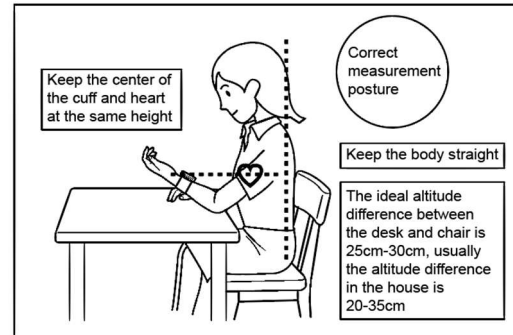



U.S.Agent

ABMED SERVICE INC

1312 17th Street Suite 692 Denver, CO US 80202

Tel: 303 8000162

E-mail: info@abmed.com.cn

Manufacturer:

Shenzhen Finicare Co., Ltd



201, No.50, the 3rd Industrial Park, Houting Community,
Shajing Street, Bao'an District, Shenzhen 518104 China

FCC ID:2A3QXFCBP200

Website: <https://www.fincare.com/>

Made in China