

尺寸：100.78×130 mm

100.78 mm

AUXILIARY LISTENING  
EARPHONES

Instruction manual



- Thank you for purchasingour products
- Please read the product Instruction Manual》carefully before use
- This manual contains the serviceguide,pleasekeep it ina safe place

Applicable population:in-ear air conduction,patients with hearing impairment (mild to moderate population),exempt from testing.  
Contraindication symptoms:acuteotitisexterna,otitis media,chronic suppurative otitis media(in the pus infection period)acutesuppurativeotitis media andmaterial

【Product Parameter】

Battery:200mAH  
Maximum OSPL90:≤110db+3db  
High frequency average:108db±4db  
Full sound gain(average high frequency):40db±5db  
Totalharmonicdistortion:≤5%+3%  
The frequency response  
range is not narrower than:300Hz~5000Hz  
Equivalent input noise lever:≤10db±3  
Bluetooth distance:>10M  
Working voltage:DC3.7V  
Bluetoothvision:BT5.3  
Transmit power:-2dbm~8dbm  
Frequencydeviation:±10KHz  
Receive sensitivity:≤-80dbm  
Hours of use:15H  
Charging time:2H  
Product size:140mm\*180MM\*250MM  
Net weight:40g

【Illustration of Product Parts】



【Operation instruction】

| Operation instruction                     | Prompt tone   | LED   |
|---|---|---|
| Auxiliary listening mode                  |   |   |
| Power on/off pairing                      | Power on/off pairin                                   | Orange and blue indicator Blinking indicatorbinkingatshortintervals |
| Click the volume“+”key                    | Whiff,totheminium prompt“minium volume”               |   |
| Click thevolume“-”key                     | Whiff   |   |
| Click mode switch key                     | Viewing mode/Universal mode/ Indoor mode/Outdoor mode |   |
| Press the switch button to mute the sound |   |   |
| Charging                                  |   | ightonfulcharghergingblue   |
| Keep press Mode switch key                | adjustment of left and right ears volume              |   |
| Bluetooth mode                            |   |   |
| Click the volume“+”key                    | Volumeup  |   |
| Click the volume“-”key                    | Volume down   |   |
| Pressthe switchbutton to mute the sound   | Play/pause music                                      |   |
| Keen nre53the volume “+” key              | Previous song   |   |
| KeA00re8sthe volum0 “-”key                | Next song   |   |
| Clickthe on/off button                    | Incoming call   |   |
| Double click theswitch                    | Hang up   |   |
| Pressand hold the switch button           | Call refusal  |   |
| Powerlowwarning                           | “Dudu”  |   |

FCC Warning

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions:  
(1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

NOTE 1: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:  
- Reorient or relocate the receiving antenna.  
- Increase the separation between the equipment and receiver.  
-Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.  
-Consult the dealer or an experienced radio/TV technician for help.

NOTE 2: Any changes or modifications to this unit not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

The device has been evaluated to meet general RF exposure requirement. The device can be used in portable exposure condition without restriction.

辅听耳机

说明书



- 感谢您购买我们的产品使用前
- 请仔细阅读《产品说明书》
- 本手册包含维修指南，请妥善保管

适用人群：入耳式气传导，听力损伤患者(轻中度人群), 免验配。

禁忌使用症状：急性外耳炎，中耳炎，慢性化脓性中耳炎(在脓液感染期) 急性化脓性中耳炎和材料过敏者。

【产品参数】

电池：200mAH

最 大OSPL90:≤110db+3db

高频平均值OSPL90:108db±4db

满档声增益(高频平均值):40db±5db

总谐波失真：≤5%+3%

频率响应范围不窄于：300Hz~5000Hz

等效输入噪声级：≤10db±3

蓝牙距离：>10M

工作电压：DC3.7V

蓝牙版本：BT5.3

发射功率：-2dbm~8dbm

频偏：±10KHz

接受灵敏度：≤-80dbm

使用时间：15H

充电时间：2H

产品尺寸：140mm\*180MM\*250MM

产品重量：40g

【产品图示】



【Operation instruction】

| 操作说明         | 提示音                 | LED           |
|--------------|---------------------|---------------|
| 辅听模式         |                     |               |
| 长按开关机键3S     | 开/关机配对中             | 橙蓝灯慢闪/橙蓝灯快闪   |
| 单击音量“+”键     | 滴一声，到了最大提示“最大音量”    |               |
| 单击音量“-”键     | 滴一声                 |               |
| 模式切换键        | 观影模式/通用模式/室内模式/室外模式 |               |
| 短按开关机按键静音    |                     |               |
| 充电           |                     | 充电橙灯常亮/满电蓝灯常亮 |
| 长按模式切换按键     | 单独调节左右耳音量           |               |
| 蓝牙模式（连接手机播放） |                     |               |
| 单击音量“+”键     | 音量加                 |               |
| 单击音量“-”键     | 音量减                 |               |
| 单击开关机键       | 播放/暂停音乐             |               |
| 长按音量“+”键     | 上一曲                 |               |
| 长按音量“-”键     | 下一曲                 |               |
| 单击开关机键       | 来电                  |               |
| 双击开关机键       | 挂断                  |               |
| 长按开关机键       | 拒听                  |               |
| 低电提示         | “滴 嘟 ”              |               |