

USER'S GUIDE


Dr.Clobo

My smart doctor
Self health-care camera



Dental & Otoscope Camera
Healthy life with Dr. Clobo!


Parts

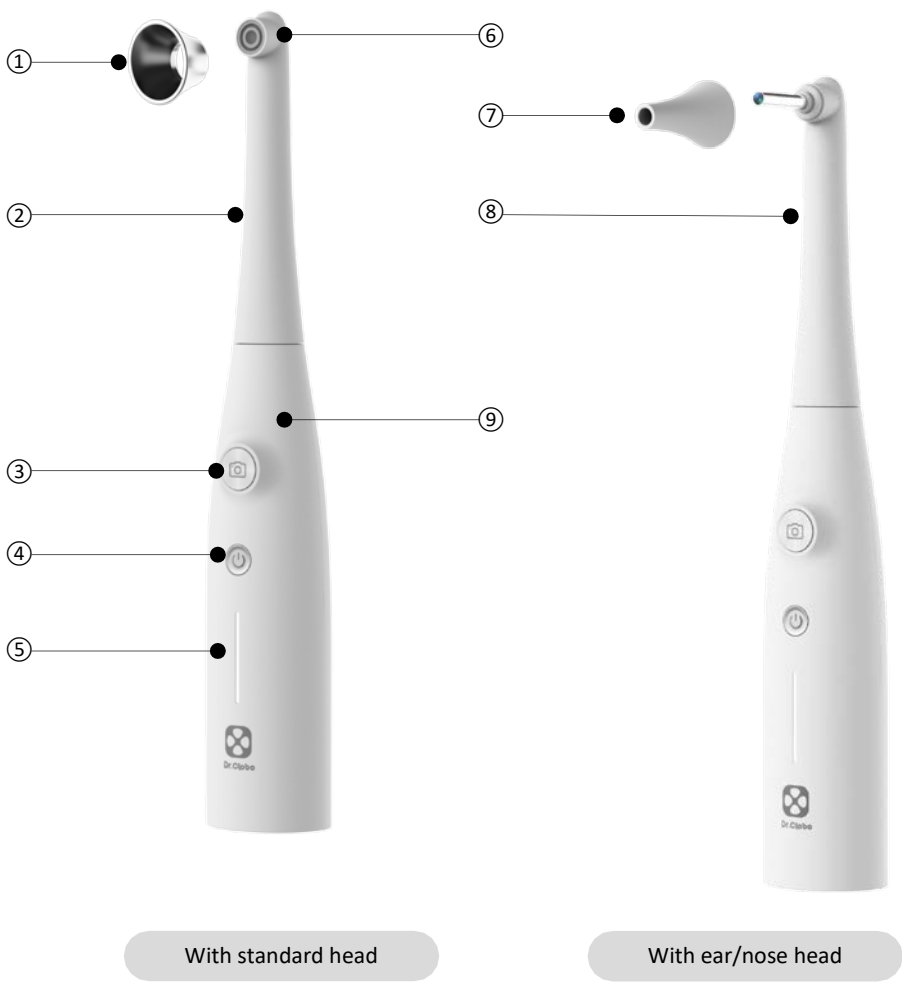


- ① Reflector
- ② Standard head
- ③ Shutter
- ④ Power
- ⑤ Battery level
- ⑥ Lens (8 LED lights)
- ⑦ Safety Cap
- ⑧ Camera head for ear/nose
- ⑨ The main Body

How to use


1. Download App
- Search for “Dr. Clobo” in the Google Play Store or App Store and install it to use the app.
2. Turn on the power
- Long press the power button for 2seconds to power on the device.(Make sure the battery level is high)
3. Connecting to Wi-Fi
- Connect Wi-Fi with Dr. Clobo Camera in Phone Setup.
4. Shooting
- Place the camera in the desired location and take pictures/videos.

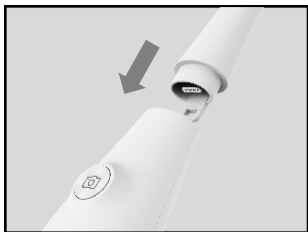

Head replacement

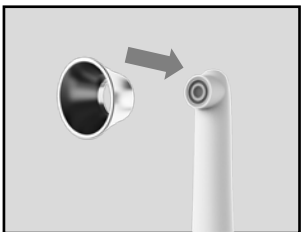

Wear a reflector

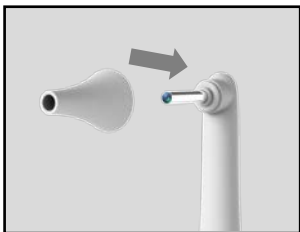

Wear a safety cap

| Detailed Information | |
|----------------------|------------------------------|
| Model Name | Dr.Clobo Camera |
| Operating System | ios / android |
| Connection | Wi-Fi 802.11 b/n(20) |
| Sensor | ¼" |
| Effective Pixels | 1920 × 1080 |
| Pixel | 2MP |
| FPS | 30FPS |
| FOV | 80° |
| DOC | 5~30mm |
| LED | 8pcs |
| Lens | Waterproof |
| Power Supply | Rechargeable Lithium Battery |
| Operating Frequency | 2 412 ~ 2 462 MHz |

Precautions

1. Please read this user manual thoroughly before using the device.
2. This device is intended for domestic use and is not to be used in a medical environment.
3. Keep out of reach of children without adult supervision.
4. This device is not designed to be explosion-proof.
5. Do not use the device in an environment where flammable gases or vapors are generated.
6. Do not attempt to disassemble, modify, tamper with or repair this device.
7. Do not touch the device with wet hands.
8. Protect the device from exposure to liquid.
9. Do not use the device if it is damaged in any way, such as exposed wires, faulty connections or damage to the casing.
10. The device may become hot while charging.

Cleaning and storage

1. The lens of this device is water-proof.
2. Remove the head to clean the lens only.
3. Wipe the head with a damp towel and cloth, or a disinfectant swab.
4. Use water and disinfecting liquid only.
5. After cleaning, dry with a dry towel or cloth.
6. Do not immerse the device in water or other liquids.
7. Keep out of reach of children
8. Avoid use and storage in high-temperature and high-humidity environments in direct sunlight for long periods.

Customer service

| Item | | Details |
|--|-------------------------------|-------------|
| Malfunction | Defect (Before use) | Exchange |
| | Malfunction upon delivery | Exchange |
| | Malfunction during normal use | Exchange |
| | If exchange is not possible | Re-delivery |
| Malfunction caused by intentional negligence by the consumer | | No exchange |

- Warranty period: 1 year
- Please inquire through the site.

Warranty certificate

1. Present the warranty written signed by the purchaser when exchanging the product.
2. The warranty period is one year from the date of purchase.
3. Warranty certificate will not be re-issued.

FCC Compliance Statement

This device complies with part 15 of the FCC rules. Operation is subject to the following two conditions: (1)

This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

FCC Interference Statement(Part 15.105)

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules.

These limits are designed to provide reasonable protection against harmful interference in a residential installation.

This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications.

However, there is no guarantee that interference will not occur in a particular installation.

If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to correct the interference by one of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

FCC Caution Any changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate this equipment.

This transmitter must not be co-located or operating in conjunction with any other antenna or transmitter.