GOOD

POSTURE

STARTS

HERE

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
USER MANUAL

HOW TO GET STARTED

1. Download the Bestand app. Search "Bestand" or simply scan the QR code
2. Follow the tutorial in the app for a step-by-step guide on how to get started.

How to connect cellphones with Bestand Posture Trainer & Corrector Pro?

1.Download Bestand APP.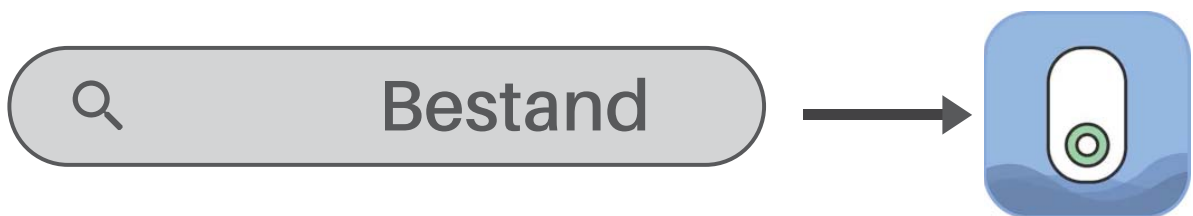


2.Log in on Bestand APP. If you do not have an account, click on “sign up” to register.

Note: Please register with your e-mail.

3.Turn on Bestand Posture Trainer & Corrector Pro:
Press the switch button of Bestand Posture Trainer & Corrector Pro.

4.Bluetooth connection: Turn on the Bluetooth of your cellphone and allow it to connect to Bestand.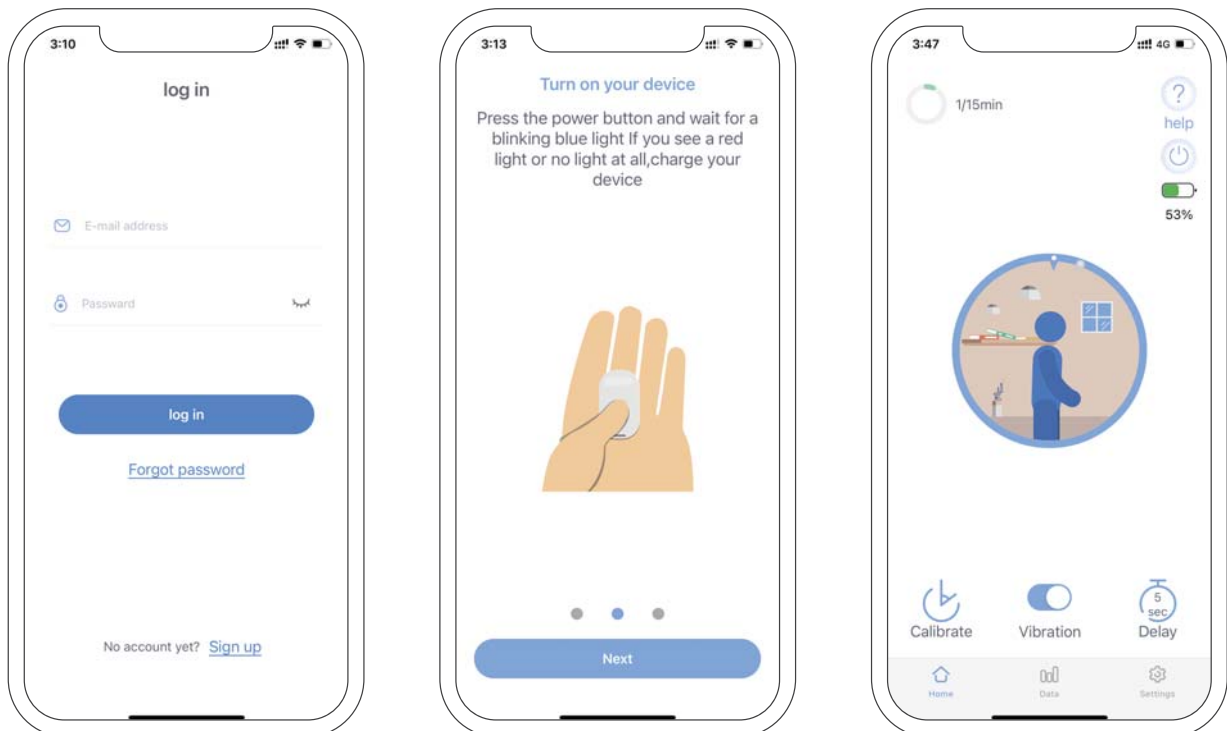

WHAT TO EXPECT IN THE APP TUTORIAL

Setting up your Bestand Posture Trainer & Corrector Pro

Your personal training plan

How to find your upright posture

How to train with your Bestand Posture Trainer & Corrector Pro

And more...

TIPS FOR SUCCESS

- In order to get the best results within 2 weeks, we strongly recommend that you use Bestand Posture Trainer & Corrector Pro daily.
- Use your Bestand Posture Trainer & Corrector Pro in a cool indoor environment.
- Put Bestand Posture Trainer & Corrector Pro on the table to remind you to train.
- Finally, don't forget to charge your Bestand Posture Trainer & Corrector Pro.

HOW TO USE BESTAND POSTURE TRAINER & CORRECTOR PRO?


1. Set your training program with Bestand APP.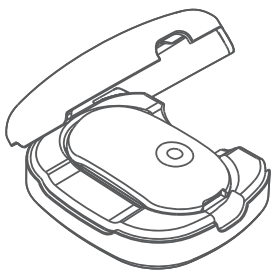

Blue icon indicates proper standing or sitting postures.




Red icon indicates improper standing or sitting postures.

Note: Laying down the Bestand Posture Trainer & Corrector Pro flat on the desk for 10 minutes, and the device will shut off automatically if nothing changes.

APP Home	Function
 1/15min	<p>The duration of the training is 15 minutes.</p> <p>1 indicates the time of keeping proper posture during the 15 minutes (find "Upright target" in settings to adjust training time and range).</p>
	<p>Click on it to shut off the device.</p>
 100%	<p>Power</p>
 Vibration	<p>Turn device vibration on or off(Find "Vibration" in the settings to adjust the vibration pattern and strength).</p>
 Calibrate	<p>Before each use, please place the product perpendicular to the desktop, and then click the calibration button in the APP to calibrate.</p>
 Delay	<p>Bestand Posture Trainer & Corrector Pro will vibrate to issue a warning if standing or sitting postures remain improper for over 5S (Click on time to change the setting as 5S/15S/30S/60S).</p>

2. View training data for the week (Find data in Bestand APP).

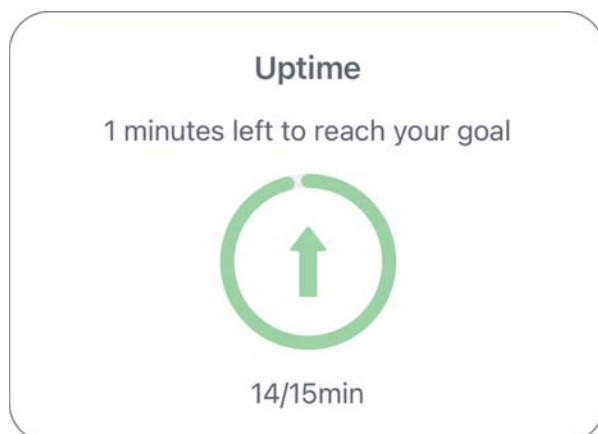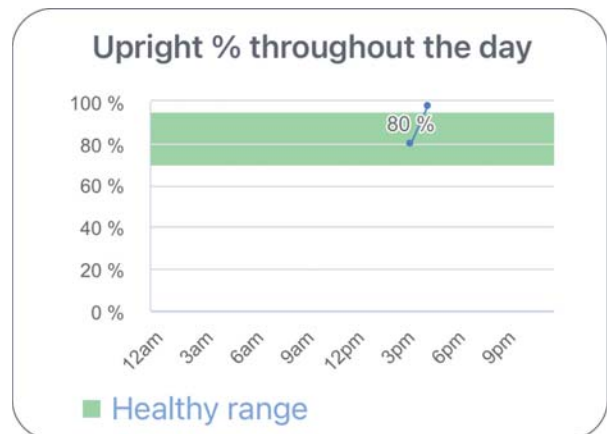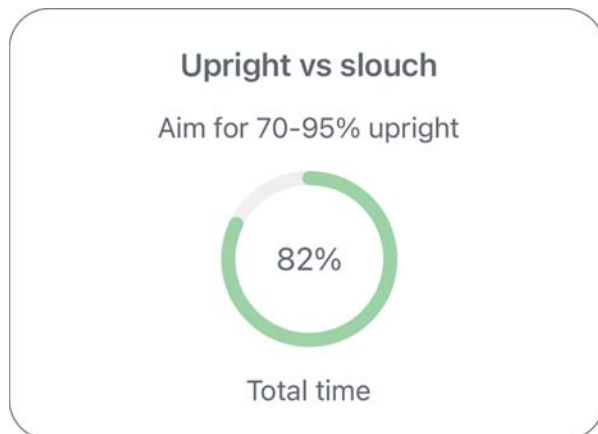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
Your personal posture trainer, which measures your posture 100 times a second.

Case

Store your Bestand Posture Trainer & Corrector Pro in the case while charging it on your desk or when you're on the go.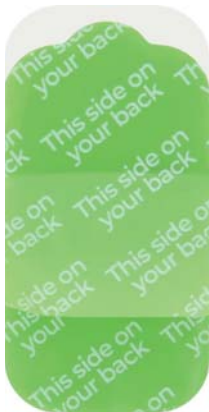


Adhesives

Attach Bestand Posture Trainer & Corrector Pro to your upper back using the reusable silicone adhesives.

Charging Cable

Type-c cable for charging Your Bestand Posture Trainer & Corrector Pro.

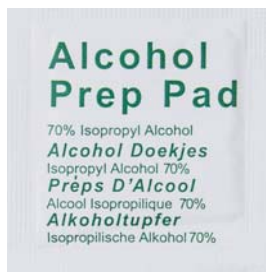Alcohol Pads

Use the alcohol pads to clean the adhesives and your skin before use.

Adhesive Removal Tool

Use it to peel off your old adhesive when replacing it with a new one.


DEVICE SHORTCUTS

Turn On, Short Click.

Turn Off , Long Click.


Set Posture (Calibration),
Double Click.

The device within 10 meters
automatically connects to
the mobile phone and enters
training mode.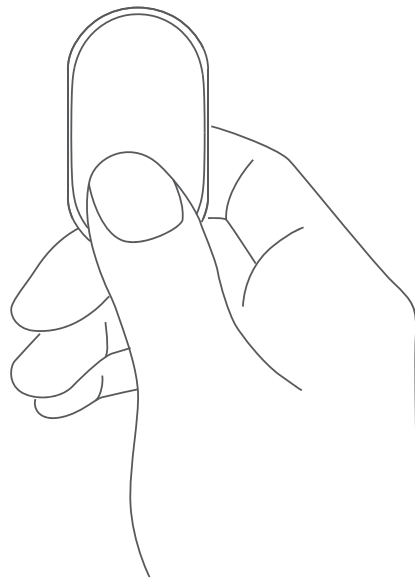


*Learn more about Set Posture, Training & Tracking
in the Bestand app.

LED INDICATORS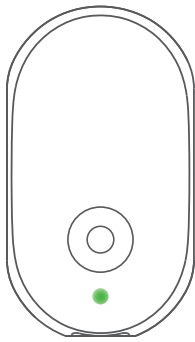


Green

Connected
to the app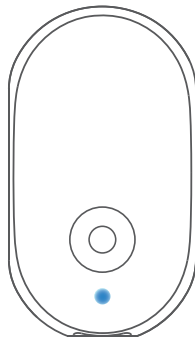

Blue

Not connected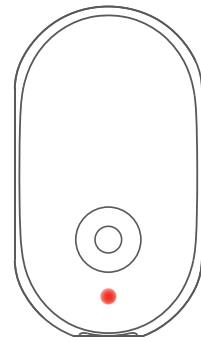

Red

Battery is in low


HOW TO CHARGE YOUR BESTAND POSTURE TRAINER & CORRECTOR PRO

Charging your battery regularly will allow you to use your Bestand Posture Trainer & Corrector Pro as often as you can.

1. Place your Bestand Posture Trainer & Corrector Pro in its case for safekeeping and plug the charging cable into the USB port.

2. It takes about 1-2 hours to fully charge.

3. Once your battery is fully charged the LED light will turn green.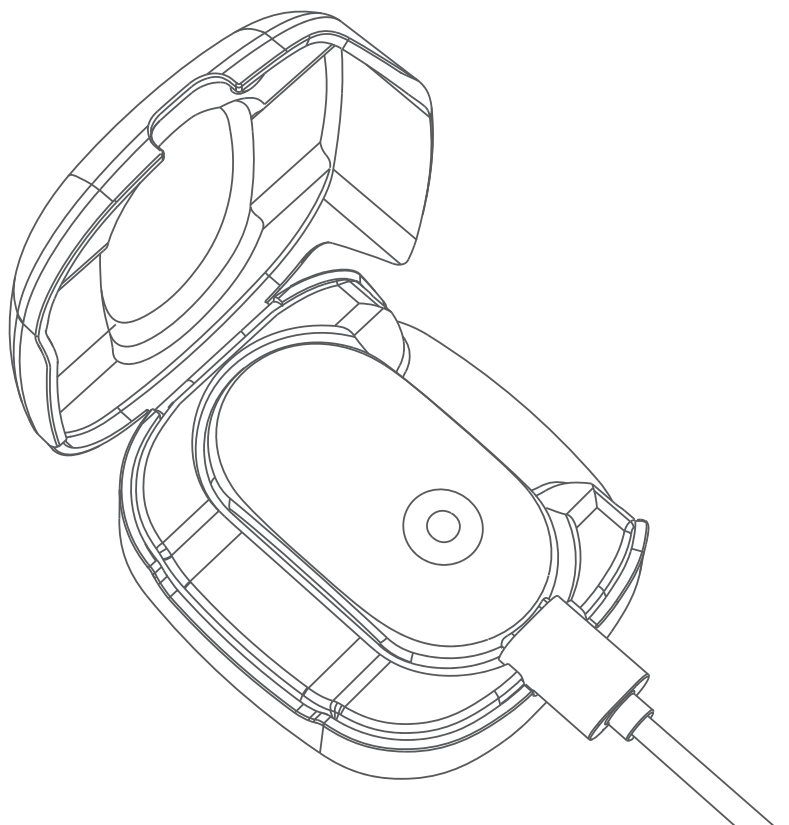

ADHESIVES

CARE & MAINTENANCE


- With proper maintenance, each adhesive can be used about 3-10 times, depending on skin type.
- Make sure your skin is clean and dry before using the Bestand Posture Trainer & Corrector Pro.

Use the alcohol pads provided to clean skin.


- The adhesive doesn't attach well to sweaty skin. Therefore, we recommend training indoors.
- When you're not using Bestand Posture Trainer & Corrector Pro, store it in its case to keep the adhesive clean.

REPLACING THE ADHESIVE


1. Remove the old adhesive from the Bestand Posture Trainer & Corrector Pro. We recommend using the removal tool provided.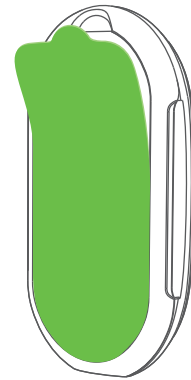


2. Important: Make sure you remove ALL parts off the old adhesive completely, so there's nothing green left on your Bestand Posture Trainer & Corrector.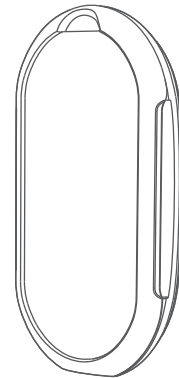


3. Take a new adhesive and just peel off the white cover from one side.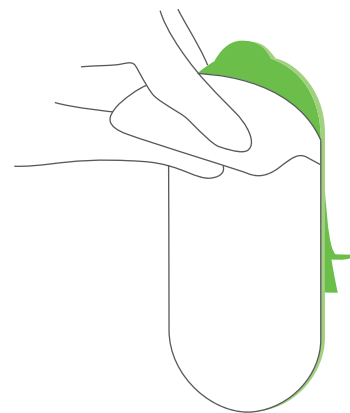


4. Attach the sticky part to the gray sunk part of Bestand Posture Trainer & Corrector Pro.

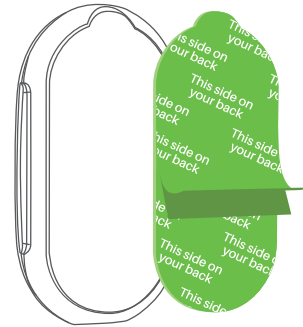5. When you want to start using your Bestand Posture Trainer & Corrector Pro, peel off the green cover which says "This side on your back".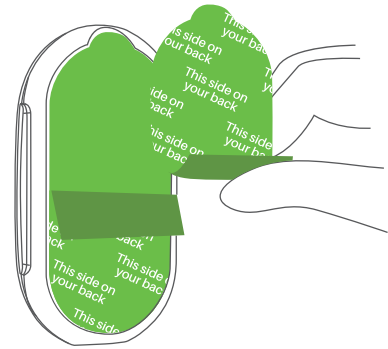

6. Attach the Bestand Posture Trainer & Corrector Pro to your back.

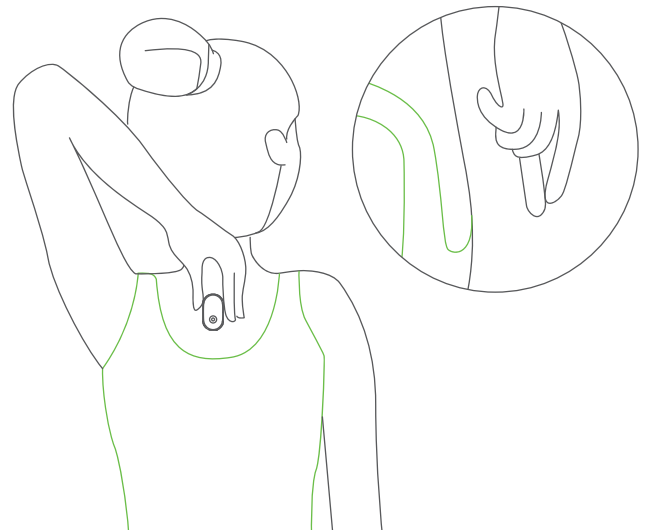Note: You could buy the adhesive from Bestand APP (Bestand APP-Settings-Purchase adhesives).

PRODUCT SPECS

Dimensions	50×28×12mm
weight	11g
Bluetooth	BLE-Bluetooth Low Energy 5.0
Battery	Single Cell, Lithium-Polymer, 3.7V, 200mAh, 0.55Wh
Battery life	45 hours
Charging cable	USB Type C
Input voltage Input current	5V, 100mA(max)
Mobile compatibility	iOS9.0 & up, Android 5.0 & up

FCC WARNING

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

Any changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

NOTE: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation.

If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

The device has been evaluated to meet general RF exposure requirement. The device can be used in portable exposure condition without restriction.

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