

User manual

Otoscope
Model: CT09

Instructions

Use steps

Open the product:

1. Scan QR code and download app;
2. Open the app and select the product model;
3. Open the cover to start the machine;

Link product WiFi:

1. Optional channel for WiFi model of mobile phone link product;
2. Return to the app and enter it for use;

Replace the ear spoon head: unscrew the ear spoon head, pull out the ear spoon, replace the ear spoon head, and then tighten it;

Hazardous substances

Part name (PB) (Hg) (CA) (Cr (VI) (PBB) (PBDE)

Printed circuit components: normal

Plastic shell: normal

Plastic rubber accessories: normal

This item is prepared in accordance with SJ / t11364

O: The content of the hazardous substance in all homogenates of this part is below the limit specified in GB / t26572-2011;

Troubleshooting

Faults and solutions:

No response after startup: please use the supporting charging cable for charging, and the charging time is 85 minutes;

Camera does not light up: the ear bar is shut down and restarted;

App flash back: uninstall the existing app and re scan the QR code for installation;

The installation of accessories and tools is not smooth: unscrew the ear spoon head, pull out the ear spoon, replace the ear spoon head, and then tighten it;

The power indicator of ear bar shows red light: please use the supporting charging line for charging, and the charging time is 55 minutes;

After the product is successfully connected to the app, the real-time image cannot be displayed normally: disconnect the mobile phone from the ear bar WiFi, restart the ear bar and reconnect the WiFi;

The WiFi connection is unstable, and sometimes it is disconnected in a few seconds: restart the ear bar and reconnect the app;

If the above operation still fails to solve the problem, please contact the customer service, and we will further provide you with practical solutions;

matters needing attention

- 1. When cleaning the lens, it is recommended to carefully wipe it with a professional alcohol cotton swab;**
- 2. Be sure to assemble the ear in place before use to prevent the ear spoon from falling;**
- 3. Please observe the surrounding environment before use. Do not use it when someone is running to avoid impact;**
- 4. Children are not allowed to use this product alone to avoid accidental injury;**
- 5. Please put the main machine body in liquid to avoid damage caused by immersion;**
- 6. The built-in charging battery of the product must also be charged once a month if it is not often used;**
- 7. Avoid direct exposure to the sun, especially the accessories in the storage box to avoid softening;**
- 8. When in use, the product temperature will rise slightly (from the laboratory data, the maximum temperature of the whole machine does not exceed about 35 °), please rest assured to use;**

Q. Can't you use the product normally after WiFi connection?

- 1. If you are using VPN service, please turn off VPN service and try again.**
- 2. If VPN cannot be set, please follow the instructions.**
- 3. If it still cannot be solved, please uninstall the software and reinstall it.**

FCC Statement



This device complies with Part 15 of the FCC rules. Operation is subject to the following two conditions: 1) this device may not cause harmful interference, and 2) this device must accept any interference received, including interference that may cause undesired operation.

Note: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation.

This equipment generates uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

FCC RF Radiation Exposure Statement

The device has been evaluated to meet general RF exposure requirement. The device can be used in portable exposure condition without restriction.