



365 Health Project

# Electronic Detection AIO

365 Health Project

Manual

Date of issue: April of 2019, Version: A01



## ■ Catalogue:

Catalogue .....	01
Instructions .....	02-06
Notices .....	07
Package Contents/Certificate .....	08
After-sale .....	09
Warranty Card .....	10



## ■ Instructions:

Product instructions :

### 1.Product instructions



Note: the pictures in this manual are for reference only. Please refer to the real object for details

## ■Instructions:



### 2.How to use

- ①After unpacking the machine, open the latches on both sides of the bottom and remove the adapter cover plate(As show in pic 1)
- ②Take out the adapter and connect the equipment, cover the cover plate, and plug the adapter into the socket(As show in pic 2)
- ③Turn on the screen of the machine and adjust the screen angle according to your preference
- ④Press and hold the power touch switch for 2 seconds to start the machine (the machine will start automatically when the power is plugged in for the first time)
- ⑤Start the machine and enter the system desktop (as shown in Figure 4)




## ■Instructions:



### 3.Software usage

- ①Connect WIFI

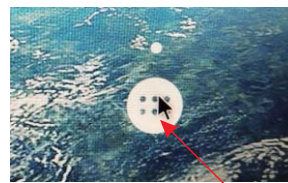

- a.Click the "menu" icon on the desktop with the left mouse button



- c.Click "WiFi" with the left mouse button



- b.Click the "Settings" icon with the left mouse button



- d.Select the WiFi to be connected and enter the password (if the password is entered incorrectly, cancel saving and re-enter the password)

- ③Click "get information" after the detection interface appears



## ■ Instructions:



④Scan QR via Wechat



⑤Enter your personal data and submit it according to the prompt information of your mobile phone

用户信息

机构: 合作机构

性别: ☒ 男 ☐ 女

用户名: 小李

年龄: 28

身高: 173 cm

体重: 60 kg

身份证: 身份证号码

微信号: 微信号

手机号: 手机号

联系人: 紧急联系人

联系人: 紧急联系人

⑥Click "start"



⑦Place both hands on the electrode according to the interface prompt, and confirm that the face is aligned with the camera until the detection is completed



Electrode

## ■ Instructions:



⑧After the test is completed, the system will prompt that the test is completed. You can view the test report on your mobile phone



⑨If the machine is not in use, press the power switch for 2 seconds to turn off the machine.



Power switch (touch) and power indicator

### 4.Common Problems and Its Resolution

①.The machine is not powered on

- Check whether the adapter is well connected with the machine and socket, and confirm whether the socket is powered.
- Reconnect the adapter (both ends of the machine and socket)

②.The instrument cannot detect normally

- Confirm whether WiFi is connected and whether WiFi can access the Internet normally
- Turn off the machine and restart it to confirm whether it is OK

③.Error or exit during detection

- Please confirm whether the palm is in good contact with the electrode
- Please confirm whether the face is aligned with the camera during the detection process, and there can only be the face of the tested person, and multiple people are not allowed to be in the detection area at the same time



## ■ Notices:

Please read below notices carefully in case of accidental injury, product damage and other possible danger.

- In case of damaging machine and wrong inspection results, please follow the instructions strictly.
- The company shall not be responsible for the damage caused by fire, earthquake, behavior of a third party, other accidents, careless and wrong use of users and use under abnormal conditions.
- Please do not disassemble and repair the machine by yourself in case of damaging machine.

Please contact our authorized distributors if you have any questions.

- Please use our appointed adapter only to avoid electric shock and damage to the machine.
- If the machine breaks down: smokes or emits a burning smell, please turn off the power immediately and contact the distributor.
- When the machine is not in use, please unplug the power supply and keep the machine moisture and dust free.
- The machine is not waterproof. Please do not spill water or any other liquid on the machine.
- In order to prevent electric shock, fire, short circuit and other hazards, please do not put the machine in a water or humid environment.
- If the machine shows signs of damage and defects, please contact distributors immediately.



## ■ Package Contents/Certificate:

### 1. Package Contents

No.	Contents	Qty	Default/Optional√	Remark
1	Machine	1	√	
2	Adapter	1	√	
3	Manual	1	√	
4				
5				

### 2. Certificate



365 Health Project

## Quality Certification

Product Name :

Model No. :

Serial No. :

Factory address :

Inspector :

Approve :



Quality control department

## ■After-sale:



When you contact us, please tell us the following information :

Model No. : XYZ-2019-3.0

Serial No. : See equipment nameplate

Date of purchase : Please tell us the date you purchased the equipment.

Fault description : Please describe the problem as much detail as possible

After-sale Address:Room 809, Building A, Rhine Hongjing center, 1439 Wuzhong

Road, Minhang District, Shanghai

After-sale Tel : 400-062-6680

## ■Warranty Card:



Dear Customer:

Thank you for purchasing our products. In order to better serve you, please read, fill in and keep this card carefully before purchasing products.

Name/Company Name			
Add.:			
Contact person		Contact No.	
Product Name		Model No.	
Serial No.		Date of Purchase	
Repair record	Date	Cause of failure	

### ※ Warranty description

If the product needs to be repaired, please cut this card along the dotted line and properly send it back to the after-sales service department of our company together with the purchased product.

### ※ Notification

- 1.From the date you (the customer) purchase the product, if you install and use it according to the product operation manual, it will be repaired free of charge within one year, and the maintenance cost will be charged after one year;
- 2.For maintenance not caused by the company's product quality, such as damage caused by improper use, improper storage, unauthorized disassembly and other reasons, the company will charge cost for maintenance;
- 3.The product is guaranteed for one year

The company reserves the right of final interpretation and modification of the contents of this warranty card

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

Any changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

NOTE: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation.

If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

The device has been evaluated to meet general RF exposure requirement. The device can be used in portable exposure condition without restriction