



To purchase more pads go to:
www.intellisnore.com/pads

For the full instruction manual, go to:
www.intellisnore.com/manual

Quick Start Guide

1. Power On/Off Instructions:

Press and hold the power button for 3 seconds to turn on the device. The power light will automatically turn off after 10 seconds. To turn the power light back on, tap the power button again. To turn the device off, simply press and hold the power button for 3 seconds.

2. Wear Instructions:

Clean the area. Apply the adhesive electrode pad under your chin, but above the throat. Point the rounded outward side of the pad up toward your chin, so that the indented side will point down toward your throat. Once applied, press and hold the pad so that it adheres strongly. Device attaches magnetically to the pad.

3. Bluetooth Connection and Data Syncing:

- Go to www.intellisnore.com/app to download the app
- Enable your phone's Bluetooth, open the app, and turn on the device.
- Follow the instructions on the app to pair the device.
- Tap the device's power button, so that the Bluetooth light will illuminate.
- Your data will now be synced to the app.

4. Adjusting Intensity:

If the out-of-the-box intensity of stimulation is too high, open the app and adjust your settings by tapping the gear in the upper righthand corner.

5. Unsuitable Users:

People with the following conditions: pacemakers, implantable defibrillators or any other implanted metal or electronic devices; acute suppurative inflammation, a tendency to bleed, or malignant tumors in the submandibular region; severe cardiovascular or cerebrovascular problems; ventilatory disorders caused by nasal disease; an allergy to electrodes or currents; skin ulcers on the site of adhesion; pregnant women or those in labor; and children under 16. Allergy Statement: This adhesive pads may contain traces of latex and/or silicone. If you have any known allergies to these components, or show signs of reaction for any reason, do not use this product or discontinue use immediately. Do not stick this product on the eyes or any other parts of the body.



FCC Caution

§ 15.19 Labelling requirements. This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions:

- (1) This device may not cause harmful interference, and
- (2) this device must accept any interference received, including interference that may cause undesired operation.

§ 15.21 Information to user.

Any Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

§ 15.105 Information to the user.

Note: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

* RF warning for Portable device:

The device has been evaluated to meet general RF exposure requirement.

The device can be used in portable exposure condition without restriction.