JIUYEE

玖益助听 JIUYEE Hearing Aid

听到爱·沟通无障碍
Hearing Love Leads to Smoothness

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Table of Contents

Introduction to the Manual : This manual guides users in the use and maintenance of the JIUYEE Hearing Aid. Please read the manual carefully, including the “Warning” section. This manual will help you fully understand how to operate the hearing aid.

Model Overview	29
Applicable Range	29
Structural Composition of the Product	29
Contraindications	30
Cautions	30
Special Instructions on Howling	32
Working Principle	32
Descriptive Graph of the Hearing Aid	33
Preparation before Wearing the Device	34
Wearing Diagram for the Hearing Aid	36
Cleaning and Maintenance of the Hearing Aid	37

Cleaning and Maintenance Methods for the Earplugs.....	38
Replacing the Earplug.....	38
Replacing the Cerumen Baffle.....	39
The Hearing Aid (the Function of the P3/P4 Model)	40
Self-check on Faults	41
General Warning	44
After-sales Service and Warranty	47
Environmental Protection.....	47
Transportation and Storage.....	48
Explanation of Graphics, Symbols and Abbreviations used in Medical Device Labels	49
Technical Description.....	50
Electromagnetic Compatibility	51
Technical Parameters.....	52
Packaging List	53

Model Overview

This manual is applicable for the following series and models of hearing aids:

REAL P1	REAL P2	REAL P3	REAL P4
---------	---------	---------	---------

Note: (1)The REAL P1/REAL P2 type does not have the functions of calling and listening to music.

(2)The REAL P3/REAL P4 type has the functions of calling and listening to music.

Applicable Range

After inspection and fitting, the hearing aid is applicable for patients with air conduction hearing loss.

Structural Composition of the Product

This product consists of the host and its accessories. The host comprises of the microphone, signal processor, battery, shell and receiver, and accessories, including the earplugs, cerumen

baffle, charging box and charging cable.

Contraindications

Patients with acute external otitis, myringitis, chronic suppurative otitis media (in the stage of suppurative infection), or acute suppurative otitis media are not suitable to wear the air conduction hearing aid. Patients who are allergic to the materials used for the product are not suitable to wear the hearing aid.

Cautions

1. Read all instructions carefully before use.
2. Before using the hearing aid, professional inspection and hearing tests should be carried out in a hospital, and the hearing aid should be used under the guidance of doctors or professional fitting personnel.
3. Keep the product away from damp areas, shock and heat.
4. Do not immerse the hearing aid in any liquid. Remove the hearing aid before swimming, washing your face, or bathing. Use the hearing aid when the auditory meatus is dry after

swimming or washing.

5. Keep the outer surface of the hearing aid clean, and regularly remove the cerumen inside the earplug.

6. The function and volume control switch should be correctly used. The hearing aid should be handled with care, and avoid excessive force on the device.

7. Do not place the hearing aid near strong magnetic fields. These will affect the sensitivity of the hearing aid.

8. If your ears are uncomfortable during use, replace the earplug in time, or consult a doctor or customer service representative (customer service hotline: xxxxxxxxxx).

9. Instructions for the safe use and maintenance of the rechargeable battery:

a. Use original charging accessories and charging cables with the same technical specifications, and select the appropriate USB for charging the product (output DC5V);

b.The hearing aid is not equipped with a medical device adapter. Please configure the adapter that meets the medical device standards (GB9706.1-2007).

c.The rechargeable battery should not be overcharged or over-discharged. This will affect the capacity and life of the rechargeable battery.

d.If the rechargeable battery is not used for a long time, the battery should be stored in its fully charged state, and charged at least once a month. This will help to extend the life of the battery.

Special Instructions on Howling

One of the problems new users often encounter when using the hearing aid for the first time is “howling” . This refers to the howling sound made by the hearing aid. This usually happen when the volume is increased to a high level.

The common causes of howling are, as follows:

- a.Cerumen is present in the auditory meatus;
- b.The hearing aid is not properly worn;
- c.Aging of the hearing aid and other reasons;
- d.The holes for the microphone in the hearing aid is blocked or squeezed.

Working Principle

The input transducer converts the input sound energy into electrical energy. Then, this transmitted to the signal processing unit. The signal processing unit performs the A/D conversion for the input signal, and uses the digital signal processor for a series of processes and amplifications. Then, the processed signal is transmitted to the output transducer. Finally, the output transducer transmits the signal to the auditory meatus through the earplug by air conduction.

Descriptive Graph of the Hearing Aid


Earplug: Fixes the line pipe and receiver in place in the auditory meatus

Receiver: Output the sound

Line pipe: Connects the receiver to the hearing aid

Microphone: Collects the sound

Program keys: Power on/off

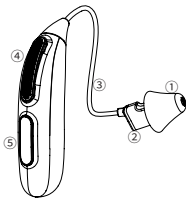For the earplug on the receiver, users can choose one of the following types, according to the ear damage:

1.tulip

2.Closed


3.double deck


Preparation before Wearing the Device

1. Activate the device


Open the cover of the charging box, take out the headphones, tear off the PE film, place the headphones back into the charging box, close and open the cover, and the hearing aid will be activated.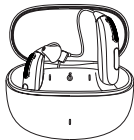

Step 1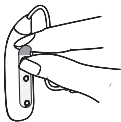

Step 2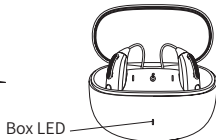

Step 3


2. Bluetooth pairing

a. When the charging box is opened, the hearing aid will turn on and automatically start to enter pairing mode. The "Bluetooth connection succeeded" will prompt


after a successful connection.

b.If the Bluetooth connection fails while wearing the headphones, press and hold the charging box button for 3 seconds, or return the two headphones back inside the charging box, and close and open the upper cover of the charging box. The box LED will flash rapidly, and the hearing aid will enter pairing mode. "Bluetooth connection succeeded" will prompt after a successful connection.

c.For hearing aids that have been successfully paired, this will automatically connect every time the charging box is opened.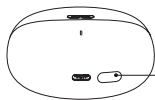


Charging box button

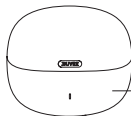Close the hearing aid

⚠ Important: The input voltage of the charging USB for the product is DC5V. Do not connect the device to a power supply port or adapter with a supply voltage that exceeds product specifications. This may damage the battery, or unexpected danger may occur (The battery used for the device is lithium battery DC3.8V).

3. Press and hold the hearing aid button for 5 seconds to turn on the hearing aid, and press and hold the same button for 5 seconds to turn off the hearing aid.

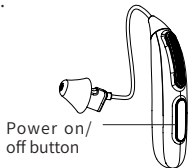Wearing Diagram for the Hearing Aid

Step 1: Place the hearing aid behind the ear.

Step 2: Hold the curved line of the receiver using your thumb and forefinger. At this time, the opening of the earplug should point to the opening of the auditory meatus.

Step 3: Gently insert the earplug into the auditory meatus until the cable of the receiver is close to the side of the head.

Caution: The sound is transmitted to the ear through the receiver. During use, the receiver should always be equipped with the earplug, and only use parts that came in with the hearing aid.


Cleaning and Maintenance of the Hearing Aid

To clean the hearing aid, place the hearing aid on a soft surface to avoid damage when it accidentally falls down.


Methods for cleaning and disinfecting the host:

- a. Please use dry soft cloth or cotton swabs to clean the hearing aid. It is recommended to clean the auditory meatus and hearing aid at least once a week. Please avoid using alcoholic solutions and cleaning solutions. These can damage the electronic circuit.
- b. When cleaning the opening of the microphone, use a clean brush to

carefully clean the residues at the opening. Do not press the brush into the opening.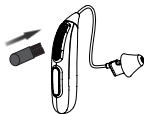

Clean the opening
of the microphone
of the hearing aid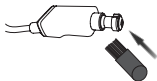

Clean the opening
of the receiver of
the hearing aid

Cleaning and Maintenance Methods for the Earplug

Remove the earplug, rinse it with clean water, and dry it, or use 70-80% (body


ratio) medical alcohol for disinfection, and dry it. If the earplug is full of cerumen, it is necessary to remove the cerumen or replace the earplug.

 **Warning:** Do not use detergent, industrial alcohol, or gasoline to clean the product. During cleaning and disinfection, prevent the liquid from flowing inside the device.


Replacing the Earplug

Please replace the earplug according to the following steps.

Step 1: Hold one end of the receiver and pull out the earplug.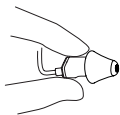

Step 2: Insert the receiver into the middle of the earplug to ensure that the connection is solid.

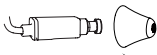
Step 3: Push hard to ensure that the earplug is firmly mounted.


Note: If the earplug is not on the receiver when removing the hearing aid, the earplug may still be in the auditory meatus. Please consult a hearing consultant for further guidance.

Replacing the Cerumen Baffle


Step 1: Open the clamps of the cerumen baffle and take out the tool. The tool has two pins: one pin is used for removing old cerumen baffles, while the other pin is used for connecting the new one.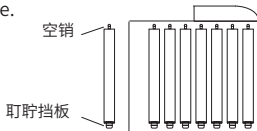

Step 2: Push the empty pin into the cerumen baffle on the hearing aid, and take out the old cerumen baffle and discard it.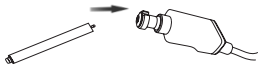

Step 3: Use the other pin to insert the new cerumen baffle, and remove the tool and discard it.

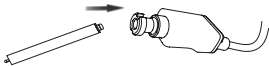Step 4: After replacing the cerumen baffle, the receiver should be similar to that shown in the figure.

The Hearing Aid (Function of the BT Model)

1. Call mode

When answering a call using the hearing aid, short press the button once to answer the call, and short press the button again (during the call) to hang up.

2.Music mode

When playing music, short press the button once to pause playing, and short press the button again to resume playing.

Self-check on Faults

For problems that may occur during the use of the hearing aid, please refer to the following table for simple solutions.

(If the solutions in the table does not work, please consult a customer service representative for assistance.)

Fault	Possible cause	Solutions
Cannot hear any sound from the hearing aid	The ear mold or earplug is blocked	Clean or replace the cerumen baffle
	When opening the charging box, the LED lamp does not light up	Charge the battery
	Low battery	Charge the battery
The sound is very weak	The volume is set too low	Turn up the volume
The sound is very weak	The cerumen baffle is blocked	Clean the baffle
	The battery of headphone has no power	Return the headphone back into the charging box to charge
	The earplug is loose	Wear the earplug properly

Can not connect to Bluetooth	The pairing during the previous Bluetooth connection was not successful	Press and hold the charging box button for 10 seconds to force the Bluetooth pairing
The hearing aid howls	There is too much cerumen in the auditory meatus	Clean the auditory meatus, and consult a doctor when necessary
	The hearing aid is not worn in place	Please wear the hearing aid properly
	Due to aging of the hearing aid and other reasons	Replace the hearing aid

General Warning

To ensure your safety and that the hearing aid correctly used, please be familiar with the following general warnings.

Please note that the hearing aid neither recovers a person's normal hearing, nor prevents or improves the hearing loss caused by organic diseases. The hearing aid only assists during listening, which may need to be supplemented by listening training and lip reading guidance. In addition, please note that in most cases, the user cannot obtain the full benefits of the hearing aid when not used frequently. It is recommended to wear the hearing aid for a long period of time, and maintain good wearing habits to maximize the benefits of the device.

- The hearing aid can only be used and adjusted under the guidance of a customer service representative. Improper use of the device can lead to sudden or permanent hearing loss.
- It is strictly prohibited to allow other users to wear the hearing aid, because each user has different hearing conditions. The improper use of the device may result to permanent damage.
- Swallowing the hearing aid and its parts can cause injury. For children or

individuals who may have been injured, the hearing aid and its parts would be out of their reach.

- Active implant: The hearing aid has been comprehensively tested and designed for human health, in accordance with international human exposure standards (absorbance - SAR), and induced electromagnetic power and voltage standards for the human body.

- The exposure values are far below the safety limits specified in the internationally recognized SAR, human-induced electromagnetic power and voltage in human health standards, and the safety limits specified in the standards for coexistence with active medical implants, such as pacemakers and cardiac defibrillators.

- If you have an active implant implanted in your brain, please consult the related manufacturer for specific interference risks.

- Cochlear implant: If wearing the cochlear implant (CI) on one ear and the hearing aid on the other, please ensure that the distance between the CI and hearing aid is always more than 1 cm. The magnetic field from the CI's sound processor, coil and magnet may permanently damage the speaker parts in the hearing aid. Do not place the two equipment too close when removing these (for example, when cleaning

these). Do not place the CI and hearing aid in the same box.

- The hearing aid may be damaged in places with strong electromagnetic fields. Please remove the device during X-ray, CT, MRI, PET, scanning, electrotherapy, or surgery sessions.

- The hearing aid must not be exposed to extreme heat, such as in a vehicle exposed to the sun.

-The hearing aid should not be dried in the microwave oven or other ovens.

- Chemicals from cosmetics, hair gels, perfumes, aftershave lotions, sunscreens and insect repellents can damage the hearing aid.

- Before using such products, remove the hearing aid and allow it to dry before use.

- Care should be given when selecting, installing and using the hearing aid with a maximum sound pressure capacity of over 132dBSPL (2CC coupled cavity), because there may be a risk of damaging the residual hearing of the user.

- The hearing aid, earplug and ear mold may accelerate the accumulation of cerumen.

- In rare cases, other non-allergic substances used in the hearing aid may cause

skin irritation or other side effects.

- If these conditions occur, please consult a doctor.
- For the hearing aid, the comprehensive interference test has been carried out according to the strictest international standards. However, the hearing aid may be interfered by other equipment, such as some mobile phones, citizens band series, store alarm systems, and so on. If this happens, please increase the distance between the hearing aid and the interference equipment.

After-sales Service and Warranty

Hunan JIUYEE Medical Technology Co., Ltd. is the after-sales service unit in charge of the hearing aid device. Consult a hearing customer service representative for relevant service policies.

You can initially consult a customer service representative on the use and troubleshooting of the hearing aid. The representative will provide guidance or assistance to immediately solve the problem.

Environmental Protection

Properly discard obsolete batteries, hosts and accessories. The disposal of wastes and residues should conform to corresponding national laws and regulations.

Transportation and Storage

1.Environmental conditions for normal operation:

- a. Ambient temperature: 0°C to 40°C ;
- b. Relative humidity: $\leq 90\%$;
- c. Atmospheric pressure: 86 kPa-106 kPa.

2. The product should be stored in a well-ventilated room without corrosive gas, and the environment should meet the following conditions:

- a. Ambient temperature: -10°C to 40°C ;
- b. Relative humidity: $\leq 90\%$;
- c. Atmospheric pressure: 86 kPa-106 kPa.

3.Handle with care to prevent collisions or strong impact.

Explanation of the Graphics, Symbols and Abbreviations used in Medical Device Labels

	Refer to the attached documents		Manufacturer
	B-type application equipment		This side up
	Production batch number		Prevent from getting wet from rain
	Service life		Prevent exposure to sunlight

	Fragile! Handle with great care		Stacking layers
	Dispose waste electrical and electronic equipment separately (Please comply with local laws and regulations)		

Technical Description

Classification for electrical:

Classification for the type of electric shock: Internal power supply

Classification of the level of electric shock: B-type

Degree of protection against liquid ingress: B-type application

The equipment is not AP-type or APG-type.

Operation mode of equipment: Continuous operation.

Externally connected equipment: The external equipment connected to this product must comply with GB9706.1-2007 or equivalent safety standards when city power is used.

Software version: V1.0

Electromagnetic Compatibility

The device should meet the requirements in GB/T25102.13-2010.

The hearing aid complies with the hearing aid immunity specified in GB/T25102.13, and the field strength of the RF test signal					
Neighbor compatibility IRIL $\leq$ 55 dB at the following field strengths Field strength (V/m)					
Frequency range (GHz)	0.08-0.80	0.80-0.96	0.96-1.40	1.4-2.0	2.0-3.0
Microphone mode	/	3	/	2	/

Technical Parameters

Model/Specification	Maximum OSPL ₉₀	High frequency average value OSPL ₉₀	Full on gain (high frequency average value)
JY01LE	$\leq 118\text{dB}+3\text{dB}$	$113\text{dB}\pm 4\text{dB}$	$45\text{dB}\pm 5\text{dB}$
JY02LE	$\leq 111\text{dB}+3\text{dB}$	$106\text{dB}\pm 4\text{dB}$	$35\text{dB}\pm 5\text{dB}$
JY03BT	$\leq 118\text{dB}+3\text{dB}$	$113\text{dB}\pm 4\text{dB}$	$45\text{dB}\pm 5\text{dB}$
JY04BT	$\leq 111\text{dB}+3\text{dB}$	$106\text{dB}\pm 4\text{dB}$	$35\text{dB}\pm 5\text{dB}$

Equivalent input noise level	Total harmonic distortion	Frequency response range (not less than)	Rated power and current consumption	mode
$\leq 25\text{dB}+3\text{dB}$	$\leq 3\%+3\%$	200Hz-6000Hz	3mA	BLE
$\leq 25\text{dB}+3\text{dB}$	$\leq 3\%+3\%$	200Hz-6500Hz	3mA	BLE
$\leq 25\text{dB}+3\text{dB}$	$\leq 3\%+3\%$	200Hz-6000Hz	3mA	BT
$\leq 25\text{dB}+3\text{dB}$	$\leq 3\%+3\%$	200Hz-6500Hz	3mA	BT

Packaging List

Product name	Quantity	Product name	Quantity
Main engine	2	Earplug	18
Charging box	1	Charging cable	1
Cleaning brush	1	Cerumen Baffle	8
Instructions	1	Certificate	1

Note: The host is optionally available, and there are 2 hosts in general. Please refer to the actual situation before purchasing.

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Registrant / Manufacturer / After-sales service unit:
Hunan JIUYEE Medical Technology Co., Ltd.
Contact information of manufacturer: XXXXXXXX
Postal code: 421008
Official website: www.jiuyee.cn
Medical device registration certificate no. / Product technical requirement no.: XXXXXXXX
Production license no.: XXXXXXXX
Address of manufacturer: Floor 2, No. 5 Factory, North Area of Comprehensive Bonded Area, First Ring South Road, High-tech Zone, Hengyang City, China
Service life: 3 years
Production date: Refer to the qualified label in the package
Date of preparation of the instructions: June 14, 2022

FCC regulatory conformance:

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions:

- (1) This device may not cause harmful interference.
- (2) This device must accept any interference received, including interference that may cause undesired operation.

NOTE: This equipment has been tested and found to comply with the limits for a **Class B** digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation.

This equipment generates uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help

NOTE: The manufacturer is not responsible for any radio or TV interference caused by unauthorized modifications to this equipment. Such modifications could void the user's authority to operate the equipment.

The SAR limit of USA (FCC) is 1.6 W/kg for head averaged over one gram of tissue. Device types REAL P1 (FCC ID:2A9JP-REALP1P2) has also been tested with 0mm against this SAR limit.

The SAR limit of USA (FCC) is 1.6 W/kg for head averaged over one gram of tissue. Device types REAL P3 (FCC ID:2A9JP-REALP3P4) has also been tested with 0mm against this SAR limit.