


80X85MM

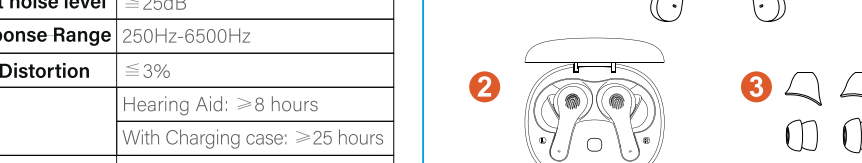

FCC ID:2A9O9-T002

GET HELP SERVICE

After-sale E-Mail: DiggingSoundAid@outlook.com

Website: www.diggingSound.com

WhatsApp: +86 13823673781


After-sale E-Mail: DiggingSoundAid@outlook.com

2

1 TECHNICAL SPECIFICATION

Model	T002
Maximum saturation sound	≤115dB+4dB
FOG50 MAX Gain	35dB ± 4dB
Equivalent input noise level	≤ 25dB
Frequency Response Range	250Hz-6500Hz
Total Harmonic Distortion	≤ 3%
Work Time	Hearing Aid: ≥ 8 hours With Charging case: ≥ 25 hours
Standby Time	≥ 150 hours
Charge Time	Hearing Aid: 1.5 hours Charging case: 2.5 hours
Power Input	Hearing Aid: DC 3.7V/60mA Charging Case: DC 3.7V/500mA
Bluetooth Distance	≥ 10M
Waterproof Grade	IPX6

2 CHECK YOUR HEARING AIDS


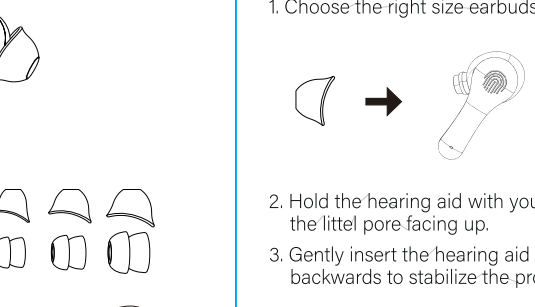


- 1 - Hearing Aid
- 2 - Charging case
- 3 - Silicone dome (Extra six sets)
- 4 - Type-c charging cable
- 5 - Cleaning tool

After-sale E-Mail: DiggingSoundAid@outlook.com

3

3 HOW TO WEAR YOUR HEARING AID

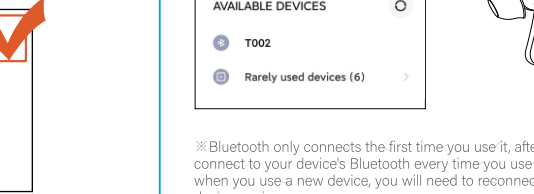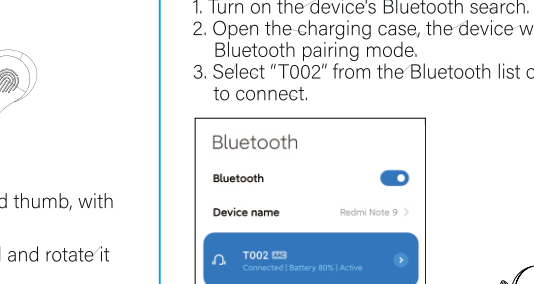


After-sale E-Mail: DiggingSoundAid@outlook.com

4

4 BLUETOOTH CONNECTION

1. Turn on the device's Bluetooth search.
2. Open the charging case, the device will automatically enter Bluetooth pairing mode.
3. Select "T002" from the Bluetooth list of your device and click to connect.

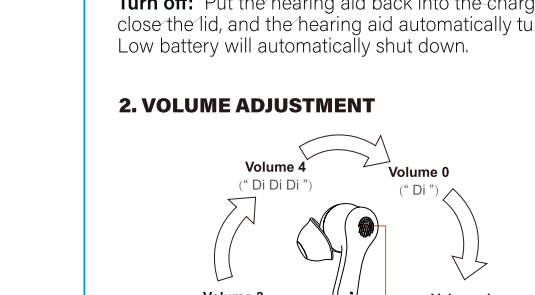

※ Bluetooth only connects the first time you use it, after that it will automatically connect to your device's Bluetooth every time you use your hearing aid. Of course, when you use a new device, you will need to reconnect Bluetooth to your new device again.

After-sale E-Mail: DiggingSoundAid@outlook.com

5


5 OPERATION TOUCH COMMANDS

1. POWER ON AND OFF

Turn on: Take out the hearing aid, it will turn on automatically. Or press and hold the touch button for 3S to turn on the hearing aid. Wait 10 seconds to enter hearing aid mode.

Turn off: Put the hearing aid back into the charging case, close the lid, and the hearing aid automatically turns off. Low battery will automatically shut down.

2. VOLUME ADJUSTMENT

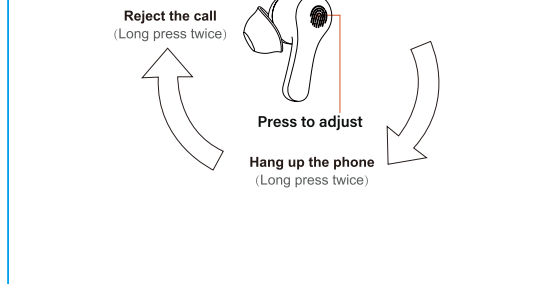


After-sale E-Mail: DiggingSoundAid@outlook.com

6

The volume goes from lowest (Volume 0) to highest (Volume 4), and then back to lowest (Volume 0)

3. PHONE CALL

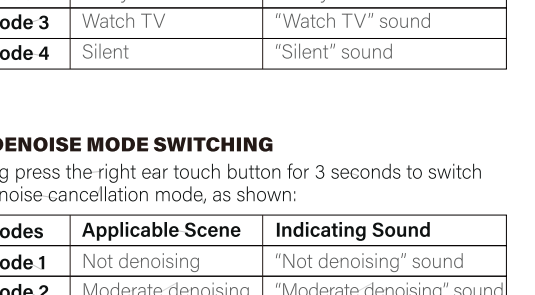

After-sale E-Mail: DiggingSoundAid@outlook.com

7

4. SCENE MODE SWITCHING

Long press the left ear touch button for 3 seconds to switch the scene mode, as shown:

Modes	Applicable Scene	Indicating Sound
Mode 1	Default	"Default" sound
Mode 2	Noisy	"Noisy" sound
Mode 3	Watch TV	"Watch TV" sound
Mode 4	Silent	"Silent" sound

5. DENOISE MODE SWITCHING

Long press the right ear touch button for 3 seconds to switch the noise cancellation mode, as shown:


Modes	Applicable Scene	Indicating Sound
Mode 1	Not denoising	"Not denoising" sound
Mode 2	Moderate denoising	"Moderate denoising" sound
Mode 3	Depth denoising	"Depth denoising" sound

After-sale E-Mail: DiggingSoundAid@outlook.com

8

6 CHARGE YOUR HEARING AIDS CASE

1. Put your Hearing Aids into the charging case.
2. Connect the charging case to a USB A/C adapter with the provided USB Type-C cable and charge for 3 hours.
3. Blinking Blue: Low battery in charging compartment. Red: Fully Charged.
4. Reset the Factory: Press and hold the reset button of the charging compartment for 5 seconds.

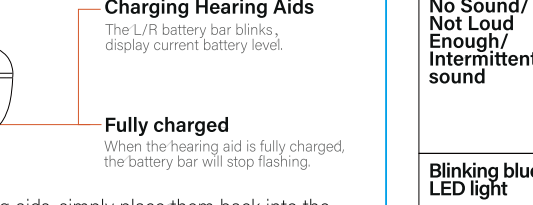


※ The charging time may vary slightly depending on your environment.

After-sale E-Mail: DiggingSoundAid@outlook.com

9

7 CHARGER YOUR HEARING AIDS

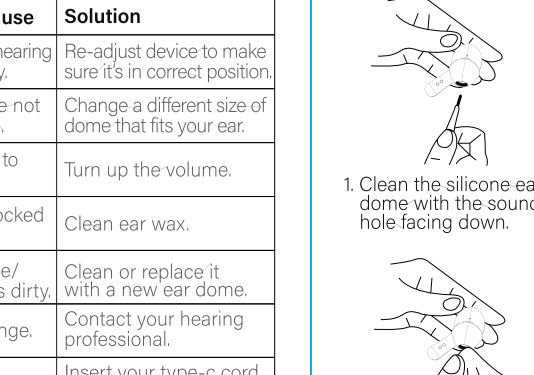

To charge your hearing aids, simply place them back into the charging case, which will charge both headphones 2-3 times.

※ When the charging compartment is less than 10%, the power display starts flashing and cannot charge the hearing aids.

After-sale E-Mail: DiggingSoundAid@outlook.com

10


8 TROUBLESHOOTING

Problem	Possible Cause	Solution
Feedback	Not wearing hearing aids correctly.	Re-adjust device to make sure it's in correct position.
	Ear dome are not the right size.	Change a different size of dome that fits your ear.
No Sound/ Not Loud Enough/ Intermittent sound	Volume is set to be low.	Turn up the volume.
	Ear canal blocked with dirt.	Clean ear wax.
	The ear dome/ sound hole is dirty.	Clean or replace it with a new ear dome.
Blinking blue LED light	Hearing change.	Contact your hearing professional.
	Low battery	Insert your type-c cord to charge the case.
Hearing aids whistling in the charging case	Dead battery	Put the hearing aid into charging case, insert your type-c cord to charge the case.

After-sale E-Mail: DiggingSoundAid@outlook.com

11

9 CLEANING AND MAINTENANCE

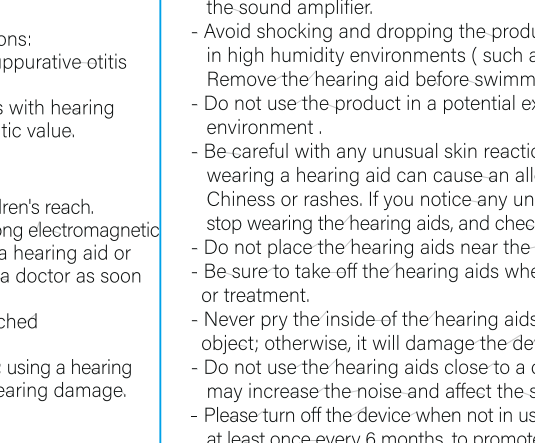

After-sale E-Mail: DiggingSoundAid@outlook.com

12

10 CAUTIONS

1.CONTRAINDICATION

- The device is unsuitable in the following situations:
- Usage is prohibited for patients with chronic suppurative otitis media and congenital ear atresia.
- This hearing aid can only compensate patients with hearing impairment and do not have hearing therapeutic value.

2.WARNINGS

- Keep small swallow-able accessories out of children's reach.
- Dropping, immersing in liquid, and accessing strong electromagnetic fields or over heat will damage hearing aids. If a hearing aid or accessories remain in the ear, please go to see a doctor as soon as possible, do not take it out by yourself.
- Warning not to use in explosive or oxygen-enriched atmospheres; it may damage the device.
- Warning that only the intended person can use it; using a hearing aid designed for someone else may result in hearing damage.

3.NOTICE

- Please do not use it at high volume for a long time. Otherwise, it will injure your ear. If you feel uncomfortable, please stop using it.

After-sale E-Mail: DiggingSoundAid@outlook.com

13

11 CUSTOMER SERVICE

If you have any problem in the process of using our products, please contact us in time. We are happy to help you with all questions about our products. And we will get back to you within 24 hours.

There are four ways to contact us:

1. Our customer service email: DiggingSoundAid@outlook.com
2. Our customer service whatsapp: +86 13823673781
3. Our Official Website: www.diggingSound.com
4. Enter our store that below the button "Add to cart" and find the button "Ask a question", which can also contact us directly.

We sincerely hope you can enjoy our products all the time!

12 RF EXPOSURE STATEMENT

The device has been evaluated to meet general RF exposure requirement. The device can be used in portable exposure condition without restriction.

The product is too small to place complete label information, so the label is placed in the user manual.

After-sale E-Mail: DiggingSoundAid@outlook.com

15

11 CUSTOMER SERVICE

If you have any problem in the process of using our products, please contact us in time. We are happy to help you with all questions about our products. And we will get back to you within 24 hours.

There are four ways to contact us:

1. Our customer service email: DiggingSoundAid@outlook.com
2. Our customer service whatsapp: +86 13823673781
3. Our Official Website: www.diggingSound.com
4. Enter our store that below the button "Add to cart" and find the button "Ask a question", which can also contact us directly.

We sincerely hope you can enjoy our products all the time!

12 RF EXPOSURE STATEMENT

The device has been evaluated to meet general RF exposure requirement. The device can be used in portable exposure condition without restriction.

The product is too small to place complete label information, so the label is placed in the user manual.

After-sale E-Mail: DiggingSoundAid@outlook.com

16

11 CUSTOMER SERVICE

If you have any problem in the process of using our products, please contact us in time. We are happy to help you with all questions about our products. And we will get back to you within 24 hours.

There are four ways to contact us:

1. Our customer service email: DiggingSoundAid@outlook.com
2. Our customer service whatsapp: +86 13823673781
3. Our Official Website: www.diggingSound.com
4. Enter our store that below the button "Add to cart" and find the button "Ask a question", which can also contact us directly.

We sincerely hope you can enjoy our products all the time!

12 RF EXPOSURE STATEMENT

The device has been evaluated to meet general RF exposure requirement. The device can be used in portable exposure condition without restriction.

The product is too small to place complete label information, so the label is placed in the user manual.

After-sale E-Mail: DiggingSoundAid@outlook.com

17