


## Keep Track of Your Injections

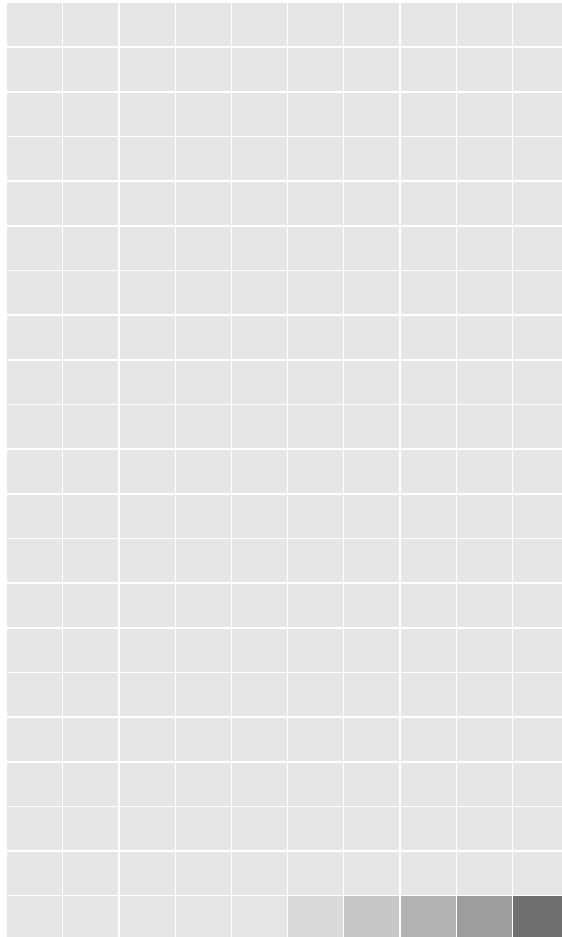


Device expiration

02 2024-08-27 8020100530\_04

## This is your Instructions for Use

- For quick overview of steps, read your Quick Reference Guide
- For training video, go to [www.skytrofa.com](http://www.skytrofa.com)

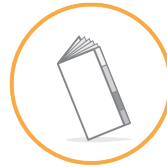

Instructions  
for Use



Quick  
Reference Guide



[www.skytrofa.com](http://www.skytrofa.com)

Training  
Video

Ascendis Pharma Customer Support



1-844-44ASCENDIS  
(1-844-442-7236)



### Distributed by:

Ascendis Pharma Endocrinology Inc.  
Princeton, NJ 08540  
USA



Phillips-Medisize A/S  
Gimsinglundvej 20  
DK-7600 Struer  
Denmark

This Instructions for Use has been approved by  
[Placeholder]  
Approved: Month, Year

# Instructions for Use

**Skytrofa® Auto-Injector**   
For SKYTROFA cartridges

Single-patient use

Watch Video



[www.skytrofa.com](http://www.skytrofa.com)



## Getting Started

## How to Use



## Troubleshooting and Care

## Product Information

### Status Icons



**Green Check Mark**  
Injection successfully completed



**Green Eye**  
Mixed medicine is ready for inspection in window



**Green Mixing**  
Medicine mixing is in progress

➤ Read more about status icons on pages 26-45 (and pages 52-63 for troubleshooting)



**Constant Green**  
➔ Battery is fully charged  
➔ The auto-injector is ready to use



**Flashing Green**  
➔ At least 1 injection remaining, but charging recommended after use



**Flashing Orange**  
➔ Battery needs charging

➤ Read more about battery status icons on page 25

## Parts Overview

### Green Top

Starts injection when pressed against skin

### Inspection Window

Allows for inspection of mixed medicine

### Progress Bar (with six light elements)

Shows progress of mixing and injection

➤ Read more on pages 30-35 and 42-43 (and pages 52-53 for troubleshooting)

### Status Icons

### Green Button

To turn on the SKYTROFA Auto-Injector or to reject mixed medicine

### Needle

To be screwed onto SKYTROFA Cartridge before inserting the cartridge in green top

### SKYTROFA Cartridge

Contains medicine and water for injection. If refrigerated, leave it at room temperature for 15 minutes before use

### Battery Icon

