OrCam **Read.**
TRANSFORM THE WAY YOU READ.

List of Languages.

English -----	02	Português -----	80	Magyar -----	158	简体中文 -----	236
Français -----	15	Čeština -----	93	Български -----	171	한국어 -----	249
Deutsch -----	28	Svenska -----	106	Türkçe -----	184	日本語 -----	262
Español -----	41	Norsk -----	119	Polski -----	197	አማርኛ -----	275
Nederlands -----	54	Dansk -----	132	Română -----	210	العربية -----	288
Italiano -----	67	Suomi -----	145	Русский -----	223	עברית -----	301

© 2022 OrCam Technologies Ltd. All rights reserved.

OrCam® is the trademark and brand name of OrCam Technologies Ltd. ("OrCam") and OrCam is the sole proprietor of all the rights and all the intellectual property rights in this guide and/or any other document produced by OrCam relating thereto. The documents are protected by applicable intellectual property laws including and without limitation copyrights, patents, design patents, trademarks, know how, trade secrets, and international treaties. No license or rights are hereby transferred to the user of this guide and said guide shall remain the sole and exclusive property of OrCam. No part of this guide shall be used, reproduced, translated, converted, modified, adapted, displayed, stored in a retrieval system, communicated, or transmitted by any means, for any commercial purpose, including and without limitation, sale, resale, distribution, reproduction, license, rental, or lease without receiving written permission from OrCam.

A list of OrCam's patents can be found in <https://www.orcam.com/en-us/patent>. OrCam makes no expressed or implied representation or warranty as to the accuracy or completeness of the contents of this guide, which is provided "as is", and assumes no responsibility or liability for any errors or inaccuracies that may appear in this document or for any product that may be provided in association with this document.

OrCam devices are intended to enhance the day-to-day lives of individuals with visual impairments, by helping them to study, work, socialize, navigate and experience the world around them. OrCam devices help improve their quality of life by empowering them with increased accessibility and independence. They are not substitutes for existing methods or forms of assistance and should not be relied upon in circumstances that are potentially life- or health-threatening or that could lead to financial loss. OrCam devices should especially never be relied upon to drive vehicles or operate heavy machinery of any kind. Please refer to the OrCam website for additional safety information: www.orcam.com/safety. To contact OrCam Support, please email support.email@orcaml.com. For the online user manual and tutorial videos, or for tips, advice, and troubleshooting support, visit <https://knowledgebase.orcam.com/knowledge>.

f i t y | www.orcam.com | info@orcaml.com

Corporate Headquarters

OrCam Technologies
3 Kiryat HaMada St.
Jerusalem 9777513
Israel
+972.2.591.7800

United States and Canada

OrCam Inc.
1115 Broadway, 11th floor
New York, NY 10010
1.800.713.3741

Germany

OrCam GmbH
c/o Regus Waidmarkt Cologne
Waidmarkt 11/4
50676 Köln
+49 (0) 211 540 138 11

United Kingdom

OrCam UK. Ltd.
Regus - Heathrow Bath Road
450 Bath Rd
London UB7 0EB
+44(20)304-82937

CE Compliant

Medical
Device

Before using your OrCam Read device for the first time and to ensure safe and proper use, visit our website at [**https://knowledgebase.orcam.com/knowledge**](https://knowledgebase.orcam.com/knowledge) to access the more detailed OrCam Read User Manual, tutorial videos, tips, advice, EMC declarations, and support.

Table of Contents.

Support.	06
What's in the Box?	07
Parts and Buttons.	08
Get Started.	09
Charge OrCam Read.	
Power On OrCam Read.	

Functions.	10
Press the Trigger Button.	
Press One Volume Button.	
Customize and Connect.	10
Connect to WiFi and Software Updates.	
Safety and Warnings.	11

Terms and Conditions.	12
Privacy.	
Warranty.	
Refund Policy.	
FCC Compliance.	
FCC Warning.	

Support.

Submit a Support Case.

If you are experiencing any issues, you can submit a support case by going to the following link or by using the QR code on the left.

USA/UK: <https://www.orcam.com/en-us/submit-case>

Germany: <https://www.orcam.com/de-de/submit-case2>

France: <https://www.orcam.com/fr-fr/submit-case>

Contact Support.

You can contact us directly through the following methods.

Email.

support.email@orcam.com

Telephone.

USA/Canada: +1-800-713-3741





UK/Ireland: +44(20)304-82937

Germany: +49-211-540-138-11


France: +33-800-903921

What's in the Box?

- 1.** OrCam Read device **2.** Quick User Guide **3.** Wall charger
4. Charging cable **5.** Earphones **6.** Lanyard **7.** Cleaning cloth **8.** Case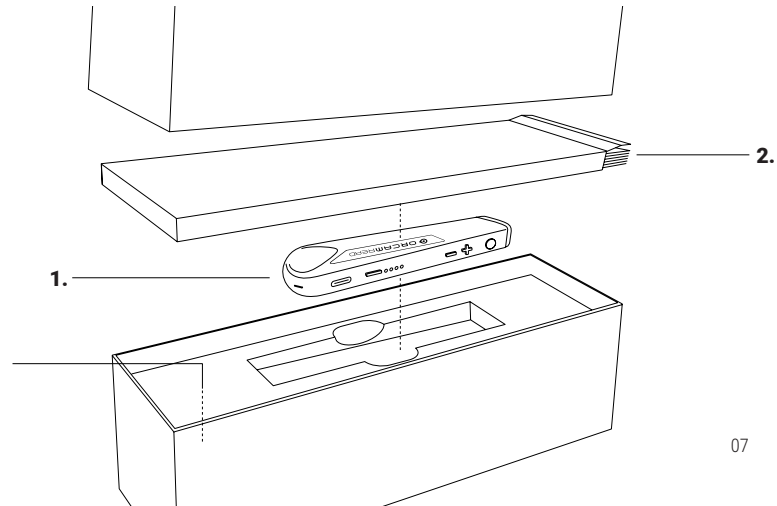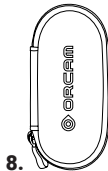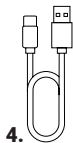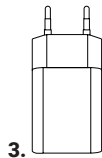

Parts and Buttons.

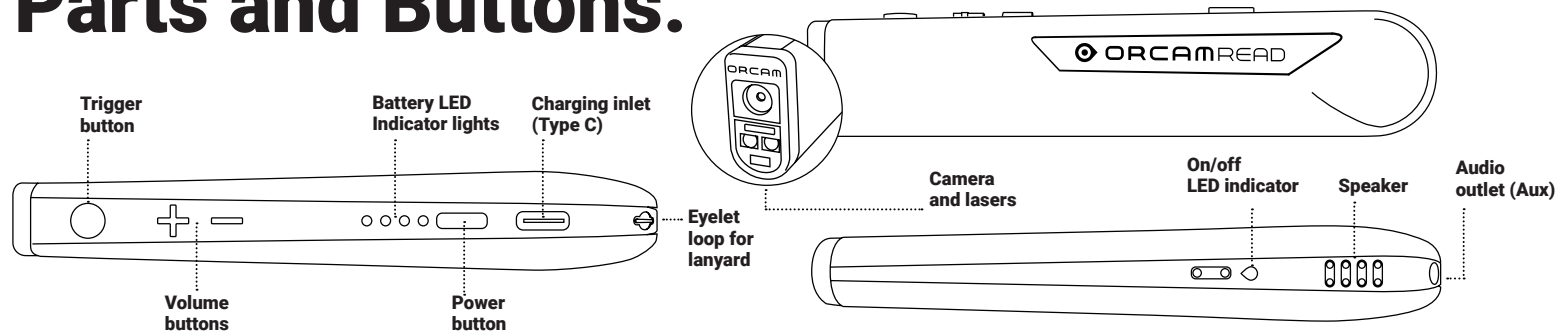Get Started.


Charge OrCam Read.

- To charge OrCam Read, insert the charging cable into the charging inlet. Plug the USB end of the charging cable into the wall charger. Plug the wall charger into an electrical outlet.
- Before turning on the device for the first time, charge your device for three hours.
- OrCam Read fully charges in approximately 90-120 minutes.
- The OrCam Read charger supports 100-240 V, 50/60 Hz.


Power On OrCam Read.

Press the power button for 2 seconds until you hear a beep.
Less than a minute later, the device will announce: *"OrCam Version [xx] is ready. Battery is [XX] percent charged"*.
Double click the power button to shut down.

Charge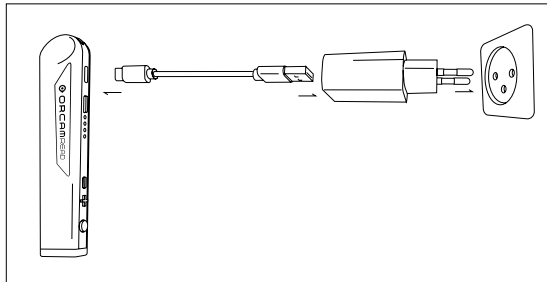

Power On

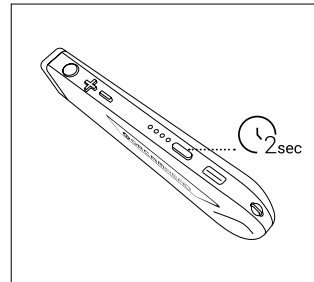Functions.

Press the Trigger Button.

Press the trigger button once and OrCam Read will read captured text, recognize barcodes or banknotes, stop reading, or make selections in the audio settings menu.

Press One Volume Button.

Press either volume button to adjust the volume, rewind or fast forward through text, or scroll through the audio settings menu.

Connect to WiFi and Software Updates.

Connecting your OrCam Read to a WiFi network ensures that your device will install the latest software updates.

The device must be connected to the charger throughout the downloading process. Please note that the OrCam Read supports 2.4 GHz WiFi, and the OrCam Read 3 also supports 5 GHz WiFi.

To learn how to connect to WiFi go to orcam.com

Safety and Warning.

Before using your OrCam device, read these precautions carefully to avoid dangerous situations and to ensure ideal performance of your device:

- Handle your OrCam device with care. It can be damaged if dropped, burned, punctured, or crushed.
- Check that there is no physical damage to your OrCam device such as any cracks or dents in the outer casing of the device. Do not open the outer casing of the device.
- Check that the camera lens of your OrCam device is not cracked, damaged, or scratched.
- Confirm that the camera lens is not covered or dirty. Read the cleaning instructions below on how to properly clean your OrCam device.
- Use only the charger supplied with your OrCam device. Using another charger may void your warranty and could cause severe injury to you or damage to the OrCam device.
- Check that the wall charger and charging cable are not damaged or broken.
- Do not attempt to replace the battery; you may damage it, which could cause overheating and even injury. The battery is designed to last for two years of average use (one charging cycle per day on average) after which it may need to be replaced.
- The OrCam device is designed to work in ambient temperatures between 32°F and 104°F (0°C and 40°C) and in normal conditions (Relative Humidity of 10% to 90% RH and Atmospheric Pressure of up to 2000m above sea level (700hPa)). The

device can be damaged if used outside of this range. Store and transport in a cool, dry place (32°F to 104°F (0°C to 40°C) and Relative Humidity of 5% to 90% and atmospheric pressure of 700hPa to 1060hPa)). If the OrCam device overheats, the OrCam device will perform its powering-off procedure.

- The OrCam device is water-resistant but not waterproof. This means the device can withstand light rain or drizzle for short periods of time, although it is not recommended. Do not use the OrCam device in heavy rain, at the beach, near a swimming pool, or in or near a bathtub or shower. Do not submerge the OrCam device in any liquid. The OrCam device meets the standards of IP Code: IP22.
- Do not force a connector into a port or apply excessive pressure to a button. This may cause damage that is not covered under the warranty. If a connector does not insert easily, check for obstructions in the port and verify that you have the correct connector for that port. Do not combine with software, hardware, or other equipment not supplied by OrCam. Exposing the device to magnetic fields may damage the unit or erase your data.
- When using the OrCam device or charging the battery, it is normal for the device to get warm. If the unit becomes excessively hot, leaks, or gives off a burning smell, turn the device off immediately and contact OrCam Support.
- If you experience any issues operating the OrCam device, or if the device or any of its components, such as the camera lens or outer casing or charger, becomes damaged or stops working, please contact OrCam Support and do not attempt to repair it by yourself. Only qualified OrCam personnel may repair or service your

device. Unauthorized repairs may void your warranty.

- OrCam recommends that the user should stay away from background noise so as to better hear the OrCam device. OrCam recommends that the user use a Bluetooth audio device such as earphones to better hear the OrCam device in loud environments.
- If you have specific questions about use alongside an internal pacemaker and/or defibrillator and/or cochlear implant, please consult your medical professional.
- If you experience an allergic reaction to any of the elements of the OrCam device, contact a medical professional immediately.
- Do not use your OrCam device to read medication labeling.
- OrCam Read meets the standard requirements in the US and EU for Class 1 medical devices.
- Do not stare into the laser beam.

Cleaning.

Clean your OrCam device if you feel it is dirty or has been transferred between users. OrCam recommends cleaning your OrCam device at least once every two weeks, especially the camera lens. Never touch the camera lens with your finger. Clean the lens only with the supplied microfiber cloth and eyeglass cleaning fluid. Do not use any abrasive materials as this may damage the OrCam device. Do not expose the device to chemicals, such as household cleaners, aerosol sprays, solvents, alcohol, or ammonia as these may damage the OrCam device.

Terms and Conditions.

Your use of your OrCam device (the "Product") is subject to the Terms and Conditions found at [orcam.com/en-us/orcam-products-terms-conditions](https://www.orcam.com/en-us/orcam-products-terms-conditions). Your agreement to these Terms and Conditions is a condition to your use of the Product.

Please read the User Manual in full, including the important Safety Instructions, before using your OrCam device. Do not use your OrCam device until you have received in-person or online training by an authorized OrCam Trainer™ or OrCam Training Program.

OrCam can take no responsibility if the Product was used other than in accordance with the instructions in the User Manual. Products should not be relied upon in circumstances that are potentially life- or health-threatening or that could lead to financial loss. OrCam devices should especially never be relied upon to drive vehicles or operate heavy machinery of any kind. OrCam shall not be responsible in the event that Products are used in such situations.

OrCam may receive information from third parties (i.e. Barcodes of Grocery Products in selected countries). OrCam received Barcode information from Barcodes of Grocery Products in Australia and New Zealand from GS1 Australia. OrCam accepts no responsibility for the accuracy of the information obtained from third parties.

Privacy.

OrCam thinks highly of your privacy and we are committed to protecting and safeguarding our clients' and users' personal information. Please read our Privacy Policy found at <https://www.orcam.com/en-us/orcam-products-privacy-statement>, which describes how we process personal data regarding our clients and users in connection with services provided by us, our website, and the OrCam Read mobile application (the "Services" and the "App", respectively); the types of personal data we collect and process; to what purposes we use and process your personal data; how your personal data is protected; and with whom your personal data may be shared.

It is important for us to enable you to understand our privacy policy and your rights in connection with the personal data we hold about you. You should also be aware that we use cookies and similar technologies to collect and save information when you visit our website, available at www.orcam.com, to ensure that we give you the best possible experience. To learn more about the use of such technologies, please see our Cookie Policy at <https://www.orcam.com/en/cookies-policy/>.

If you feel that a concern you have in connection with your privacy has not been met by the Privacy Statement, please contact us at: privacy@orcam.com and let us know. If you choose to download the App or use, interact with, register, or provide data to us in connection with the Services, you explicitly agree to the use of such data in accordance with this Privacy Statement. You may not use the Services or submit any data to us if you do not agree to any of the terms hereunder. This Privacy Statement shall not be construed in any manner to derogate from our Terms of Conditions or any other agreement made by and between OrCam and you.

Warranty.

Customer acknowledges that, other than as explicitly stated herein, OrCam does not make any representations or warranties to Customer regarding the Product, its functions or any images it produces.

The OrCam end-user limited product warranty (the "Limited Warranty") covers the owner of an OrCam product against defects in material or workmanship at the time of its original purchase, for a subsequent period of one (1) year, and for any additional applicable legal warranty period. Original proof of purchase/delivery is required from the owner in order to obtain this coverage.

OrCam will repair or replace, at its discretion, any product that is determined by OrCam to be covered under this Limited Warranty. OrCam will not be responsible for conditions arising as a result of intentional damage or misuse of the product. The owner is responsible for the costs of shipping the product to the location specified by OrCam.

This Limited Warranty is the sole and exclusive warranty applicable to OrCam products. No third party has the authority to make any representation, warranty, or agreement on behalf of OrCam with respect to OrCam products. No warranty of any kind or nature is made by OrCam beyond those expressly stated herein. In no event shall OrCam be liable for any special, collateral, indirect, punitive, incidental, consequential, or exemplary damages in connection with or arising out of the use of the product. In no event shall OrCam's liability exceed the price paid for the product.

Without limiting any provisions limiting liability under this agreement, the limited warranty set forth herein will not apply, and the owner will reimburse OrCam for any costs and expenses incurred in the event that:

- (i) the product has been used other than in accordance with the product documentation and terms of use or other written operating instructions provided by OrCam
- (ii) the product has been subject to tampering, misuse, negligence, or accident
- (iii) the product or parts identification labels have been removed or the product has been otherwise altered
- (iv) the product has been opened, modified, repaired, serviced, maintained, or altered by someone other than an authorized OrCam representative
- (v) the product has been combined with software, hardware, or other equipment not supplied by OrCam or not approved in writing by an authorized OrCam representative
- (vi) the product has been intentionally damaged or exposed to water, heat, or high voltage
- (vii) the product has been used by a person who has not completed training by an authorized OrCam Trainer™

Any parts replaced pursuant to this warranty will be under warranty for the remaining time of the original warranty period or for ninety (90) days from the date of repair, whichever is longer.

To register your device for warranty, please visit the warranty registration website.

If you believe that your product needs warranty support, contact support.email@orcaml.com. You will be responsible for shipping costs. However, if it is determined that the warranty request is valid, OrCam will pay the total cost of shipping the repaired or replaced product.

ORCAM DISCLAIMS ANY AND ALL WARRANTIES, WHETHER EXPRESSED OR IMPLIED, INCLUDING WITHOUT LIMITATION, THE IMPLIED WARRANTIES OF NON-INFRINGEMENT OF THIRD-PARTY RIGHTS, MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. ORCAM DOES NOT WARRANT THAT THE PRODUCT WILL BE ERROR FREE OR "BUG" FREE, WILL OPERATE WITHOUT INTERRUPTION, THAT ANY IMAGES OR SOUNDS PRODUCED BY THE PRODUCT WILL BE ACCURATE OR THAT THE OPERATION OF THE PRODUCT WILL BE SECURE AND HEREBY DISCLAIMS ANY AND ALL LIABILITY ON ACCOUNT THEREOF. THIS SECTION SHALL BE ENFORCEABLE TO THE FULLEST EXTENT ALLOWED BY APPLICABLE LAW.

Refund Policy.

You can return your undamaged device for a refund within the period required by local law or as agreed between you and OrCam at the time of purchase. Training fees are non-refundable. Please contact your local representative or OrCam Support to initiate a return process.

When you return the OrCam device, please follow OrCam Support instructions carefully. The device must be in the same condition that you received it and undamaged in any way. Your refund and how it is issued may differ based on how the device was purchased,

its condition, and how long you've had the device.

OrCam will process your return only after the carrier has received your device and delivered to OrCam. It usually takes up to three weeks for OrCam to receive (for international shipping involving local customs it may take longer) and process your return. Once OrCam issues your refund, it may take additional time for your funds to be made available in your account by your financial institution. For credit card payments, it may take 5 to 10 business days.

If you have any questions, please feel free to contact the OrCam Support team.

FCC Compliance.

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.

- Increase the separation between the equipment and receiver.
- Connect the equipment to an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician.

This device complies with Part 15 of the FCC Rules.

Operation is subject to the following two conditions:

1. This device may not cause harmful interference.
2. This device must accept any interference received, including interference that may cause undesired operation.

FCC Warning.

Modifications not expressly approved by the manufacturer could void the user authority to operate the equipment under FCC Rules.

NOTE: THE MANUFACTURER IS NOT RESPONSIBLE FOR ANY RADIO OR TV INTERFERENCE CAUSED BY UNAUTHORIZED MODIFICATIONS TO THIS EQUIPMENT. SUCH MODIFICATIONS COULD VOID THE USER'S AUTHORITY TO OPERATE THE EQUIPMENT.

ISED Compliance:

This device contains licence-exempt transmitter(s)/receiver(s) that comply with Innovation, Science and Economic Development Canada's licence-exempt RSS(s). Operation is subject to the following two conditions:

1. This device may not cause interference.
2. This device must accept any interference, including interference that may cause undesired operation of the device.

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Authorized Swiss Community Representative:

Arazy Group Switzerland GmbH
Bruderholzallee 53 4059, Basel, Switzerland
Telephone Number: +41 335332267

UK Responsible Person:

MEDES LIMITED
5 Beaumont Gate, Shenley Hill, Radlett, Hertfordshire, London WD7 7AR, UK
medes@arazygroup.com

Applicable Standards.

OrCam Read meets all applicable requirements of EU MDR 2017/745 and 2007/47/EEC and the essential requirements set out in Annex 1.
EN IEC 60601-1. EN IEC 60601-1-2. EN IEC 60601-1-6. EN IEC 60601-1-11. ETSI EN 301 489-1. ETSI EN 301 489-17. FCC Part 15, Subpart B. IEC 62366-1:2015. IEC 60825:2014. EN IEC 62304:2006. RoHS Directive 2011/65/EU. WEEE. EN ISO 14971:2016. ISO 13485:2016. ISO 20417 2021. EN ISO 15223-1:2016.

Waste Electrical and Electronic Equipment Directive Compliant
Contact OrCam for proper disposal.

RoHS

Restriction of Hazardous Substances Directive Compliant (materials and components)

FCC

FCC ID:
2AAMI-READ2WAY
2AAMI-READ

Refer to User Instruction Guide and Manual

Class II equipment according to IEC 60536

Max output of laser radiation: 0.371 mW
Emitted wavelength: 635 nm
Name of the standard: IEC 60825-1

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English-----	02	Português-----	80	Magyar -----	158	简体中文-----	236
Français -----	15	Čeština -----	93	Български -----	171	한국어-----	249
Deutsch -----	28	Svenska-----	106	Türkçe-----	184	日本語-----	262
Español-----	41	Norsk-----	119	Polski-----	197	አማርኛ-----	275
Nederlands-----	54	Dansk -----	132	Română-----	210	العربية-----	288
Italiano-----	67	Suomi -----	145	Русский-----	223	עברית-----	301

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