


# monbaby

## SMART WEARABLE BABY MONITOR

works with iPhone® and Android™ phones

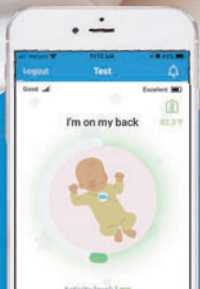

# CONTENT

1. WHAT'S IN THE BOX -----	3
2. QUICK START GUIDE -----	4
3. TROUBLESHOOTING & SUPPORT -----	8
4. WARNING -----	9
5. DOWNLOADING THE APP -----	10
6. USING THE APP -----	10
6.1 Calibration -----	10
6.2 Dashboard -----	11
7. ALERT SETTINGS -----	12
7.1 General Options -----	12
7.2 Alert Options -----	13
8. FASTENING THE SMART BUTTON -----	14
9. REMOVING THE SMART BUTTON -----	14
10. SAFETY AND OTHER IMPORTANT NOTES -----	14
11. LIMITED WARRANTY (US) -----	16
12. LIMITED WARRANTY (EU AND OTHER JURISDICTIONS; EXCLUDING CANADA AND THE US) -	20
13. LIMITED WARRANTY (CANADA) -----	24
14. RETURN POLICY (US PURCHASES MADE THROUGH MONBABY.COM) -----	29
15. DISCLAIMERS AND LEGAL -----	30

If you are having any issues with setting up the device, please contact us at [support@mondevices.com](mailto:support@mondevices.com).

# WHAT'S IN THE BOX

The MonBaby Retail Bundle includes

- 1 One (1) MonBaby Smart Button™
- 2 One (1) MonBridge BLE to Wi-Fi Base
- 3 One (1) Wireless charger DC 9V 1.1A
- 4 One (1) USB Cable
- 5 One (1) 2.5A Power Supply
- 6 Supporting Documentation including the User Manual

MONBABY DOES NOT REPLACE PARENTING.

Your MonBaby product is a baby care aid, not a medical device, and it is not intended to diagnose, treat, cure or prevent any disease, including Sudden Infant Death Syndrome (SIDS).

MONBABY IS A SMART WEARABLE BABY MONITOR AND IS NOT INTENDED TO BE USED AS A TOY.



© 2020 MonDevices Inc. All rights reserved. The MonBaby word mark and logo are trademarks of MonDevices Inc. Apple, the Apple logo, and iPhone are trademarks of Apple Inc., registered in the U.S. and other countries. App Store is a service mark of Apple Inc. Android is a trademark of Google Inc. The Android robot is reproduced or modified from work created and shared by Google and used according to terms described in the Creative Commons 3.0 Attribution License. The Bluetooth® word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. and any use of such marks by MonDevices, Inc. is under license. IOS is a registered trademark of Cisco Systems, Inc and/or its affiliates in the U.S. and certain other countries.

FCC ID:


# QUICK START GUIDE

## FOR DIRECT BLUETOOTH CONNECTION

(Direct Bluetooth Connection allows one smart phone to connect to the button and is ideal for same room use and works at a maximum of 40 ft or 12m)

- 1 Unpack the MonBaby 2 Smart Button, Wireless Charger, USB cable.
- 2 Download the MonBaby application from the AppStore (Apple devices) or Google Play (Android devices).
- 3 Plug the wireless charger into a USB compatible plug or the USB port on your laptop or home computer.
- 4 Place the MonBaby 2 logo side up at the center of the wireless charger. The number of times the LED blinks indicates its power level.
  - 3 blinks = Battery life is at about 25%
  - 2 blinks = Battery life is at about 50%
  - 1 blink = Battery life is at about 75%
  - No blinks = Battery is at full power
- 5 When the MonBaby battery level is at 100% remove it from the wireless charging pad and place it on a flat immobile surface such as a table. Please make sure the Bluetooth function is enabled on your smart phone or tablet.
- 6 Launch the MonBaby application and fill out the registration information.
- 7 Tap 'Connect' at the connection screen and wait for the application to scan your MonBaby. It should appear in blue text and please tap connect. You can now name the MonBaby device and then tap 'Calibrate.'

- 8 Your MonBaby 2's LED should flash several times as it connects and calibrates to your smart device. The calibration process memorizes the position of the button so please do not touch or move it during this time. When it is finished it will bring you to the main dashboard. From here you can tap the image of the baby to upload or take a photo of your baby to replace the stock image.
- 9 You can now tap any of the icons at the bottom of the screen to access: main dashboard, graphs and history, nighttime mode, alert settings, and general settings and information.
- 10 You are now ready to place the button on your baby's clothing and begin monitoring their activity! We recommend placing it between their chest and lower belly. You may need to place it in an area where their breathing movement is most pronounced.

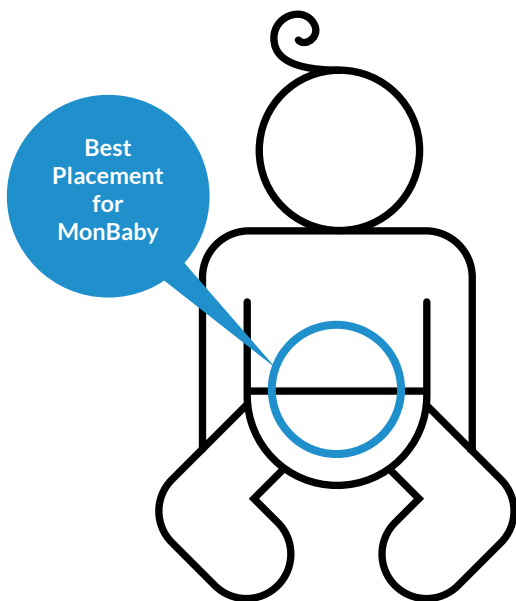

## FOR WI-FI CONNECTION

(Wi-Fi Connection allows several smart phones to connect to the button and is ideal for use anywhere in your home as long as you receive a good wi-fi signal)

- 1 Unpack the MonBaby 2 Smart Button, MonBridge Base Wireless Charger, USB cable, and 2.5A Power supply.
- 2 Download the MonBaby application from the AppStore (Apple devices) or Google Play (Android devices).
- 3 Plug the wireless charger into a USB compatible plug or the USB port on your laptop or home computer.
- 4 Place the MonBaby 2 logo side up at the center of the wireless charger. The number of times the LED blinks indicates its power level.
  - 3 blinks = Battery life is at about 25%
  - 2 blinks = Battery life is at about 50%
  - 1 blink = Battery life is at about 75%
  - No blinks = Battery is at full power
- 5 When the MonBaby battery level is at 100% remove it from the wireless charging pad and place it on a flat immobile surface such as a table. Please make sure your phone is connected to your wi-fi router and that the Bluetooth function is turned on.
- 6 Plug the USB 2.5A Power Supply into a wall socket within 5-10 feet of where you plan to use the MonBaby 2. Attach the cable to the back of the MonBridge base all the way to the right under the "power" symbol, and place the MonBaby 2 next to it. You can bring the MonBridge to different parts of your home, but for the best results we suggest always keeping it within 5-10 feet of the MonBaby 2 while it is being used.



- 7 Please allow 1 to 2 minutes for the MonBridge base to connect to the MonBaby 2 button (The base will connect to the button automatically).
- 8 Launch the MonBaby application on your smart device. After registering, please choose "Configure Wi-Fi."
- 9 If your MonBaby button and MonBridge base are next to each other and ready to setup please tap "Next." The application will now scan for the MonBridge base. Once it discovers it please tap on the "Connect to MonBridge and Configure." If the application does not find it via the scan please make sure the Bluetooth is enabled on your smart device and try again.
- 10 Please choose the security protocol of your Wi-Fi router (most routers use WPA2 which is the default setting), and enter your wifi SSID and password. Both fields are case sensitive. Tap connect when finished.
- 11 The application will finish the setup and connection process in a pop up window. When the text "WiFi connection OK" appears please tap "Continue." This should bring you back to the connection screen. The "Configure WIFI" should have a green check mark next to it.
- 12 You are now ready to connect to your MonBaby via Wi-Fi. Tap "Connect" from the connection page and choose "Connect via WIFI" in green type. Choose a name for the button and allow it to calibrate and connect. You are now ready to place the button on your baby's clothing and begin monitoring their activity! We recommend placing it between their chest and lower belly. You may need to place it in an area where their breathing movement is most pronounced.
- 13 If you move or purchase a new router you can reset the MonBridge base by tapping the "Configure WIFI" button and then tapping "MonBridge Configured." This will now give you the option to reset the connection and create a new one if necessary.

We recommend that you keep the MonBaby 2 button and the MonBridge base within 10 feet of each other for best results. You can connect multiple phones to the same button simultaneously and receive the data from the button. Once the MonBridge/Monbaby button setup is complete other smart devices in your household can download the application and connect to the Wi-Fi signal.

**IMPORTANT FOR iOS USERS:** The MonBaby application must be kept active on your screen or the connection will be lost. If you hit the home button or use a different application the MonBaby application will lose the signal.

The app features and layout described herein may be subject to change as we continue to optimize the performance of the application and develop new functionality for our users. For our latest app and MonBaby features, please contact us at [support@mondevices.com](mailto:support@mondevices.com) and/or please check the instructions incorporated into the app.

## TROUBLESHOOTING & SUPPORT

If you are having any problems with the setup or operation of your MonBaby 2 DO NOT RETURN THE DEVICE, email our support team at [support@mondevices.com](mailto:support@mondevices.com).

To find the latest version of the MonBaby manual, please check out the support page or our FAQ on [www.monbaby.com](http://www.monbaby.com).





# WARNING

**THIS DEVICE CONTAINS ELECTRICAL EQUIPMENT THAT COULD CAUSE INJURY IF HANDLED IMPROPERLY. IMPROPER HANDLING COULD ALSO AFFECT THE PERFORMANCE OF THIS DEVICE:**

- DO NOT** disassemble your MonBaby 2 product.
- DO NOT** disassemble your MonBridge product.
- DO NOT** disassemble the wireless charger.
- DO NOT** use your MonBaby product if you are not able to snugly attach the device pursuant to the instructions provided.
- DO NOT** use your MonBaby product if the Smart Button or enclosure is warped, cracked or otherwise damaged.
- DO NOT** place your MonBaby 2 product in a dishwasher, washing machine or dryer. Do not submerge your MonBaby 2 in water or wash it under a faucet. Clean carefully with a clean, damp cloth.
- DO NOT** expose your MonBaby product to extremely high or low temperatures.
- DO NOT** expose your MonBaby product to direct sunlight for an extended period of time.

# DOWNLOADING THE APP

To download the Monbaby App, visit MonBabyApp.com using your mobile phone browser, and you will be directed to the iPhone App Store® or Google Play™ app listing.

You can also scan the QR Code on the right:



You can find our video manual located on our website:  
<https://monbaby.com/video-manual/>


## USING THE APP

Once you have your MonBaby Smart Button fastened and connected to the app, you can use the app to monitor your baby's sleeping habits. You can see the details of the app features below.

### Calibration

Please allow your device to calibrate on a flat immobile surface during the connection process.

Please wait until calibration is completed. If you close the calibration window, you will exit the dashboard, and log off the device.

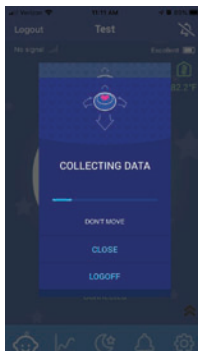



For best results, place the MonBaby Smart Button near the center of your child between the child's chest and abdomen, with the baby lying on his or her back on a firm sleep surface.

# Dashboard

Your Baby's  
current body  
position

Tap  
to snooze  
all alerts

Tap  
to change  
from F to C

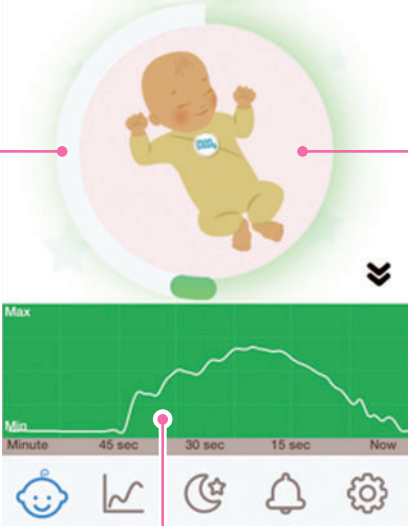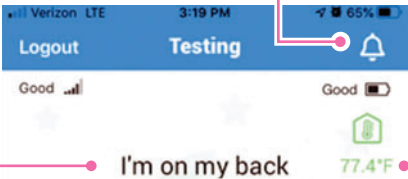

Activity  
level  
bar

Real time  
Activity  
Graph

Tap  
to upload  
photo

## Activity Bar

This bar shows you the activity level of your baby

## Status Text

This text will let you know the current position of your baby (on the back, stomach or side)

## Temperature

Current ambient temperature around your baby's body

## Image of the Baby

Tap to upload or take a new photo

## Current Battery Level

This will tell you when it is time to recharge the MonBaby 2

## Signal Strength

This will tell you the signal strength of the Bluetooth or Wi-Fi signal

## Real-Time Activity Graph

The Real-Time Activity chart shows you your baby's movement activity for the past minute. Tap the black arrows to show or hide the graph

# ALERT SETTINGS

## General Options

### Optional Calibration

Set to on if you want to skip calibrating the button each time you connect

### Night Mode

Change the background colors to make it easier to see in a dark room

### MonBaby Light

Set to on to activate the MonBaby 2 LED to make it easier to find in the dark

## Brightness

Change the screen brightness to a comfortable level

## Alert Options

(You can double tap the alert description to change the alert sounds)

### Breathing movement alert

This alert will activate if it does not detect any breathing movement after 15 consecutive seconds

### Temperature Alert

Double tap the description to change the reading from F to C, and set the upper and lower temperature limits. The alert will activate if the temperature exceeds the upper or lower limits

### Orientation Alert

The Alert will activate if your baby rolls over onto his or her stomach

### Activity Alert/High Activity Alert

Alerts you if the baby wakes up or, in the case of high activity alert, adjusts the sensitivity level of the application when this alert is used on the baby who is active.

### Proximity Alert

Alerts you if the MonBaby Smart Button (and your child) is out of range of your smart phone device

### Fall Detection Alert

Alerts you if the baby has fallen

### Battery Alert

Alerts you if the MonBaby battery is running low (please recharge the MonBaby 2 using the wireless charger)

# FASTENING THE SMART BUTTON

- 1 Place the MonBaby 2 snap ring on the inside of your baby's pajamas or onesie.
- 2 Align the MonBaby 2 Button on the outside of your baby's pajamas (logo side facing out) with the snap ring, and press it into the enclosure so that it is fastened to the clothing.

# REMOVING THE SMART BUTTON

To remove the MonBaby Smart Button, place your hand under your baby's pajamas and, using your index finger, push the smart button out through the hole in the center of the Enclosure. Be sure to have your other hand on the MonBaby Smart Button to collect it after you push it out, and collect the snap ring. Place the button inside the snap ring when not in use.

# SAFETY and other IMPORTANT NOTES

**MONBABY DOES NOT REPLACE PARENTING.**

Your MonBaby product is a baby care aid, not a medical device, and it is not intended to diagnose, treat, cure or prevent any disease, including Sudden Infant Death Syndrome (SIDS).

**MONBABY 2 IS A SMART WEARABLE BABY MONITOR  
AND IS NOT INTENDED TO BE USED AS A TOY.**

Your MonBaby 2 product is not a substitute for responsible direct supervision by parents and adult caregivers. Check on your baby at regular intervals. Create a safe sleep environment for your baby, get training in infant resuscitation and immediately call the emergency assistance hotline in your area if your baby has stopped breathing.

Parents and caregivers should consult their qualified medical professional regarding sleep safety or SIDS risk reduction guidelines and check the relevant government issued recommendations in their area regarding this topic on an ongoing basis. Common guidelines for infant sleep and SIDS reduction, such as those issued by the American Academy of Pediatrics available at [www.aap.org](http://www.aap.org), include, but are not limited to the list below. Ensure to consult with your qualified medical professional prior to following any of the listed guidelines.

- Place your baby on his or her back for every sleep time.
- Always use a firm sleep surface.
- Your baby should sleep in the same room as you do, but not in the same bed (room-sharing without bed-sharing).
- Keep pillows, blankets, stuffed toys and other soft objects or loose bedding out of your baby's crib.
- Choose a safe crib without drop-side rails and never use a crib that has been recalled or that is broken or missing parts.
- Offer a pacifier at nap time and bedtime.
- Avoid covering your baby's head or overheating.
- Keep your baby away from smokers and places where people smoke.
- Consult your pediatrician as he or she may recommend variations from these guidelines based on your baby's individual circumstances.

### CAREFULLY FOLLOW INSTRUCTIONS

- Failure to follow instructions will result in erroneous MonBaby 2 operations and could affect connectivity or accuracy of monitoring information.

WE RECOMMEND THAT MONBABY BE PLACED AT THE CENTER BETWEEN THE CHEST AND ABDOMEN OF THE BABY

Even though MonBaby can be placed anywhere and stores its position, we recommend that the MonBaby be placed near the center of the baby, between the baby's chest and abdomen.

Although the MonBaby may work well at longer distances, environmental factors such as dense wall material made out of brick or metal or other devices in the home that may be

## Cleaning the Device

In case your MonBaby gets dirty, it can be cleaned carefully with a damp cloth or paper towel. Do not forget to take off the MonBaby before you throw dirty clothes in the washer.

## LIMITED WARRANTY (US)

MonDevices, Inc ("MonDevices") warrants to the original purchaser only that the MonBaby device you purchased (the "Product") shall be free from defects in materials and workmanship under normal use for a period of one (1) year from the date of retail purchase by the original purchaser ("Warranty Period").

### 1. Remedies:

If a hardware defect arises and a valid claim is received by MonDevices within the Warranty Period, MonDevices will, at its option and to the extent permitted by law, either (1) repair the Product at no charge, using new or refurbished replacement parts, or (2) exchange the Product with a new or refurbished Product. If MonDevices, in its sole discretion, determines it is not reasonable to replace the defective Product, MonDevices may refund to you the purchase price paid for the Product. In the



event of a defect, to the extent permitted by law, these are your sole and exclusive remedies. Shipping and handling charges may apply except where prohibited by applicable law. This Limited Warranty is valid only in the jurisdictions where the Products are sold by MonDevices itself or through its authorized reseller or agent, and is valid to the extent permitted by the applicable laws of such jurisdictions. Any replacement hardware product will be warranted for the remainder of the original warranty period or thirty (30) days, whichever is longer, or for any additional period of time that may be applicable in your jurisdiction.

## 2. How to Obtain Warranty Service:

To obtain warranty service, you must obtain a Return Merchandise Authorization (RMA) from our customer service team and deliver the Product, in either its original packaging or packaging providing an equal degree of protection, together with proof of purchase, to the address specified by MonDevices. You agree to pay the cost of shipping the Product to MonDevices. By sending the Product, you agree to transfer ownership to MonDevices. If the claim is justified based on this limited warranty, MonDevices shall bear the cost of shipping the repaired or replacement Product to you. Any Product returned to MonDevices without a valid warranty claim or without a RMA may be rejected, returned at sender's cost (subject to prepayment) or kept for 30 days for sender's pick-up and then disposed of in MonDevices' sole discretion. For specific instructions on how to obtain warranty service on your Product, visit the MonBaby website <https://monbaby.com/support>. It is your responsibility to backup any data, software, or other materials you may have stored or preserved on the Product. It is likely that such data, software, or other materials will be lost or reformatted during service, and MonDevices will not be responsible for any such damage or loss.

## 3. Exclusions and Limitations:

This Limited Warranty applies only to the Product manufactured by or for MonDevices that can be identified by the "MonBaby" trademark, trade name, or logo affixed to it. The Limited Warranty does not apply to any (a) MonDevices products and services

other than the Product, (b) consumables (such as batteries), or (c) software, even if packaged or sold with the Product or embedded in the Product. Please refer to the Terms of Service that accompany the MonDevices mobile application for details of your rights with respect to its use.

THE PRODUCT IS NOT A MEDICAL DEVICE; IS NOT REGULATED OR APPROVED BY THE U.S. FOOD AND DRUG ADMINISTRATION; AND IS NOT DESIGNED TO DETECT OR PREVENT CAUSES OF SUDDEN INFANT DEATH SYNDROME OR ANY OTHER MEDICAL CONDITION. THE PRODUCT

IS INTENDED TO BE USED WITH MONDEVICES' MOBILE APPLICATION TO HELP YOU MONITOR YOUR BODY DATA AND IS NOT A SUBSTITUTE FOR MEDICAL CARE OR ADULT SUPERVISION.

MonDevices does not warrant that the operations of the Product will be uninterrupted or error-free. MonDevices is not responsible for damages arising from failure to follow instructions relating to the Product's use.

This warranty does not apply to a Product or part of a Product that has been altered or modified (e.g., to alter functionality or capability) by anyone who is not a representative of MonDevices. In addition, this Limited Warranty does not apply to: (a) damage caused by use with non-MonDevices products; (b) damage caused by accident, abuse, misuse, spillage of food or liquid

or other external causes; (c) damage caused by operating the Product outside the permitted or intended uses described by MonDevices or with improper voltage or power supply; (d) use for medical, healthcare or treatment purposes, or (e) damage caused by service performed by anyone who is not a representative of MonDevices. Recovery and reinstallation of software programs and user data are not covered under this warranty. This warranty is void if a Product is returned with removed, damaged or tampered labels or any alterations (including the unauthorized removal of any component or external cover).

No MonDevices reseller, agent, or employee is authorized to make

any modification, extension, or addition to this Limited Warranty. If any term is held to be illegal or unenforceable, the legality or enforceability of the remaining terms shall not be affected or impaired.

4. Implied Warranties and Conditions:

EXCEPT TO THE EXTENT PROHIBITED BY APPLICABLE LAW, ALL IMPLIED WARRANTIES AND CONDITIONS (INCLUDING WARRANTIES AND CONDITIONS OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE) SHALL BE LIMITED IN DURATION TO THE DURATION OF THIS LIMITED WARRANTY. Some jurisdictions do not allow limitations on the duration of an implied warranty or condition, so the above limitation may not apply to you. This warranty gives you specific legal rights, and you may also have other rights that vary from state to state.

5. Limitation of Damages:

EXCEPT TO THE EXTENT PROHIBITED BY APPLICABLE LAW, MONDEVICES SHALL NOT BE LIABLE FOR ANY INCIDENTAL, INDIRECT, SPECIAL, OR CONSEQUENTIAL DAMAGES, OR ANY LOSS OF PROFITS, REVENUE OR DATA, RESULTING FROM ANY BREACH OF EXPRESS OR IMPLIED WARRANTY OR CONDITION, OR UNDER ANY OTHER LEGAL THEORY, EVEN IF MONDEVICES HAS BEEN ADVISED OF THE POSSIBILITY OF SUCH DAMAGES AND REGARDLESS OF THE FORM

OF ACTION WHETHER IN CONTRACT, TORT (INCLUDING NEGLIGENCE) OR ANY OTHER LEGAL OR EQUITABLE THEORY. Some jurisdictions do not allow the exclusion or limitation of special, indirect, incidental or consequential damages, so the above limitation or exclusion may not apply to you.

6. Governing Law:

This Limited Warranty shall be governed by the laws of the State of New York, USA, without giving effect to any conflict of laws principles that may provide the application of the law of another jurisdiction.

## 7. National Statutory Rights:

Consumers in some jurisdictions may have legal rights under applicable national legislation governing the sale of consumer goods, including, without limitation, national laws implementing

EC Directive 99/44. These rights are not affected by the warranties in this Limited Warranty.

If you have any questions, please contact MonDevices at [support@mondevices.com](mailto:support@mondevices.com).

## LIMITED WARRANTY (EU AND OTHER JURISDICTIONS; EXCLUDING CANADA AND THE US)

MonDevices, Inc (“MonDevices”) warrants to the original purchaser only that the MonBaby device you purchased (the “Product”) shall be free from defects in materials and workmanship under normal use for a period of one (1) year from the date of retail purchase by the original purchaser (“Warranty Period”).

### 1. Remedies:

If a hardware defect arises and a valid claim is received by MonDevices within the Warranty Period, MonDevices will, at its option and to the extent permitted by law, either (1) repair the Product at no charge, using new or refurbished replacement parts, or (2) exchange the Product with a new or refurbished Product. If MonDevices, in its sole discretion, determines it is not reasonable to replace the defective Product, MonDevices may refund to you the purchase price paid for the Product. In the event of a defect, to the extent permitted by law, these are your sole and exclusive remedies. Shipping and handling charges may apply except where prohibited by applicable law. This Limited Warranty is valid only in the jurisdictions where the Products are sold by MonDevices itself or through its authorized reseller or agent, and is valid to the extent permitted by the applicable laws

of such jurisdictions. Any replacement hardware product will be warranted for the remainder of the original warranty period or thirty (30) days, whichever is longer, or for any additional period of time that may be applicable in your jurisdiction.

## 2. How to Obtain Warranty Service:

To obtain warranty service, you must obtain a Return Merchandise Authorization (RMA) from our customer service team and deliver the Product, in either its original packaging or packaging providing an equal degree of protection, together with proof of purchase, to the address specified by MonDevices or by its authorized reseller or agent. You agree to pay the cost of shipping the Product to MonDevices. By sending the Product, you agree to transfer ownership to MonDevices or its authorized reseller or agent. If the claim is justified based on this limited warranty, MonDevices or its authorized reseller or agent shall bear the cost of shipping the repaired or replacement Product to you. Any Product returned to MonDevices or its authorized reseller or agent without a valid warranty claim or without a RMA may be rejected, returned at sender's cost (subject to prepayment) or kept for 30 days for sender's pick-up and then disposed of in MonDevices' or its authorized reseller's or agent's sole discretion. For specific instructions on how to obtain warranty service on your Product, visit the MonBaby website <https://monbaby.com> or contact us at [support@mondevices.com](mailto:support@mondevices.com).

It is your responsibility to backup any data, software, or other materials you may have stored or preserved on the Product. It is likely that such data, software, or other materials will be lost or reformatted during service, and neither MonDevices nor its authorized reseller or agent will not be responsible for any such damage or loss.

## 3. Exclusions and Limitations:

This Limited Warranty applies only to the Product manufactured by or for MonDevices that can be identified by the "MonBaby" trademark, trade name, or logo affixed to it. The Limited Warranty does not apply to any (a) MonDevices products and services other than the Product, (b) consumables (such as batteries),

or (c) software, even if packaged or sold with the Product or embedded in the Product. Please refer to the Terms of Service that accompany the MonDevices mobile application for details of your rights with respect to its use.

THE PRODUCT IS NOT A MEDICAL DEVICE; IS NOT REGULATED OR APPROVED BY ANY GOVERNMENT AGENCY IN YOUR JURISDICTION AS A MEDICAL DEVICE; AND IS NOT DESIGNED TO DETECT OR PREVENT CAUSES OF SUDDEN INFANT DEATH SYNDROME OR ANY OTHER MEDICAL CONDITION. THE PRODUCT IS INTENDED TO BE USED WITH MONDEVICES' MOBILE APPLICATION TO HELP YOU MONITOR YOUR BODY DATA AND IS NOT A SUBSTITUTE FOR MEDICAL CARE OR ADULT SUPERVISION.

MonDevices does not warrant that the operations of the Product will be uninterrupted or error-free. MonDevices is not responsible for damages arising from failure to follow instructions relating to the Product's use.

This warranty does not apply to a Product or part of a Product that has been altered or modified (e.g., to alter functionality or capability) by anyone who is not a representative of MonDevices. In addition, this Limited Warranty does not apply to: (a) damage caused by use with non-MonDevices products; (b) damage caused by accident, abuse, misuse, spillage of food or liquid

or other external causes; (c) damage caused by operating the Product outside the permitted or intended uses described by MonDevices or with improper voltage or power supply; (d) use for medical, healthcare or treatment purposes, or (e) damage caused by service performed by anyone who is not a representative of MonDevices. Recovery and reinstallation of software programs and user data are not covered under this warranty. This warranty is void if a Product is returned with removed, damaged or tampered labels or any alterations (including the unauthorized removal of any component or external cover).

No MonDevices reseller, agent, or employee is authorized to make any modification, extension, or addition to this Limited Warranty. If any term is held to be illegal or unenforceable, the legality or

enforceability of the remaining terms shall not be affected or impaired.

#### 4. Implied Warranties and Conditions:

EXCEPT TO THE EXTENT PROHIBITED BY APPLICABLE LAW, ALL IMPLIED WARRANTIES AND CONDITIONS (INCLUDING WARRANTIES AND CONDITIONS OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE) SHALL BE LIMITED IN DURATION TO THE DURATION OF THIS LIMITED WARRANTY. Some jurisdictions do not allow limitations on the duration of an implied warranty or condition, so the above limitation may not apply to you. This warranty gives you specific legal rights, and you may also have other rights that vary from state to state.

#### 5. Limitation of Damages:

EXCEPT TO THE EXTENT PROHIBITED BY APPLICABLE LAW, MONDEVICES SHALL NOT BE LIABLE FOR ANY INCIDENTAL, INDIRECT, SPECIAL, OR CONSEQUENTIAL DAMAGES, OR ANY LOSS OF PROFITS, REVENUE OR DATA, RESULTING FROM ANY BREACH OF EXPRESS OR IMPLIED WARRANTY OR CONDITION, OR UNDER ANY OTHER LEGAL THEORY, EVEN IF MONDEVICES HAS BEEN ADVISED OF THE POSSIBILITY OF SUCH DAMAGES AND REGARDLESS OF THE FORM

OF ACTION WHETHER IN CONTRACT, TORT (INCLUDING NEGLIGENCE) OR ANY OTHER LEGAL OR EQUITABLE THEORY.

Some jurisdictions do not allow the exclusion or limitation of special, indirect, incidental or consequential damages, so the above limitation or exclusion may not apply to you.

#### 6. Governing Law:

This Limited Warranty shall be governed by the laws of the jurisdiction where it was sold to you, without giving effect to any conflict of laws principles that may provide the application of the law of another jurisdiction.

## 7. National Statutory Rights:

Consumers in some jurisdictions may have legal rights under applicable national legislation governing the sale of consumer goods, including, without limitation, national laws implementing EC Directive 99/44. These rights are not affected by the warranties in this Limited Warranty.

If you have any questions, please contact MonDevices at [support@mondevices.com](mailto:support@mondevices.com).

## LIMITED WARRANTY (CANADA)

MonDevices, Inc (“MonDevices”) warrants to the original purchaser only that the MonBaby device you purchased (the “Product”) shall be free from defects in materials and workmanship under normal use for a period of one (1) year from the date of retail purchase by the original purchaser (“Warranty Period”).

### 1. Remedies:

If a hardware defect arises and a valid claim is received by MonDevices within the Warranty Period, MonDevices will, at its option and to the extent permitted by law, either (1) repair the Product at no charge, using new or refurbished replacement parts, or (2) exchange the Product with a new or refurbished Product. If MonDevices, in its sole discretion, determines it is not reasonable to replace the defective Product, MonDevices may refund to you the purchase price paid for the Product. If the original Product has been discontinued, MonDevices may, in its sole discretion, replace the Product with a newer version provided that, you shall be responsible to pay for any difference in retail value between the original Product and its replacement. Shipping and handling charges may apply except where prohibited by applicable law. Any replacement Product will be under warranty for the remainder of the Warranty Period or thirty (30) days, whichever is longer, or for any additional period of time that may be applicable in your jurisdiction.



## 2. How to Obtain Warranty Service:

To obtain warranty service, you must obtain a Return Merchandise Authorization (RMA) from our customer service team and deliver the Product, in either its original packaging or packaging providing an equal degree of protection, together with proof of purchase, to the address specified by MonDevices. You agree to pay the cost of shipping the Product to MonDevices. By sending the Product, you agree to transfer ownership to MonDevices upon receipt of the Product by MonDevices. You shall assume all risk of loss or damage to the Product while in transit to MonDevices. If the claim is justified based on this Limited Warranty, MonDevices shall bear the cost of shipping the repaired or replacement Product to you. Any Product returned to MonDevices without a valid warranty claim or without a RMA may be rejected, returned at your cost (subject to prepayment)

or kept for no more than thirty (30) days for your pick-up and then disposed of in MonDevices' sole discretion. For specific instructions on how to obtain warranty service on your Product, visit the MonBaby website <https://monbaby.com/support>. It

is your responsibility to backup any data, software, or other materials you may have stored or preserved on the Product. It is likely that such data, software, or other materials will be lost or reformatted during service, and MonDevices will not be responsible for any such damage or loss.

## 3. Exclusions and Limitations:

This Limited Warranty applies only to the Product manufactured by or for MonDevices that can be identified by the "MonBaby" trademark, trade name, or logo affixed to it. This Limited Warranty does not apply to any (a) MonDevices products

and services other than the Product, (b) consumables (such as batteries), (c) software, even if packaged or sold with the Product or embedded in the Product, (d) damage caused by accident, abuse, misuse, spillage of food or liquid or any acts of God, (e) damage caused by operating the Product outside the permitted or intended uses described by MonDevices or

with a non-compatible mobile device or mobile application, (f) use for medical, healthcare or treatment purposes, (g) damage, alterations or modifications caused by anyone who is not a representative of MonDevices or (h) use of replacement parts or software not supplied by MonDevices. Please refer to the Terms of Service that accompany the MonDevices mobile application for details of your rights with respect to its use.

THE PRODUCT IS NOT A MEDICAL DEVICE; AND IS NOT REGULATED OR APPROVED BY HEALTH CANADA; AND IS NOT DESIGNED TO DETECT OR PREVENT CAUSES OF SUDDEN INFANT DEATH SYNDROME OR ANY OTHER MEDICAL CONDITION. THE PRODUCT IS INTENDED TO BE USED WITH MONDEVICES' MOBILE APPLICATION TO HELP YOU MONITOR YOUR BODY DATA AND IS NOT A SUBSTITUTE FOR MEDICAL CARE OR ADULT SUPERVISION.

MonDevices does not warrant that the operation of the Product and the mobile application will be without interruption or otherwise be free of anomalies, errors or viruses, and MonDevices will not be responsible for damages or loss arising from any such interruption, anomalies, errors or viruses.

This Limited Warranty is void if a Product is returned with removed, damaged or tampered labels or any alterations (including the unauthorized removal of any component or external cover). This Limited Warranty does not cover any alteration or damage to any other software or hardware as a result of installing and using the mobile application provided by MonDevices. Recovery and reinstallation of software programs and user data are not covered under this Limited Warranty.

REPAIR OR REPLACEMENT AS PROVIDED IN THIS LIMITED WARRANTY IS THE EXCLUSIVE REMEDY OF THE PURCHASER. THIS WARRANTY IS IN LIEU OF OTHER WARRANTIES, EXPRESSED OR IMPLIED, INCLUDING, BUT NOT LIMITED

TO, ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR USE OR PURPOSE, AND MONDEVICES NEITHER ASSUMES NOR AUTHORIZES ANY PERSON TO ASSUME FOR IT ANY OTHER OBLIGATION OR

LIABILITY IN CONNECTION WITH THE SALE OR SERVICE OF THE PRODUCT. MONDEVICES DOES NOT WARRANT THAT THE PRODUCT OR ITS MOBILE APPLICATION WILL OPERATE WITH ANY OTHER PRODUCT, SOFTWARE OR HARDWARE EXCEPT THOSE INDICATED AS COMPATIBLE BY MONDEVICES. MONDEVICES IS NOT RESPONSIBLE FOR CHARACTERISTICS OF THIRD PARTY HARDWARE OR SOFTWARE WHICH MAY AFFECT THE OPERATION OF THE PRODUCT OR THE MOBILE APPLICATION PROVIDED BY MONDEVICES.

WITHOUT LIMITING THE FOREGOING, MONDEVICES DOES NOT MAKE ANY CLAIMS OR WARRANTIES OF ANY KIND WHATSOEVER REGARDING THE PRODUCT'S POTENTIAL, ABILITY OR EFFECTIVENESS TO PREVENT, MINIMIZE, OR IN ANY WAY AFFECT PERSONAL INJURY.

MonDevices reserves the right to make changes to this policy

at its sole discretion without advance notice. The most current version of this policy shall be available on [MonBaby.com](http://MonBaby.com).

MonDevices reserves the right to make changes to its Products without incurring any obligation to modify any Product that has already been manufactured.

#### 4. Implied Warranties, Conditions and Limitation of Damages:

EXCEPT TO THE EXTENT PROHIBITED BY APPLICABLE LAW, ALL IMPLIED WARRANTIES AND CONDITIONS (INCLUDING WARRANTIES AND CONDITIONS OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE) SHALL BE LIMITED IN DURATION TO THE WARRANTY PERIOD.

EXCEPT TO THE EXTENT PROHIBITED BY APPLICABLE LAW, MONDEVICES SHALL NOT BE LIABLE FOR ANY INCIDENTAL, INDIRECT, SPECIAL, OR CONSEQUENTIAL DAMAGES, OR ANY LOSS OF PROFITS, REVENUE OR DATA, OR INJURIES TO ANY PERSON, ARISING FROM THE USE OF THE PRODUCT OR ARISING FROM THE MALFUNCTIONING OR NON-FUNCTIONING OF THE PRODUCT, OR RESULTING FROM ANY BREACH OF EXPRESS OR IMPLIED WARRANTY OR

CONDITION, OR UNDER ANY OTHER LEGAL THEORY, EVEN IF MONDEVICES HAS BEEN ADVISED OF THE POSSIBILITY OF SUCH DAMAGES AND REGARDLESS OF THE FORM

OF ACTION WHETHER IN CONTRACT, TORT (INCLUDING NEGLIGENCE) OR ANY OTHER LEGAL OR EQUITABLE THEORY. ANY AND ALL CLAIMS OR STATEMENTS, WHETHER WRITTEN OR VERBAL, BY SALESPeOPLE, RETAILERS, DEALERS OR DISTRIBUTORS TO THE CONTRARY ARE NOT AUTHORIZED BY MONDEVICES, AND DO NOT AFFECT THIS PROVISION OF THIS LIMITED WARRANTY.

## 5. Governing Law:

This Limited Warranty shall be governed by and construed in accordance with the laws of Canada. Without limit to the foregoing, certain provinces do not allow the exclusion or limitation of implied warranties or the limitation of incidental or consequential damages for certain products supplied to consumers or the limitation of liability for personal injury, so the above limitations, exclusions and disclaimers may be limited in their application to you. If implied warranties are not allowed to be excluded in their entirety for any reason, they will be limited to the duration of the Warranty Period. This Limited Warranty gives you specific legal rights which may vary depending on laws of the province in which the Product was purchased. By keeping this Product you agree to the terms of this Limited Warranty without objection or addition, and agree that your sole remedy for any disputes over any elements of this Limited Warranty shall be limited to MondeVICES' performance of its obligations under this Limited Warranty. You may have other rights under provincial or federal laws and where the whole or part of any item of this Limited Warranty is prohibited by such laws, it shall be deemed null and void. The remainder of the warranty shall remain in effect.

## 6. National Statutory Rights:

Consumers in some jurisdictions may have legal rights under applicable national or federal legislation governing the sale of consumer goods, including, without limitation, the Consumer

Protection Act and the Canada Consumer Product Safety Act. These rights are not affected by the warranties in this Limited Warranty.

If you have any questions, please contact MonDevices at [support@mondevices.com](mailto:support@mondevices.com).

## RETURN POLICY (US PURCHASES MADE THROUGH MONBABY.COM)

### MONBABY.COM 14 DAY SATISFACTION GUARANTEED RETURN POLICY:

If you are unsatisfied with your purchase from MonBaby.com for any reason, you have 14 days from the date of purchase to request a full refund of any product purchased on Monbaby.com

#### **1. To qualify for a refund, all the following conditions must be met:**

- A Return Merchandise Authorization (RMA) must be requested from our customer service team within 14 days of the date on which you purchased the Product.
- To request an RMA, you must contact our customer service team via our website at <https://monbaby.com/contact-us-2/> or via email addressed to [support@mondevices.com](mailto:support@mondevices.com).
- Any accessories originally included with the Product you purchased must be included with your return.
- MonDevices may require that you furnish proof of purchase details to obtain an RMA.  
Returned Product(s) must be in good physical condition (not physically broken or damaged), in either its original packaging or packaging providing an equal degree of protection. The RMA number must be included along with your returned Product.

#### **2. Additional terms and conditions:**

- Shipping and handling charges, gift wrap fees, and taxes paid (e.g., state, customs, VAT) are not refundable.

- You are responsible for and must prepay all shipping charges, and you shall assume all risk of loss or damage to the Product while in transit to MonDevices. If you return a Product to MonDevices (a) without an RMA from MonDevices
- or (b) without all parts included in the original package, MonDevices retains the right to either (1) refuse delivery of such return or (2) charge you a restocking fee equal to the higher of fifteen (15) per cent of the original price of the Product or the retail value of the missing parts. Refunds will be processed and paid within 2 weeks of MonDevices' receipt of the Product.

## DISCLAIMERS AND LEGAL

### FCC (North America)

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

**No Unauthorized Modifications:** Do not make any changes or modifications to this product without the prior express written approval of MonDevices, Inc. Any changes or modifications made without express written approval could void the user's authority to operate this product.

**Note:** This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

## IC (Canada)

This Class B digital apparatus complies with Canadian ICES-0031/ Cet appareil numérique de la classe B est conforme à la norme NMB-003 du Canada This device complies with Industry Canada licence-exempt RSS standard(s). Operation is subject to the following two conditions: (1) this device may not cause interference and (2) this device must accept any interference, including interference that may cause undesired operation of the device/ Le présent appareil est conforme aux CNR d'Industrie Canada applicables aux appareils radio exempts de licence. L'exploitation est autorisée aux deux conditions suivantes: (1) l'appareil ne doit pas produire de brouillage, et (2) l'utilisateur de l'appareil doit accepter tout brouillage radioélectrique subi, même si le brouillage est susceptible d'en compromettre le fonctionnement

### INDUSTRY CANADA RADIATION EXPOSURE STATEMENT

**IMPORTANT NOTE:** This equipment complies with IC radiation exposure limits/ Cet équipement est conforme aux limites d'exposition aux rayonnements IC établies

## Environmental Protection



This electronic equipment contains reusable material and must not be disposed with domestic waste. Please take it to a collecting point for electronic equipment (ask your local authorities for details).

A woman with long dark hair, wearing a light-colored lace top, is leaning over a white wooden crib. She is smiling and looking down at a baby lying on its back in the crib. The baby is wearing a light-colored long-sleeved shirt and pants. The crib has a white wooden frame with vertical slats. The background is a dark blue wall with white abstract shapes.

**safesleep**

[safesleep.monbaby.com](https://safesleep.monbaby.com)