

JPD-FR409

CE 0598

JUMPER

Infrared Thermometer
Instruction Manual

Shenzhen Jumper Medical Equipment Co., Ltd

Manual Version: 1.0

Date of Issue: 2025/01

Product Information

Product Name: Infrared Thermometer

Model: JPD-FR409

Manufacturer: Shenzhen Jumper Medical Equipment Co., Ltd

Address: D Building, No. 71, Xintian Road, Fuyong Street, Baoan, Shenzhen, Guangdong, China.

Copyright

Copyright © Jumper Medical. All rights reserved.

Statement

Shenzhen Jumper Medical Equipment Co., Ltd. owns and reserves all of the rights comprised in the copyright of this document. No part of this document may be changed, excerpted, copied, reproduced, or imitated in any form or by any means without the prior consent of Shenzhen Jumper Medical Equipment Co., Ltd.

All statements, information, and recommendations in this document are provided "AS IS" without warranties, guarantees or representations of any kind, either express or implied. The information in this document is subject to change without notice. Shenzhen Jumper Medical Equipment Co., Ltd. reserves the right of final interpretation of this document.

Introduction

Thank you for purchasing this Infrared Thermometer. Please read the User Manual carefully to make sure safe and proper use of this thermometer.

Please read and fully understand the Safety Precautions before use.

Keep the Instruction Manual with this thermometer for future reference.

Contents

Unpacking Check	1
Package Contents	1
Safety Precautions	1
Warning	3
Symbols	4
Body Temperature Basics	5
Product Description	7
Features	8
Product Structure	9
Display Description	10
Sounds and Indicator Color Instructions	10
Display and Operating Instructions	11
Measuring Ear Temperature	19
Measuring Forehead Temperature	21
After measurement	22
Replacing Batteries	23
Cleaning and Disinfection	24
Maintenance	26
Troubleshooting	26
Specifications	28
Security Class	30
Storage and Transportation	30
EMC Information-Guidance and Manufacture's Declaration	31
FCC Compliance Statements	34
US Radiation Exposure Statement	35
Warranty and After-Sale Service	35

Unpacking Check


Please open the package carefully before use, check whether all accessories are available or not and whether any component is damaged during transportation, and perform installation and operation following this user manual. In case of any damage or operation problem, please contact the dealer or contact Jumper directly. You will need the following information when making your claim: device model, serial number, purchase date, and your contact information and address.

Package Contents

No.	Name	Quantity
1	Infrared Thermometer	1
2	Pouch	1
3	Battery (AAA)	2
4	Instruction Manual	1
5	Probe covers	20

Safety Precautions


Read the following precautions carefully before using the thermometer.

 Attention	
◆	Take care of the temperature probe lens, which is fragile.
◆	No service /maintenance while the equipment is in use.
◆	Dispose used batteries with care. To protect the environment,

	you are recommended to send the used batteries to a designated collection point.
◆	The thermometer doesn't need recalibration.
◆	<ul style="list-style-type: none"> - Make sure that the thermometer is not exposed to lint, dust, light (including sunlight), etc. - Please note the effects of degraded sensors that can degrade performance or cause other problems. - Make sure that the thermometer is not exposed to pets, pests.
◆	If the thermometer is soiled or its infrared optical components is damaged, please stop using it.
◆	The lay operator or lay responsible organization should contact the manufacturer or manufacturer's representative on the following issues: -assistance in setting up, using, or maintaining the equipment or system when needed, or to report unexpected operation or events.
◆	The IR thermometer is identified as intended for consumer use. The patient is an intended operator.
◆	The effect that the following actions could be unsafe as applicable: use of accessories, detachable parts, and materials not described in the instructions for use or modification of the equipment.
◆	The hazards that can result from unauthorized modification of the me equipment.
◆	The manufacture can provide the circuit diagram, component part list, description and calibration instructions to assist service personal for parts repair.
◆	Do not subject the thermometer to vibration or impact.
◆	Do not take body temperature readings within 20 minutes after you do physical exercises or get excited.
◆	Do not use the thermometer for continuous temperature monitoring purposes.
◆	Do not use the thermometer for purposes that are not specified in this User's Manual. Follow the instructions in the "Measurement Process" chapter and carefully operate the







	thermometer when measuring children's temperature.
◆	Do not immerse the thermometer in water or other liquid. Clean and disinfect the thermometer as described in the "Cleaning and Disinfection" chapter.
◆	Do not touch the tip of the temperature probe, on which a precise temperature sensor resides.
◆	Keep the temperature probe clean to make sure accurate readings.
◆	Before measuring the temperature from the ear canal, clean the earwax, if any.
◆	The ambient temperature must not be extremely high or low. To make sure accurate readings, keep the thermometer under room temperature for more than 30 minutes before use.
◆	Do not use the thermometer under an ambient temperature higher than 40°C (104°F) or lower than 10°C (50°F), which is beyond the operating temperature range of the thermometer.
◆	Risk of pollution! The user is recommended to send the overdue thermometer to local garbage disposal site or send it back to us.
◆	2 AAA batteries of 1.5V are the only replaceable accessories of the thermometer. Please do not use the batteries of other voltages or specifications.







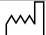


Warning

 Warning	
●	Do not force the temperature probe of the thermometer into an ear canal. Otherwise, the ear canal may get injured.
●	Do not use the thermometer if the ear is infected with otitis or suppuration.
●	Keep the thermometer out of the reach of children.
●	The result may be inaccurate if you use the overdue thermometer.
●	The thermometer is not intended to diagnose or treat any health

	problem or disease. The measurement results are for reference only.
●	It is dangerous to make a self-diagnosis or self-treatment based on the obtained measurement results. For such purposes, please consult a doctor.
⊘	Do not charge an alkaline dry-cell battery or throw it in fire. Otherwise, the battery may explode.
⊘	Do not disassemble the thermometer or attempt to repair it. Otherwise, the thermometer may be damaged permanently.
⊘	Do not take temperature measurements on body parts other than forehead and ears. Otherwise, the temperature readings may be inaccurate.
⊘	During measurement, do not use a mobile phone or any other device that may cause electromagnetic interference.
⊘	Do not use the thermometer in an environment where flammable anesthetic mixture with air or with oxygen or nitrous oxide is available.
⊘	Do not open the battery cover when using the thermometer.

Symbols

Symbol	Description
	Type BF applied part.
	Attention must be paid.
	The action is prohibited.
	Information about the manufacturer.
	Date of manufacture.
	Consult the instructions for use.

Symbol	Description
	Medical Device
	Authorized European Representative
	This product complies with the Regulation (EU) 2017/745 requirements.
	Waste electrical materials should be sent to a dedicated collection point for recycling.
IP22	Degree of protection against the Ingress of water.
	Humidity
	Temperature limitation
	Atmospheric Pressure
 Warning	A personal injury or damage to the thermometer may occur if the thermometer is not used correctly.
 Attention	Inaccurate reading or damage to the thermometer may occur if the thermometer is not used correctly.

Body Temperature Basics


- The normal body temperature is a range.
- The normal range varies from person to person and can fluctuate throughout the day.
- The normal range also varies by body site. Therefore, measurements from different sites should not be compared directly.

To determine if an individual is experiencing an elevated body temperature and/or having a fever, it is critical to know the individual's normal body temperature when he/she is well. Take multiple readings to obtain the normal body temperature range and note the specific body site measured, for example: forehead or eardrum temperature.

Body Site	Normal Temperature Range
Forehead	34.7°C-37.3°C (94.5°F- 99.1°F)
Eardrum	35.8°C-38.0°C (96.4°F-100.4°F)
Mouth	35.5°C-37.5°C (95.9°F- 99.5°F)
Armpit	34.7°C-37.3°C (94.5°F- 99.1°F)
Rectal	36.6°C-38.0°C (97.9°F-100.4°F)

The normal body temperature range varies slightly with age and gender. Generally, newborns or children have higher body temperatures than adults, and adults have higher body temperatures than the elderly. Women's body temperatures are approximately 0.3°C (0.5°F) higher than men's.

Variation in body temperature

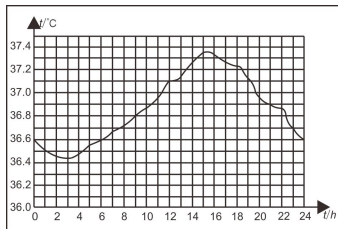

Normal body temperature fluctuates throughout the day and is also affected by external factors. The body temperature of an individual is the lowest between 2:00 a.m. and 4:00 a.m. and the highest between 2:00 p.m. and 8:00 p.m. An individual's body temperature typically changes by less than 1°C (1.8°F) each day.

Product Description

1) Overview


Infrared Thermometer JPD-FR409 measures the body temperature based on the infrared energy emitted from the eardrum or the forehead. Users can quickly get measurement results after positioning properly the temperature probe in the ear canal or forehead.

2) Structure

The thermometer consists of a shell, an LCD, a measure button, a beeper, an infrared temperature sensor, and a Microprocessor.

3) Operating principle

The infrared temperature sensor collects infrared energy emitted by the eardrum or the skin surface. After being focused by a lens, the energy is converted into a temperature reading by the thermopiles and measurement circuits.

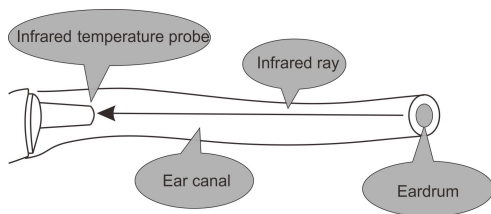

4) Intended use

The Infrared Thermometers take human body temperature via the eardrum or forehead for both professional and home use.


5) Contraindications

The device has no side-effects if administered correctly and residual risk is acceptable.

Features

- 01 Good safety
 - Passive infrared receiving technology.
- 02 Easy operation
 - Ergonomic design
 - One button measurement
 - Automatic switching mode
- 03 Quick measurement
 - 1-second measurement
- 04 High accuracy
 - Advanced infrared temperature sensor, with high sensitivity
 - High accuracy with automatic temperature calibration
- 05 Diverse functions
 - 20 reading recalled
 - Fever alert
 - Switching between °C and °F
 - Automatic power-off, power saving
- 07 Pairing the 4G network
 - Automatically assess the network
 - Automatically store and upload data
- 06 Intended patient population
 - Forehead mode applies to all age groups
 - Ear mode applies to people above 3 months


Product Structure

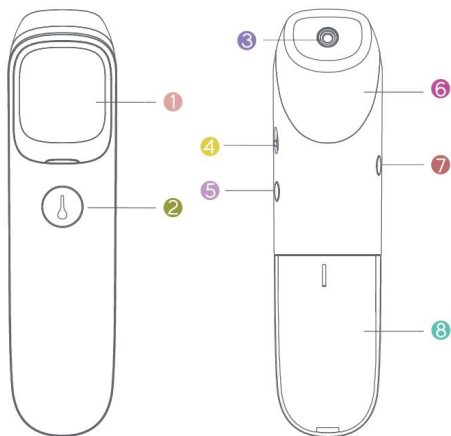

- (1) LCD display screen
- (2) Power button/Measure button
- (3) Probe
- (4) Unit switch (°C/°F)
- (5) Mute button
- (6) Probe cap
(put the cap on automatic switching to the Forehead mode)
(take the cap off automatic switching to the Ear mode)
- (7) Memory button
- (8) Battery cover

Display Description

- 1.Forehead temperature mode
- 2.Probe cover attached
- 3.Ear temperature mode
- 4.Mute / un-mute
- 5.Fahrenheit / Celsius degrees
- 6.Memory recall
- 7.Low battery
- 8.Transmit successfully Icon
- 9.Data transmit Icon
- 10.Signal indication Icon
- 11.Temperature value

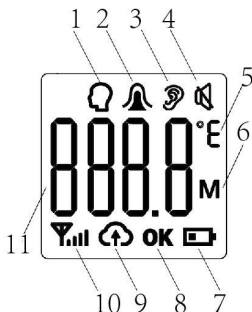


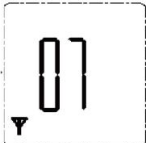


Sounds and Indicator Color Instructions


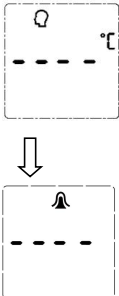



Range	Sounds	Indicator Color
Forehead temperature		
22.0°C-37.5°C/71.6°F-99.5°F	A long beep	Green
37.6°C-43°C/99.6°F-109.4°F	3 short double beeps	Red
Ear temperature		
34.0°C-37.5°C/93.2°F-99.5°F	A long beep	Green
37.6°C-43.0°C/99.6°F-109.4°F	3 short double beeps	Red





Note: When the forehead temperature is between 22.0°C/71.6°F and 37.5°C/99.5°F, the ear temperature is between 34.0°C/93.2°F and 37.5°C/99.5°F, there will be a long beep and a green light. This indicates that your body temperature is normal.


When the forehead and ear temperature is between 37.6°C/99.6°F and 43.0°C/109.4°F, there will be 3 short double beeps and a red light. This indicates that your body temperature is a little high. You may have a fever. Please consult your doctor if you are not sure.



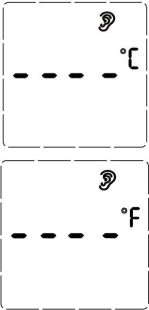
Display and Operating Instructions


Screen Display	Operating Instructions Displayed State	Sound and Indicator Color
Pairing the mobile network		
   	<p>When the device is turned on for the first time after installation or battery replacement, the device will automatically pairing the mobile network.</p> <p>The icon “OK” appears on the screen is confirms pairing the mobile network successful.</p> <p>Note:</p> <p>(1)After pairing successful , the device will automatically shut down if there is no operation for about 10 seconds.</p> <p>(2)Network pairing varies based on signal strength. With a strong signal, pairing typically takes about 15 seconds. In case of abnormal pairing or poor signal, the pairing process will be repeated and the time extended</p>	Silent


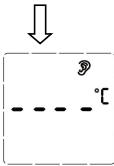
Screen Display	Operating Instructions Displayed State	Sound and Indicator Color
Pairing network fails		
	<p>If pairing network fails, the Error symbol “E3” appears on the screen, press the Measure button again to start measurement.</p> <p>Note:</p> <p>(1)The device still can measure the temperature in Offline state.</p> <p>(2)all the 20 measurement data will automatically store in devise.</p> <p>(3)When pairing network successful, the devise will automatically upload all the data.</p>	Silent
	<p>(1)Put on the probe cap, the device will automatically switching to Forehead mode and the Head icon “” appeal.</p> <p>(2)Take off the probe cap, the probe cover attached icon “”blink on screen.</p> <p>(3) After the probe cover is placed successfully, the “” will disappears.</p>	Silent


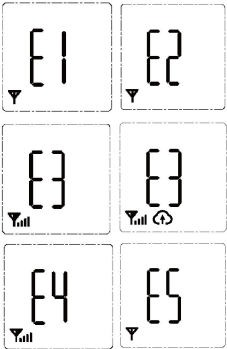
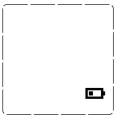
	<p>(4) Start measurement when the ear icon “” appears on the screen.</p>	
<p>Measuring Ear temperature</p>		
	<p>Take off the probe cap and placed the probe cover successfully, the ear icon “” appears on the screen.</p> <p>Press the Measure button to start a measurement.</p> <p>After the measurement, the data transmission automatically starts.</p> <p>Note::</p> <p>(1)The Ear mode Measuring only work on the probe and probe cover fit tightly, the probe cover icon disappear.</p>	<p>See the table in the “Sounds and indicator color instructions” section</p>

Screen Display	Operating Instructions Displayed State	Sound and Indicator Color
Measuring Forehead temperature		
	<p>Put probe cap on and the Head icon “Q” is appears on the screen.</p> <p>Press the Measure button to start a measurement.</p> <p>After the measurement, the data transmission automatically starts.</p>	<p>See the table in the “Sounds and indicator color instructions” section</p>

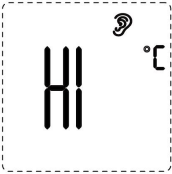Screen Display	Operating Instructions Displayed State	Sound and Indicator Color
Out of the measuring range display		
	<p>In Ear mode, a temperature reading of more than 43.0°C(109.4°F)</p> <p>In Forehead mode, a temperature reading of more than 43.0°C(109.4°F)</p>	<p>A long beeps, the indicator light is red.</p>
	<p>In Ear mode, a temperature reading of less than 34.0°C (93.2.0°F)</p> <p>In Forehead mode, a temperature reading of less than 22.0°C (71.6°F)</p>	<p>A long beeps, the indicator light is red.</p>
Switching between °F/°C		
	<p>Switching Unit by the side °C/°F switcher</p>	<p>Silent</p>

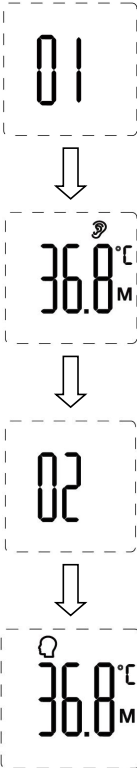Screen Display	Operating Instructions Displayed State	Sound and Indicator Color
Recall 20 memories		
	<p>In a power-on state, press the Memory button to enter the memory mode.</p> <p>When the Memory button is released, 01 will be shown, followed by the recorded reading.</p> <p>Press the Memory button again for the next recorded data. 02 will be shown, followed by the recorded reading.</p> <p>A maximum of 20 temperature readings can be recalled.</p> <p>When the maximum number of records is exceeded, the earliest memory data will be overwritten.</p> <p>Note:</p> <p>1 represents the newest data.</p>	<p>Silent, the indicator light is green.</p>


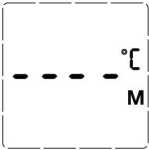
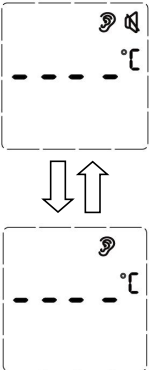
Screen Display	Operating Instructions Displayed State	Sound and Indicator Color
No memory data / Clear memory data		
	<p>The display is as shown, when there is no more data checked while recalling memories.</p> <p>Remove 2 dry batteries and re-install the power to clear all memory data.</p>	<p>When the power is turned back on, a long beep, the indicator light is green, then turn red.</p>
Switching between mute and un-mute		
	<p>Press Mute button to turn the sound on or off.</p> <p>When the sound is turned off, the "🔇" appears.</p> <p>When the sound is turned on, it will beep once and the "🔇" is disappears.</p>	<p>When the sound is turned on, it sounds a long beep and the indicator light is green.</p>

Screen Display	Operating Instructions Displayed State	Sound and Indicator Color
Error information & low battery		
	<ul style="list-style-type: none"> ●The ambient temperature is higher than 40.0°C (104.0°F) or lower than 10.0°C (50.0°F). ●An error occurs when data is being read from or written to the memory, or the temperature correction is not complete. 	A long beeps, the indicator light is red.
	<p>Error message in 4G status display.</p> <p>Follow the instructions in Troubleshooting.</p>	Silent
	<p>When the battery voltage is lower than $2.5V \pm 0.1V$, the low battery symbol will appear. Please replace the batteries.</p>	Silent

Measuring Ear Temperature

- How to attach the probe cover to thermometer probe.

- (1) Remove thermometer's probe cap
- (2) Torn along the tangent line;
- (3) Align the centers of probe and probe cover;
- (4) Insert firmly;
- (5) Lift probe upwards;

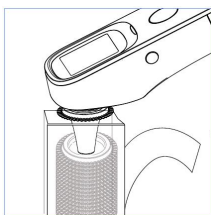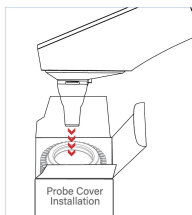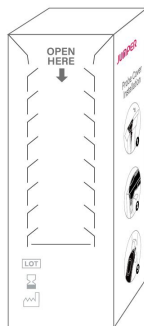

Note:

1. Probe cover is intended for single user only, remove and dispose the used cover after measurement, each time please use new one, otherwise poses a risk.
2. Replace the new probe cover when:
 - (a) cover film is damaged or dirty;
 - (b) cover is used;
 - (c) probe covers are necessary to be replaced for other circumstances.
3. The operator is responsible for checking the compatibility of thermometers with probe cover before use.


4. This probe cover (FT-001) can only be used with model JPD-FR409 thermometer manufactured by Shenzhen Jumper Medical Equipment Co.,Ltd.


● Measuring the temperature



(1) Turn on the thermometer and take off probe cap, the device will automatically switch to Ear mode(with probe cover).


Note: 1. If pairing network fails, the Error symbol “E3” appear on the screen, the device still can use in Offline state.

2. When pairing network successful, the device will automatically upload all the data.

(2) Attach the probe cover to the device when icon “” flashes on screen and it will disappear after the probe cover is placed successfully.

(3) Start measurement when the icon “” appears on the screen.

Note: Before measuring, ensure the probe cover is properly installed. If installed correctly, the probe cover icon “” on the screen will disappear, and the ear icon “” will appear.

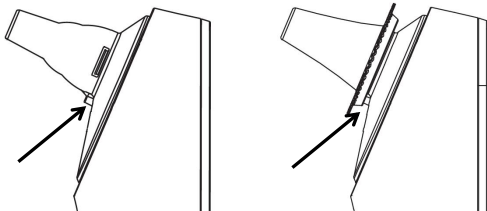


(4) Insert the temperature probe into the ear canal.

(5) Press and release the Measure button. The ear temperature reading will be appear on the screen instantly.

Note: Children under 1 year: Pull the ear straight back.

Children aged 1 year to adult: Pull the ear up and back.

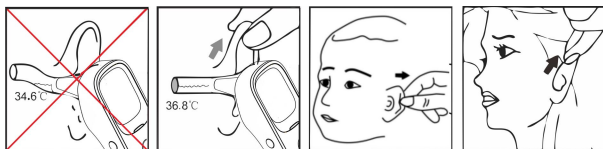


⚠ Do not force the thermometer into the ear canal. Otherwise, the ear canal may get injured.

⚠ When taking the temperature on an adult, gently pull the ear up and back to make sure the ear canal is straight, so that the temperature probe can receive an infrared ray from the eardrum.


⚠ Be careful when taking temperature on a child, whose ear canal is small.

Measuring Forehead Temperature

(1) Turn on the thermometer and put the cover on the probe, the device will automatically switch to Forehead mode.

(2) The Forehead icon “” is appear on the screen.

(3) Point the thermometer probe to the sides of the forehead, about 1-3cm away from the skin surface.

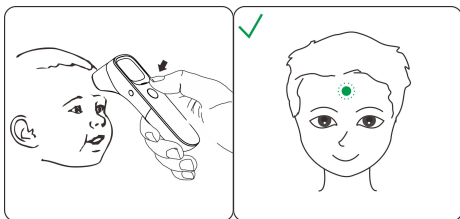

(4) Press and release the **Measure button**. The temperature reading will be displayed on the screen instantly.

After measurement

- (1) After each measurement the readings stored in the device will be automatically uploaded. Three sets of data can be uploaded at a time, until all data is uploaded
- (2) After successful data uploaded, press the button to continue with the next measurement. The device will automatically power off after 10 seconds if there is no other operation.
- (3) If the measurement data is not uploaded promptly or fails to upload, it will be saved on the device and will be sent when pairing the mobile network successful. The device can store up to 20 readings.
- (4) If the Infrared thermometer is performed without prior network configuration, the device will automatically configure the network after the measurement, synchronize data and upload.
- (5) After each measurement, you can enter the recall mode and query earlier temperature readings. For more details, see "Recall 20 memories" in the preceding table.
- (6) After each measurement, clean the temperature probe with a soft cloth,



and put the thermometer in a dry and well-ventilated place.

Notes:

1. The thermometer is suitable for an indoor environment without strong air convection (for example, winds from a fan, an air-conditioner, or a heater) between the thermometer and the person.
2. Make sure that the ear canal is clean and dry before starting a measurement. It is recommended to clean the ear canal with a cotton swab if any dirt exists. Otherwise, the temperature probe may be polluted and temperature readings may be inaccurate.
3. Do not hold the thermometer for a long time, because it is sensitive to the ambient temperature.
4. Make sure the sense head is free of foreign matters before use;
5. Make sure the forehead has no sweat and no hairs covered before measure the forehead temperature; otherwise, the result could be incorrect;
6. No intense emotion or strenuous exercises before measuring;
7. After measuring the data once, you should wait for the indicator to turn off and measure the next data.

Replacing Batteries

- (1) Slide the battery cover off along the marked direction and take it off.
- (2) Insert the two AAA batteries into the compartment according to the stated polarities.

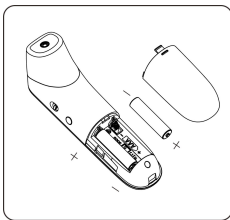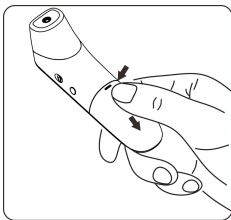

⚠ Make sure that the batteries are installed correctly. Otherwise, the thermometer may be damaged.

⚠ If the low-battery symbol is appear on the screen, replace the batteries.

⚠ Batteries of a same type should be used. Dispose the used batteries in accordance with the local environmental policies.

⚠ The thermometer is shipped with batteries. First open the battery cover, then remove the insulating piece.

Cleaning and Disinfection


Cleaning

Recommended detergents:

- * Medical detergents;
- * Home use mild detergents;

Cleaning steps:

- (1) Take the batteries out before cleaning.
- (2) Clean the temperature probe with a soft cloth. Clean the lens of the temperature probe with a cotton swab.
- (3) Wipe the thermometer body with a slightly damp soft cloth.

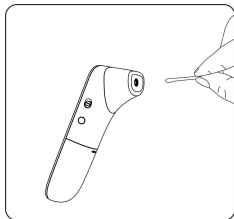

⚠ Keep water out off the lens during the cleaning process. Otherwise, the lens may be damaged.

⚠ The lens may be scratched if it is cleaned with a hard object, which might result in inaccurate readings.

⚠ Do not clean the thermometer with corrosive cleansers. During the cleaning process, do not immerse any part of the thermometer into liquid, or allow liquid to penetrate the thermometer.

Disinfection

Recommended disinfectants:

- * Isopropyl alcohol solution (concentration: 70%)
- * Medicinal alcohol (concentration: 75%)
- * Sodium hypochlorite solution (concentration: 3%)

Disinfecting steps:

- 1) Wet the clean soft cloth with a small quantity of disinfectant, wipe the thermometer and quickly dry it.
- 2) Disinfect the thermometer body and the area around the temperature probe with a cloth slightly moistened with 75% medical alcohol.

⚠ Do not use hot steam or ultraviolet radiation for disinfection.

Otherwise, the thermometer may be damaged or quickly aged.

⚠ Clean and disinfect the thermometer under the temperature of $+10^{\circ}\text{C}\sim+40^{\circ}\text{C}$ ($50^{\circ}\text{F}\sim 104^{\circ}\text{F}$), the relative humidity of 15%~85%RH (no condensation) and the barometric pressure of 86kPa~106kPa.

Maintenance

Preventive inspection & maintenance period

- 1) Ensure the safety of thermometer, and check whether it has potential safety hazards in normal use each week, e.g. whether the lens is broken, the shell has cracks and the sensing head is polluted. Do not use the thermometer with potential safety hazard. Clean the thermometer if not used for a long time.
- 2) After each use, clean the temperature probe as described in the "Cleaning and Disinfection" chapter.
- 3) Store the thermometer in a dry, dust-free, and well-ventilated place. Make sure that the thermometer is not exposed to sunlight. Make sure that the storage and transportation environments meet the requirements.
- 4) Check regularly whether safety risks exist.
- 5) Remove the batteries if the thermometer will not be used for more than two months.

Troubleshooting

Problem	Possible Cause	Solution
The thermometer fails to power on.	Low battery	Change the batteries.
	Polarities of the batteries are reversed.	Make sure that the batteries are installed correctly.
	The thermometer is damaged.	Contact the manufacturer.
"Er1" is	The ambient	Take a measurement

displayed.		temperature is lower than 10°C (50.0°F) or higher than 40°C (104°F).	under an ambient temperature between 10°C (50.0°F) and 40°C (104°F).
The temperature reading is lower than the typical body temperature range.		The lens of the temperature probe is dirty.	Clean the lens using a cotton swab.
		The thermometer probe is not aligned to the eardrum.	Reposition the thermometer probe so that it is aligned to the eardrum.
		The thermometer is used within 30 minutes after being taken from a cold environment.	Wait for more than 30 minutes after the thermometer is moved into the measurement environment.
The temperature reading is higher than the typical body temperature range.		The temperature probe is damaged.	Contact the manufacturer.
Error message in 4G status display	“E1” displayed	Communication failure	Please contact customer support for after-sale service.
	“E2” displayed	SIM card is not detected or SIM card is abnormal	Check and re-install the SIM card. If the issue persists, contact customer support.
	“E3” displayed	Data transmission error	●Re-measure at a location where you get strong cellular signal with your mobile phone. If the issue persists,contact customer

			support. ●Reinstall the battery.
	“E4” displa yed	Network registration error	Re-measure at a location where you get strong cellular signal with your mobile phone, or change a new SIM card.
	“E5” displa yed	No signal detecting	Re-measure at a location where you get strong cellular signal with your mobile phone. If the issue persists,contact customer support.

Specifications


Product Name	Infrared Thermometer
Product Model	JPD-FR409
Power Supply Mode	Internal power supply
Operating Voltage	DC 3V
Battery Model	AAA x 2
Service life	2 years
Battery Life	Alkaline dry battery for around 20,000 measurements
Operating Mode	Continuous operating
Display	Segment LCD
Measure time	About 1 second
Latency Time	About 3 second
Measuring Range	Forehead mode:22.0°C–43.0°C (71.6°F–109.4°F) Ear mode:34.0°C–43.0°C (93.2°F–109.4°F)

Accuracy (Laboratory)	Forehead mode: $\pm 0.2^{\circ}\text{C}$ (36.0°C – 39.0°C) ; $\pm 0.3^{\circ}\text{C}$ (22.0°C – 36.0°C / 39.0°C – 43.0°C); Ear mode: $\pm 0.2^{\circ}\text{C}$ (36.0°C – 39.0°C) ; $\pm 0.3^{\circ}\text{C}$ (34.0°C – 36.0°C / 39.0°C – 43.0°C);
Accuracy (Clinical)	$\pm 0.3^{\circ}\text{C}$ ($\pm 0.6^{\circ}\text{F}$)
Resolution	0.1°C (0.1°F)
Measuring site	Ear canal, Forehead (keep distance 1~3 cm from forehead)
Reference body site	Armpit
Mode of operation	Adjusted mode
Software version	4.0
Memory	20 temperature readings
Low-battery Alert	The low-battery symbol is displayed if the power voltage is lower than $2.5\text{ V} \pm 0.1\text{ V}$
Automatic Power-off	The thermometer automatically powers off if it is not used in 10 ± 1 seconds.
Outer dimensions (mm)	$155.2 \times 39.6 \times 49.1\text{ mm}$
Weight (g)	Thermometer (with batteries): 101 g
Manufacturing date	see the label
Operating Environment	Temperature: 10°C ~ 40°C (50°F – 104°F)
	Humidity: 15%–95% RH, non-condensing
	Atmospheric pressure: 86–106 kPa
Storage and Transportation	Temperature: -20°C to 55°C (-4°F – 133°F)
	Humidity: 15%- 95% RH, non-condensing
	Atmospheric pressure: 50–106 kPa

The infrared thermometer has been tested and conforms to the standard ASTM E1965-98. ASTM laboratory accuracy requirements in the display range of 96.8°F to 102.2°F (36°C-39°C) for ear canal IR thermometers is $\pm 0.4^{\circ}\text{F}$ ($\pm 0.2^{\circ}\text{C}$). Note that for mercury-in-glass and electronic thermometers, the requirement per ASTM Standards E667-86 and E1112-86 is $\pm 0.2^{\circ}\text{F}$ ($\pm 0.1^{\circ}\text{C}$).

Security Class

Type of protection against electric shock: internally powered equipment.

Degree of protection against electric shock:  Type BF applied part.

- Degree of protection against ingress of water: IP22
- Safety degree of using in flammable anesthetic gas blending with air, oxygen or nitrous oxide: Non-AP/APG
- No application parts of the thermometer prevents defibrillation charge effect.
- No application parts of the thermometer output signal.
- The thermometer is impermanent installed device.

Storage and Transportation

The thermometer can be transported using general transportation tools.

Severe vibration, shock, or rain must be avoided during transportation.

The thermometer must be packaged and then stored in a well-ventilated room without corrosive gas.

EMC Information-Guidance and Manufacture's

Declaration

1* WARNING: Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.”

2* WARNING: Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.”

3* WARNING: Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the ME equipment, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.”

Table 1

declaration - electromagnetic emission	
Emissions test	Compliance
RF emissions CISPR 11	Group 1
RF emissions CISPR 11	Class B
Harmonic emissions IEC 61000-3-2	Not applicable
Voltage fluctuations/ flicker emissions IEC 61000-3-3	Not applicable

Table 2

declaration - electromagnetic immunity		
Immunity test	IEC 60601 test level	Compliance level
Electrostatic discharge (ESD) IEC 61000-4-2	±8 kV contact ±2 kV, ±4 kV, ±8 kV, ±15 kV air	±8 kV contact ±2 kV, ±4 kV, ±8 kV, ±15 kV air
Electrical fast transient/burst IEC 61000-4-4	± 2 kV for power supply lines ± 1 kV for input/output lines	Not applicable
Surge IEC 61000-4-5	± 0.5kV, ± 1 kV line(s) to lines ± 0.5kV, ± 1 kV, ± 2 kV line(s) to earth	Not applicable
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	0 % UT; 0.5 cycle At 0°, 45°, 90°, 135°, 180°, 225°, 270°and 315° 0 % UT; 1 cycle and 70 % UT; 25/30 cycles Single phase: at 0° 0 % UT; 250/300 cycles	Not applicable
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	30 A/m	30 A/m
NOTE: UT is the a.c. mains voltage prior to application of the test level.		

Table 3

declaration - electromagnetic immunity		
Immunity test	IEC 60601 test level	Compliance level

Conducted RF IEC 61000-4-6	3 V 0.15 MHz to 80 MHz 6 V in ISM bands between 0.15 MHz and 80 MHz	Not applicable
Radiated RF IEC 61000-4-3	10V/m 80 MHz to 2.7 GHz	10V/m

Table 4

declaration - IMMUNITY to proximity fields from RF wireless communications equipment					
Immunity test	IEC60601 test level				Compliance level
	Test frequency	Modulation	Maximum power	Immunity level	
Radiated RF IEC 61000-4-3	385 MHz	**Pulse Modulation: 18Hz	1.8W	27 V/m	27 V/m
	450 MHz	*FM+ 5Hz deviation: 1kHz sine	2 W	28 V/m	28 V/m
	710 MHz 745 MHz 780 MHz	**Pulse Modulation: 217Hz	0.2 W	9 V/m	9 V/m
	810 MHz 870 MHz 930 MHz	**Pulse Modulation: 18Hz	2 W	28 V/m	28 V/m
	1720 MHz 1845 MHz 1970 MHz	**Pulse Modulation: 217Hz	2 W	28 V/m	28 V/m

	2450 MHz	**Pulse Modulation: 217Hz	2 W	28 V/m	28 V/m
	5240 MHz 5500 MHz 5785 MHz	**Pulse Modulation: 217Hz	0.2 W	9 V/m	9 V/m
<p>Note* - As an alternative to FM modulation, 50 % pulse modulation at 18 Hz may be used because while it does not represent actual modulation, it would be worst case.</p> <p>Note** - The carrier shall be modulated using a 50 % duty cycle square wave signal.</p>					

FCC Compliance Statements

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

Note: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation.

If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

Caution: Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

US Radiation Exposure Statement

This equipment complies with FCC radiation exposure limits set forth for an uncontrolled environment. End user must follow the specific operating instructions for satisfying RF exposure compliance. This transmitter must not be co-located or operating in conjunction with any other antenna or transmitter.

Warranty and After-Sale Service

The device is under warranty for one year from the date of purchase.

The batteries, the packaging, and any damage caused by improper use are not covered by the warranty.

Excluding the following user-caused failures:

- 1.Failure resulting from unauthorized disassembly and modification.
- 2.Failure resulting from an unexpected dropping during application or transportation.
- 3.Failure resulting from not following the instructions in the user's manual.

After-sale service unit: Shenzhen Jumper Medical Equipment Co., Ltd.
Address: D Building, No. 71, Xintian Road, Fuyong Street, Baoan,
Shenzhen,Guangdong,China
Tel: +86-755-26696279
E-mail: info@jumper-medical.com
Website: www.jumpermed.com
www.jumper-medical.com
Postal Code: 518103

Authorized European Representative:



MedPath GmbH

Mies-van-der-Rohe-Strasse 8, 80807 Munich, Germany

JUMPER



Shenzhen Jumper Medical Equipment Co., Ltd.
D Building, No. 71, Xintian Road, Fuyong Street,
Baoan, Shenzhen, Guangdong, China 518103.
E-mail: info@jumper-medical.com
Tel: +86-755-26696279
Website: www.jumpermmed.com
www.jumper-medical.com