

Intentions of use:

Baby Check is a wearable thermometer and motion dector sensor. It's intentions for use are to be worn on the upper part of the child's inner arm (near armpit) and monitor the general temperature and movement. This is not intended to replace your general temperature readings. Full functions and usability can be found in the features section of this manual.

*BABY CHECK IS NOT TO BE USED ORALLY OR RECTALLY IN ANY MANNER.

How to use:

- **1.** Download the Baby Check App from either iTunes (Apple / iOS) or Google Play store (Android).
- **2.** Remove Baby Check from the packaging.
- 3. Install battery:
- Remove rubber cap cover
- Remove the battery cover using a small flat blade screw driver.
- · Open casing and install battery
- Replace casing and rubber cap




- **4.** Give Baby Check a quick shake to wake it up a flashing light will indicate its active.
- **5. Open app** in phone and ensure that Bluetooth has been turned on. Baby Check app will walk you through setup.
- **6.** Place the rubber face and silver sensor against inside of child's arm. Ensure the sensor is in contact with the skin and use band to secure it in place.
- *Review placement diagram to ensure it has been properly secured.

7. With app open, check that device is connected.

Notes:

- * Baby Check will be monitoring until the device is disconnected from app.
- * Baby Check will take a few minutes to reach body temperature.
- * Multiple children can be added within app and used to monitor multiple devices.

Placement Diagram:

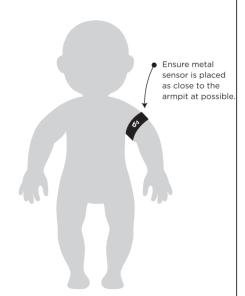

Warnings:

Seek Medical Advise if your child is not feeling well. This device is for monitoring only, do not use device if your child is sick and needs medical attention.

BY USING THIS DEVICE YOU ACCEPT ALL RESPONSIBILITY FOR THE HEALTH AND MEDICAL CONDITION OF YOUR CHILD.

Visit www.babycheckmonitor.com for complete manual, warnings, and precautions.

The device has been evaluated to meet general RF exposure requirement. The device can be used in portable exposure condition without restriction. **WARRANTY:** This product has a 1 year warranty from the manufacturer. Product must be registered through babycheckmonitor.com and follow warranty procedures to qualify for any return claims.

received, including interference that may cause undesired operation. Warning: Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

NOTE: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by

turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

(1) Reorient or relocate the receiving antenna. (2) Increase the separation between the equipment and receiver. (3)

Connect the equipment into an outlet on a circuit different from that to which the receiver is connected. (4) Consult the dealer or an experienced radio/TV technician for help.

RF warning statement:

The device has been evaluated to meet general RF exposure requirement. The device can be used in portable exposure condition without restriction.



Baby Check is a registered trademark of DK Technologies, LLC. Distributed & sold by DK Technologies, LLC. P.O. Box 170 Wellborn Texas 77881

6

7