NEUROON

Sleep Mask Smart System

Product Manual

Version 1.0

Table of Contents

Getting Started.....	3
What is in the box?.....	3
What's in this document?.....	3
Getting to know your Neuroon.....	4
Inside the Neuroon.....	4
Learn how to speak Neuroon.....	5
Wearing your Neuroon.....	5
Using the Neuroon.....	6
Tapping on the on/off button.....	6
Sleep Analytics.....	6
Restarting your Neuroon.....	6
Neuroon General info & Specifications.....	7
Battery life and charging.....	7
Care instructions.....	7
Memory.....	7
Environmental conditions.....	7
Help.....	7
Return policy and warranty.....	8
Regulatory & Safety Notices.....	9
USA.....	9
Canada.....	10
European Union (EU).....	10
Japan.....	11
Safety Statement.....	12

Getting Started

Welcome to the Neuroon: the intelligent sleep mask that uses cutting edge technology paired with a smart application to give you the productive rest you need.

What is in the box?

Your Neuroon box includes:

Neuroon Mask

Charging Cable

Quick Start Guide, Warranty Card & Safety Instructions

What is in this document?


This manual gets you started quickly setting up your Neuroon. Setup ensures that your tracker can synchronize its data with [Neuroon.com](#) or the Neuroon mobile app, where you can get detailed information on your stats, view historical trends, and much more. As soon as setup is complete you're ready to start moving.

The remainder of the manual walks you through every function on your Neuroon and also explains how to keep your Neuroon up to date with free software updates. Throughout the manual you'll notice that for tasks that can be done on the Neuroon mobile app—if you have the Neuroon app for iOS or Android, you can easily find the comparable task in the app without needing tap-by-tap instructions here. That said, everything you need to know about our mobile apps can be found in our articles. Just select Neuroon and then browse the iOS or Android categories.

The manual concludes by explaining how to restart your Neuroon in the event you encounter difficulty with your mask. All other troubleshooting suggestions, as well as detailed information about how Neuroon trackers and services work, can be found in the articles at [Neuroon.com](#).

Getting to know your Neuroon

Inside the Neuroon

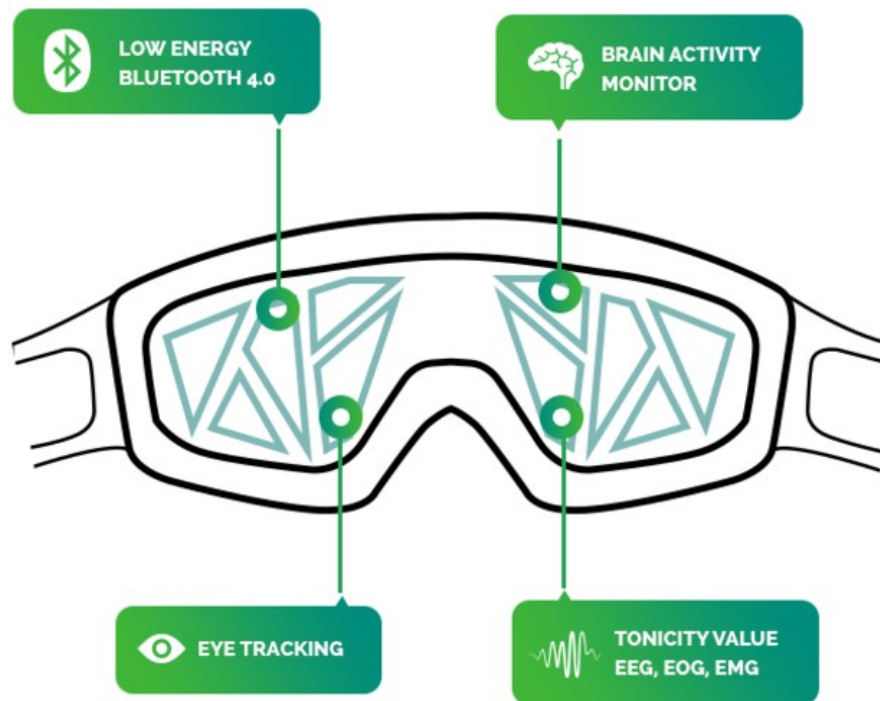- 1 - EEG Sensors
- 2 - Pulse oximeter
- 3 - Smart Pack
- 4 - Belt regulation

Learn how to speak Neuroon

LIGHTS:

DOUBLE LED-CONSTANT BLUE LIGHT - your Neuroon is on standby; use your mobile App to choose the action you would like to take.

DOUBLE LED-FLASHING BLUE LIGHT - your Neuroon is connecting to your Mobile App or sending data using the Bluetooth. Please do not turn it off, until it stops blinking.

DOUBLE LED-CONSTANT GREEN LIGHT - the Neuroon is ready to start a chosen therapy. Put your mask on the forehead and benefit from the Neuroon power! The light stops shining when you put the Neuroon correctly on your forehead. Please see the chapter:

SINGLE LED-FLASHING RED LIGHT - the Neuroon is turning off. Neuroon will turn off automatically after sending the data to the mobile app or manually if you click on the on/off button.

SINGLE LED-CONSTANT RED LIGHT (when connected to a charger) - the Neuroon is charging. Please do not disconnect the charger until the Neuroon is fully charged.

SINGLE LED-CONSTANT GREEN LIGHT (when connected to a charger) - your Neuroon is fully charged.

Wearing your Neuroon

Your Neuroon is designed to be most accurate when worn on your forehead.

It should stick to your forehead, lie comfortably on your nose. After starting a program if there's a good connection of electrodes, the **DOUBLE LED-CONSTANT GREEN LIGHT** will disappear.

The Neuroon mask is designed for adult's head. The size can be regulated by changing the length of the belt.

Using the Neuroon

Tapping on the on/off button

1 short click (approx.. 1 sec) - turn on / turn off the device.

1 long click (approx.. 8 sec) - reset the device. NOTE: read more in the section.

Sleep Analytics

The Neuroon tracks your biological data during your sleep and every nap. After syncing, you can view a daily and weekly accumulation of the following:

- Sleep Score - score is a way to measure your sleep quality based on sophisticated algorithm
- Sleep Duration
- Time to fall asleep
- Number of awakenings
- Heart rate
- Amount of REM Sleep

After each sleep or nap, you will get some tips & advices based on your sleep stats.

Restarting your Neuroon

If you experience one of the following problems, it may be fixed by restarting your tracker:

- Your mask isn't syncing
- Your mask has a battery charge but isn't lighting up
- Your mask isn't responding to button presses

To restart your Neuroon:

1. Press and hold the button for 8 to 10 seconds.
2. Let go off the button.
3. The Neuroon will flash color lights, ending with vibration. After the vibration, the reset is complete.

Now your Neuroon should work normally.

For additional troubleshooting suggestions or to contact Customer Support, see: neuroon.com

Neuroon General info & Specifications

Battery life and charging

Your Neuroon contains rechargeable lithium-ion batteries.

With normal use, your fully charged Neuroon should last about 5 to 7 days before needing a charge. You can check the level of your battery by looking at the mobile app.

After use of the Neuroon, the mobile app will inform you if you should charge your battery before the next use or not.

To charge your Neuroon, plug the charging cable into the USB port on your computer and plug the other end into the port in the Neuroon Smart Pack.

You may also use any standard AC adapter with USB output as long as it meets the following requirements:

- Rated output voltage: 5V DC, rated output current: min. 0,5A
- Class 2 certified device, conforming to Limited Power Source requirements covered in IEC60950-1

Care instructions

Regularly clean your mask —especially after sweating or exposing your skin to substances like soaps and lotions that can get trapped beneath the mask. To clean, gently wipe with a soft cloth moistened with warm or mild detergents and then remove all moisture with a soft, dry cloth. Do not use harsh solvents or chemicals for cleaning.

We also recommend occasionally cleaning the charging contacts on the back of your Neuroon Smart Pack. They can become obstructed by microscopic dirt or debris that prevents them from making a good connection to the charger. Use a toothpick or toothbrush to clean the charging contacts; do not scrape them with anything metal. Scraping may damage the plating and result in corrosion.

Memory

The Neuroon holds up to 7 days of your activity data.

Sync the Neuroon regularly to ensure the most detailed data has been downloaded onto the app. Data is transferred automatically after the end of each program.

Environmental conditions

Operating Temperature +5 - +35 °C

Non-operating Temperature: TBD

Maximum Operating Altitude: 5000m above sea level

Water resistance: NO

Help

Troubleshooting and assistance for your Neuroon can be found at neuroon.com

Return policy and warranty

Warranty information and the Store Return Policy can be found online at neuroon.com

Regulatory & Safety Notices

Model Name: Neuroon

The FCC ID & IC numbers are located on the Smart Pack.

To access the FCC ID and IC numbers, remove the foam mask by sliding the Smart Pack upwards, out of the enveloping foam. The numbers are printed on a label visible through the silicone layer, on the side of the Smart Pack that is normally facing away from the user when the mask is worn (see pictures below).

WARNING: TO PREVENT FIRE OR SHOCK HAZARD, DO NOT EXPOSE THIS UNIT TO RAIN OR MOISTURE.

USA

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

No changes shall be made to the equipment without the manufacturer's permission as this may void the user's authority to operate the equipment.

NOTE: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

This device meets the FCC requirements for RF exposure in public or uncontrolled environments.

FCC ID: 2AF73-NEUR1

Canada

This device complies with Industry Canada's licence-exempt RSSs. Operation is subject to the following two conditions: (1) This device may not cause interference, and (2) this device must accept any interference, including interference that may cause undesired operation of the device.

This device complies with the safety requirements for RF exposure in accordance with RSS-102 Issue 5 for portable use conditions.

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IC: 20848-NEUR1

European Union (EU)

Declaration of Conformity with Regard to the EU Directive 1999/5/EC Inteliclinic is authorized to apply the CE Mark on NEUROON (Model: Neuroon), thereby declaring conformity to the essential requirements and other relevant provisions of Directive 1999/5/EC and other applicable Directives.

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CAUTION

**RISK OF ELECTRIC SHOCK
DO NOT OPEN**

CAUTION: TO REDUCE THE RISK OF ELECTRIC SHOCK, DO NOT REMOVE COVER (OR BACK). NO USER-SERVICEABLE PARTS INSIDE. REFER SERVICING TO QUALIFIED SERVICE PERSONNEL.

DANGEROUS VOLTAGE: The lightning flash with arrowhead symbol within an equilateral triangle is intended to alert the user to the presence of uninsulated dangerous voltage within the product's enclosure that may be of sufficient magnitude to constitute a risk of electric shock to persons.

ATTENTION: The exclamation point within an equilateral triangle is intended to alert the user to the presence of important operating and maintenance (servicing) instructions in the literature accompanying the appliance.

IMPORTANT SAFETY INSTRUCTIONS

1. Read these Instructions.
2. Keep these Instructions.
3. Heed all Warnings.
4. Follow all Instructions.
5. Do not use this apparatus near water.
6. Clean only with a dry cloth.
7. Do not block any of the ventilation openings. Install in accordance with the manufacturer's instructions.
8. Do not install near any heat sources such as radiators, heat registers, stoves, or other apparatus (including amplifiers) that produce heat.
9. Do not defeat the safety purpose of the polarized or grounding-type plug. A polarized plug has two blades with one wider than the other. A grounding-type plug has two blades and a third grounding prong. The wide blade or the third prong are provided for your safety. When the provided plug does not fit into your outlet, consult an electrician for replacement of the obsolete outlet.
10. Protect the power cord from being walked on or pinched particularly at plugs, convenience receptacles, and the point where they exit from the apparatus.
11. Only use attachments/accessories specified by the manufacturer.
12. Use only with a cart, stand, tripod, bracket, or table specified by the manufacturer, or sold with

the apparatus. When a cart is used, use caution when moving the cart/apparatus combination to avoid injury from tip-over.

PORTABLE CART WARNING
(symbol provided by RETAC)

13. Unplug this apparatus during lightning storms or when unused for long periods of time.
14. Refer all servicing to qualified service personnel. Servicing is required when the apparatus has been damaged in any way, such as power-supply cord or plug is damaged, liquid has been spilled or objects have fallen into the apparatus, the apparatus has been exposed to rain or moisture, does not operate normally, or has been dropped.
15. **WARNING:** To reduce the risk of fire or electric shock, do not expose this apparatus to rain or moisture. Do not place objects filled with water, such as a vase, on the apparatus.
16. The AC mains plug is used as the power disconnect device and it should remain readily accessible and operable during intended use. In order to completely disconnect the apparatus from power, the mains plug should be completely removed from the AC outlet socket.
17. **CAUTION:** To reduce the risk of fire or explosion, do not expose batteries to excessive heat such as sunshine, fire or other heat sources.