
ISM-Module TruSystem ANY

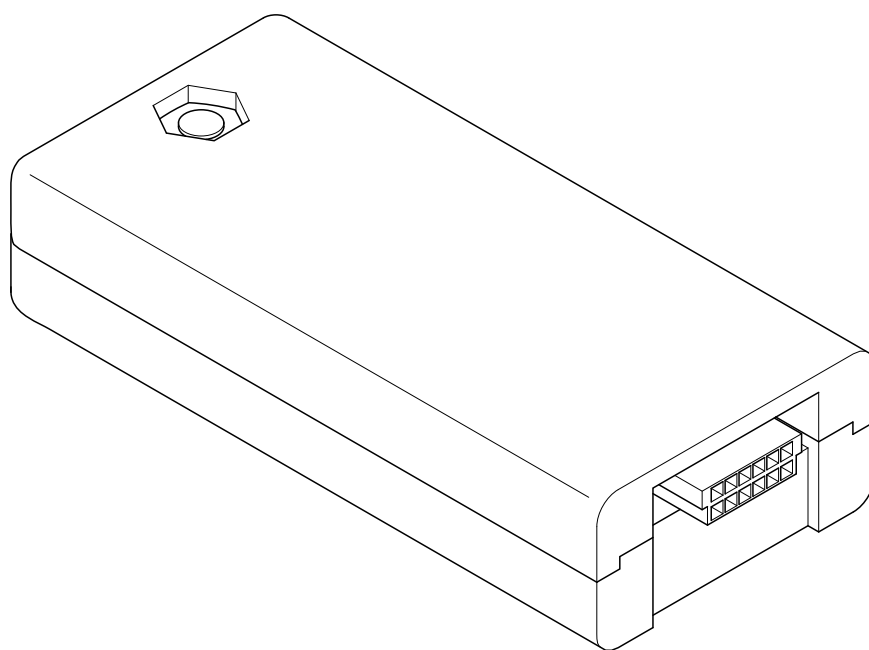#2056413

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Manufacturer and distributor/sales

Trumpf Medizin Systeme GmbH + Co. KG
Carl-Zeiss-Straße 7-9
07318 Saalfeld
Germany

www.trumpf-med.com

Telephone: +49 3671 586-0
Fax: +49 3671 586-41 166

Technical Customer Service

Telephone: +49 3671 586-0
Fax: +49 3671 586-41 175

Email: Service.med@de.trumpf.com

We are constantly working on the developing of our products. We therefore reserve the right to make changes to the scope of supply with respect to the format, equipment and technology at any time.

Reprinting, copying, or translating this document, in whole or in part, is prohibited without the express written permission of TRUMPF Medizin Systeme GmbH + Co. KG!

TRUMPF Medizin Systeme GmbH + Co. KG expressly reserves all rights under copyright law.

As part of the legal provisions, the manufacturer is only responsible for the technical safety characteristics of this device if the maintenance, service and modifications to this device are performed by him or by someone appointed by him in accordance with his instructions.

This instruction manual is applicable for the following service assemblies:

(S) ISM-Module TruSystem ANY (OPT) ...#2060655

(S) ISM-Module TruSystem ANY (FB).....#2060656

Device assembly

ISM-Module TruSystem ANY.....#2056413

Prior to using the product, read this user manual carefully from beginning to end in order to familiarise yourself with the product and its functions in a step-by-step manner. Store the user manual near the product!

Glossary

Term	Explanation
#	Part number
Trumpf Medical	TRUMPF Medizin Systeme GmbH + Co. KG, Saalfeld
Operating table	Operating table by Trumpf Medical

Content

Intended use	7
Safety information	8
ISM-Module pictograms	8
Installing the ISM-Module	8
ISM-Module TruSystem ANY (OPT)	8
ISM-Module TruSystem ANY (FB)	8
Function	9
Technical data	9
Regulatory approvals	10
European Union	10
Australia/New Zealand	10
Japan	10
USA/Canada	10
Care	11

1 Intended use

The ISM-Module TruSystem ANY may only be used in products or OEM components of Trumpf Medical.

The ISM-Module TruSystem ANY is built into an end device or OEM component of Trumpf Medical, and this is the only way it will function. The function of the ISM-Module is to connect an operating table with an external device in order to control the device and/or pass on information.


Any other use or extension of use is regarded as improper use. Liability claims do not apply for damages resulting from improper use.

ENGLISH

2 Safety information

- Please also read the instruction manual of the relevant end device before using this product.
- Pay attention to the pictograms on the product! All pictograms (adhesive labels) on the device must be complete and legible! Replace damaged or loose adhesive labels!

3 ISM-Module pictograms

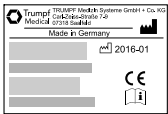Pictogram	Position, meaning
	<p>on the top of the ISM-Module</p> <p>The nameplate contains the following information:</p> <ul style="list-style-type: none"> - Manufacturer - Material number (with bar code) - Serial number (with bar code) - Date of manufacture (year/month) - CE marking - Note "Follow the user manual"
	<p>Approval codes</p>

4 Installing the ISM-Module

There is no need to install the respective software version of the end device on the ISM-Module. The use and operation of the end device defines the respective version of the ISM-Module TruSystem ANY with the assigned software version. There is a difference between the ISM-Module TruSystem ANY of the operating table (#2060655) and that for the remote control (#2060656).

The label with the authorisation codes is attached to the operating table and to interface assembly TAM-ISI. For remote controls, the label with the authorisation codes is kept in the packaging.

4.1 ISM-Module TruSystem ANY (OPT)

The ISM-Module TruSystem ANY (OPT) is installed in the holder provided on the operating table, and is connected to the operating table using an integrated connector. Refer to the operating table service manual for specific installation information. The following figure represents a possible installation within an operating table.

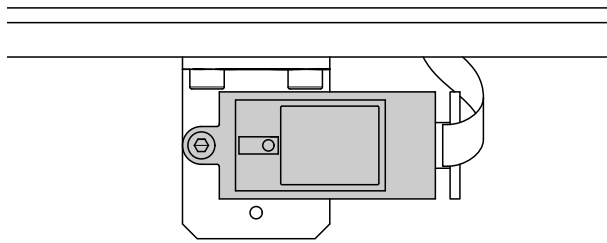
Figure: ISM-Module TruSystem ANY (OPT), without housing, connected to the operating table

4.2 ISM-Module TruSystem ANY (FB)

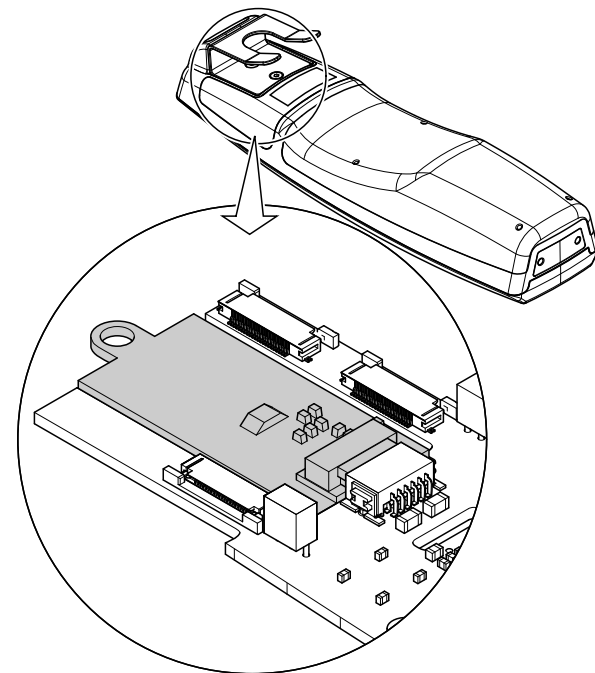
Figure: ISM-Module TruSystem ANY (FB), without housing, connected in the remote control

The ISM-Module TruSystem ANY (FB) is attached to the upper printed circuit board of the remote control.

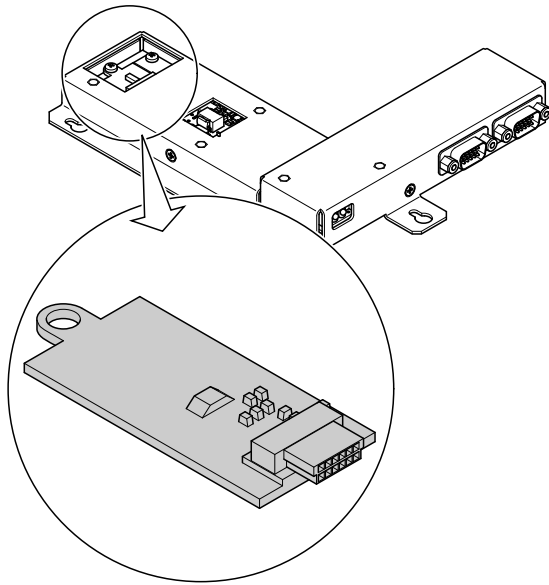
Figure: ISM-Module TruSystem ANY (FB), without housing, connected in the TAM-ISI interface assembly

The ISM-Module TruSystem ANY (FB) is installed in the TAM-ISI interface assembly, and is connected using an integrated connector.

5 Function

The ISM-Module TruSystem ANY in a remote control or interface assembly has a search function. After the ISM-Module is installed in a remote control or the interface assembly, or if the external device is taken into a different room, the radio module searches for active operating tables in the room. The active operating tables that are found provide the ISM-Module in the external device with information for a direct connection. The ISM-Module of the external device connect to the operating table on its own using the information provided. Staff confirm the correct connection. Just as the ISM-Module TruSystem ANY (FB) can connect to an operating table on its own, it can disconnect in the same way. The connection between the operating table and the external device can be terminated actively or passively.

Actively terminating the connection

There are two ways to actively terminate an existing wireless connection:

- Select a different operating table with the external device.
- Switch off the operating table or external device.

Passively terminating the connection

Move the operating table or external device out of communications range.

6 Technical data

- Dimensions (length x width x height):
(57.5 x 26 x 15) mm
- Weight 15 g
- Connection via UART connector
- Has its own MAC address
- Integrated ceramic antenna
- Antenna amplifier 0.5 dBi (max.)
- Power supplied via end device
- Transmission frequency band:
2.405 GHz to 2.480 GHz
- DSSS radio transmission
(Direct-Sequence Spread Spectrum)
- Modulation rate O-QPSK
(Offset Quadrature Phase Shift Keying)
- HF bandwidth 2 MHz (IEEE 802.15.4)
- Output power +3.5 dBi (max.)
- Emission designation 1M67G1D
- Encryption via AES128-PSK

Operating and storage conditions

Temperature	
Operation	+10 °C to +40 °C
Storage/transport	-20 °C to +70 °C
Humidity	
Operation	20 % to 80 %
Storage/transport	5 % to 95 %
Air pressure	
min. 700 mbar to 1060 mbar	

ENGLISH

7 Regulatory approvals

NOTE: Any changes or modifications made to this equipment not expressly approved by TRUMPF Medizin Systeme GmbH + Co. KG may void the following authorizations to operate this equipment.

The ISM-Module TruSystem ANY complies with the following legal requirements:

7.1 European Union

This equipment complies with the essential requirements and other relevant requirements of the R&TTE Directive (1999/5/EC) in consideration of the intended use. Please contact TRUMPF Medizin Systeme GmbH + Co. KG if the Declaration of Conformity is required.

7.2 Australia/New Zealand

With respect to electrical safety, EMC, and radio frequency spectrum, the product complies with the regulatory requirements for products intended for the Australia/New Zealand market.

7.3 Japan

This product is for indoor use only!

Ⓜ 202-SMD086

This equipment is granted pursuant to the Japanese Radio Law (電波法) and the Japanese Telecommunications Business Law (電気通信事業).

Test Standard: Ordinance Regulating Article 2 clause (1) Item (19)

7.4 USA/Canada

FCC ID 2AGCQANY2400SC1REV0

Note: This equipment has been tested and found to comply with the limits for a Class A digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced technician for help.

IC ID 20829-0ANY2400SC1

Note: This device complies with Part 15 of the FCC Rules and with Industry Canada license-exempt RSS standard(s).

Operation is subject to the following two conditions:

1. this device may not cause harmful interference, and
2. this device must accept any interference received, including interference that may cause undesired operation.

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8 Care

The ISM-Module TruSystem ANY is protected by the cover of the end device and is not visible.

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TRUMPF Medizin Systeme Österreich GmbH
TRUMPF Medical Systems Ltd.
TRUMPF Medical Systems, Inc.
TRUMPF Pte Ltd (Medical Division)