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Pregnant woman & Fetal ECG Monitor

Blissbuds Baby's fetal Heartbeat Smart Monitor
(BBSMT01)

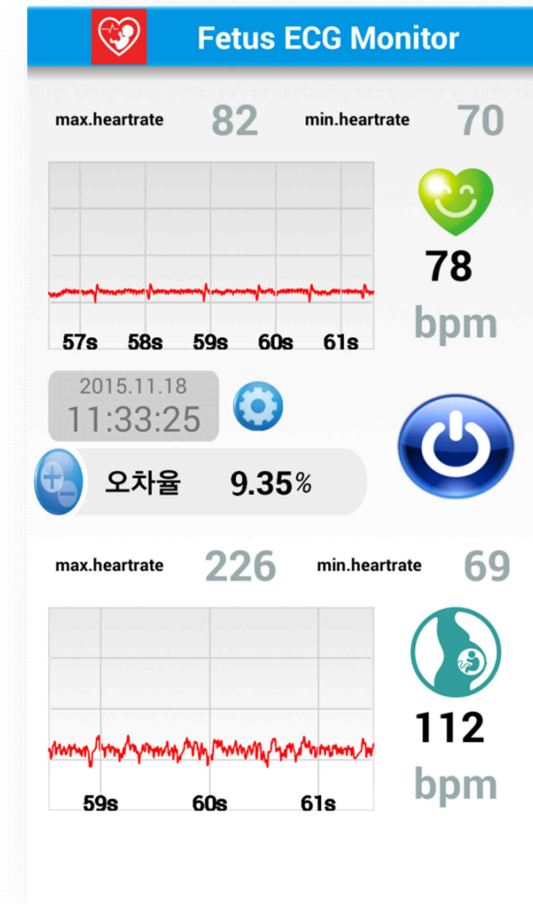
Pregnant woman & Fetal ECG Monitor is not a medical device, wellness device for health check.

CAUTION
RISK OF EXPLOSION IF BATTERY IS REPLACED
BY AN INCORRECT TYPE.
DISPOSE OF USED BATTERIES ACCORDING
TO THE INSTRUCTIONS

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Hi! Heartrate is the U-Health care device which monitors pregnant women and fetal health in real time, along with a smartphone application. It records ECG from the heartbeat of pregnant women and it extracts the fetal ECG and heartbeat in real time.

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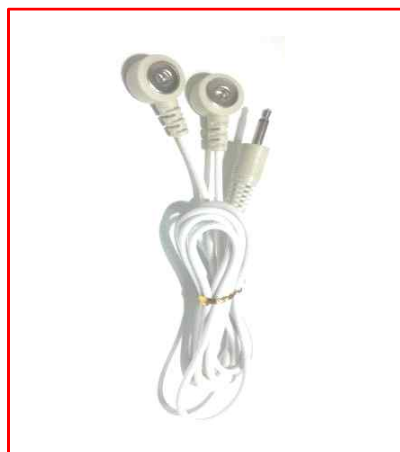
Hi! Heartrate is

- Easy to use
- Simple necklace type. Easy to carry and check yourself anytime, anywhere
- Check your real time data, using Bluetooth, anywhere
- Measurement time can be short or done continuously 24 hours

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
(기기)ECG Monitor

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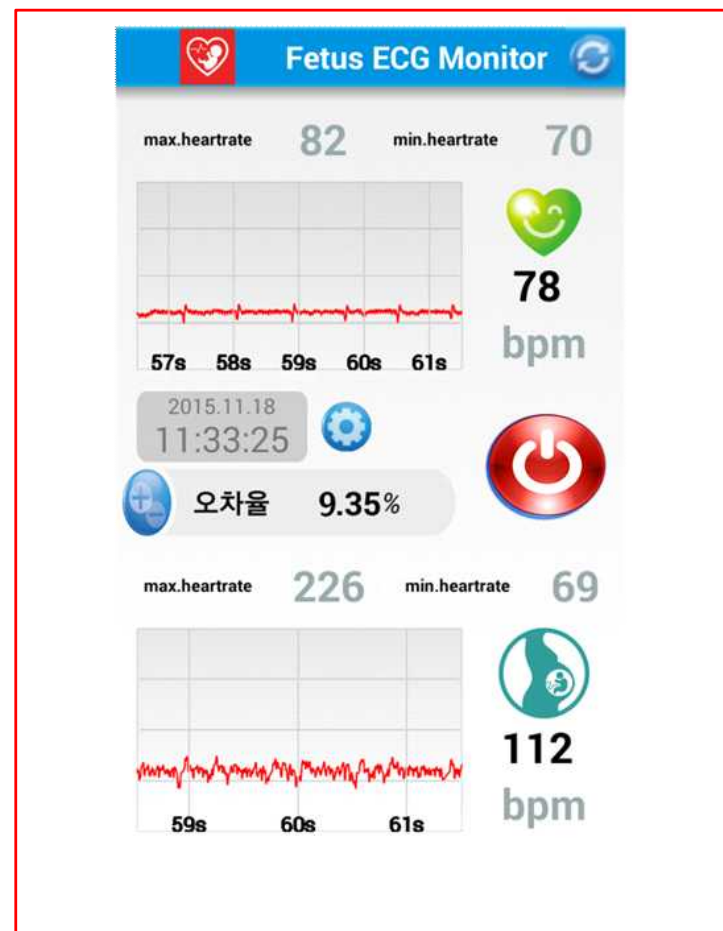
- ECG Monitor device
- Snap electrode wire cable & pad
- Application : Android & iOS

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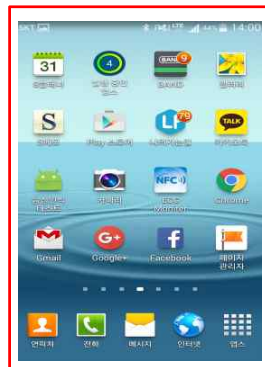Before using Hi! Heartrate,

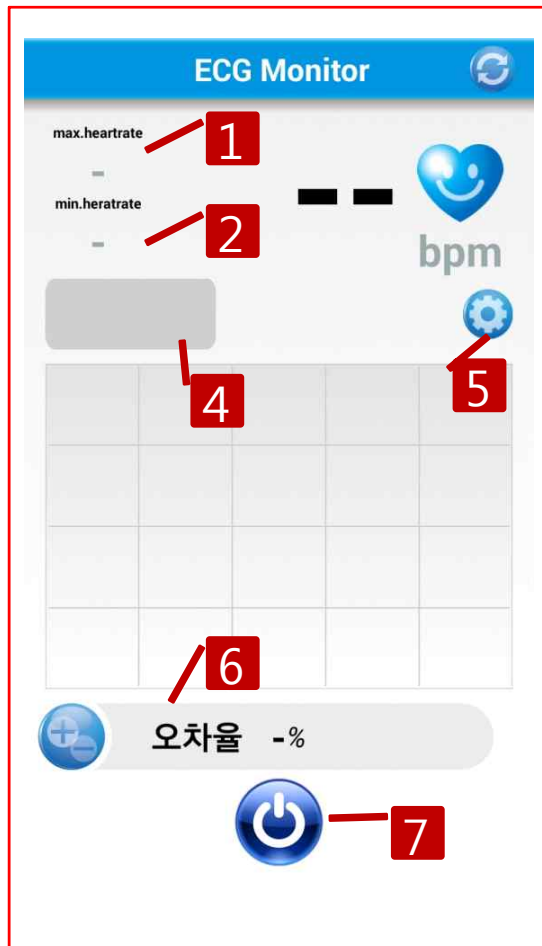
- 1) Turn power on at the bottom

- 2) Start the application on your phone


- 3) Choose the measurement menu and connect to Bluetooth

- Change the battery if you cannot find the Bluetooth while the power is on

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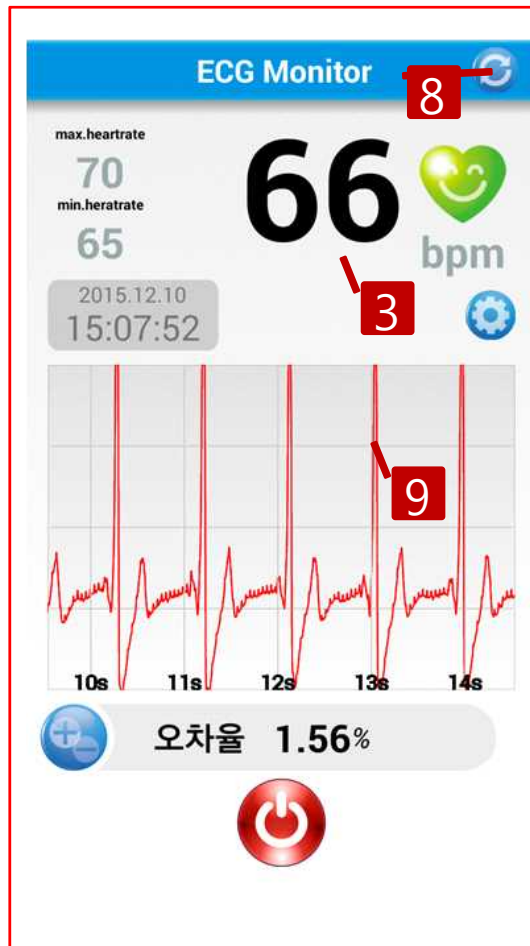- Press stop when you want to complete the measurement
- The Measurement data is saved automatically.

NOTES:

You should attach snap electrode pads to your chest.

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(Measurement screen for pregnant woman/fetus)

1. Pregnant woman/fetal max. heart rate
2. Pregnant woman/fetal min. heart rate
3. Pregnant woman/fetal average heart rate
4. Time
5. View data
6. Margin of error
7. Start/Stop
8. Change to adult measurement

- Press stop when you want to complete the measurement
- The Measurement data is saved automatically.

NOTES:

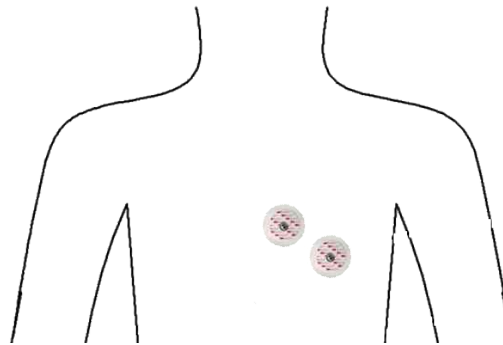
You should attach snap electrode pads on pregnant woman's abdomen.

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How to use Hi! Heartrate Application

- (1) Connect the device and snap electrode cable and turn power on
- (2) Electrode pad placement point for adults
Upper heart about an angle of 45 degrees
Space between pads: 10cm
- (3) Electrode pad placement point for pregnant women
Right and left side of the navel
Space between pads: 10cm
- (4) Start application
- (5) Choose ECG measurement for adults
- (6) Choose ECG measurement for pregnant women/fetus

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Hi! Heartrate Products Specification

List	Details
Product name	Blissbuds Baby's fetal Heartbeat Smart Monitor
Category	Pregnant women & Fetal ECG Monitor
Model	BBSMT01
power	3.0V / 10mA
Tma	-10 ~ 50°C
Design function	BLE (Wireless Range: 10m)
	LED(Low battery)
	Power on/off switch
	Lithium Battery 3V (TOSIBA)
	Snap electrode 3.5Ø

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Cautions

- Check products for defects
- Ask manufacturer or seller for inspection when you find defects
- Check connection of electrode pads and the device
- The electrode pad is disposable. Replace it after you check once.
- Turn power off after measurement
- Make sure the electrode pads do not touch metallic items such as a necklace or belt buckle
- Maintain moderate temperature and humidity. Keep away from dust or flammable materials.
- Keep out of reach of children
- Make sure electrode pad is on the placement points when you use it
- Do not use on wounds or contaminated skin

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1. Connect Connect the device and snap electrode cable
2. Attach electrode pads on placement points
3. Choose ECG measurement menu

FCC Information to User

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

Caution

Modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

FCC Compliance Information : This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.