

PAGE 1 INSIDE

JLAB

WELCOME TO THE LAB

This Lab is where you'll find real people, developing really great products, in a real place called San Diego.

PERSONAL TECH DONE BETTER

Designed for You
We build devices to what you want and we're always looking for ways to make everything better and better for you.

Lab Quality Sound
Choose from different settings and customize your sound style to match exactly what you're doing.

Surprisingly Awesome Value
Our design focus is on great functionality and fun only when product isn't a truly accessible price.

#yourkindoftech

WE GOT YOUR BACK

We're passionate about making the best possible experience when using our products. If you have any questions, comments, or feedback, we'll have the pros develop a real answer for you (of course, customer support takes time).

Website: info@yourkindoftech.com **Email:** support@yourkindoftech.com
Phone (US, EU): +1 (858) 445-7172 (toll-free) **Phone (US, EU):** +1 (858) 445-7172 (toll-free)
Text: +1 (858) 445-7172 (toll-free) **Text:** +1 (858) 445-7172 (toll-free)

LATEST AND GREATEST

Our team is constantly developing new products and services. This could mean new features or controls which are not included in this guide.

For the latest version of this manual, scan the QR code below.



© 2023 JLAB Inc. All rights reserved. Model: JLAB-HA-001

PAGE 1 OUTSIDE

JLAB




HEAR
OTC HEARING AID

QUICK START GUIDE

PAGE 2 INSIDE

1 CONNECT

CONNECT YOUR DEVICE
(Don't need Bluetooth? Follow steps 1-2 and turn to page 3 for Hearing Aid Only Mode.)

1. Remove stickers. Place earbuds back in case and close door.

2. Blue lights on case indicate charging.

CHARGING CASE
Connect charging cable to computer or USB-C charging device.

LEDs glow white when charging and turn red when fully charged.

CHARGING EARBUDS
Earbuds will power OFF and start charging when placed in case. Blue lights = Earbuds charging.

LEFT EARBUD RIGHT EARBUD


PAGE 2 OUTSIDE

WITH LOVE FROM THE LAB
We have lots of different ways to show we care.

GET STARTED + FREE GIFT
Product updates | How-to tips | FAQs & more
Go to the [JLAB website](#) to unlock your customer benefits including a free gift.
See us online for additional details.

LET'S GET STARTED ▶

Having trouble?



Scan QR for Interactive Video

1 CONNECT

PAGE 3 INSIDE

2 CONTROLS

CONTROLS: HEARING AID

LEFT VOLUME CHANGE: Press Hold
RIGHT VOLUME CHANGE: Press Hold
SOUND MODE CHANGE: 2x Press

RIGHT VOLUME CHANGE: Press Hold
SOUND MODE CHANGE: 2x Press

VOLUME: 1 2 3 4

OUND MODES

LOUD ENVIRONMENT
Decreases surrounding noise and white noise to reduce ear fatigue.

RESTAURANT
Decreases background noises in crowded environments.

CONVERSATION
Enhances voices facing you.

QUIET ENVIRONMENT
Enhances all noises to clearly hear surroundings and voices.

CONTROLS: BLUETOOTH MEDIA

PLAY/PAUSE: 1x Press
VOLUME DOWN: Press Hold
TRACK BACK: 2x Press
ANSWER/PAUSE UP: 2x Press

PLAY/PAUSE: 1x Press
VOLUME UP: Press Hold
TRACK FORWARD: 2x Press
ANSWER/PAUSE UP: 2x Press

FITTING

Cush Fit™

Closed Ear Tip: Best for a more focused sound

Open Ear Tip: Lets in more air for more natural sound

3 REPEATABLE USE

PAGE 3 OUTSIDE

3 REPEATABLE USE

HEARING AID ONLY MODE
(Does not connect to Bluetooth and maxes battery life)

Press Hold 2x seconds in case. Earbuds will blink white.


HEARING AID + BLUETOOTH MODE
(Connects to Bluetooth and has the best)

Press Hold 2x seconds in case. Earbuds will blink blue.

Remove earbuds from case. One earbud blinks. Blue-white = Ready to Pair.

Select "JLAB Hear" in your Bluetooth device settings.

DOWNLOAD THE APP
Customize JLAB Hear and update to the latest software. Download the JLAB Hearing Health App on Apple/Google Play Store.



2 CONTROLS