

QUICK START GUIDE

Refer to the User Guide for detailed information on operations and use.

Model: NW-1000

Offered **exclusively** by PROTECUUS*

Equine Health 8.5 Safety



BEFORE YOU START

- Fully charge your device before first-time use.
- DO NOT turn on your device until you are ready to set up.
- Visit <u>account.nightwatch24.com</u> to register your account.
- Refer to the User Guide for complete instructions.

IN THE BOX







ACCOUNT

VISIT account.nightwatch24.com

1. Register account

· Click "REGISTER" on the log-in page and follow prompts to create a username and password.

2. Activate device

Log into your account and select "ADD NEW DEVICE" within the device tab. Enter your Activation Code
and the SSID/Serial # for your device, and select an annual monitoring contract by accepting the terms
and conditions of the license agreement.

3. Customize settings

 Log into your account to add horses, groups, and users, to assign a device to your horse, and to set alert thresholds. Please refer to the User Guide for complete instructions.



INSTALL MOBILE IOS APP*

- 1. Download the NIGHTWATCH® iOS app from the iTunes App Stores $^{\rm SM}$.
- Log into the iOS app using the username and password you created when setting up your account.

POWER

FULLY CHARGE YOUR DEVICE BEFORE FIRST TIME USE OR IF DEVICE HAS BEEN STORED FOR A PERIOD OF 1 OR MORE MONTHS



 To turn **OFF** your device, hold down the power button for <u>5 seconds</u> until LED rapidly blinks **WHITE** and then turns off*

- $\underline{\textbf{ALWAYS}}$ remove device from animal when charging.
- \bullet $\underline{\text{DO NOT}}$ charge within 20 cm of any person or animal.
- **DO NOT** charge device near any flammable substances.

^{*} Please refer to the User Guide for complete LED details.

CONNECT



CONNECT TO WIFI (with cellular)*

- 1. Launch the NIGHTWATCH® iOS app or access the Web Portal.
- 2. Choose "connect to WiFi" within device settings.
- 3. Select preferred WiFi network and enter the network password. Once connected, your device's SSID/Serial # will disappear. Repeat these steps to connect to a new WiFi network.

CONNECT TO WIFI (without cellular)*

- Turn OFF your device and then hold down the power button for <u>5 seconds</u> until LED is solid RED.
- Enable WiFi on any handheld device or computer within range (30-50 ft) of your device.
- 3. Select your device by its SSID/Serial # (ie, NW-XB-0001000) and enter the authorization code (contained within your shipping confirmation email) as the password.
- Go to https://connect-nightwatch from a web browser and choose "connect to WiFi" within device settings.
- 5. Select preferred WiFi network and enter the network password.

^{*} To maximize performance, connect your device to a WiFi network (if available) even when there is cellular coverage.

FIT


A LOOSE FITTING DEVICE MAY RESULT IN SKIN IRRITATION, IMPROPER OR INCONSISTENT RESULTS, AND/OR DAMAGE TO YOUR DEVICE.

CONTACT CUSTOMER CARE IF YOU ARE UNSURE IF YOUR HALTER/COLLAR FITS PROPERLY.

HALTER: Attach <u>crown</u> to <u>headstall</u> with ID plate appearing on left side of horse. Tighten and adjust double buckles on the headstall for a firm yet comfortable fit

COLLAR: Attach <u>crown</u> to <u>coupler</u> with ID plate appearing on left side of horse. Tighten and adjust double buckles on the headstall coupler for proper fit.



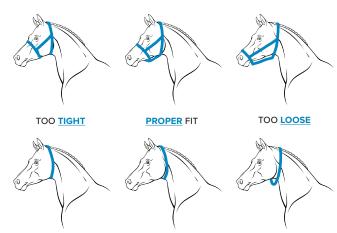

Firmly bend coupler with mesh inlay (for collar) to the contour of your horse's throat latch to prevent rotation or slippage over your horse's ears.

Need support? We're here to help.

Customer Care

800-757-3856 (+512-515-1095) M-F 8:30 AM - 5:30 PM CST support@protequus.com