

The Bluetooth® word mark and logos are registered trademarks owned by the Bluetooth SIG, Inc. and any use of such marks by SC Industries is under license. Other trademarks and trade names are those of their respective owners.



Designed, tested and assembled in the USA.
Made in the United States with domestic
and foreign components.

 **LifeEar®**
STREAM



Simple Device. Enhanced Life.
Read manual before use.

LifeEar®STREAM Box Contents*

Size 312 Batteries
(Brown-Tab)



Hub
USB Charger

easy-CLICK™ Tips†



Carrying Case



Cleaning Brush



*Contents will vary depending on product ordered.

†easy-CLICK™ is a registered trademark of Hearing Components.



Left ear = Blue receiver
Right ear = Red receiver

- 1 easy-CLICK™ Tip
- 2 Receiver
- 3 Receiver Wire
- 4 Microphones
- 5 Program Button
- 6 Battery Door
- 7 Right/Left Indicator

Power and Volume Control

Power

ON = battery door closed


OFF = battery door open

Battery Type

Size 312 zinc-air battery
(brown tab).

*Wait **one FULL minute**
after removing tab for
battery to fully charge.*

Inserting Battery

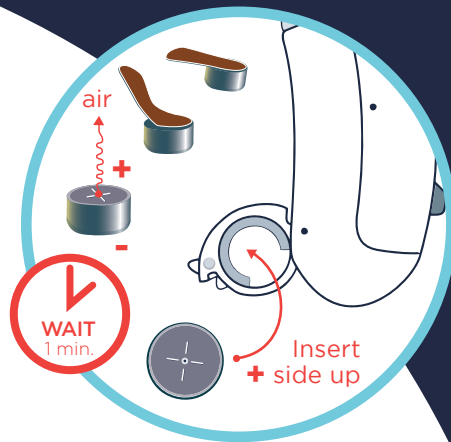


Wearing Your LifeEar® STREAM

1 Hold onto receiver.


2 Place tip in entrance of ear canal.




2

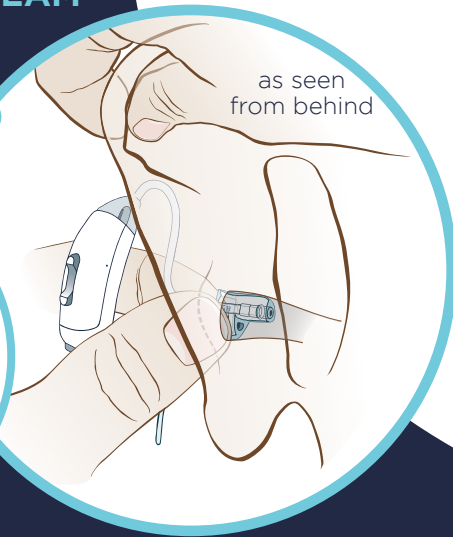


3 Place behind ear.

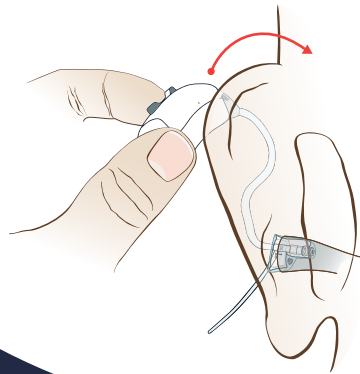

4 Move ear lock into bowl of ear.

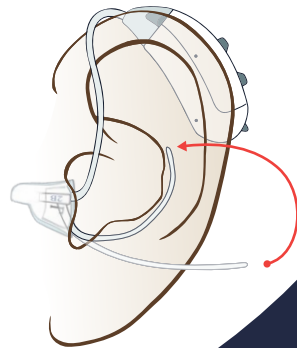

Program Control

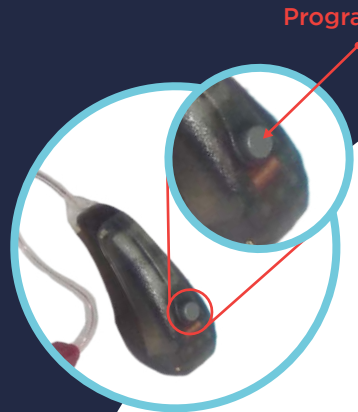


Press Program Button
to change Programs

- ♪ = Program 1
- ♪♪ = Program 2
- ♪♪♪ = Program 3
- ♪♪♪♪ = Program 4
- ♪♪♪♪♪ = Program 5

Program Settings

- 1 Automatic: all frequencies emphasized — up to XX dB.
- 2 Quiet: all frequencies emphasized — up to XX dB.*
- 3 Conversation: speech frequencies emphasized — up to XX dB.*
- 4 Automatic: high-frequencies emphasized — up to XX dB.
- 5 Quiet: High-frequencies emphasized — up to XX dB.



*To avoid whistling, use foam tips at higher volumes.

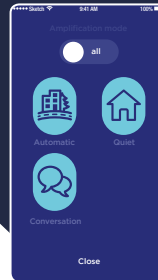

Replacing Tubing and Tips





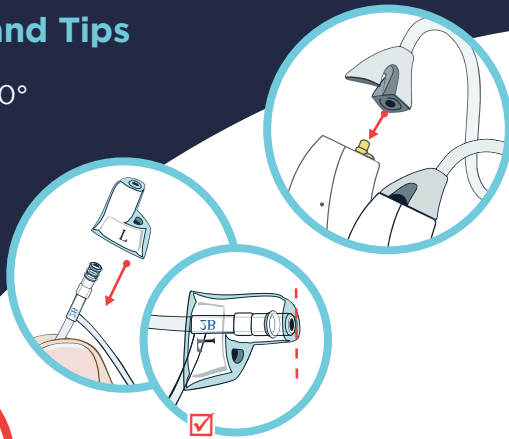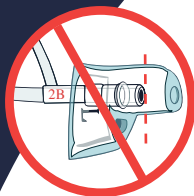

Tubing: Rotate tubing 90° and slide off. Push new tubing on.

Tip: Pull tip off tubing. Push new tip on until flush with end of tubing.



Replace every three months for best sound at www.lifeear.com

Fitting Chart

	Average Ear Canal	Large Ear Canal
Natural Sound 	included  LifeEar Tip Standard	 * LifeEar Tip Large
Maximum Output 	 * Power Tip Standard	included  Power Tip Large

*Available at www.lifeear.com

Cleaning and Care

Wipe daily with tissue.

Regularly use cleaning brush and wire loop to remove debris.

Avoid exposure to moisture.

Store in a cool, dry place.



Support

www.lifear.com

Troubleshooting

Problems? Issues? Questions?
Visit www.lifear.com

Note: If experiencing weak or no sound, clean easy-CLICK™ tip thoroughly or replace.

Warranty

To activate your one year warranty from date of purchase, register your LifeEar at: www.lifear.com/warranty

FCC Statement

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation.

This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one of the following measures:

Reorient or relocate the receiving antenna.

Increase the separation between the equipment and receiver.

Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.

Consult the dealer or an experienced radio/TV technician for help.

To assure continued compliance, any changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate this equipment. (Example - use only shielded interface cables when connecting to computer or peripheral devices).

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions:

- (1) This device may not cause harmful interference, and
- (2) This device must accept any interference received, including interference that may cause undesired operation.

FCC Radiation Exposure Statement:

(if mobile device (generally 20 cm distance) is applicable, MPE calculation)

"This equipment complies with FCC RF radiation exposure limits set forth for an uncontrolled environment. The antenna(s) used for this transmitter must be installed to provide a separation distance of at least 20 cm from all persons and must not be co-located or operating in conjunction with any other antenna or transmitter".