SNORE CIRCLE

Please scan the QR code to
download SnoreCircle application.

FCC Caution

§ 15.19 Labelling requirements. This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions:

- (1) This device may not cause harmful interference, and
- (2) this device must accept any interference received, including interference that may cause undesired operation.

§ 15.21 Information to user.

Any Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

§ 15.105 Information to the user.

Note: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

* RF warning for Portable device:

The device has been evaluated to meet general RF exposure requirement. The device can be used in portable exposure condition without restriction.

English Product Instruction	01~12
中文产品说明书	13~26


TABLE OF CONTENTS

1	Product overview and components - - - - -	01
2	Power and battery - - - - -	02
3	Wearing guide - - - - -	04
4	Working principle - - - - -	05
5	Connecting with App - - - - -	06
6	After-sales service - - - - -	08
7	Precautions - - - - -	08
8	Troubleshooting - - - - -	09
9	The Manufacturer - - - - -	12
10	Hazardous Substance - - - - -	12

1 PRODUCT OVERVIEW and COMPONENTS

The fourth generation Yunzhongfei VVFLY SnoreCircle is an intelligent wearable consumer product, which provides a better sleep and quality of life.

Schematic Specification

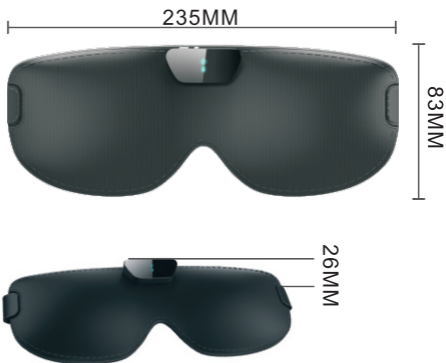Package components

- Eye mask X 1 (approximately 38 g per device)
- Instruction manual X 1
- Charging/data-transferring micro-USB line X 1 (port applicable with Android's line)

2 POWER and BATTERY

Power on

Switch the knob to ON to start the power. After turning on, the Bluetooth indicator will flash for 10 seconds. The flash disappears if it fails to connect with your phone. After checking the power indicator remains green, please wear the eye mask and lie down to rest. To keep others sleep well preventing from light bothering, the power LED will turn out automatically.

Switch the knob to ON

To self-test if the product works properly:

After turning on the product and wearing it, continuously imitate a snoring sound in a quiet environment and you will feel a weak vibration.

Power off

Switch the knob to OFF to shut down. And all the indicator lights will turn out.

Battery display

While the product is powered on, simply press the FUNC button to confirm battery level by checking the power LED light color.

- 1) Power light in red means low battery, please recharge it.

- 2) Power light in green means sufficient battery.

- 3) You can get battery remaining percentage through connecting with SnoreCircle App on your smart phone.

Concerns about battery

- 1) The rechargeable built-in lithium battery can be charged through phone chargers, portable chargers and computer-connecting USB.
- 2) It can be fully charged in two hours. After charging, it lasts for two nights. We suggest those heavy snorers charge it every day.
- 3) Once it's fully charged, the power indicator remains green.

3 WEARING GUIDE

- 1) Once the product is powered on, wear it properly according to the following instruction.
- 2) Make sure the main unit is closely put to your forehead.


Technical hint

If the product is not worn properly, it would affect the precision of snore recognition and physical intervention. Therefore, we recommend wearing the product according to our instruction.

4

WORKING PRINCIPLE

- A) The product precisely detects and identifies snore sounds through bone-conducted technology.
- B) Once the snore is recognized, our intelligent product delivers physical intervention through 36-level of tiny vibration to sleeping users in a non-perceived manner.
- C) To see if this product improves your sleep, you can record and track the snoring sound or frequency through the product App, family observations, your own perception, recordings and so on to compare the changes.

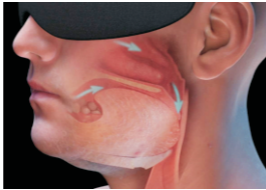5 CONNECTING with APPLICATION

SnoreCircle App is applicable for

iOS system: iPhone 4S or later

Android system: 4.3 or later, with Bluetooth 4.0

How to download the App

iOS: Go to App Store and search for SnoreCircle

Android: Go to Google Play and search for SnoreCircle

scan the QR code to download

App features of SnoreCircle

- 1) View snoring times, intervention times , and snore-stopping rate
- 2) View quality of your sleep with detailed table, graph and chart
- 3) View historical snoring archive
- 4) Adjust the intensity and sensitivity of physical intervention, or activate the snore-stopping delay function to personalize the product.

How to use the App

- 1) After the App is installed, turn on the Bluetooth on your smart phone and open the App. Meanwhile, make sure the product is powered on and press the FUNC button to start Bluetooth that will flash in blue light. Bind the product with your phone, then you can view your snoring and sleep quality through the App.
- 2) Based on individual's snoring severity and history, you need to set up your ideal working mode for the product. The default mode may not suitable for everyone. We recommend adjusting the mode for your own on the first few days of using the product.
- 3) For light sleepers, we recommend not to give physical intervention during your light sleep cycle. Through activating the snore-stopping delay function, the product does not interfere with your sleep.

6

AFTER-SALES SERVICE

- 1) The warranty lasts for one year from the date of original purchase for the whole product.
- 2) Artificial damages are out of warranty.

We highly recommend an observation of the wearing for two weeks. It's about 10% users need to adapt to the product for overcoming the wakening-up and foreign body sensation.

7

PRECAUTIONS

- 1) Use and store the product within specific temperature range, -10°C to 50°C . The extreme temperature may cause malfunction of the product.
- 2) Avoid put the product near heating sources, such as heaters, microwaves, or roasters.
- 3) Keep the product from heavily shock or vibration, it may damage the product.
- 4) Avoid cleaning the product with chemicals, detergents or water. Always keep it dry.
- 5) Please do not remove or replace the built-in lithium battery of the product. It may damage the product. Do not throw the product into fire, it may cause an explosion.
- 6) This is not a toy. Please place it in a proper location and keep it out of reach of children. Avoid mis-operation by children that may cause hazards.

8 TROUBLESHOOTING

Battery

- 1) The product contains a rechargeable built-in lithium battery, which can be charged through phone chargers, portable chargers and computer-connecting USB.
- 2) It can be fully charged in two hours. After charging, it lasts for two nights. We suggest those heavy snorers charge it every day.
- 3) Once it's fully charged, the power indicator remains green light.
- 4) When the power indicator shows in red light, it means low battery. Please charge it before using it. Otherwise, it may shut down automatically.

What if the product wakes me up?

Through App binding with the product by Bluetooth, you can adjust the working mode to moderate or low level of intensity and sensitivity. Meanwhile, you can activate the snore-stopping delay function to prevent intervention during your light sleep.

How long does it take to have an effect?

We suggest you follow up weekly as an observation cycle to compare the snoring records, snoring times, snore sounds decibel and your own feelings.

How to know if the product works properly?

- 1) Switch the power knob to ON.
- 2) After turning it on, please wear the product without shaking your head. Then continuously imitate a snoring sound for 10 seconds. If the product detects the snore, it delivers tiny vibration.
- 3) After wearing overnight, view the snoring result through the App. And you will know if it works properly.


Is it necessary to connect the product with phone all the time when using it?

The product can be used independently without connecting with your phone. The product collects and saves the data in the built-in CPU during your sleep. After you wake up, you simply synchronize it with the App to view data. Even if you don't use the APP, the product is allowed to work solely.

How to charge the product?

There is a charging port next to the power knob. Uncap the seal-cover of the port and charge it with phone chargers, portable chargers or computer-connecting USB.

Schematic of the functions

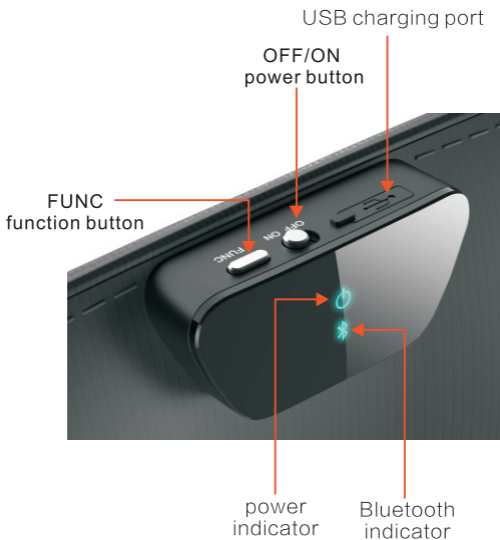9 THE MANUFACTURER

Manufacturer:

Shenzhen Yunzhongfei VVFLY Electronics Co. Ltd.

Address:

Room 1310, Changhong Science and Technology Building, Southern District in High Tech Zone, 518000 Nanshan District, Shenzhen, China

Service e-mail:

service@vvfly.cn

10 HAZARDOUS SUBSTANCE

According to the relevant provisions and requirements of "Electronic Information Products Pollution Control Regulations" of People's Republic of China, provide the following parts including the name of electronic products, toxic and harmful substances in the style.

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1、“○” Indicates that the content of the hazardous material in all homogeneous materials of the part is required by the standard of SJ/T11363-2006

2、“×” Indicates that the amount of the hazardous substance contained in at least one of the homogeneous materials in the part exceeds the limit specified in the standard of SJ/T11363-2006

The safety period of the use of part of the environment

Component part	Safety period for environmental use
Smart snoring MASK	10

* The lifespan in the above symbol is based on normal conditions of use (temperature / humidity)

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