



ENURESIS ALARM

(Built-in wireless device)

MODEL : UA433

INSTRUCTION MANUAL

I-DING MEDICAL EQUIPMENT CO., LTD

Website : www.iding.url.tw

TEL:07-5818679

**Company address : No.66, Pizitou Rd, Zuoying Dist.,
Kaohsiung City 813, Taiwan (R.O.C.)**

**Factory address : 8F.-2, No.288-7, Xinya Rd, Qianzhen
Dist., Kaohsiung City 806, Taiwan (R.O.C.)**

Able Of Content

Suitable.....	3
Product Description.....	3
Product Item.....	4
Operating Instructions.....	4
Daily Preserved Method.....	8
Precautions.....	9
Specification Table.....	10
NCC Warning / FCC Warning.....	11
Return Notes.....	12
Product warranty book.....	13

Suitable

Unconscious long bedridden(For example : Stroke 、 Brain damage 、 Spine damaged patients or human vegetable).

Product Description



Launcher



Alarm



Spring clip



Rubber Wrap



Sensing cable



Product Item





- ◆One Launcher
- ◆One Alarm
- ◆Two Sensing cable
- ◆Three Rubber Wrap
- ◆Two Spring clip
- ◆Two AA battery
- ◆One A23 battery
- ◆One sterile gauze


Operating Instructions

When the sensor line is sensed, the LED lamp of the transmitter will light up. The alarm will sound a warning sound or light after receiving the signal, and remind the operator to do the appropriate treatment.

※ Installation Notes :

Illustration	Description
	1. Put the sensing cable sensing surface up on the rubber wrap.
	2. Connect the tabs that sense the rear of the cable to the slots on the launcher.

	<p>3. There are four hooks on the back of the launcher to help secure the launcher on the spring clip. Press the spring clip with both fingers down the launcher hook slowly push inside, until the spring clip stuck on the launcher so far.</p>
	<p>4. Will have been put on the rubber wrap sensor cable sleeve in the spring clip.</p>
	<p>5. Put the diaper on the user's body.</p>
	<p>6. Will have completed the above steps launcher, sensing cable and spring clip along the urethra below the urethra placed close to the position before the perineum, you can start the sensing.</p>

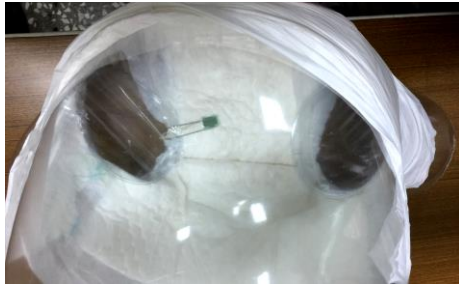

7. Sensing the cable laying inside the diaper perspective.

※Product keys and light description :

Launcher button light function



Alarm button lights function



※ Instructions :

1. When the sensing cable is wet when the launcher LED lights will light up, while alarm will also issue a warning sound and light reminder.
2. There are two buttons on the alarm, one is to adjust the volume, one is to select the alert music. Volume is large, medium, small and silent four options. Caution music has more than 10 kinds of music to choose from.
3. To remove the sensing cable and launcher from the diaper, turn on or off the diaper before proceeding.

※ How to determine the lack of electricity :

When you press the button on the launcher, if you find that the LED light on the launcher becomes dark and the alarm does not sound a warning, it means that the launcher has insufficient power. Please replace the battery with a new one.

※ How to replace the battery :

1. Launcher : Remove the screw on the back of the launcher, open the launcher case, remove the old battery, replace it with a new battery, and replace the screw in the launcher case.
2. Alarm : Remove the cover on the back of the alarm battery, unplug the old battery, put on a new battery, and then buckle back the battery cover.

Daily Preserved Method

1. Launcher and alarm :

Launcher and alarm are precision electronic components inside, do not arbitrarily disassemble, heavy fall, so as to avoid damage to the failure.

2. Sensing cable :

- (1) In use or after use, do not pull the sensing cable drastically to avoid damage to the cable.
- (2) Sensing cable sensing surface, please do not be worn, damaged, any improper use or external force is likely to degrade, undermining its sensing capabilities.
- (3) Sensing cable as a consumable, if contaminated with urine, excrement or has worn, broken, weakens the sense of force, please discard the replacement of the new sensing cable to use.

3. Rubber Wrap :

This is a consumable, if damaged, broken or contaminated with excrement, urine, please use a new rubber wrap.

4. Spring clip :



This is a consumable, if contaminated with excrement, urine, rust, damage, deformation and other conditions, please replace the new spring clip use.

5. For a long time do not use, please remove and remove the battery launcher and alarm.

Precautions

1. To place the product in front of the diaper, do the following two tests first to ensure that the urine is properly sensed:
 - (1) Click the launcher on the black button, launcher LED lights will light up, while the alarm will also sound and light, which means launcher and alarm are all with power.
 - (2) Dropping water in the sensing cable, the alarm will immediately sound and light, which means that the sensor cable and launcher have the correct link.
2. Will be exposed to the skin of the spring clip place a sterile gauze to avoid direct skin contact.
3. Sensory cable and spring clip should be placed along the groin below the urethra in the position near the perineum, male and female positions are different, please put appropriate position discretion, to avoid placing the sensor cable position or distance is too far can not be caused by sensing .
4. To take off the launcher and sense the cable, please open the diaper or take off the diaper before removing the cable and launcher.
5. This product is for personal hygiene and to avoid cross-infection, sensing cable, rubber wrap and spring clip for personal use only, please do not for different users.
6. Do not reuse the sensing cable, rubber wrap and spring clip to avoid infection.
7. When the sensing cable is unplugged from the launcher, remove the sensing cable manually according to the socket end. Do not pull the cable part directly. Otherwise,

the cable may break or damage the sensing cable.

8. Symbols and Symbols :  (Please refer to the instruction manual) 、  (BF category application site).

9. Products Please dispose of electronic waste in accordance with local ordinances.

Specification Table

Name	ENURESIS ALARM (Built-in wireless device)	
Model	UA433	
Country of manufacture	Taiwan	
	Launcher	Alarm
Size	4.8×3.5×1.5cm(±1cm)	10.5×7.5×4cm(±1cm)
Weight	15g(The battery is not included)	65g(The battery is not included)
Voltage	12VDC	3VDC
Use the battery	A 23 battery	AA battery
Frequency	433.91MHz	
Transport / storage of the temperature and humidity conditions	Temperature:-15℃ ~55℃ Humidity: ≤ 50%RH	
Temperature and humidity conditions of the operating environment	Temperature:-15℃ ~55℃ Humidity: ≤ 75%RH	

NCC Warning

This product complies with the provisions of Article 12 and Article 14 of the Measures for the Management of Low-power Radiated Radio Motor

1. The low-power RF motor qualified by type certification shall not change the frequency, increase the power or change the original design features and functions without the permission of the company, trade name or user.

2. The use of low-power radio frequency motors shall not affect flight safety and interfere with legitimate communications. When interference is found, it shall be immediately deactivated and shall be continuously used until there is no interference.

Legal communication in the preceding paragraph means radio communication in accordance with the provisions of the Telecommunications Law.

Low-power RF motors must withstand the interference of legitimate communications or industrial, scientific and medical radiowaved electrical equipment.

NCC Certificate :



Federal Communications Commission (FCC) Statement

§15.105

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

§15.21

1. Any changes or modifications not expressly approved by the grantee of this device could void the user's authority to operate the equipment.

§15.19

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions:

- (1) this device may not cause harmful interference, and
- (2) this device must accept any interference received, including interference that may cause undesired operation.

Return Notes

Please check the product quality immediately after purchasing the product, whether the product accessories are short or not, whether the product can be used normally (please test the product first, combine the product according to the above steps, and see if the LED light on the launcher is on the sensor), If you have any questions, please contact us immediately, apply for return to protect your rights. In addition, please cooperate with the following points:

1. Returns of goods, shall be within seven days from the date of purchase (including Saturday, national holidays, etc.), please keep the original product packaging, goods, warranty complete, together with accessories, gifts, invoices and all accompanying documents or Information returned at the same time, if the short parts, packaging damage or man-made damage to the goods, will not be able to provide return.
2. Send the issue of the return, the consumer is required to bear the return shipping costs and derivatives.
3. Based on health considerations, if you have used the product, can not be returned after the sale of goods, are unable to provide return.

Product warranty book

No : _____

Name		TEL	
Address			
Product	Enuresis Alarm	Date of purchase	
Confirmation of business		Issuing a card	

★Service guarantee content :

1. Within one year from the date of purchase, our company will be repaired free of charge in case of malfunction in normal use.
2. To be sent by mail to repair, the shipping costs back and forth by the customer's own responsibility.
3. Although the warranty period, if the following conditions are not free repair :
 - (1) Failed to produce a guarantee.
 - (2) Improper use of human factors or unauthorized modification.
4. This card without the dealer seal to prove, and can not be provided proof of purchase will not provide warranty.



I-DING MEDICAL EQUIPMENT CO., LTD

Company address : No. 66, Pizitou Rd., Zuoying Dist., Kaohsiung City 813, Taiwan (R.O.C.)

Factory address : 8F.-2, No. 288-7, Xinya Rd., Qianzhen Dist., Kaohsiung City 806, Taiwan (R.O.C.)

Tel : 07-5818679

Fax : 07-5856787

Web : www.iding.url.tw

mail : mail@iding.url.tw

★Please take good care of this product warranty to ensure your rights★