



Health Equipment Manufacturer . Human Body Data Service Providers

Beijing Tomboh Science and Technology Co.,Ltd

Made in China

Website: www.tomboh.com

Service Line: 400-808-2602

Address: Room 101, Block A, No.10 Xinghuo Road, Fengtai District, Beijing.

The final interpretation of the upgrades, alterations and trademarks involved
in the retention of the product by Tomboh



ShengWa 1.0 Product Manual

Beijing Tomboh Science and Technology Co.,Ltd

Health Equipment Manufacturer . Human Body Data Service Providers

名称：圣娃温率宝

Name: ShengWa Thermometer

型号：T16WXR01

Model: T16WXR01

公司：北京汤博科技有限责任公司

Company: Beijing Tomboh Science and Technology Co.,Ltd.

网址：www.tomboh.com

Website: www.tomboh.com

电话：86-010-4008082602

Telephone: 86-010-4008082602

地址：北京市丰台区星火路10号A座101室

Address: Room 101, Block A, No.10 Xinghuo Road, Fengtai
District, Beijing.



FCC statement

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

Caution: Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

NOTE: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio

frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

低功率电路辐射性电路管理办法

第十二条

经过认证合格的低功率射频电路，未经许可，公司或个人均不得擅自更改频率、加大功率或改变设计特性与功能。

第十四条


低功率射频电路之使用不得影响航空安全及干扰合法通信；发现有干扰现象时，应改善至无干扰时才可继续使用。合法通信，指依据电信法规定作业的无线作业通信。低功率射频电路需兼容合法通信或工业、科学及医疗用电波辐射性电路设备之干扰。



IC statement



This device complies with Industry Canada license-exempt RSS standard(s).

Operation is subject to the following two conditions: (1) this device may not cause interference, and (2) this device must accept any interference, including interference that may cause undesired operation of the device.

Le présent appareil est conforme aux CNR d'Industrie Canada applicables aux appareils radio exempts de licence. L'exploitation est autorisée aux deux conditions suivantes : (1) l'appareil ne doit pas produire de brouillage, et (2) l'utilisateur de l'appareil doit accepter tout brouillage radioélectrique subi, même si le brouillage est susceptible d'en compromettre le fonctionnement.

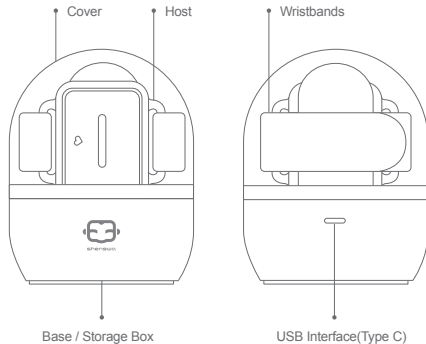


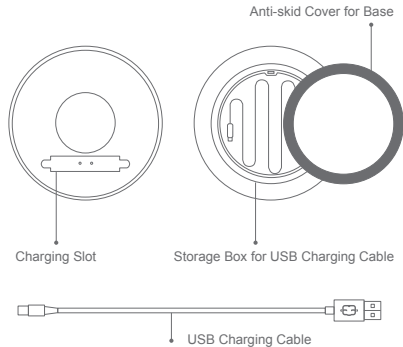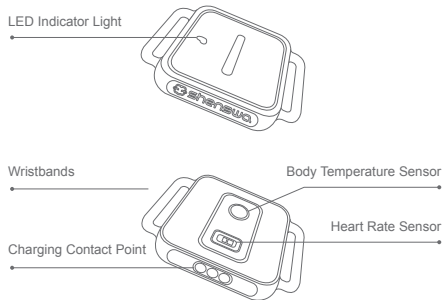

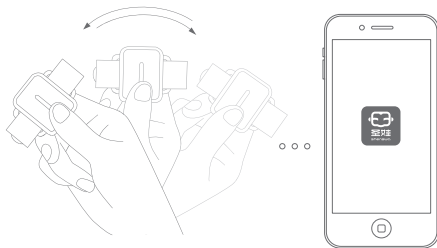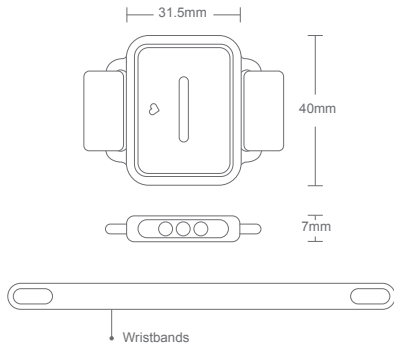

Power On by Shaking
(Disconnect from APP for 3 minutes
and power off automatically)


Download and Open APP
Connecting ShengWa
Thermometer-Device

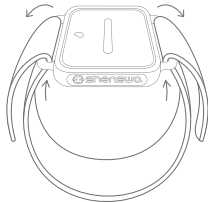


- 1、 The device is weared on the inside of the upper arm
- 2、 The wristbands is tightened properly



- 1、 Power On:The green light flashes three times
(Insufficient Power,red light flashes 3 times)
- 2、 Binding:Red,yellow,green light alternately flashing
- 3、 Charging:Red light flashing
- 4、 Full Charging:Green light is always on

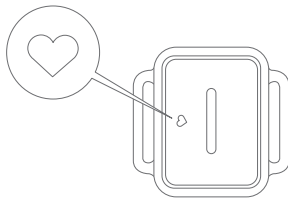

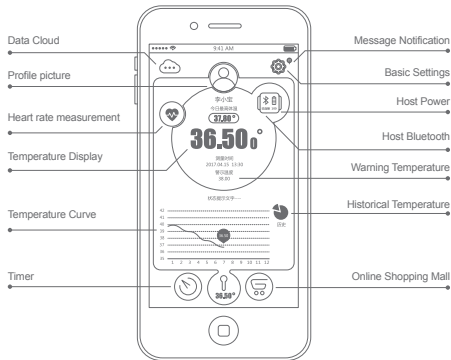

Data Cloud: Data cloud connection status (Upload and download human data status)

Profile Picture: Current user, Click the picture to edit or switch other users

Heart rate measurement: Click to switch to the heart rate measurement interface

Temperature Display: The real-time temperature of the user being measured

Temperature Curve: Real-time temperature change within 2 hours

Timer: Medication, eating, drinking, learning, exercise, entertainment and other arrangements for the time

Message Notification: User message reminder

Basic Settings: Basic functions to view and operate, set

Host Power: Display battery power


Host Bluetooth: Bluetooth connection status with mobile phone

Warning Temperature: Temperature of overrun warning Value (suggest 38.0°C)

Historical Temperature: Query the history date of temperature

Online Shopping Mall: Buy products, accessories online

Note: The current measurement user must be consistent with the host user

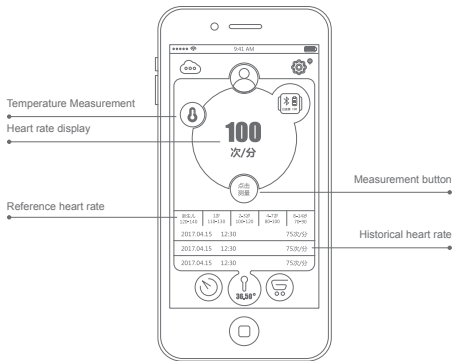

Using food grade materials, please feel free to use!

The wristbands is made of natural biological material, Skin care, breathable, sweat, can be cleaned.



Implementation of standards: GB/T21416-2008



This product meets the EU ROHS environmental requirements, CE/FCC quality certification.

Disclaimer: Although we have fully considered the safety and certification in the production process, because of the uncertainty of the child's behavior, the product requires the use of adult surveillance.



Using wireless upgrade mode,

APP is automatically upgraded when the app store publishes new apps

APP interface and operation are constantly updating and adding more features ,So that user have a better experience!

According to the People's Republic of China on the protection of consumer rights and the quality law of the People's Republic of China,Implementation of three Guarantees after-sales service.

- 1、 7 days from the date of receipt, if the failure list of the situation,confirmed by the after-sales service center,can be free for return or replacement service.
- 2、 8-11 days from the date of your receipt,if the failure in the case, confirmed by the after-saler service center, free of charge for replacement or maintenance services.
- 3、 Within 12 months from the date of your receipt, if the situation in the list of failures, confirmed by the after-sales service center, free of charge for maintenance services.

Non-warranty regulations

- 1、 Unauthorized repair,alteration, disassembly.
- 2、 Improper use of the goods accessories, tear or modify the label and anti-counterfeit labels
- 3、 More than three Guarantees of validity.
- 4、 Damage caused by force majeure.
- 5、 Failure or damage caused by human Artificially caused by failure or damage.

ShengWa-Fault List

Name	Fault	Failure and Handling
Host	Unable to Start	Less power: Please charge; Equipment damage: Contact manufacturers maintenance.
	Temperature and heart rate function failure	Sensor damage: Contact manufacturers maintenance; Not powered on.
	The indicator function failure	LED damage: Contact manufacturers maintenance; Less power: Please Charge.
	Connection phone failed	Phone Bluetooth is not open: Turn on the Bluetooth function; Distance beyond communication range; No device binding; damage: Contact manufacturers maintenance.
	Alert function is not normal	No ringtones: Turn on or adjust the volume or Update the program.
	Data cannot be displayed properly	Not powered on; Distance beyond communication range; The temperature is too high or too low.
	Battery Failure	After fully charged, the device cannot be used for 8 hours, please replace the battery.
	Damage before receipt	Contact logistics or shipping manufacturers.
Charging Box	Charging Box Failure	Check the interface; Check the charging electric shock; Damage: Contact manufacturers maintenance.
Charging Cable	Charging Cable Failure	Check the cable interface; Check the charger; Damage: Contact manufacturers maintenance.

Parts Name	Hazardous Substance					
	(Pb)	(Hg)	(Cd)	(Cr(VI))	(PBB)	(PBDE)
Printed circuit board	X	O	O	O	O	O
Battery	O	O	O	O	O	O
Shell	O	O	O	O	O	O
Accessories	O	O	O	O	O	O
Charging Cable	X	O	O	O	O	O

This table is based on the provisions of SJ/T 11364

O: Indicating that the content of the hazardous substance in all materials is below the requirements specified in GB/T 2572.

X: Indicates that the content of the harmful substance in at least one of the materials exceeds the requirements specified in GB/T 26572.



This product needs to be used under the supervision of parents!!!

- 1、The data of this product is not based on the diagnosis of human disease, only as a supplementary reference.
- 2、Body temperature will be affected by ambient temperature, skin differences, wear tightness, the length of time and so on. There may be difference in measured values. We are trying to develop a temperature compensation algorithm to ensure the accuracy of the temperature.
- 3、When measuring, wear the appropriate position to tighten, otherwise the measurement results will fluctuate.
- 4、The accuracy of temperature will be affected when the ambient temperature changes greatly , such as from room to outside.
- 5、Temperature sensor is a precision component, the temperature change is slow, take 5 to 8 minutes to stabilize.
- 6、The device automatically enters the sleep state in the absence of operation, power on by shaking.

- 7、Please adjust the over-temperature warning to indicate that the sound is large enough to avoid being missed due to the close or the volume is too small.
- 8、In order to avoid false warning notification, cannot exit APP, Bluetooth communication with the device cannot be interrupted.
- 9、Check if there is sufficient power before use to ensure that the equipment is working properly
- 10、After use, with a clean cotton or cotton swab dipped in a little alcohol to wipe the surface disinfection.
- 11、If you do not use device within six months, please charge to keep the equipment is intact.



Shengwa APP
QR code



Shengwa Healthy
QR code



Shengwa Healthy microblogging
QR code

Please download and install APP, you can search ShengWa in the official website, Android application store, Apple APP Store, or scan APP QR code, or scan QR code in the lower part of the charging box.

Thank you for reading, buying, using the ShengWa product!