



智能护眼 睛彩视界

Intelligent eye care vision

便携式智能眼部按摩仪使用说明

Portable intelligent eye massager



一、适用范围

Scope of application

- 【1】 缓解眼疲劳，预防近视

relieve eye fatigue and prevent myopia

- 【2】 美容祛黑，活化眼肌肤

beauty remove black, activate eye skin

- 【3】 改善睡眠，抑制神经衰弱

improve sleep, inhibit neurasthenia

- 【4】 仿真按摩太阳穴，缓解头疼

massage the temples to relieve headache

- 【5】 脉冲振动，疏通眼部经络

pulse vibration, dredge the eye meridian

- 【6】 远红外热敷，促进眼部血液循环

far infrared hot compress, promote eye blood circulation

- 【7】 恢复睫状肌弹性，提升视力

to restore ciliary muscle elasticity, improve eyesight

- 【8】 内置磁石促进细胞代谢，活化细胞

the built-in magnet promotes cell metabolism and activates cells

二：适用人群

Intended for

- 上班族 久坐不动，血液流通慢，脑供血受阻，视力疲劳严重，长时间对着屏幕，容易导致眼部干涉，视力下降。

Office workers sedentary, blood circulation is slow, the brain blood supply blocked, visual fatigue serious, long time toward the screen, easily lead to eye interference, visual acuity decline

- 熬夜族 经常熬夜，眼睛本身超负荷工作，长时间用眼，黑眼圈严重、容易导致眼睛干涩、胀疼、疼痛、视疲劳加剧，造成视力下降。

Stay up late the family often stay up late, the eye itself overload work, long time with the eye, dark circles are serious, easily lead to dry eyes, swelling pain, pain, visual fatigue increased, resulting in decreased vision.

- 司机族 驾驶员长时间，过度用眼，熬夜会导致眼部视觉过度疲劳。

Drivers long time, excessive use of eyes, stay up late will lead to excessive visual fatigue of the eye.

- 低头族 长时间使用手机，及对着手机屏幕，会导致视力疲劳及下降

bow heads, time to use the phone, and toward the phone screen, will lead to visual fatigue and decline

- 学生族 长期的学习及用眼，会使视觉疲劳，视力下降

Students long-term learning and use of eyes, will cause visual fatigue, vision loss

- 爱美族 本产品对美容祛黑眼圈有很好效果

The beauty of this product for the beauty of black eye circles have a very good effect

- 失眠族 容易失眠、睡眠质量不好的群体

insomnia group prone to insomnia, sleep quality is not good

- 近视族 近视或者高度近视群体

A group of myopia or high myopia

- 产妇族 产后或产中群体


Maternity or postpartum group

- 懒人族 不注意眼部保健群体

Pay no attention to eye care groups

三：使用功能说明

Use function description

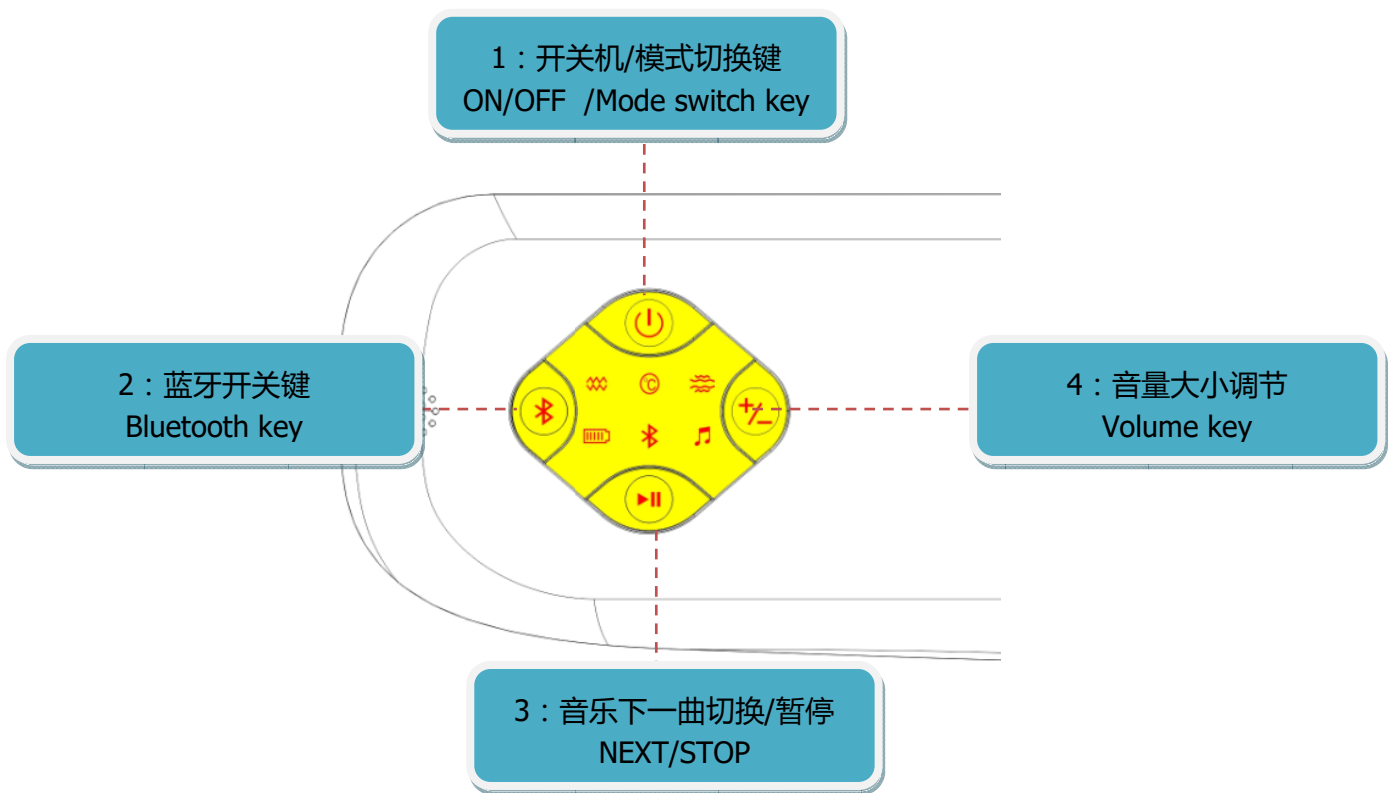

1: 开关机键 Switch key

长按 2 秒进行开关机。短按 1 秒进行模式切换。

Switch the machine for 2 seconds long. Switch mode short by 1 second.

2: 蓝牙键 Bluetooth key

长按蓝牙开启/关闭并提示音

Long press Bluetooth to turn on / off and prompt tone.

3: 下一曲键 Next key


按 1 秒下一曲音乐，长按 2 秒音乐暂停

Press one second to play the next piece of music. Long press 2 seconds, music pause。

4: 音量键 Volume key

音量大小分 4 档调节，开机默认 2 档。可以按小到大 1—4 档。每按 1 次 1 档。

The volume size is divided into 4 files, and the boot defaults 2 files. You can press small to large 1 - 4 files. Each press 1 times, 1 files

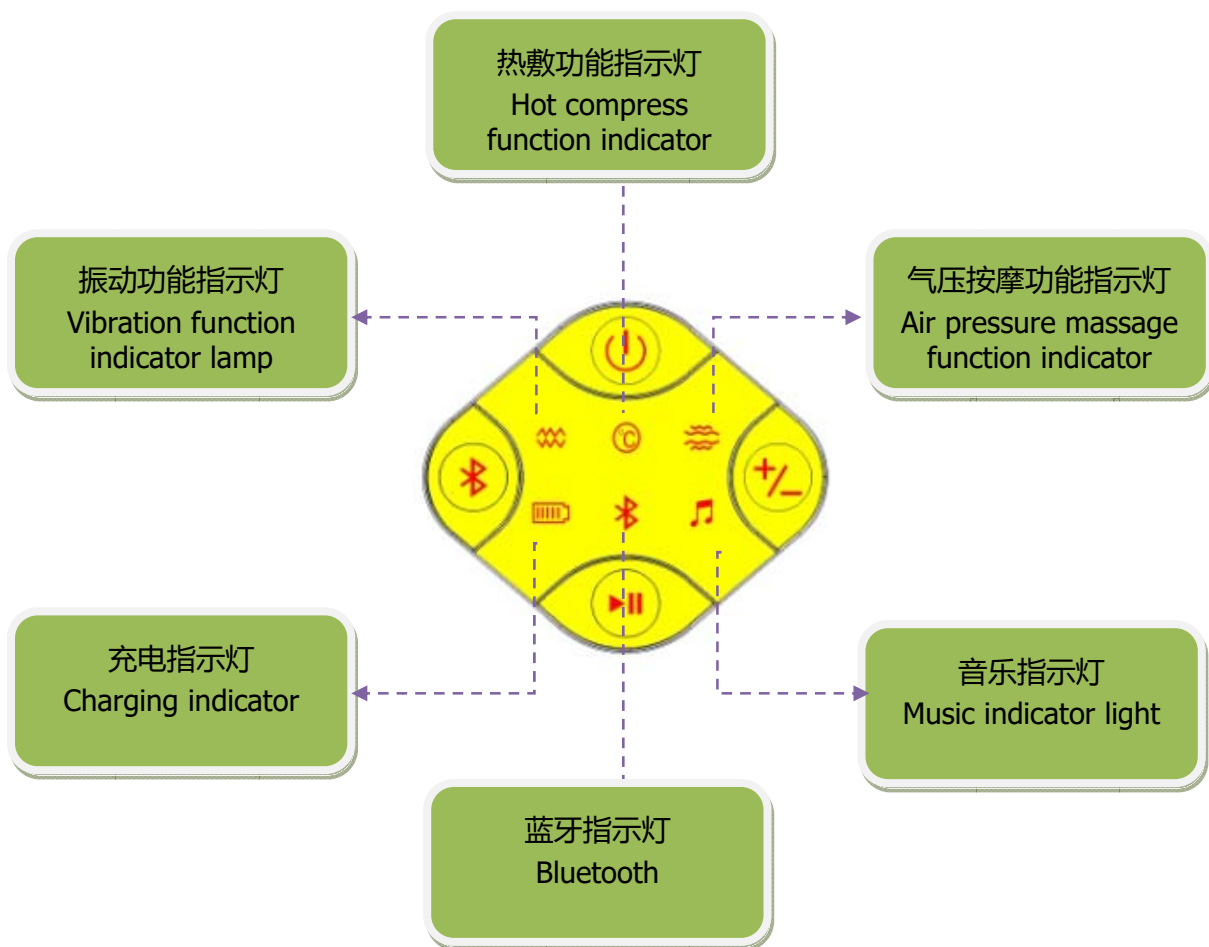


喇叭出音孔
Horn sound hole



可调节头带扣
Adjustable headband buckle

四：五大智能功能模式

Five modes of intelligent function

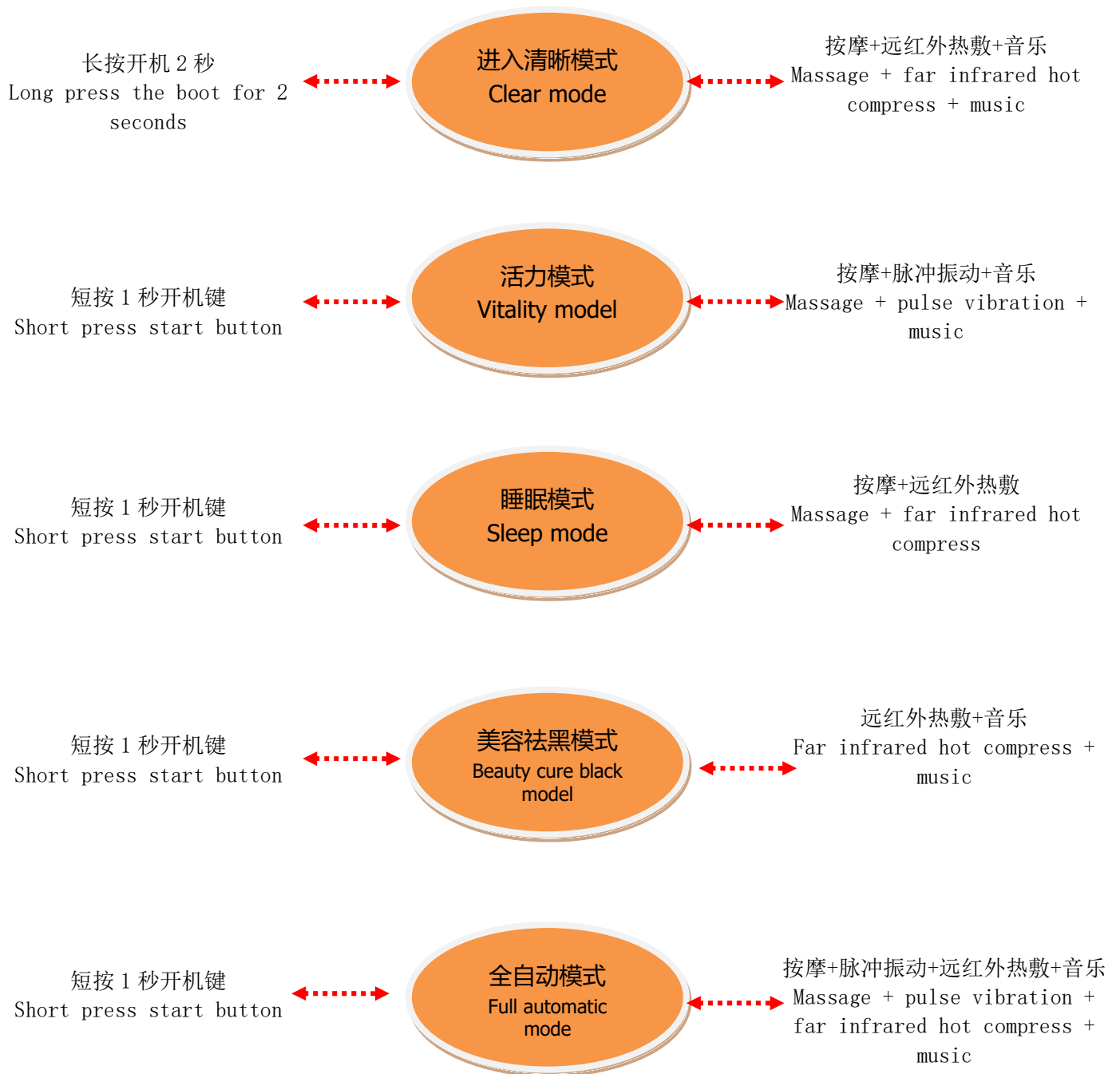

温馨提示：每个模式默认 15 分钟自动关机, 当电池电压过低会出现提示音。

Warm prompt: each mode defaults to 15 minutes automatic shutdown. A warning tone occurs when the battery voltage is too low.

五：使用注意事项

Notice of use

- 1) : 使用前，请先为本产品进行充电 2 小时。产品出厂前不会满电。

Please charge the product for 2 hours before use. The product won't be full until it leaves the factory.

- 2) : 禁止边充电边使用，充电时间不宜过长，过长充电会影响电池寿命。

Prohibit charging side use, charging time should not be too long, too long charge will affect battery life

- 3) : 使用前请摘下眼镜或者隐形眼镜。

Please remove your glasses or contact lenses before use.

- 4) : 使用前请先将头带调整到最舒适位置，面罩对准双眼带上本产品

Before use, please adjust the headband to the most comfortable position. The mask is aligned with the product

- 5) : 如果全家人共同使用，建议每使用一次用干净软纸巾垫绒布上，更干净

If the whole family is used together, it is recommended to use a clean soft paper pad every time

- 6) : 使用该产品建议采用坐姿或者卧姿，使产品更贴合眼部，效果更佳

The use of the product by sitting or prone position, make the product more fit for the eye, the effect is better

- 7) : 使用完该产品后，请闭目养神 3—5 分钟，让眼部穴位经络放松舒展，视野更清晰

After using this product, please close your eyes for 3 - 5 minutes, make eye acupuncture meridians to relax stretch, clearer vision

- 8) : 眼部刚做完手术禁止使用，术后在医生指导下使用

The eyes have just been operated on and are prohibited from being used. The doctor is under the guidance of the doctor

- 9) : 使用本产品前，需要连接蓝牙设备，请先连上蓝牙设备在佩戴使用，禁止在使用过程中进行走动或开车。

Before using this product, you need to connect the Bluetooth device. Please connect the Bluetooth device and use it. It is forbidden to walk or drive in the process of using.

- 10) : 该产品禁止水洗或者暴晒，以免影响电子部件使用寿命。

This product banned washing or explosion so as not to affect the electronic component Sai, service life.

六：售后服务及支持

After sales service and support

YAOEOAY 智能眼部按摩仪产品保修条例

YAOEOAY intelligent eye massager product warranty regulations

本产品售后服务严格依据《中华人民共和国消费者权益保护法》《中华人民共和国产品质量法》实行售后三包服务，服务内容如下：

The after-sale service of this product is strictly based on the <<law of the People's Republic of China on the protection of consumers' rights and interests >>and the <<quality of domestic products of the people's Republic of China>>, and the after-sale Three Guarantees service is as follows:

1: 自您签收日期起 7 日内，本产品出现《产品性能故障表》所列故障的情况，经由香港天杰国际有限公司售后技术服务中心检查确定，可以享受免费退货或者换货服务。

Since your receipt within 7 days from the date of this product, "fault products listed in table" performance fault situation, determined via Hongkong Tian Jie International Limited Customer Service Technical Service Center, you can enjoy a free return or replacement service.

2: 自您签收日期起 8—30 日内，本产品出现《产品性能故障表》所列故障的情况，经由香港天杰国际有限公司售后技术服务中心检查确定，可以享受免费换货或者维修服务。

Since you sign 8 within 30 days from the date of this product, "fault products listed in table" performance fault situation, determined via Hongkong Tian Jie International Limited Customer Service Technical Service Center, you can enjoy a free replacement or repair service.

3: 自您签收日期起 12 个月内，本产品出现《产品性能故障表》所列故障的情况，经由香港天杰国际有限公司售后技术服务中心检查确定，可以享受免费维修服务。

Since you sign within 12 months from the date of this product, "fault products listed in table" performance fault situation, determined via Hongkong Tian Jie International Limited Customer Service Technical Service Center to check, can enjoy free maintenance service.

附：产品性能故障表

Product performance fault table

产品名称 Product name	性能故障 Performance failure	备注 Remarks
眼部按摩仪整机 Eye massager	1: 产品无法开机 The product cannot boot	
	2: 产品无法连接蓝牙或者不稳定 The product cannot be connected to Bluetooth or unstable	
	3: 产品指示灯不亮 The product indicator is not on	
	4: 产品无法连接电脑 The product cannot connect to the computer	扩展内存版本才可以下载 Extended memory version before you can download
	5: 产品无法播放音乐 The product cannot play music	
	6: 产品按摩气囊或者气压不正常 Product massage air sac or abnormal air pressure	
	7: 产品马达振动异常 Abnormal motor vibration of the product	
	8: 电磁阀不放气, 或者不工作 The solenoid valve does not work	
	9: 电池破损或者漏液 Battery damaged or leaking	
	10: 发热片破损或者不发热 The heating sheet is broken or not hot	
	11: 整机外壳损坏或破损 Damaged or damaged outer casing	非人为损坏 Not man-made damage
	12: 头带脱落 Headband abscission	非人为损坏 Not man-made damage
	13: 无法充电 Cannot charge	
	14: 主板功能程序紊乱 Main board function program disorder	
	15: 按键脱落 Button drop	
	16: 气囊破损 Airbag breakage	
	17: 内置配件少数 Inadequate accessories	
	18: 其他出厂硬件导致功能无法使用 Other hardware problems cause functionality to be unavailable	

FCC Warning Statement

Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment. This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions:

- (1) This device may not cause harmful interference, and
- (2) This device must accept any interference received, including interference that may cause undesired operation.

七：非保修条例

Non warranty regulation

1: 未经授权的维修, 误用、碰撞、疏忽、滥用、进液、事故、改动、不正确使用产品、或者撕毁、涂改标贴、防伪标记

Unauthorized maintenance, misuse, collision, omission, misuse, entry, accident, alteration, incorrect use of the product, or tear up, alter, mark, anti fake mark

2: 已经超过三包保修期限

More than three packages warranty period has been exceeded

3: 因不可抗力造成的损坏

Damage due to force majeure

4: 不符合产品性能表所列的性能故障

Failure to comply with performance faults listed in the product performance table

5: 人为原因导致产品产生《产品性能故障表》所列故障的情况

Human factors lead to product failure in the product performance fault table



产品名称: 便携式智能眼部按摩仪

Item name: portable intelligent eye massager

产品型号: YEY-2.0

Item number: YEY-2.0

电池容量: 1100mAh

Battery capacity: 1100mAh

额定电压: 5V/1A

Rated voltage: 5V/1A

配件内容: 整机、使用说明书、保修卡、合格证、绒布产品袋

Accessories: complete machine, instruction manual, warranty card, certificate, flannelette bag

客户热线: 4008358005

Customer hotline: 4008358005

广东省东莞市长安镇涌头社区宏业中路4号

Changan town of Guangdong city of Dongguan province chongtou village Hongye Road No. 4

香港天杰国际有限公司

Hongkong Tian Jie International Co., LTD

东莞市天杰实业有限公司

Dongguan Tian Jie Industrial Co., Ltd.

所有技术信息 以实体性能为准 香港天杰国际有限公司保留最终解释权

All technical information shall be subject to substantial performance. Hongkong Tian Jie International Co., Ltd. reserves the right of final interpretation