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<FeMon(피몬) 스마트 전자체온계>

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1. Introduction

<FeMon Smart Thermometer>

1) Real-time temperature measurement and recording via app

Through the link the app, you can check your body temperature easy and conveniently.

Even if the internet is disconnected, it will be saved in the internal memory of the device and the data will be automatically updated when the app is reconnected.

2) Setting alarm

You can set the alarm temperature you want to receive notifications, you will be notified when the set temperature is reached.

3) Low power Bluetooth communication(72 hours + a)

It is designed low power control, ensures continuous operating time of over 72 hours(ISO/IEEE 11073 Personal Health data standards)

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4) Safety and hygiene(Banana type patch)

Adhesion, compound harmless human body has been tested, user can use with confidence.

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3. Instruction

Please read this user manual carefully before using a patch.

1) Before use

- ① After user stabilize the condition, please remove the moisture from the armpit area.
- ② Before running the app, please Enable Bluetooth and location services on user phone.

※ Please check the battery level of user phone.

If your phone is turned off due to battery, the FeMon app will also shut down.

If the app is shut down from the connection between the Patch and the app, it will be saved in the internal memory of the device and the data will be automatically updated when the app is reconnected.

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(Power On : LED flashing)

② App Download

You can install FeMon apps for iOS, Android.

③ App and Patch connection

- After start the app, accept the usage of Bluetooth.
- In the 'Scan Patch' page, 'Start' Blue button click.
- Check the patch
- If you can't search a patch, please press the 'BT' button on patch once, then try b~c course again.

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④ Pin Code

After select the patch, the Bluetooth connection request confirmation screen will appear. Please input the number on the PIN Code in the box.

⑤ Attach the patch

Attach the patch to the skin so that the temperature sensor for body temperature detection is completely attached to the armpit area.

⑥ Check a temperature

When the patch is connected, it switches to the main screen and the body temperature to be measured is displayed in the center of the main screen. The sensor will take approximately 4~5 minutes to detect the correct temperature.

⑦ Fever Check

If you press the "Fever check" button on the app, you can monitor the temperature intensively by changing the temperature measuring once every 5 seconds to the once per second for 20 seconds.

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⑧ Alert setting

- You can set notification (enable / disable / setting for high / low temperature) through the bell shape button on the app.

(Enable / Disable notification in the settings at the top right on the main screen)

- Setting temperature : 35℃ ~ 42℃
- If the body temperature exceeds the set temperature, the warning sound and measurement faster.
- Alarm : Vibration, Sound, Push message

※ The sound notification are heard only when you raise the volume of the phone.

⑨ Connect the Patch

: Connect the Patch and app

: Disconnect the Patch and app

- Press the "이미지" button, switch to the Bluetooth connect.
- If reconnection is not possible, press the BT button once on the patch. However, there should be at least one connection history between app and Patch.

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- The patch can be re-scanned using the Scan Patch button in the upper right-hand setting menu.

⑩ Graph

Press the "" button on the main screen to check the temperature change over time.

⑪ Manual Input

When you press the "" button at the bottom on the main screen, you can input time and temperature by yourself.

(Same as Graph screen manual input function)

⑫ History high temperature

Through the scroll at the bottom on the main screen, you can check the high-temperature history for 72 hours.

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When you touch a color range on the scroll bar at the bottom, the temperature of the record that is highest is displayed as pop-up message.

The data received from the patch and entered by the user are displayed.

Please check "" for detailed temperature data.

The patch and app are not connected and low temperature notifications are not displayed. The patch and app must be linked at least once for accurate time display.

⑬ Patch Off

- If you want to save battery, press the on the patch "" button for 3 seconds.

(Saving mode : LED off)

If you want to re-use a patch, press the on the patch "" button once.

(Operating mode : LED flashing)

- To completely turn off the patch, hold down the "" button for 6 seconds.

(Power Off mode : LED off)

At this time, the saved data can not be saved or restored.

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⑭ Guide

The "" button on the top-right of the app provides a description of the app's use.

⑮ Setting

User can set function through the "" button on the upper right.

※ Note : Range of Body temperature

The table shows the average of the normal temperature range. It may vary depending on the body, environment temperature, and physical activity.

Normal Temperature Range(Armpit)

Age 0 ~ 2

3 ~ 10

11 ~ 65

Over 65

4. Maintenance and Inspection

- ① If you turn on/off the patch or press the Bluetooth check switch button, remove the patch on body.

When you press the switch with the patch attached, the person to be measured may be put under pressure.

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② If your app and patch connection fail, please double-check that the following are true :

- Smartphone Bluetooth, location activation status
- Power on state of patch
- When searching patch, press Bluetooth check switch button
- 15m or less recommended distance between patch and smartphone
- If there is a wall between the patch and smartphone, the distance of use may be significantly reduced

③ Excessive patching for extended periods of time may cause skin irritation

It is recommended to check the attachment position after using the patch for 8 hours and move the attachment area if there is a problem with the skin.

④ You may not be able to beep when you use the media, such as music or video, or adjust the smartphone volume during body temperature measurement

⑤ Depending on your smartphone type and OS version, FeMon app may not work properly. Please check compatible devices.

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⑥ Be careful not to damage the sensor and keep it clean.

⑦ If you use the patch alone (offline mode) instead of linking it with your app, it may be lost in the order of past data after 24 hours.

To prevent this, at least one connection should be done before 24 hours after measurement.

5. How to keep

① Avoid direct sunlight and keep it away from dust and moisture

② Avoid storage near firearms or chemical agents.

③ Please keep the vibration and shock

④ Keep out of reach of liquids or foreign objects

⑤ Please keep in mind the steady state such as inclination, vibration, shock(including during transportation).

⑥ Usage time : The recommended total usage after opening the product is 72 hours. When not in use continuously, you can turn off the power. After use, it can be used until the battery is completely discharged or weakened.

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6. Cautions

※ Warning

- If an abnormality is found in the patch, immediately stop the operation of the device, remove the device from the body, and ask the manufacturer for inspection.
- Use a smartphone that supports Bluetooth 4.0 Low Energy standard.
- If you turn on/off the patch or press the Bluetooth check switch button, remove the patch on body. When you press the switch with the patch attached, the person to be measured may be put under pressure.

※ General caution

- ① This product is available for all ages.

However, please pay attention to the following when using infants.

- If the infant and thermometer enter the room from outside or the temperature environment moves to another place, please use it after 30 minutes or more.
- When storing the product during use, be careful to keep the adhesion of the adhesive on the back of the device so that the adhesive does not fall off.

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- Please use after calming because there is a gap between sensors on the skin that should be adhered to by the struggle of infant etc and measurement error may occur.

② Be careful not to let anyone other than you touch the patch.

③ Please contact the manufacturer if any of the following conditions occur

- If you have an abnormal skin reaction(hives, allergies, etc.)
- When there is an abnormality in the whole device
- In case of using with other medical devices, change of body temperature is remarkable.

④ Be careful not to damage this patch.

⑤ If a fever is detected in the patch, be sure to remove it from your body.

⑥ Be careful that the baby does not put the patch into your mouth or bite it.

⑦ If the fever is severe, be sure to follow your doctor's instructions

⑧ When discarding a patch, discard it as it is original

(If you fold or cut the patch, it may cause injury due to internal parts)

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※ Be careful when using together

Use with other electronic or electrical medical devices may result in reduced effectiveness, nonconformity or harmful effects of the medical device.

※ The following patients should be used with caution

- ① Patients with injury or disease at the attachment site
- ② Patient with sensitive skin
- ③ Patients using other medical devices

※ Stop using the following patients

Please contact the manufacturer if any of the following situations occur

- ① If you have an abnormal skin reaction(hives, allergies, etc.)
- ② When there is an abnormality in the whole device
- ③ In case of using in parallel with other medical equipments

※ Treatment of a product accident

In the event of an accident caused by malfunction of the product, please contact the list of Customer Support.

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Lithium coin-cell

Power consumption

Type and degree of protection
against lightning

2) Date : June, 2017

8. Customer Support

If you have any problems or need to
improve the product, please contact
the below contact information and we
will kindly consult with you.

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FCC Information to User

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

Caution

Modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

FCC Compliance Information : This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

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- Company : Life Science Technology Inc.