

將手錶充電

Charge Watch


1. 將充電座連接USB電源或電腦。
Connect magnetic charging dock with USB cable.

2. 將Heartisans 手錶取出，並放在充電座上，當正確連接時你會感覺到它攝在充電座上。
Remove Heartisans Watch from band and place onto charging dock, it will snap into place.


3. Heartisans 手錶充電時，屏幕上會顯示電量增加的圖像。
A charging battery icon appears on the display while charging.

放入手錶的方法

Insert Watch

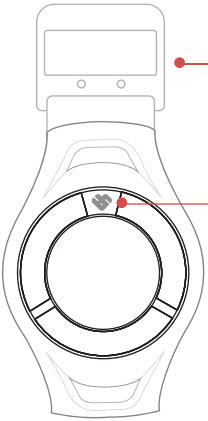


錶扣與Heartisans 圖標同時在上方，放入時從下而上推。
Place the strap over the watch, and align Heartisans logo with the buckle.

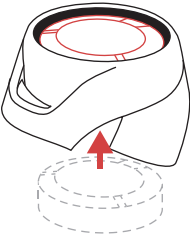

啟用手錶

Activate Watch

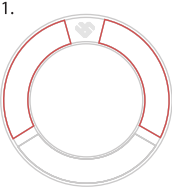1. 

手錶第一次使用時，請按左面按鈕或右面按鈕開啟手錶屏幕。
When you first use your heartisans watch, press the left or right button to start the watch.

2. 

當看到「手機」圖示（左圖），請下載 Heartisans App 並按照 App 內指示完成啟用程序。
When "Smartphone" icon appears, please download Heartisans App and follow the instructions to activate the watch.

3. 於iPhone的App Store或Android手機的Google Play搜索“Heartisans”或掃描以下QR碼。
Search “Heartisans” on App store or Google Play Or scan QR Code below.



GET IT ON
Google Play

Download on the
App Store

2017 Heartisans Ltd. All Rights Reserved.

Heartisans® and logo are trademarks for Heartisans Ltd. in Hong Kong.
Apple and the Apple logo are trademarks of Apple Inc., registered in the U.S. and other countries. App Store is a service mark of Apple Inc., registered in the U.S. and other countries. Google Play and the Google Play logo are trademarks of Google Inc.
Bluetooth logos and icons are owned by the Bluetooth SIG, Inc. and any use of such marks by Heartisans® is under license.

WWW.HEARTISANS.COM

手錶功能

Features

Heartisans 手錶是一隻多功能的健康手錶，啟用後你可以使用以下功能：
Heartisans Watch is a multifunction watch for health and wellness:



量度血壓
BLOOD PRESSURE



計步
STEPS



心電圖
ECG



卡路里消耗
CALORIES BURNT



心率
HEART RATE



時間
TIME

校正手錶

Calibration

校正手錶需要Heartisans App 以及一個經認證的手臂式血壓計，只需數分鐘就能完成整個校正程序。
Calibration requires Heartisans app and a validated blood pressure monitor. This take approximately few minutes.

第一周 First Week	每天校正一次，為期七次 once a day x 7 times
七次校正之後 After 7 times	每月校正一次 once a month

解鎖

Unlock




當「鎖」圖示出現時，請按左面按鈕或右面按鈕解鎖。
When “Lock” icon appears, press the left or right button (shown above) to unlock.

快速使用手錶

Quick Guide

返回時間顯示
Back to time

轉換功能
Switch Features

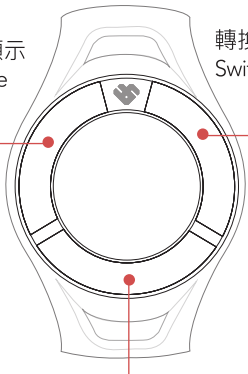

解鎖後長放30秒量度血壓及心電圖
After unlock, hold press for 30 seconds to measure blood pressure & ECG

HEARTISANS

快速使用手冊
QUICK USER GUIDE

Front

File Name: user guide-v5-op3_PMTV1

Path: Production \ Hcode

Customer: Hcode

Project Name:

Artist: Jake

Date: 2017 - 11 - 24

Color Information:

Special Treatment:



RRD Diecut B

rrd

Packaging Premedia

No.28 Huang Sha He West Road,
Xin Jiu Wei Village, Liaobu Town
Dongguan, China, 523410
E-mail: hongchao.shu@rrd.com
Tel: +86 0769 81126 525