


<p><b>CONTACTS</b></p> <p><b>Company</b> Olive Union, Inc.</p> <p><b>Homepage</b> www.oliveunion.com</p> <p><b>Email</b> office@oliveunion.com</p> <p><b>Tel.</b> + 82 - 2 - 3454 - 0701</p> <p><b>Address</b> Seocho-daero 286 Unit 706 Seocho-gu, Seoul, Republic of Korea</p> <p><b>Olive cradle is not a basic component of Olive.</b></p> <p>© 2017 by Olive Union, Inc. All rights reserved.</p>	<p><b>SPECIFICATIONS</b></p> <p><b>Product Category</b> Air Conduction Hearing Aids</p> <p><b>Model</b> OLV - 01</p> <p><b>Bluetooth</b> Version 4.1 Frequency Band 2.4GHz Operating Distance Up to 10m</p> <p><b>Audio</b> Channel 16 band Max Output 110dB Full-On Gain 35dB FRR 315Hz – 7,100Hz</p>	<p><b>CAUTION</b></p> <ol style="list-style-type: none"> <li>1. This device may cause signal interference so it is not recommended for use around sensitive tele-communications equipment.</li> <li>2. Avoid exerting physical pressure on the Olive or the charging dock such as sitting on it.</li> <li>3. Avoid exposing the Olive to water (heavy snow or rain or extreme humidity).</li> <li>4. If you are currently undergoing any hearing treatment, please consult your audiologist before using the Olive.</li> <li>5. Please consult a physician or audiologist before using the Olive if the user is a minor or suffers from any medical conditions including mental disorders.</li> <li>6. Keep it out of reach of children under 5. If any parts are swallowed, seek immediate medical attention.</li> </ol>	<p><b>WARRANTY</b></p> <p>The Olive comes with a limited warranty of one (1) year starting from the date of purchase. Unless otherwise specified, this warranty covers the hardware components of the Product as originally supplied and does not cover, or partially covers, software, consumable items, or accessories, even if packaged or sold together with the Product.</p> <p>The limited warranty only covers product defects caused by workmanship or build materials. To make a claim, contact Olive Union Customer Support at office@oliveunion.com.</p> <p>Olive Union may use rebuilt, reconditioned or new parts and components when repairing any product. Alternatively, we may replace the defective product entirely with a rebuilt, reconditioned or new Olive product.</p>									
 <p><b>User Manual</b></p>	<p><b>Usage Time</b> Charging Time 1H Total Usage Time 16H Standby Up to 10 days</p> <p><b>Power</b> Battery Rechargeable Li-ion Polymer Battery</p> <p><b>STORAGE AND CARRY</b></p> <table border="1" data-bbox="672 1114 1079 1251"> <thead> <tr> <th></th> <th>Storage</th> <th>Carry</th> </tr> </thead> <tbody> <tr> <td>Temperature</td> <td>10 – 40°C</td> <td>-20 – 60°C</td> </tr> <tr> <td>Relative Humidity</td> <td>10 – 80%</td> <td>5 – 90%</td> </tr> </tbody> </table>		Storage	Carry	Temperature	10 – 40°C	-20 – 60°C	Relative Humidity	10 – 80%	5 – 90%	<ol style="list-style-type: none"> <li>7. Do not put the hearing aid in the microwave oven.</li> <li>8. Avoid using maximum volume.</li> </ol> <p><b>WARNING</b></p> <ol style="list-style-type: none"> <li>1. If you find any skin irritation or discharge from your ear, discontinue use until discussing with your physician.</li> <li>2. The Olive contains magnetic parts. If you have a pacemaker or any other internal medical devices, please consult your physician before using the Olive.</li> <li>3. Use caution when near NMR, MRI, CT scanner, or other medical machines. Other low-energy electromagnetic machines such as anti-theft EAS will not affect the Olive but may cause temporary interference.</li> </ol>	<p><b>The limited warranty does not cover:</b></p> <ol style="list-style-type: none"> <li>1. Defects or damage resulting from accidents, neglect, misuse or abnormal use; abnormal conditions or improper storage; unusual physical, electrical or electromechanical stress.</li> <li>2. Scratches, dents and cosmetic damage, unless caused by Olive Union.</li> <li>3. Devices that have the serial number removed, defaced, damaged, altered or made illegible.</li> <li>4. Ordinary wear and tear.</li> <li>5. Defects or damage resulting from the use of the Product in conjunction or connection with accessories, products, or ancillary/peripheral equipment not furnished or approved by Olive Union.</li> <li>6. Defects or damage resulting from cellular signal reception or transmission, or viruses and other software problems introduced into the Product.</li> </ol>
	Storage	Carry										
Temperature	10 – 40°C	-20 – 60°C										
Relative Humidity	10 – 80%	5 – 90%										

**FCC Information to User**

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

**Caution**

Modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

**FCC Compliance Information** : This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation