**MagniSmarttech**

OPERATING AND MAINTENANCE MANUAL

**magniflex**<sup>®</sup>  
A small graphic element at the bottom of the Magniflex logo, consisting of a blue square with a white horizontal line and a white square to its right.

## MAGNISMARTECH ©SYSTEMS

"MagniSmartech" version (complete system) with a motorized slatted bed frame, motor control & anti-snoring functions, a vital signs acquisition system and chromotherapy

"MagniSmartech ©Mattress" version with vital signs acquisition system and chromotherapy

"MagniSmartech ©Motor" version with a motorized slatted bed frame, motor control & anti-snoring functions

## PRODUCT AND MANUFACTURER IDENTIFICATION DETAILS

<b>Manufacturer:</b>	Alessanderx S.p.A.
<b>Address:</b>	Via San Leonardo da Porto Maurizio, 24/26/28 59100 Prato (PO) - ITALIA
<b>Version:</b>	<ul style="list-style-type: none"><li>• "MagniSmartech" (complete system) with a motorized slatted bed frame, motor control &amp; anti-snoring functions, a vital signs acquisition system and chromotherapy</li><li>• "MagniSmartech ©Mattress" with vital signs acquisition system and chromotherapy</li><li>• "MagniSmartech ©Motor" with a motorized slatted bed frame, motor control &amp; anti-snoring functions</li></ul>
<b>Input voltage:</b>	AC 240 V / 100 V, $\pm 10\%$ (per tutte le versioni)
<b>Output voltage:</b>	DC 12 V for "MagniSmartech ©Mattress" version DC 29 V for motor and anti-snoring control system in versions "MagniSmartech ©Motor" and "MagniSmartech"

## **INTRODUCTION**

The following groups of people are referred to in this manual:

### **User**

Buyer(s) or person(s) authorized by the buyer(s) who is/are able to use the product in all its versions.

### **Specialized personnel**

Qualified personnel made available by the manufacturer to carry out complex tasks in particular situations, or any operation that requires a specific qualification (installation, testing/commissioning, repairs, etc.).

### **Seller**

Individual or legal entity responsible for the design and development of the product in all its versions. The seller is responsible for ensuring its conformity and placement on the market under his/its name or under his/its own brand.

## **ATTENTION**

Before activating or operating the MagniSmartech ©systems in the version you've purchased, carefully read the technical instructions contained in this manual and carefully follow them. Keep this manual and all its attachments in a place that is accessible and known to the user.

This operating and maintenance manual was drafted in compliance with the applicable ce directive for MagniSmartech ©systems and afferent to the version you have purchased, which is available at the following website:

[www.magnismartech.com](http://www.magnismartech.com)


The product declarations of conformity for MagniSmartech ©systems and afferent to the version you have purchased is available at the following website: [www.magnismartech.com](http://www.magnismartech.com)

The declarations of conformity for the parts supplied by third-party manufacturers that form the MagniSmartech ©systems and afferent to the version you have purchased are available at the following website: [www.smartech.com](http://www.smartech.com)

**Reproduction of the above documents, even partial reproduction, is prohibited without the written permission of Alessanderx S.p.A.**

Overview motorized slatted bed frame with "MagniSmartech ©Motor" system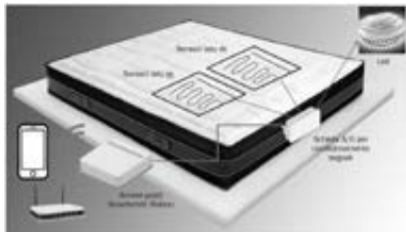

Overview of the "MagniSmartech ©Mattress" system

For further details, please refer to the operating and maintenance manual supplied with the product

Company:

Alessanderx S.p.A.

Address: Via San Leonardo da Porto Maurizio, 24/26/28 - 59100 Prato (PO) - ITALY

Phone: +39 0574 51011 Fax: +39 0574 5101235

Version:

Input voltage: AC 240 V /100 V,  $\pm 10\%$

Serial number:

Output voltage: DC 12 V for "MagniSmartech ©Mattress" version  
DC 29 V for the motors and anti-snoring control system provided in versions "MagniSmartech ©Motor" and "MagniSmartech"

Date of manufacture

## REGULATORY COMPLIANCE

### **FCC Declaration of Conformance**

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

### **FCC Class B Information**

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. Operation of this equipment in a residential area is likely to cause harmful interference in which case the user will be required to correct the interference at his own expense.

If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference

**FCC Caution**

Any changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate this equipment.

**Important Note: FCC Radiation Exposure Statement**

This equipment complies with FCC radiation exposure limits set forth for an uncontrolled environment. This equipment should be installed and operated with minimum distance 20cm between the radiator & your body. This transmitter must not be co-located or operating in conjunction with any other antenna or transmitter.

**This device complies with the following requirements.**

- FCC (Federal Communications Commission) - Part 15
- ISED (Industry Canada) RSS-102 and RSS-247 and RSS-Gen

Operation is subject to the following two conditions

1. This device may not cause harmful interference, and
2. This device must accept any interference received, including interference that may cause undesired operation of the device.

Changes or modification not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

The sensor is designed for use at least 200 mm (7.87 in.) away from the operator in normal operating conditions.

FCC ID: 2AP5WMASSMARTMOT1  
IC: 23971-MASSMARTMOT1

CAN ICES 3(B) / NMB 3(B)

# 1. SAFETY

## 1.1 FOREWORD

The technicians of Alessanderx S.p.A. will make sure that, during installation of the product, the user is duly informed about the risks of injury, the devices designed to ensure his safety and about the risks of noise emission, in accordance with the provisions of the international directives and of the legislation applicable in the country where the product is installed (see Chap. 3 Par. 3.10 "REFERENCE LAWS AND STANDARDS" and Par. 3.11 "SAFETY SIGNS"). The behaviour of the user when performing maintenance, cleaning, inspections, etc. must, at any rate, comply with the above laws, regulations and standards in force in the country where the product is installed. Before activating and operating the product, the user must be fully aware of all the controls and their features, and must also have read this manual in its entirety. The product must only be used by users who have participated in the training course organized by Alessanderx S.p.A. when installing the product and who have fully read and understand the instructions contained in this manual. Users are required to fully follow the instructions, warnings and general rules given in this manual. The tampering/unauthorized substitution of one or more parts or units of the product, and the use of accessories other than those recommended by the manufacturer, may involve the risk of injury and relieves the manufacturer from all civil and criminal liability.

## 1.2 INTENDED USE OF THE PRODUCT

### **ATTENTION**

- The MagniSmartech ©system in its purchased version has been designed for "self care" use.
- The MagniSmartech ©system in its purchased version features electrical components that can cause damage or injury if not used properly.
- The MagniSmartech ©system in its purchased version is not a medical device and is not capable of performing diagnoses and treatments, or of preventing disease.
- Consult your doctor if you suffer from diseases that may be affected by the use of the MagniSmartech ©system in its purchased version.

The MagniSmartech ©system is meant to be used in the following configuration and operating conditions: “MagniSmartech” version (complete system) with a motorized slatted bed frame, motor control & anti-snoring functions, and a vital signs acquisition system and chromotherapy. Other types of use are not compatible with the specific characteristics of the product.

### **1.3 GENERAL SAFETY RECOMMENDATIONS**

It is absolutely forbidden for unauthorized personnel to perform any operation whatsoever. Never bypass the protective devices, which are designed to protect the safety of the user. Routine and special maintenance tasks must be performed while the product is not operating and without power supplied to the system. Any repairs of the electrical systems must be carried out with the power off. Do not put hands or other foreign objects on moving parts, let alone liquid containers near the electrical parts. The user must avoid unsafe operations on the product that could compromise its balance. It is strictly prohibited to remove or disengage the safety devices.

### **1.4 CONNECTIONS**

The main electrical supply should be adequate to withstand the maximum power consumption of the machine, as indicated.

The electrical connection includes the introduction of the plug of the connection cable/s of the supplied transformer/s into two 10 A power supply outlets (depending upon the electrical system in use ) located as close as possible to the area of installation of the product.

### **1.5 EARTHING SYSTEM**

The earthing and short circuit of the metal structures the product consists of is ensured through the power supply through the low voltage transformer (the current flows toward the earthing lead-in wire of the user’s electrical system). The earthing connection must be also be performed for the low-voltage systems which are located in areas that are usually wet and very humid (if the voltage exceeds 25 V to earth for alternate current and 50 V towards earth for direct current).

## 1.6 DIAGRAM OF THE RISKY AREAS

Area "L":RIGHT/LEFT side of the motorized slatted bed frame (versions "MagniSmartech ©Motor" and "MagniSmartech")

At adjustment stage (raising/lowering of the front/rear part), the user may contact the moving parts of the motorized slatted bed frame.



The "L" areas show risks of:

- abrasion
- dragging
- crushing

External/lower parts provided with power/control connections and diagram of the MagniSmartech ©system drive motors.

All the electrical connections in the distribution system of the "MagniSmartech ©Motor" and "MagniSmartech" are provided in compliance with the harmonized and technical standards of Directives 2006/42/CE, 2014/35/UE and subsequent amendments and integrations and 2014/30/UE and subsequent amendments and integrations. Furthermore, the user must pay attention to make sure that they do NOT plug the holes of the motor control units located in the lower part of the bed, in order to ensure the greatest dissipation.


Motor control and drive unit box ( "MagniSmartech ©Motor" and "MagniSmartech" versions)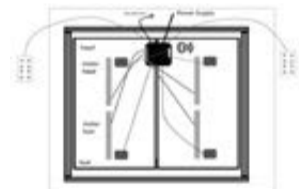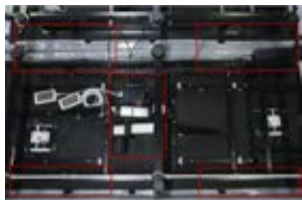

Any tampering of the electrical connections or cables by the user may generate risks of:  
electrocution  
fire

External/lower parts provided with power/control connections and diagram of the MagniSmartech ©system drive motors ("MagniSmartech ©Mattress" version)

Furthermore, the MagniSmartech ©Mattress system, contains a battery in the bed table unit, which operates in case the main power supply fails.  
The life of the battery is approximately 15 hours. If the main power supply fails, the chromotherapy system will not operate.

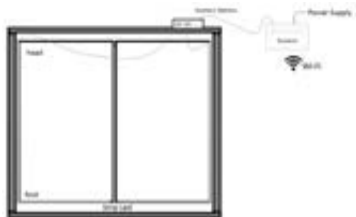## NOISE

The product was designed and developed to reduce the sound power level at the source and show the noise level within the limits of the standards in force - further details are available in the documentation that can be found on the product website” .

## SAFETY SIGNS

As related to the risks of abrasion, dragging and hand crushing when handling the motorized slatted bedframe, no safety signs are shown on the product for aesthetic reasons, but the risks are reported in this manual for cautionary and warning purposes.

Sign of electrical hazard/risk

Sign of hand crushing hazard/risk

## **1.7 STANDARD REFERENCES**

Directives concerning the safety of the product

Directive 2006/42/CE and subsequent amendments and integrations relevant to the machinery and that modifies directive 95/16/CE (recast).

Directive 2014/35/UE and subsequent amendments and integrations concerning the harmonizing of the Member State legislation relevant to the availability on the market of electrical material to be used within some voltage limits .

Directive 2014/30/UE and subsequent amendments and integrations concerning the harmonizing of the Member State legislation relevant to electro-magnetic compatibility (recast).

## **1.8 DIRECTIVES AND DECISIONS CONCERNING THE PROTECTION OF THE ENVIRONMENT**

Directive 2012/19/EU on waste electrical and electronic equipment (WEEE)

Directive 2011/65/UE on the restriction of the use of certain hazardous substances in electrical and electronic equipment.

Decision of the Commission 2000/532/CE and subsequent amendments and integrations that supersedes decision 94/3/CE, which defines a list of waste in compliance with article 1, letter a), of directive 75/442/CEE of the Council related to waste and decisions 94/904/CE of the Council, which defines a list of hazardous waste in compliance with article 1, section 4, of directive 91/689/CEE of the council relevant to hazardous waste.

## **1.9 SAFETY WARNINGS FOR THE USER**

### **ATTENTION**

- Before operating the MagniSmartech ©systems in the purchased version, we suggest removing the packages from the areas where the system is to be installed in order to prevent any potential risks for children.
- Children or animals must not play or stay in close proximity to the moving parts of the bed.
- In order to prevent potential hazards from the mechanical moving parts, the children cannot use the system without the supervision of an adult.

The most important part of the safety warning is to prevent damage to people and/or things.

- Use the motorized slatted bedframe provided with the "MagniSmartech ©Motor" and "MagniSmartech" versions only if in a good state and only by trained personnel.
- In case of extended use of the motorized slatted bedframe supplied with the "MagniSmartech ©Motor" and "MagniSmartech" versions, after a certain period of time (we recommend at least once a year), we recommend performing an in-depth check to verify its functionality and no visible damage.
- Do not connect multiple outlets underneath the motorized slatted bedframe provided with the "MagniSmartech ©Motor" and "MagniSmartech" versions.
- In case of adjustments, make sure that there are no hindrances such as furniture or ceiling inclines.
- In case of use of external electrical components, such as - for instance - reading lights, make sure that the power supply cable does not entangle with the moving parts of the motorized slatted bedframe supplied with the "MagniSmartech ©Motor" or "MagniSmartech", or that it does not get damaged.
- If you suspect any presence of damage or inappropriate operation, put the product out of service by disconnecting the power supply cable/s.
- The power supply cable/s must be set so that it is not dragged away, walked on or damaged by moving parts during its operation.
- Regularly check the power supply cables by checking for any mechanical damage (abrasions, uncovered wires, bending, dents, etc.) and - in particular - in case of mechanical overload (e.g. crushing motorized slatted bedframe) or after any change of position of the same motorized slatted bedframe supplied with the "MagniSmartech ©Motor" and "MagniSmartech" versions before plugging and regularly during the operations by the user.
- Make sure that the low voltage connection of the transformer/s is not damaged by the movement of the motorized slatted bedframe supplied with the "MagniSmartech ©Motor" and "MagniSmartech" versions.
- Do not use the free room underneath the motorized slatted bedframe provided with the "MagniSmartech ©Motor" and "MagniSmartech" versions as a storage and/or deposit area.
- When not used, make sure that the push-button board cannot fall inadvertently. Make sure the push-button board is out of children's reach in order to protect them from any accidental electrical injuries.
- Avoid locating vessels containing liquids in close proximity to the control equipment and/or electrical connections.

- The bedframe drive motors are not designed for continuous use. The full functionality and lifespan expected of the system requires that the motors are not used continually for more than 5 minutes every 30 minutes or with a duty cycle greater than 15%.
- Never cover the warning labels by hooks or bars.
- Do not open the device or try to disassemble it.

#### **1.9.1 PRECAUTIONS RELEVANT TO THE BATTERY**

- The MagniSmartech ©Mattress system, contains a battery in the bed table unit, which operates in case the main power supply fails.
- The battery charger included in the system packaging was selected to charge the Li-Po battery in safe conditions.

#### **WARNING: CAREFULLY FOLLOW THE INSTRUCTIONS AND WARNINGS ENCLOSED.**

- The inappropriate use of the Li-Po batteries may cause injuries to people and/or damage to things.
- The user takes all off the risk associated with the installation, charge and use of the enclosed Li-Po battery.
- A variation of the MagniSmartech ©Mattress system envelope is visible while it charges. Cancel the charge by unplugging the transformer and turning the device off.
- During transport or temporary storage, the temperature of the battery must remain between 5 and 50° C. Do not store the MagniSmartech ©Mattress system in the car or under direct sunlight.
- If kept inside an overheated vehicle, the battery may experience damage or even catch fire.
- NEVER USE a different power supplier than the supplied one. The use of different or non-compatible transformers may cause fires, injuries to people and/or damage to things.
- The system contained in the product and battery may damage the environment and/or the health of the battery if managed and disposed of inappropriately.
- Do not dispose of the MagniSmartech ©system in a fire, the battery may blow.
- Disconnect the power of the MagniSmartech ©system if it gets too hot.
- The MagniSmartech ©system contains a battery that the user cannot replace.
- The opening or the attempt to open the product forfeits the warranty and may cause of safety hazard.
- Only use the recharging cable supplied with the product, when charging the battery.

## 2. INSTRUCTIONS FOR INSTALLATION AND START-UP

### 2.1 DESCRIPTION OF THE ASSEMBLY SEQUENCE

#### 2.1.1 ARRIVAL OF THE PRODUCT TO THE CUSTOMER'S LOCATION

Upon arrival of the product, it is recommended that the customer:

- 1) check that the material delivered matches the relevant bill of lading
- 2) check the integrity of the packages and their contents
- 3) store the packages in a humidity-free, clean environment that is far from any corrosion agents.

#### 2.1.2 PRELIMINARY CHECKS AND PREPARATION OF THE INSTALLATION

Once at its destination, the product is installed and tested under the direct control of Alessanderx S.p.A specialized personnel.

Checking operations are performed prior to installation in compliance with the modalities reported hereinunder:

- 1) check the components to be assembled: ascertain the presence and integrity of all the material that the product consists of in order to prevent useless, costly losses in time of the subsequent assembly stages
- 2) ascertain the professionalism of the personnel involved in the installation operations, confirm that they have the basic technical background and experience for assembly
- 3) presence of the installer: during the assembly stages, the user must be present to receive basic knowledge of routine maintenance.

The list of the components included in the MagniSmartech ©systems is as follows:

“MagniSmartech” version (complete system) with a motorized slatted bed frame, motor control & anti-snoring functions, a vital signs acquisition system and chromotherapy

- transformer
- transformer
- MagniSmartech ©Station
- MagniSmartech ©Mattress
- ethernet data cable
- LED cable LEFT and RIGHT
- ethernet remote cable LEFT
- ethernet remote cable RIGHT
- microphone
- double mattress
- push-button board
- motorized slatted bedframe including drive motor and massage system (in fixed mode)

### 2.1.3 INSTALLER'S GUIDE FOR COMPONENT WIRING

"MAGNISMARTECH ©MATTRESS" VERSION WITH A VITAL SIGNS ACQUISITION SYSTEM AND CHROMOTHERAPY

1. Connect the MagniSmartech ©Station to the MagniSmartech ©Mattress through the ethernet data cable

2. Connect the MagniSmartech ©Mattress to the LEFT and RIGHT LED cables.

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

4. Connect the 12 V transformer to the MagniSmartech ©Station  
Connect 12 V transformer to the 10 A power outlet

## "MAGNISMARTECH ©MOTOR" VERSION WITH A MOTORIZED SLATTED BED FRAME, MOTOR CONTROL & ANTI-SNORING FUNCTIONS

1. Make sure that all of the connection cables on the motors and on the vibrating systems are connected as appropriate to the MagniSmartech ©Motor by means of the color verification systems .
2. Connect the MagniSmartech ©Motor to the LEFT and RIGHT ethernet remote cables.
3. Connect the two push-button boards to the two ethernet remote cables. The position has no influence.
4. Connect the microphone to the MagniSmartech ©Motor.
5. Detect the optimum microphone position by accounting for the function of the anti-snoring system.
6. Connect the transformer to the MagniSmartech ©Motor.
7. Connect the transformer to the 10 A power outlet and make sure that the LED lights up.

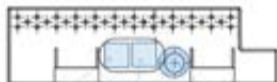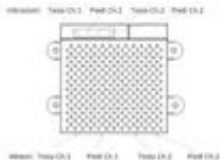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#### **2.1.4 USER'S GUIDE FOR THE FIRST POWER UP**

##### **"MAGNISMARTECH ©MATTRESS" VERSION WITH VITAL SIGNS ACQUISITION SYSTEM AND CHROMOTHERAPY**

1. Press the "A" button located in the lower part of the MagniSmartech ©Station. Press the button for approximately 4 seconds or until the relevant status LED light lights up and turns green.
2. Wait 30 seconds for the system to start - the mobile phone holder should light up.
3. Press the "B" button located in the lower part of the MagniSmartech ©Station. Press the button for approximately 5 seconds or until the relevant status LED light lights up in light blue. Then, after switching it off, it will slowly flash in an amber (orange) color.
4. Download the "MagniSmartech" APP from the AppStore (soon available for Android).
5. Choose among the available wifi networks on the mobile phone. Select the network that has an SSID with the item "Smartech ", followed by an alphanumeric series.
6. Through the APP, follow the connection instructions to the system and LogIn with your registered user credentials. For further details, consult the following website: [www.magnismartech.com](http://www.magnismartech.com)
7. If the user connects the "MagniSmartech ©Mattress" system to their own wifi network, the "B" button's reference status LED light located on the lower part of the MagniSmartech ©Station, will shift from a slow flashing amber color to a slow flashing green color, which indicates that the connection to the remote server is correct.
8. To reset the connection and take the system back to the access point mode, press the "B" button located in the lower part of the MagniSmartech ©Station, press it down for approximately 5 seconds or until the relevant status LED light lights up in light blue and then, after switching off, slowly flashes in an amber (orange) color.

##### **"MAGNISMARTECH ©MOTOR" VERSION WITH A MOTORIZED SLATTED BED FRAME, MOTOR CONTROL & ANTI-SNORING FUNCTIONS**

1. Make sure that the LED light located on the transformer is lit, along with the green light, which can be seen under the bedframe of the MagniSmartech ©Motor.
2. Download the "MagniSmartech" APP from the AppStore (soon available for Android)
3. Through the APP, follow the connection instructions to the system and perform the LogIn with your registered user credentials.

4. Turn on the Bluetooth on your device. The system is based on BLE 4.0 technology. Make your Smartphone visible to the other devices and proceed with pairing it to the other BLE. This will be performed automatically.
5. Make sure that the system confirms the connection between the Smartphone and the MagniSmartech ©Motor of the purposely-allocated APP screen.
6. Check the appropriate operation of the push-button boards.
7. If you wish to shift to the automatic operating screen (“Program”), the first time you have to wait for the system to automatically perform the self-calibration (the flat program starts automatically and you need to wait until the buttons go back to rest mode - the max. waiting time for this stage is approximately 30 seconds). If this procedure is not complied with, the location of the automatic programs might be inaccurate.
8. You can activate the anti-snoring function through the App or by means of the purposely-allocated button on the button’s board that is relevant to the side of the bed where you intend to intervene. In both cases, the orange LED light on the button’s board will light up to confirm that the anti-snoring system is operational.
9. Once triggered, the anti-snoring system will take the bedframe back to the original position after 60 seconds.

## 3. OPERATION DESCRIPTION

### 3.1 PRODUCT DESCRIPTION

As already described, the MagniSmartech ©systems are available in the following three configurations and conditions of use:

- “MagniSmartech” version (complete system) with motorized slatted bed frame, motor control & anti-snoring functions, a vital signs acquisition system and chromotherapy
- “MagniSmartech ©Mattress” version with a vital signs acquisition system and chromotherapy
- “MagniSmartech ©Motor” version with a motorized slatted bed frame, motor control & anti-snoring functions

#### 3.1.1 DESCRIPTION OF MAGNISMARTECH ©SYSTEMS FUNCTION

The “MagniSmartech ©Motor” version is provided with a motor control system that allows users to operate the double bedframe. The system also allows users to manage the incline of the head and legs through a remote control or cell phone “APP” using a BLE4 connection (4th generation Bluetooth).

A microphone is located close to the head of the bed and connected to the control unit. The system can hear when a user is snoring.

If activated, the MagniSmartech ©anti-snoring system will start the motors and move the user to a position that will reduce or completely eliminate the snoring, which is often due to the sleeping position of the neck.

It must be taken into consideration that at least 20 seconds of snoring activity is required for the signal to be acknowledged; at that point, the motorized slats will incline by approximately 10% versus its starting position. The system will go back to its initial position after 1 minute (throughout this time interval, the discrimination system of the signal remains in stand-by).

The MagniSmartech ©anti-snoring system may not be accurate and will activate involuntarily if there is continuous noise in the room.

An improper microphone position (with the ideal position at the center of the head of the bed, 40 cm above the height of the pillow) may affect the optimal performance of the MagniSmartech ©anti-snoring system.

Furthermore, this version is also equipped with a massage system (in fixed mode) which, if operational, will activate the massage function in the front and/or rear position of the motorized slatted bedframe.

The "MagniSmartech ©Mattress" version collects vital parameters (breathing and heartbeat), during the rest stage, which are determined through a set of sensors connected to a central control card, which allows for the following functions:

- 1) Detection of the average heart and breathing frequencies;
- 2) Detection of the humidity within the room and the ambient temperature;
- 3) Detection of the mattress temperature;
- 4) Detection of the luminosity in the room, ambient sound pressure and atmospheric pressure;
- 5) Detection of the frequency of movements of a person laying in the bed.

Chromotherapy system based on the lighting of RGB LED lights, which are located under the bed frame;

- 6) The lighting can be set through the cell phone "APP".

The "MagniSmartech ©Mattress" continuously records data without any need for the user to wear sensors. The sensors are inside the mattress, allowing the processing of data and other relevant access to be recorded in Real Time, either in proximity of the device or remotely. The smartphone is only used to control the beginning and the end of the data recording session. It is not required to place the phone in close proximity to the bed.

### **3.1.2 OPERATION OF THE DATA ACQUISITION SYSTEM**

From the moment the system is connected to the power supply (green LED lights under the MagniSmartech ©Station

ON), the system continuously acquires all the data coming from the sensors. Humidity readings in the room, ambient temperature, ambient luminosity, sound pressure and atmospheric pressure are all detected through the sensors which are integrated in the control unit. The temperature of the mattress is read by a sensor directly placed inside the mattress itself.

### **ATTENTION**

The reading of the vital parameters (breathing and heartbeat) might be affected by inaccuracy in the following conditions:

- Continuous movement in bed by the user.
- Massage system on
- Incline of the motorized slatted bed frame in the front area (head and shoulders) – for “MagniSmartech ©Motor” and “MagniSmartech” versions.
- Detection of the optimal position for the microphone in the room is determined by the quality of the operation of the anti-snoring system. The optimum position is on the bed head, at the center of the mattress.

Optimum position to read the vital parameters. The dashed area indicates the location area of the sensors inside the mattress

Optimum microphone position to allow the anti-snoring system to function properly

Furthermore, “MagniSmartech ©Mattress” is programmed to ensure that the user has the best sleep and the softest wake up, thanks to two special functions:


- **CELLPHONE HOLDER** - the MagniSmartech AccessPoint device is a tray where the user can locate his/her Smartphone. The rear side of the system is provided with a USB port [A], signified with the writing PW-USB, dedicated to recharging the Smartphone.
- **CHROMOTHERAPY** - The bed is provided with LED's that can be turned on and adjusted as you wish, or you can select one of the pre-set programs: relax, wakeup or hot. Side push-button [B] allow the user to turn off the LED's of the cell phone tray without having to use the App.
- **ALARM CLOCK** - It includes three modes: Simple raising of the mattress. Raising of the mattress accompanied by massage and a full alarm. Raising of the mattress, massage and chromotherapy lighting

### 3.1.3 OPERATION OF THE PUSH-BUTTON CONTROL BOARD

The MagniSmartech ©system is provided with 2 remote controls with cable RJ45 to allow adjusting - by handling through electrical drive motors - the incline of the head, shoulders ("Head up/down") and legs ("Feet up/down") of the user.

Furthermore, controls are available to:

- start the massage in fixed mode ("high massage" for the cervical area or "low massage" for the lumbar area) to activate the massage function in the front and/or rear part of the motorized slatted frame;
- take the motorized slatted bedframe into horizontal position ("Flat")
- activate the anti-snoring system ("Anti-Snore"). The activation of the system is highlighted by the lighting of the orange LED light (normally OFF).

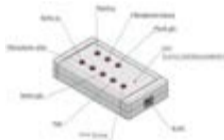The MagniSmartech ©system is also provided with a set of pre-set programs that have different configurations which can be exclusively activated through an App that can be downloaded from the Manufacturer's website:

- **ZERO GRAVITY** - position that optimizes muscular rest
- **RAISED LEG RELIEF** - for the relaxation of the lower limbs
- **READING** - ideal position for reading
- **ANTI-SNORING** - the mattress changes position to help stop snoring

The programs can be individually managed to only one side of the bed, so you do not disturb your partner's rest. In order to have access to the real time view with the remote, you need to log in with your own credentials as a recorded user, using a smartphone that is connected to the web.

### **Recording of a session/reading of the historical records**

To start a data recording, a user within the domestic network uses his/her application where he/she is logged in by his/her personal account.

After the login, the user can start a recording session from the mobile "APP" at the moment he/she is going to sleep. When waking up -again through the application - he/she will activate the function to stop the recording. The cell phone does not need to be beside the bed while sleeping.

The different records (individual session and statistics) can be displayed through the website and/or the APP, by accessing it with your own credentials.

The APP allows:

- User's to login
- Association of the device either to the individual Smartphone or to the Wi-Fi Network
- Session recording ON/OFF
- Data reading in real time
- LED control with either manual or chromotherapy functionality
- Activation of the chromotherapy programs
- Handling of the bed
- Activation of the automatic handling programs
- Activation of the anti-snoring system
- Setting of the interactive alarm clock

### **ATTENTION**

As related to the recording, setting and operational modes of the app, please refer to the on-line help service available within the app itself.

## 4. MAINTENANCE

### 4.1 GENERAL RECOMMENDATIONS FOR MAINTENANCE

- **Before proceeding to any kind of action, carefully read the instructions contained in this publication.**
- All the routine and extraordinary maintenance operations are to be performed after disconnecting the product from the main power sources.
- When the electrical equipment is operational, some parts of the equipment use hazardous voltage.
- Behaviors that are not in compliance with the safety instructions in utilizing this equipment can therefore cause death or severe damage to people and/or things.
- The extraordinary maintenance operations must be entrusted to specialized competent personnel only.
- The user's and maintenance instructions contained in this publication and the warnings reported on the product are therefore to be complied with.
- Comply with the time intervals indicated for the maintenance interventions.
- To ensure the perfect operation of the product, make sure that any replacement, if needed, is exclusively performed with Alessanderx S.p.A. original spare parts.
- Once the maintenance operations are performed, before restarting the product, make sure that: the replaced parts, if any, and/or the tools used for the maintenance operations have been removed from the product
- All the use and maintenance operations of the purchased parts of the product that are not reported in this manual are contained in the relevant publications enclosed with this manual.

### 4.2 MECHANICAL MAINTENANCE

#### 4.2.1 ROUTINE MAINTENANCE

The maintenance tasks that should be performed on the product are; Every month:

- check the operation of the drive mechanisms and if necessary, lubricate them with silicone sprays, products that are easily found on the market visually
- check the condition of the product's structure in order to locate any possible damage and/or deformation.

## **4.3 ELECTRICAL MAINTENANCE**

### **4.3.1 MODES OF ACTION**

Inspection, testing and any maintenance tasks of the product's control and display units are to be performed in the manner described below:

- Control and inspection of units/devices. The user must check that units/devices are working properly in the manner described (see the task description provided below).
- Calibration maintenance or replacement of parts. In the event of a system failure or rupture of a control device, the maintenance task must be carried out by the manufacturer's "Specialized personnel" only at the specific request of the user.

### **4.3.2 ROUTINE ELECTRICAL MAINTENANCE**

Before carrying out maintenance or a replacement of electrical or electronic equipment, cut off the power supply to the machine.

## **5. TYPE OF PROBLEM: CAUSES - SOLUTIONS**

The maintenance and inspection tasks to be performed on the machine are: Every month:

- Visual inspection of electrical components
- Visual inspection of the control pad functions and status of the connecting cables
- Visual inspection of the electric power cable

### **5.1 FOREWORD**

This section of the manual is intended to provide useful information for the troubleshooting problems most frequently encountered when using the machine.

## 5.2 PROBLEM DESCRIPTION

Problem:	The control pad and/or the drive system is/are not working
Possible causes:	The control pad plug is not inserted properly / The transformer plug is not inserted properly Faulty control pad or drive system / Control pad functions locked
Solution:	Check the connection of the plug / Plug the transformer into the socket / Request a maintenance/ repair service
Problem:	The LED on the transformer does not light up and the drive system is not working
Possible causes:	The plug is not inserted properly / Voltage missing in socket / Defective transformer
Solution:	Insert the plug / Check the socket / Request a maintenance/repair service
Problem:	The LED on the MagniSmartech ©Motor lights up, but the anti-snoring system does not work
Possible causes:	Improperly connected microphone / Microphone positioned incorrectly / The connecting plug is not inserted properly / Control pad functions locked
Solution:	Replace the control pad / Check the connections / Request a maintenance/repair service
Problem:	The drive mechanisms only work in one direction
Possible causes:	Faulty control pad or drive mechanisms
Solution:	Request a maintenance/repair service
Problem:	Wifi/Bluetooth signal loss between the various system units
Possible causes:	Internal signal loss
Solution:	Reboot the internal router/system units and make sure the connection is resumed. Switch your smartphone OFF and ON again. Make sure that the bluetooth and wifi connections are switched on.

## 6. WASTE DISPOSAL AND DEMOLITION

### 6.1.6 THE DISPOSAL PROCEDURES

Since this product is electrically adjustable, this product is classified as an electrical device of industrial use in compliance with Directive 2012/19/UE on waste electrical and electronic equipment (WEEE).

Therefore, the replaced electrical components (drives, control devices, push-button boards, etc.) of these products must be treated as electrical waste according to the WEEE directive and disposed of accordingly.

The symbol on either the product or the package means that, at the end of the life cycle, the product must be disposed of separately from the household waste.

The disposal of the special and/or toxic harmful waste must be:

- entrusted by contract to expressly authorized companies and those who physically perform the transport must hold the appropriate authorizations and be registered in the Register of the Forwarding Agents.
- directly performed by the user in compliance with the municipality regulations and entrusted to the purposely-allocated collection centers.

In case of direct disposal, the user must ascertain that no component has deteriorated.

In case the product is scraped, the worn plastic and metal must be disposed of separately and appropriately.

Attention should be paid to the drive units, since they are potentially hazardous (before the disposal, check the indications provided by the manufacturer in the relevant user's and maintenance manual).

The hydraulic units, if any, mainly consist of metal and plastic. Before disposing, comply with the Manufacturer's indications as related to the recovery of the oil contained for the subsequent disposal.

In case of doubts, address the municipality, the disposal companies or Alessanderx S.p.A.

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