
1

Welcome!

Attach your watch to the charger, and connect it to an USB outlet. Let it charge for at least 15 min.

Remove your watch from the charger and press any button to wake up the display. Select your language and your watch is ready for pairing.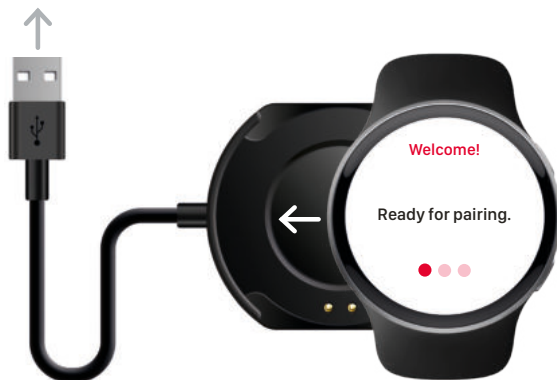

EN


BE3330 Watch Receiver

2

**Pair your watch**

- ◀ First, make sure your bridge is installed correctly. Then, hold down the top test button until the front LEDs blink alternately to initiate pairing.

A screen appears to confirm when your watch has been paired. ▶

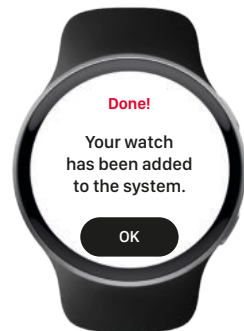3

**Test the connection**

- ◀ Press the bridge top test button to send a signal to your watch.

A status screen confirms that your watch is connected. ▶

4

Position your watch

Wear your watch comfortably on your wrist, yet firmly enough to allow it to generate distinct vibrations.

5

Wake-up

Raise your wrist or press any of the side keys to wake up the screen.

Navigation

Tap the screen to e.g. press a button.

Watch face

Indicates the battery level.

Shows the day and date.

Displays missed notifications.

6

Home key

Press to go back to the watch face.

Press and hold to turn your watch on or off.

Back key

Press to open the menu or go back one step.

Press and hold for 3s to call for attention.

Discover all features!

BE3330_048MAN4.0

bellman.com/watch-guide

BE3330
Watch Receiver

EN

Warranty and Safety information

Before using your Watch Receiver from Bellman, please review the Watch User Guide at bellman.com/watch-guide. If you have additional questions about your product, contact your point of purchase for further assistance.

Safety and Handling

For information about safety and handling, see "Safety and Handling" in the Watch User Guide at bellman.com/watch-safety.

Exposure to Radio Frequency

For radio frequency exposure information, see "Exposure to radio frequency energy" in the Bellman Watch User Guide at bellman.com/watch-safety.

Battery and Charging

For your safety, do not remove the battery incorporated in the product. Battery replacement by non-qualified professional can cause damage to your device. If you need to replace the battery, take it to the nearest authorized service center for assistance. Batteries must be recycled or disposed of separately from household waste. For information about battery disposal and recycling, see "Correct disposal of batteries in this product" in the Watch User Guide at bellman.com/watch-guide.

Medical Device Interference

Persons with pacemakers should:

- ALWAYS keep the product more than six (6) inches, (15 cm), from their pacemaker when the product is turned ON.
- Not carry the product in a breast pocket.
- Use the wrist opposite the pacemaker to minimize the potential for Radio frequency (RF) interference.

- Turn the product OFF immediately if there is any indication that interference is taking place.
- If you use any other personal medical device, consult your physician or the manufacturer of your device to determine if it is adequately shielded from external RF energy.

Two-Year Limited Warranty Summary

The company guarantees this product for two (2) years from date of purchase against any defects that are due to faulty materials or workmanship. This guarantee only applies to normal conditions of use and service, and does not include damage resulting from accident, neglect, misuse, unauthorized dismantling, or contamination howsoever caused.

This guarantee excludes incidental and consequential damage. Furthermore, the warranty does not cover Acts of God, such as fire, flood, hurricanes and tornadoes.

This warranty gives you specific legal rights and you may also have other rights that vary with territory. Some countries or jurisdictions do not allow the limitation or exclusion of incidental or consequential damages, or limitations on how long an implied warranty lasts, so the above limitation may not apply to you. This guarantee is in addition to your statutory rights as a consumer. The above warranty may not be altered except in writing signed by both parties hereto.

For Australian Consumers

Our goods come with guarantees that cannot be excluded under the Australian Consumer Law. You are entitled to a replacement or refund for a major failure and for compensation for any other reasonably foreseeable loss or damage. You are also entitled to have the goods repaired or replaced if the goods fail to be of acceptable quality and the failure does not amount to a major failure.

Cleaning

Wipe your watch clean with a nonabrasive, lint-free cloth. If necessary, lightly dampen the cloth with fresh water. If your watch has substances or debris visible in places such as the charging ports, you can hold it under lightly running, warm, fresh water until it's clean. This device does not require sterilization.

Service and Support

If the device appears to be damaged or doesn't function properly, follow the instructions in the Watch User Guide at bellman.com/watch-guide. If the product still doesn't function, contact an authorized service center for information on service and warranty.

Regulatory

For regulatory certification information, see the Bellman Watch User Guide at bellman.com/watch-safety.

Technical Specifications

Dimensions	49.5 × 44.5 × 10.2 mm, Weight: 42g, 1.5 oz.
Screen Size	1.3" TFT full touch screen
Screen Resolution	240 × 240 px
Battery Power	200 mAh
Operating time	Up to 1 week depending on use
Material	ABS + PC (watch), Silicone (wrist band)
Waterproof	IP67
Charger	USB charger
Bluetooth Freq.	2400 – 2483.5 MHz, Max transmission power: 3 mW
Coverage	Up to 250 m open range. Obstacles, such as walls, bodies or electrical equipment can reduce the Bluetooth coverage significantly.

In the box

- BE3330 Watch Receiver
- BE9295 Pocket Charger
- BE9419 Wrist Band, Black
- Getting Started Guide

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EN FCC compliance statement

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

Warning

Changes or modifications to this unit not expressly approved by Bellman & Symfon Group AB could void the user's authority to operate the equipment.

Note

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful

interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

IC compliance statement

This device contains license-exempt transmitter(s)/receiver(s) that comply with Innovation, Science and Economic Development Canada's license-exempt RSS(s). Operation is subject to the following two conditions:

- (1) This device may not cause interference.
- (2) This device must accept any interference, including interference that may cause undesired operation of the device.

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Product disposal and recycling information

According to local laws and regulations your product should be disposed of separately from household waste. When this product reaches its end of life, take it to a collection point designated by local authorities. The recycling of your product will help conserve natural resources and ensure that it is recycled in a manner that protects human health and the environment.

Battery disposal information

This product contains batteries that according to local laws and regulations must be disposed of separately from household waste. Take used batteries to a collection point designated by the authorities.

FCC ID: 2APAKBE3330

IC: 6693A-BE3330

Manufacturer

Head Office

Bellman & Symfon Group AB
Södra Långebergsgatan 30
436 32 Askim
Sweden
Phone: +46 31 68 28 20
E-mail: info@bellman.com

Distributor | Distribuidor | Distributeur

USA & Canada

Bellman & Symfon North America Inc.
PO Box 1388, Morrisville
North Carolina 27560, USA
Phone: (910) 239-5796
Customer support: (833) BELLMAN
or (833) 235-5626

Australia & New Zealand

Word of Mouth Technology
PO Box 1001
Croydon VIC 3136
Australia
Phone: 03 9723 0660
Email: info@wom.com.au

© and TM 2025 Bellman & Symfon AB. All rights reserved. Revision: BE3330_047WCD4.0. Date of issue: 2025-02-25