Safety Guide

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Thank you for purchasing this product. Before using this product, please read the Safety Guide and Quick Use Guide carefully to ensure your safety and experience in use.

[Safe Use]

1. While using this product, please follow the basic precautions below:

a) Please use this product at -10°C to 50°C and charge it at 0°C to 45°C.

b) Never expose this product to water, rain, liquid, snow, etc.

c) If the voltage of the power supply or charger used is not 5V, fire or personal injury may occur.

d) Do not use this product if damaged or modified.

e) Do not disassemble or squeeze this product or keep it in fire or at high temperatures.

Exposure of this product to fire or high temperatures above 100°C will cause an explosion.

f) Maintenance must be performed by competent maintenance personnel with the same spare parts.

2. Never modify this product without authorization.

3. It is recommended to shut down this product and keep it in the supplied bag if it is not in use.

4. Use this product with the approval of relevant authorities and the power supply (e.g. UL, CSA, VDE and CCC) conforming to local regulatory requirements.

5. In case of overheat, shut down this product and contact our customer service center.

6. If you are under the age of 18, please read this document together with your legal guardian, and follow the precautions for use by minors.

[Basic Daily Maintenance]

In order to ensure the normal operation of this product and extend its service life, please carry out cleaning and maintenance regularly.

1. Do not wash this product with water. Wipe dirt (if any) on the headband with dry soft cotton cloth. If there is still dirt, soak cloth in water, wring it and wipe this product. Then wipe it with dry cloth.
2. In the event of any defects (e.g. damage) found in daily inspection, please contact the Customer Service Center of BrainCo for repair.
3. Keep this product clean and dry. Do not dry this product with external heating devices such as hair dryers or microwave ovens.
4. Do not use this product in contact with dark clothing in order to avoid staining.
5. Prior to cleaning and maintenance, please shut down this product and disconnect all cables from other devices.
6. While you are wearing this product, please keep it dry, breathable and in proper contact with the skin. In case of any discomfort in long-time use, please take off this product and have a rest. If you still feel uncomfortable, it is recommended to stop using this product. Instead, consult a doctor in time.

[Precautions]

1. Do not wipe this product and its accessories with liquids such as volatile oil, thinner or gasoline.
2. Do not store and use this product in direct sunlight or in hot, wet and dusty places or

where there is a lot of vapor or corrosive gas.

3. Do not keep this product and its accessories at too high or low temperatures. Otherwise, failure, fire or explosion may occur.
4. Do not expose this product and its accessories to intense impact, vibration, scratching, hard or sharp objects and the like, in order to avoid failure arising from damage to this product and its accessories.
5. Do not clean this product or its accessories with virulent chemicals, cleaning agents or potent detergents. Please wipe them with clean and dry soft cloth.
6. Do not expose this product to any magnetic cards (such as bank cards and phone cards) for a long time. Otherwise, magnetic cards may be damaged by magnetic fields.
7. Do not disassemble or modify this product and its accessories without authorization, otherwise, our warranty for this product and its accessories will be invalid. Please contact the Customer Service Center of BrainCo in case of failure.
8. If this product is broken as a result of collision against hard objects or external violent impact, it is forbidden to touch or attempt to remove the broken part. Please shut down this product immediately and contact the Customer Service Center of BrainCo in time.

Do not use this product where it is expressly prohibited to use wireless devices. Otherwise, interference may be caused to other electronic devices or there may be other hazards. This product is not a medical device. All health data and recommendations provided are for reference only and must not be used as a basis for diagnosis and treatment. Considering physical differences, please stop using this product in case of any skin discomfort, and seek medical advice immediately in serious cases.

Those suffering from severe mental disorders (e.g. severe schizophrenia, bipolar disorder, PTSD, anxiety or depression) must not use this product without evaluation and guidance by a physician. Those who were once diagnosed with a psychiatric disorder but are in a stable phase shall keep close contact with their therapist.

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

1. For devices approved under Part 15, the user's manual or instruction manual for an intentional or unintentional radiator shall caution the user about changes or modifications to the device (Section 15.21).

Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation.

If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

[Certification Information]

R XXX-XXXXXX

Model	FCC ID	IC	CMIIT ID	MIC
CL1A	2AQA9-CL1A	24164-CL1A	24J332RR6789	XXX-XXXXXX

FCC Radiation Exposure statement

This equipment complies with FCC radiation exposure limits set forth for an uncontrolled environment. This transmitter must not be co-located or operating in conjunction with any other antenna or transmitter.

ISED Regulatory Compliance

This device contains licence-exempt transmitter(s)/receiver(s) that comply with Innovation, Science and Economic Development Canada's licence-exempt RSS(s). Operation is subject to the following two conditions: (1) This device may not cause interference. (2) This device must accept any interference, including interference that may cause undesired operation of the device.

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This equipment complies with IC RSS - 102 radiation exposure limits set forth for an uncontrolled environment.

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[Lithium battery]

WARNING:

This appliance contains a rechargeable lithium battery, and the scrapped battery is harmful to the environment, please follow the following tips before discarding the appliance:

- The battery must be removed from the appliance before discarding it;
- When removing the battery, the appliance must be disconnected from the power supply;
- Batteries should be disposed of safely. Be careful not to damage the battery casing to prevent danger. Please take the removed battery to a designated professional and qualified battery recycling facility;
- Replacement of a battery with an incorrect type that can defeat a safeguard (for example, in the case of some lithium battery types);
- Disposal of a battery into fire or a hot oven, or mechanically crushing or cutting of a battery, that can result in an explosion;

- Leaving a battery in an extremely high temperature surrounding environment that can result in an explosion or the leakage of flammable liquid or gas;
- A battery subjected to extremely low air pressure that may result in an explosion or the leakage of flammable liquid or gas .

CAUTION

Disposal of a battery into fire or a hot oven, or mechanically crushing or cutting of a battery, that can result in an explosion; leaving a battery in an extremely high temperature surrounding environment that can result in an explosion or the leakage of flammable liquid or gas; a battery subjected to extremely low air pressure that may result in an explosion or the leakage of flammable liquid or gas.

RISK OF EXPLOSION IF BATTERY IS REPLACED BY AN INCORRECT TYPE. DISPOSE OF USED BATTERIES ACCORDING TO THE INSTRUCTIONS.

[Limitation of Liability]

1. In order to protect the legitimate rights and interests of users, please read the Quick Use Guide and Safety Guide carefully before using this product. BrainCo reserves the right to update these documents.
2. Once you use this product, it is deemed that you have read, understood and accepted all terms and contents of the Safety Guide. Users shall undertake the responsibility for all your actions and consequences arising therefrom. Users shall also promise to use this product for proper purposes only and agree with all terms herein and other related policies or

criteria that may be developed by BrainCo.

3. While using this product, please strictly follow and implement the requirements in the Safety Guide, but without limitation. In the event of any personal injuries, accidents, property losses, legal disputes and other adverse events causing conflicts of interest due to violations of the Safety Guide or force majeure, users shall bear the responsibilities and losses arising therefrom, and BrainCo shall not assume any liability.

4. BrainCo shall not assume any liability for direct or indirect violations of legal provisions during the use of this product.

5. Other exemptions stipulated in the laws and regulations.

Notes:

1. Users under the age of 18 shall carefully read this document and use the product with the supervision, guidance and consent of their guardians.

2. Minor users and their guardians shall be aware that they bear all legal responsibilities arising from violations of the laws and regulations or requirements in this document.

Notes:

Thank you for purchasing our product. The product warranty is one year from the date of purchase.

1. This product is entitled to three guarantees (for repair, replacement or compensation of faulty product). If there is any discrepancy between the service terms and the provisions of three guarantees, the provisions of three guarantees and the contents in the certificate of three guarantees shall prevail. Where a special catalogue of three guarantees is applicable,

our company will observe relevant local regulations.

2. The warranty applies to the host only. Accessories such as the soft headband and power cord are not covered by warranty.

The following cases are excluded from three guarantees:

- Expiration of three guarantees;
- Loss caused by failure in use, maintenance and storage of this product according to the operating instructions;
- Loss arising from disassembly that is not performed by those stipulated in the requirements of three guarantees;
- Absence of a valid certificate of three guarantees or valid invoice (except those proving that this product is within the validity period of three guarantees);
- Unauthorized alteration of the certificate of three guarantees;
- Inconsistency between the product or number in the certificate of three guarantees with the actual product;
- Absence of the factory name, factory address, production date or product certificate;
- Damage caused by force majeure.

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