Model: extense

HeLo extense

USER GUIDE

Hejo extense

Congratulations on your new HELIO Extense!
The non-invasive device for blood sugar level measurement.

Thanks to sophisticated technology and the use of certified medical sensors, Helo Extense is an innovative device that allows you to take an easy blood sugar level measurement while avoiding the pain caused by pricking your finger with other methods of measurement.

USER GUIDE - EN

BOX CONTENTS

HELO Extense box contents:

- Device
- Cloth for maintenance and cleaning
- User Guide
- Micro-USB charging cable

DISPLAY AND BUTTONS

The multi-function button allows you to switch the device from on/off.

HELO Extense is equipped with a monochrome DOT-LCD display that allows you to read and see information and settings:

- **Recharge Icon**

A charging icon appears on the display when the device is connected to the charging cable and battery is charging.

- **Battery Icon**

This icon shows the current battery level on the display.

• **Bluetooth Icon**

The Bluetooth icon that appears on the display can show two operation status:

- **Flashing;** when the device is on but is not paired/connected to a smartphone.
- **Fixed;** when HELO Extense is currently paired/connected to a smartphone.

• **Messages**

The display shows the status of the Measurement in progress and firmware update procedures during the execution of these operations.

NOTE: The Extense display does not show the measurement details. Please refer to the app to view the measurements details and information.

USE

• App download

To use your Helo Extense properly, you need to use one of the compatible apps.

All the compatible Apps are available at
www.heloappstore.com


HeloAppStore

• **Measurement Tips**

In order to perform a measurement correctly with your HELO Extense, you need to follow these simple tips.

Since the device comes equipped with several measurement sensors, allocated explicitly inside the device, to ensure correct detection, you must

place your finger inside the device following the natural conformation and frame of the device. Make sure your finger covers all the sensors located on the bottom, on the left side, and the finger socket, as shown in the illustration.

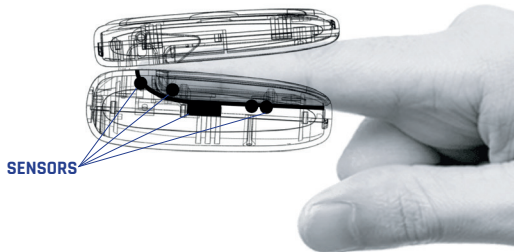CARE AND MAINTENANCE

HELO EXTENSE is a simple-to-use device that requires no special maintenance.

It is good practice to follow some tips:

- Wash hands thoroughly before measuring.
- Periodically clean the device with the supplied cloth.
- Do not insert objects into the sensor compartment.
- Do not force the opening beyond the allowed limit.
- Do not use charging cables or power supplies different from the following indicated requirements.
- Do not expose the device to water.

TECHNICAL INFORMATION

Product Name: Helo Extense

Product type: Electronic Device

Product weight: 0.06kg

Product dimensions

(L x W x H): 37.98*65*33.56mm

Box weight: 0.4kg

Box dimensions

(L x W x H): 130*75*40mm

Materials: ABS/PC

Color: Silver

Battery:

Li-ion Polymer, RZXS502548-600mAh

Charging method: USB cable, DC4.4~10.2V,
250mA.

Battery life:

- **Standby: 2 days** (operating display and BLE)
- **Normal use: 10 days** (1 hour/day - switching on and off using the operating button after each use).

Other:

Connection: Bluetooth 4.1

**Operation: white light indicators on OLED display,
128X64 pixels.**

Chipset: Toshiba MCU TZ1201

**APP version: Android - iOS (Bluetooth connection
required)**

Note: Extense is equipped with a sophisticated software able to self-adjust the functioning and to calibrate the sensors before making any measurement, analyzing the user skin type. The estimated time for this calibration can vary from 10 seconds to a minute, during this operation the device indicates the operating status. The user requires no additional action at this stage, but wait for the process to complete itself.

FCC Statement

Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help

This device complies with part 15 of the FCC rules. Operation is subject to the following two conditions (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

IC Statement

This device complies with Industry Canada's licence-exempt RSSs. Operation is subject to the following two conditions:

- (1) This device may not cause interference; and
- (2) This device must accept any interference, including interference that may cause undesired operation of the device.

The term "IC: " before the certification/registration number only signifies that the Industry Canada technical specifications were met. This product meets the applicable Industry Canada technical specifications.

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