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Instrucciones

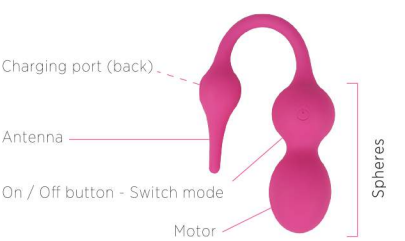
Download Joy ON **Kehel** APP (free for Android & iOS) to enjoy all the features of your Kehel.


Specifications

- **Material:** food grade silicone
- **Color:** pink
- **Size:** 2 spheres 33 mm / 1.3 in diameter
- **Charging time:** 3-4 hours
- **Working time:** 2-3 hours
- **100% waterproof**
- **1 piece**
- **Contains:** Kehel, USB cable, storage bag and user manual

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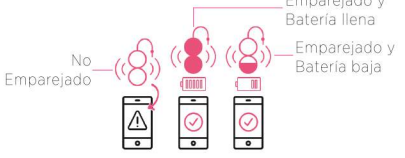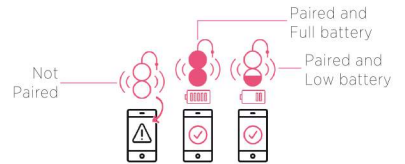
**1 On/Off:** to switch on Kehel, press and hold the power button for 5 seconds until the device vibrates. Press and hold the power button for 5 seconds to switch off Kehel. Kehel can only be turn off if it is not paired to your mobile device. If it is paired, you will need to unpair it in the App by clicking the Kehel icon in the top right corner of the Exercises or Massage screens.

**2 Using Kehel with the App:** once you switch on your Kehel, verify the LED indicator light is blinking fast and that the Bluetooth in your Smartphone or tablet is activated. Open the App and pair Kehel by clicking the Kehel icon located in the top right corner of the Exercises or Massage screens. Once paired, the Kehel icon will turn pink, showing the remaining battery life of your Kehel. If Kehel's battery is running low, please charge it before continuing. Click on **“How to use Joy ON Kehel”** to start learning about how to use Kehel's App.

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**3 Non-App manual mode:** you can still use Kehel without connecting to the App by using your pelvic muscles to squeeze the device. The harder you squeeze, the stronger the vibration will feel. With Kehel switched on, short press the power button and Kehel can be used as vibrating egg. Short press again and it will go back to pressure vibration mode. Manual modes can only be used when Kehel is not paired to your phone. If it is paired, you will need to unpair it in the App by clicking the Kehel icon in the top right corner of the Exercises or Massage screens.

**4 Kehel position:** Kehel should be positioned inside your vagina, with at least 2cm/0.8in from the vaginal opening to the lower ball. Spheres should rest over your pelvic floor muscle, with the On / Off button facing upwards. The exact distance from the vaginal opening to the pelvic floor muscle differs from woman to woman, but Kehel should be positioned in a similar way as a tampon. Kehel's tail is an antenna and must be kept outside of the vagina. If you are using Kehel with the App, please note the antenna should be pointed up or towards your mobile phone. Do not cover the antenna or sit on it and keep your Smartphone within 5m/16ft range to avoid interrupting the connection.

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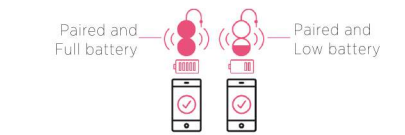
Charging Kehel and checking battery life

1. Plug the charging pin into Kehel's charging port and plug the USB end into a power adapter, computer or powerbank.
2. During charging, the LED indicator light will be on.
3. If LED indicator turns off immediately after starting the charge, please unplug it and insert it into Kehel again, making sure it is plugged in firmly.
4. It takes about 3-4 hours to fully charge. Once Kehel is fully charged, the LED indicator light will turn off.

To check the remaining battery life of Kehel, click on the Kehel icon in the top right corner of the Exercises or Massage screens. Once paired it will show the battery status.

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Cleaning and maintenance

Proper cleaning and maintenance will extend the life of your Kehel.

**Lubricant:** use water soluble lubricants only.

Cleaning and disinfection:

- Clean and disinfect Kehel before and after each use.
- Use an antibacterial, adult product or neutral soap to clean your Kehel.
- Water temperature for cleaning shouldn't be over 40°C / 104.0 °F.

**Storage:** keep your Kehel in a dry and well-ventilated place. Keep away from direct sunlight. Keep away from children.

Notice

- Stop using Kehel immediately and visit your doctor if you feel any discomfort or irritation.
- Do not use if you have a gynecological inflammation.
- If you are pregnant, check with your doctor first before using any kegel exercising device.

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Additional info and troubleshooting

For additional information, check the extended user guide available inside the App in **Account > User help** or visit the website [joyontoy.com](http://joyontoy.com).

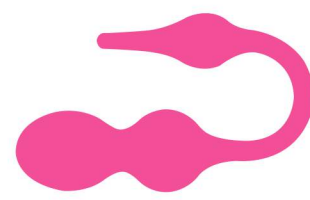
Warranty and after sales service

If you have any problem or question, please contact us directly at **support@joyontoy.com**.  
For more information on warranty and after-sales service, visit the website [joyontoy.com](http://joyontoy.com).

Authorizations and licenses

For more information on sales licenses and authorizations, visit the website [joyontoy.com](http://joyontoy.com).

USER MANUAL  
MANUAL DE USUARIO

KEHEL

JOYONTOYS.COM

Download Joy ON **Kehel** App  
Descarga Joy ON **Kehel** App

iOS  
[joyontoy.com/ioskehel](http://joyontoy.com/ioskehel)

Android  
[joyontoy.com/androidkehel](http://joyontoy.com/androidkehel)

JOYONTOYS.COM

JOY ON KEHEL FCC CERTIFICATION

FCC ID: 2AT5B – KEH-01

Certification markings can be found on Kehel's charging cable.

FCC Regulations

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

**NOTE:** This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
  - Increase the separation between the equipment and receiver.
  - Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
  - Consult the dealer or an experienced radio/TV technician for help.
- The device has been evaluated to meet general RF exposure requirement. The device can be used in portable exposure condition without restriction.