



all rejection

③ Activation and Connection

Launch the APP to menu interface, click on the activation portal, select Cling Aura, then follow the APP instructions and discover Cling Aura devices. Select Cling Aura with the ID shown on the screen and activate it. Until you reach 100% , Binding Complete.

★ Note: the "pairing" prompt of the IOS phone pops up, be sure to allow pairing, Android phone choose to cancel pairing.

Pressure Fluctuation detection feature it is essential that the Cing App using a Sphygmomanometer. Enter the reading into the Cing App as your baseline. The Cing App "asuring" mode and calibrate itself. The screen display is a is completed the Blood Pressure reading will be shown on the next detection. Hold the Start key for approximately 20 manual Blood Pressure Fluctuation reading.

acceptable device (like a Sphygmomanometer) for readings.

types of outdoor sports motion trajectory:
In order to track these two workouts, activate page touch and hold the Home Key button for switch to outdoors by clicking the Home Key for

Bluetooth and GPS shows a connection, you can still touch the "Got it" button placing the Aura in pairing mode. However under this in the Cling App does not record the motion trajectory. Touch the Key to switch between the pages

pressing the button 3 seconds, movement



Hold and press the Home Key for 3 seconds in order to quit the movement.

Three vertical panels displaying fitness metrics:

- BPM:** Warning, 175
- Steps:** 10005
- Finish:** 2500

device in the charging seat.
occupied.

es, please cancel the manual pairing in the list of Bluetooth paired devices.

open the Cling App and activate it step by step

either other mobile phones are paired or linked, you need to disconnect the
give up the Bluetooth of the device before the Cling App can search for the

You can set **Cling App running in the background?**
 Management - Cling App- to set up trust in this software. If you
 have a class or a cleanup accelerator class application, set the Cling
 program of the mobile phone system, please set "Cling" as
 display and other functions need to obtain this privilege to
 running mode.
 operation lock.

Android 4.4+ and iOS8.0+ responsible for compliance and operate the equipment.

This device complies with Part 15 of the FCC Rules, subject to the following two conditions:

- (1) this device may not cause interference
- (2) this device must accept any interference that may cause

App software and Aura firmware will be to the latest push versions available.

ser's authority to

Rules. Operation is

ence, and
received, including
eration.

(4) Damage of product is not caused by authorized maintenance
(5) Damage of product is caused by force majeure;
(6) Damage of product is caused by inappropriate use;
(7) Damage of product is caused by private disassembling.

1. The product is not a medical equipment. Please do not rely on this product for sickness. If you are feeling unwell, please consult and seek medical help.

2. This product's exercise suggestion is only for your reference. If you want to do daily routine, please seek professional help.

3. Please use this product and application according to local legislation.

4. Please scrap this product according to local legislation.