

# Muscle Sense™

- Best performance available for **Feedback and Control** using muscle activation levels.
- Bluetooth® LE 4.2.
- iPhone app (free) displays multiple Muscle Senses.
- **Accurate noise-free** coupling of electro chemical activity at neural muscular junctions.
- **Proprietary** algorithms and precision electronics.
- **Highly selective coupling** for minor muscle group differentiation.


## Users Guide:

- Muscle Sense (model M40)
  - Snap one Repositionable Skin Electrode\* into Muscle Sense.
  - Stick one Electrode on skin over target muscle.
  - Align Muscle Sense along target muscle, snap onto previously adhered Electrode and press firmly into place.
  - Press the button on Muscle Sense to turn it on.
  - During use and at all times, keep the Muscle Sense dry.\*\*
- Application
  - Pair (if necessary).
  - Set device ID (Muscle Sense indicator light colors).

\* Electrodes can be purchased at various online stores including anrcorp.com.

\*\* The Muscle Sense is not waterproof.

[www.anrcorp.com](http://www.anrcorp.com)

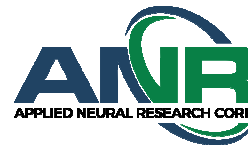

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation. Caution: changes or modifications not expressly approved by ANR Corp may void the user's authority to operate the equipment.