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MENU
1 Who Should Use Femometer
2 How to Use Thermometer
3 Alerts & Sounds
4 About the Display
5 Troubleshooting
6 Product Maintenance & Care
7 Technical Specifications
8 Product Functions & Features
9 Safety State ment
10 Disclaimer
11 Copyright Statement
12 Warranty
13 Symbol Key
14 Record of Purchase

2. Pair Femometer with Your Smartphone
Step 1: Remove the cap from the thermometer.
Step 2: Turn on your phone's Bluetooth.
Step 3: Follow the prompts on your phone.

4. Replace the Battery
Step 1: Insert the stainless steel tool into the crevice at the base of the thermometer.
Step 2: Wait for one minute, put a CR2032 battery in the battery tray with "+" facing down.
Step 3: Close the tray from the base of the thermometer.

4. About the Display
1. Remove the upper cover of the thermometer.
2. To extend the battery life, the screen will be off once "OFF" is displayed 5 seconds after the measurement.

6. PRODUCT MAINTENANCE & CARE
1. Battery care
- Replace the battery when instructed via notification in the Femometer App.
- Always dispose of used batteries according to local laws and regulations.
2. Cleaning & Disinfection
- Clean the tip of the thermometer with a solution of 75% rubbing alcohol and 25% water before and after every use.
- Do not use cleaning agents other than alcohol and water to clean the device as it may damage or decrease the lifetime of the product and/or present safety risks.

8. PRODUCT FUNCTIONS & FEATURES
1. Basal body temperature measurement & Bluetooth
- The thermometer measures basal body temperature to 1% degree Fahrenheit accuracy.
2. Data stored in the cloud
- All your data is stored in the cloud, so you don't have to worry about manually uploading or losing your information.
3. Share date with friends
- Share your data easily with your partner or doctor directly from the Femometer App via Twitter or Facebook.

11. COPYRIGHT STATEMENT
- Hangzhou Bangtang Network Technology Co.,Ltd (Bangtang), Floor 12,92 Yuexia Street, Binjiang District, Hangzhou, China
- Manufacturer/Hersteller: Joytech Healthcare Co., Ltd. No.365, Wuzhou Road, Yuhang Economic Development Zone, Hangzhou city, 311100 Zhejiang, China

FCC statements:
This device complies with part 15 of the FCC rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.
NOTE: The manufacturer is not responsible for any radio or TV interference caused by unauthorized modifications or changes to this equipment. Such modifications or changes could void the user's authority to operate the equipment.

The device has been evaluated to meet general RF exposure requirement, The device can be used in portable exposure condition without restriction
Federal Communication Commission (FCC) Radiation Exposure Statement
Power is so low that no RF exposure calculation is needed.

1. WHO SHOULD USE FEMOMETER
This product should be used by women to measure basal body temperature. Patient is the intended operator.
2. HOW TO USE FEMOMETER
1 Download App
Search "Femometer" on Apple Store or Google play to download.
Available on the App Store
GET IT ON Google Play

3. Take Your Temperature
Step 1: Set a reminder on your phone to wake up and take your temperature each morning.
Step 2: Upon waking, remove the thermometer from its case, place it under your tongue and hold it there for 1-3 minutes, until you hear 3 beeps, indicating temperature has been taken.
Step 3: Open the application, the data will be transmitted automatically.

6. Upload Date to the App
Step 1: Once your temperature has been taken, your phone's Bluetooth must be on, the data can be transmitted from the thermometer to your phone.
Step 2: The data from the thermometer will sync automatically to the Femometer App. If the data does not sync, remove the cap from the thermometer, and tap the scale ring in the Femometer App to transmit the data.

5. TROUBLESHOOTING
Symptoms Possible reasons Solutions
I never heard the end tone while the thermometer is under my tongue.
The data did not sync to the app after my measurement.
There is no sound when opening the thermometer.

7. TECHINCAL SPECIFICATIONS
Product name: Vinca II
Modell: FM-VC-201
Größe: 120.2mm*24.4mm*16.4mm
Gewicht: 24.8g (without battery)
Transmission distance: Up to 16 feet
Measurement: 99.6°F-107.6°F
Accuracy: ±0.09°F(99.0°F-100.4°F) ±0.18°F(99.6°F-99°F) & (100.4°F-107.6°F)
Battery: CR2032
Storage conditions: Temperature: -40~131°F Humidity: <63%

10. DISCLAIMER
- This manual is written in accordance with available information and contents are subject to change without notice.
- If there are discrepancies in this manual due to software upgrades, instructions and function dictated by the software should prevail.
- All symbols and logos associated with Bluetooth, including the Bluetooth word mark, figure mark, combination mark and Bluetooth Smart and Bluetooth ready marks, are trademarks owned by the Bluetooth SIG.

13. SYMBOL KEY
Device type is BF
Serial number
Read user manual before use
Manufacturer
No littering
Recyclables
Model Purchased:
Name:
Serial Number:
Phone Number:
Date of Purchase:
E-Mail Address:
No.2019-bongmi-Vinca II.001
Version number: A

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