




Massage Chair

LC3200-S
Manual

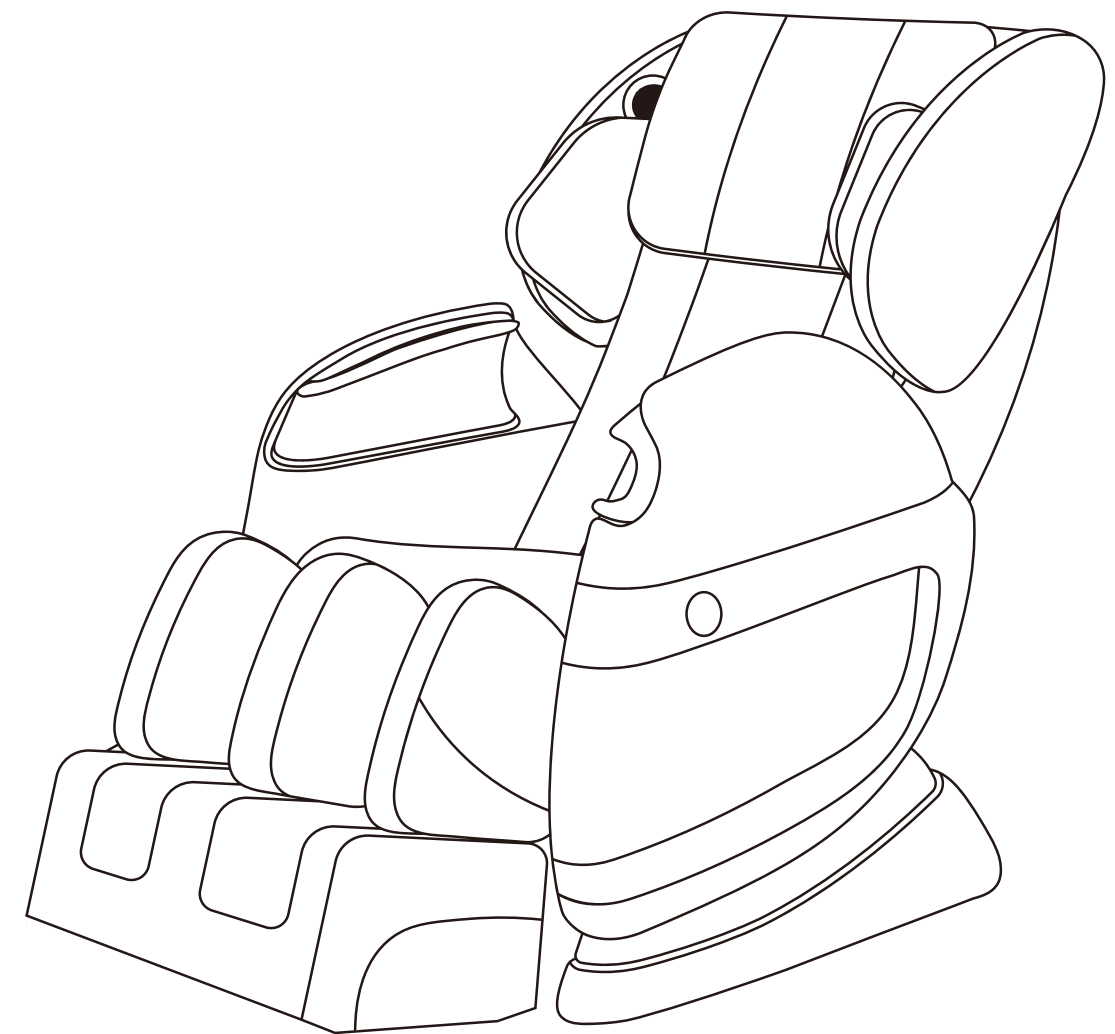

Questions, problems, missing parts? Contact technical support department at **657-341-0362, 9:00 a.m. - 5 p.m., PST, Monday - Friday** or visit us online at www.esmartpro.com
Please have your model and serial number ready



M_LC3200-S_2019
Printed in China

Contents

Thank you for purchasing eSmart massage chair.
Please read the owner’s manual (operating manual) before use.

Safety Precautions.....1-4

Grounding or double-insulation5

Components & Descriptions.....6-7

Before Use..... 8-13

Adjusting Massage Chair.....13

Start Massage.....14

After Use.....14

Troubleshooting..... 15

Safety Check..... 15

Storage and Care..... 16

Specification.....17

- This product is intended for residential use only.
- Use this product only as described in this manual. Any other use is not recommended by the manufacurer and may cause fire, electric shock or injury to persons.
- Remember to keep the manual in an easily accessible place.

Safety Precautions

IMPORTANT SAFETY INSTRUCTIONS

This massage chair is intended for household only .When using an electrical appliance, basic precautions should always be followed, including the following:
Read all instructions before using this massage chair.

DANGER-To reduce the risk of electric shock:

- 1)Always unplug this appliance from the electrical outlet immediately after using and before cleaning.
- 2)Never use pins or other metallic fasteners with this appliance.
- 3)Carefully examine the covering before each use. Discard the appliance if the covering shows any sign of deterioration, such as checking, blistering, or cracking.
- 4)Keep Dry - Do not operate in a wet or moist condition.

WARNING-To reduce the risk of burns, fire, electric shock, or injury to persons:

- 1) An appliance should never be left unattended when plugged in. Unplug from outlet. when not in use, and before putting on or taking off parts.
- 2) Do not operate under blanket or pillow. Excessive heating can occur and cause fire, electric shock, or injury to persons.
- 3) Close supervision is necessary when this appliance is used by, on, or near children, invalids, or disabled persons. Keep children away from extended foot support.
- 4) Use this appliance only for its intended use as described in this manual. Do not use attachments not recommended by the manufacturer.
- 5) Never operate this appliance if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water.
Return the appliance to a service center for examination and repair.
- 6) Do not carry this appliance by supply cord or use cord as a handle.
- 7) Keep the cord away from heated surfaces.
- 8) Never operate the appliance with the air openings blocked. Keep the air openings free of lint, hair, and the like.
- 9) Never drop or insert any object into any opening.
- 10)Do not use outdoors.
- 11)Do not operate where aerosol (spray) products are being used or where oxygen is being administered.
- 12)To disconnect, turn all controls to the off position, thenremove plug from outlet.
- 13)Temperatures sufficiently high to cause burns may occur regardless of the control setting. Do not use on an infant or invalid or on a sleeping or unconscious person. Do not use on insensitive skin or on a person with poor blood circulation. Check the skin in contact with the heated area of the appliance frequently to reduce the risk of blistering.
- 14)Do Not Crush - Avoid sharp folds.
- 15)Do not use massager in close proximity to loose clothing or jewelry.
- 16)Keep long hair away from massager while in use.
- 17)Connect this appliance to a properly grounded outlet only. See Grounding Instructions.
- 18)Remove control box when not in use, and store out of reach of children.
- 19)Burns will result from improper use, read instructions before using, do not wet-do not use pins, never remove cover.

Safety Precautions

IMPORTANT SAFETY INSTRUCTIONS

● Safety Precautions

- To ensure safe and correct use of the massage chair, do not operate it without reading this operation manual carefully.
- Do not use the massage chair when the upholstery or seat is removed. This may cause injury or malfunction.
- Do not use the massage chair in combination with other therapeutic equipment or electric blanket, etc., as this may result in ineffectiveness or injury.
- Do not use the massage chair an hour before or after eating as this may have adverse effects on the user.
- Do not allow children or pets to play around the massage chair, e.g., behind the backrest, under the seat or leg rest of the massage chair. This may result in injury.
- Do not rest or place heavy objects on the armrests, leg rest or backrest as this may result in malfunction or injury.
- Do not use the massage chair when your body is wet.
- Do not operate the massage chair with wet hands.
- It is not recommended to use the massage for more than 15 minutes.
- Do not perform continuous massage on the same spot of your body for more than 5 minutes at a time as this may result in excessive stimulation and may have adverse effects.
- Do not insert your hand or foot along the paths of the massage rollers during use as this may cause injury.
- Stop operation immediately if you experience any discomfort.
- This product is not intended for self-treatment of conditions that should be managed by a qualified health care provider.
- People on medication or with medical conditions, please consult your doctor before use. Please do not use the product if you are not feeling well.
- Remove all scarves, neckties, necklaces and jewelry before using the massager.

People with Medical Conditions

Consult your doctor before use if you:

- are under medical rest as ordered by a doctor.

have spinal disorders, an abnormal spinal condition or have suffered a spinal injury.

have back problems.

have diabetes, osteoporosis or sensory impairment.

have joint dysfunction.

have a pacemaker or other electronic medical devices.

may be pregnant.

have phlebitis or thrombosis.

have an increased risk for blood clots.

had recent surgery.

have surgical pins, screws, or anything mechanical implanted in your body.

Safety Precautions

SAVE THESE INSTRUCTIONS

Safety Instruction

● WARNING

The power plug must be fully inserted into and appropriate outlet. Improper connection of the equipment-grounding conductor could result in an electric shock.

Check with a qualified electrician and servicemen if you are in doubt as to whether the product is properly grounded. Do not modify the power plug provided by the product. If the outlet is not suitable, please ensure that a proper outlet is installed by qualified Electrician.

When using the massage chair, basic precaution should always be followed to reduce the risk of electric shock, an injury or burns to the person.

Read the following details before usage:

1. Do not use the massage chair outdoors and avoid places with direct sunlight.
2. Do not use the massage chair in places where the humidity is high .
3. Do not use the massage chair near any heating device or fire .
4. Do not use the mass. • chair when smoking
5. Recommended to unplug the power supply after use.
6. Please ensure that a proper outlet is installed by qualified electrician, Improper connection of the equipment-grounding conductor could result in an electric shock.

⚠ Caution

- The massage chair is to be used indoors only.
- If you use the massage chair in a cold room, do not increase the room temperature abruptly. It is recommended that you increase the temperature gradually to a normal level.
- If the massage chair is stored in a cold place and is being brought into a warm environment, it is advisable to wait one hour before use. This is because its performance may not be at optimum level, as there may be water droplets on the mechanical parts due to condensation. Using the massage chair under such conditions may result in malfunction.

User-Maintenance Instruction

User-maintenance instructions shall include:

- a) Instructions for cleaning and user maintenance operations recommended by the manufacturer, such as lubrication or nonlubrication , and a statement to the user that any other servicing should be performed by an authorized service representative or that the appliance has no user serviceable parts.
- b) Instructions for an appliance employing an automatically reset thermal limiter that shuts off the entire appliance shall inform the user what to expect if the thermal limiter operates
- c) Specific instructions for the proper method of storage of the cord, the total appliance, and the like, when the appliance is not in use, and for care of the cord while in use (for example, for a hand-supported product, untwisting)
- d) In the case of an appliance intended to be used with water additives, conditioners, or other solutions with or without water, specific instructions regarding the proper liquid or additive to use and the amount to be used in conjunction with the appliance

Safety Precautions

⚠ Caution

- The massage chair is not intended for use by children or persons with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the massage chair by a person responsible for their safety.
- Children should be supervised to ensure that they do not play with the massage chair.
- Avoid using the massage chair under the influence of alcohol.
- Do not apply massage rollers directly on the head, elbow or knee joints, torso or abdomen.
- If your legs or feet slip off the leg rest during massage, do not force them back in as this may result in injury.
- Remove the power plug from the electrical outlet immediately should liquid be accidentally spilled onto the massage chair. Do not use wet hands to connect or disconnect the power plug from the electrical outlet.
- Do not tamper with the power cord or use the massage chair with a damaged cord as this may result in malfunction and electric shock.

● Instances When the massage chair Should Not Be Used

In the following events, please switch off the main power switch immediately and disconnect the power cord from the electrical outlet:

- If water is accidentally spilled onto the massage chair. This may result in electric shock or malfunction.
- If the fabric of the backrest is torn and the internal components are exposed.
- If pain or discomfort is felt during massage, stop operation immediately and consult your doctor.
- If you detect any malfunction or any other abnormal condition during operation.
- If there is a power failure. Injury may occur when power is restored unexpectedly.
- If there is lightning.

● Assembly and Repair of the massage chair

- Do not disassemble the backrest cover of the massage chair. Touching the internal components may result in malfunction or electric shock.
- The massage chair must only be professionally serviced or repaired. Do not attempt to disassemble or repair the massage chair yourself.

● Things to Note About the Power Plug and Cord

- Check that the Alternating Current (AC) voltage corresponds with the specifications indicated for the massage chair.
 - Do not connect or disconnect the power plug from the electrical outlet with wet hands. This may result in malfunction or electric shock.
 - When disconnecting the power plug, pull it by holding the plug, not the cord.
 - Although operation will be stopped automatically by the Auto-Timer function, always remember to switch off the main power switch after use.
 - Do not place the power cord under the massage chair or any other heavy objects.
 - Do not wind the power cord around the massage chair as this may damage the cord and result in a fire or electric shock.
 - Do not operate the massage chair with a damaged cord or extension cord.
 - Do not use the massage chair if the electrical outlet is loose.
- If the power cord is damaged, it should be replaced by a qualified technician.

Grounding or double-insulation.

This product must be grounded. If it should malfunction or breakdown, grounding provides a path of least resistance for electric current to reduce the risk of electric shock. This product is equipped with a cord having an equipment-grounding conductor and a grounding plug. The plug must be plugged into an appropriate outlet that is properly installed and grounded in accordance with all local codes and ordinances.

DANGER Improper connection of the equipment-grounding conductor can result in a risk of electric shock. Check with a qualified electrician or serviceman if you are in doubt as to whether the product is properly grounded. Do not modify the plug provided with the product—if it will not fit the outlet, have a proper outlet installed by a qualified electrician.

b) For a grounded, cord-connected product rated less than 15 A and intended for use on a nominal 120-v supply circuit, the instructions in either (1) or (2):

1) This product is for use on a nominal 120-v circuit, and has a grounding plug that looks like the plug illustrated in sketch A in Figure 86.1. A temporary adapter that looks like the adapter illustrated in sketches b and c may be used to connect this plug to a 2-pole receptacle as

shown in sketch B if a properly grounded outlet is not available. The temporary adapter should be used only until a properly grounded connected outlet (sketch A) can be installed by a qualified electrician. The green colored rigid ear, lug, or the like extending from the adapter must be held in place by a metal screw.

2) This product is for use on a nominal 120-v circuit and has a grounding plug that looks like the plug illustrated in sketch a in Figure 86.1. Make sure that the product is connected to an outlet having the same configuration as the plug. No adapter should be used with this product.

C) For all other grounded, cord-connected products:

This product is for use on a circuit having a nominal rating more than 120 V (or This product is rated more than 15 a and is for use on a circuit having a nominal rating of 120v") and is factory-equipped with a specific electric cord and plug to permit connection to a proper electric circuit. Make sure that the product is connected to an outlet having the same configuration as the plug. No adapter should be used with this product. If the product must be reconnected for use on a different type of electric circuit the reconnection should be made by qualified service personnel.

d) For a permanently connected product

