

ABOUT EV-06

This safety watch is designed specifically for the elderly and isolated workers. The watch combines a variety of intelligent functions, such as SOS alarms, two-way calling, heart rate, blood oxygen, and step tracking. Its operation is very simple. Please read this operation guide carefully before using it.



Accessories

- Front



- Side



- Back



SIM Card Installation

Please do as below picture shows.



Charging your Watch

- Connect the USB cable from charging base to the AC adaptor.
- Place the watch into the charging base correctly, you will hear spoken confirmation "your watch is charging".
- Please fully charge the battery daily for around 2 hours or when battery is low.



The watch battery is not removable.

After a long period of inactivity, the watch may not be turned on due to low battery. Please charge it every day to ensure that the watch can work normally.

Power On/Off

- Press and hold the power button for 5~10 seconds to turn on the device until the watch face appears on the screen.



5-10sec.

- Enter the Settings menu, tap on the Power Off button, and wait until the screen turns off.



Functions and Basic Settings

When the watch is turned on, you can swipe left or right to view all available functions, such as contacts, watch faces, heart rate, blood oxygen, step tracking, alarm, settings, and more.

On the watch face home screen, you can see the time, date, and battery information.



Making a phone call

- Slide rightwards to enter the interface of Contact, select the contact person, and make a call.



Steps

- Keep sliding right to enter the interface of Steps, step counting is enabled by default and opened all day. You can also view distance and consumed calories.



Heart Rate Measurement

Wear the watch on the left hand and click heart rate, after several seconds checking, the screen will display your heart rate BPM.



Blood oxygen monitoring

After the watch is worn steadily, click on the "Blood Oxygen" icon. After a few seconds, the screen will display your blood oxygen saturation percentage.



Activating an SOS Alarm

Press and hold the button for 3 seconds, the device will vibrate and display the SOS alarm status on the screen.



Notes on Voicemail

We advise emergency contacts to switch off any voice mail facility on their phone to prevent delay in reaching help. However, the following safe-guards are used if a voicemail is reached:

- The wearer can end the current call by clicking the SOS button and the calling sequence to continue if the voice mail does not automatically disconnect.
- An emergency contact can call back to the watch if they are disconnected. Remember that the watch maybe already calling the next contact unless the calling sequence has been stopped.




Fall Alarm

The watch can automatically detect a fall. When this occurs the watch will give a warning message to the user.



- some daily activities like sports or even sitting down abruptly can trigger the fall alarm.
- The wearer has a short time to cancel a fall alarm by clicking the Cancel on the watch. If the alarm is not cancelled, emergency contacts will be called.

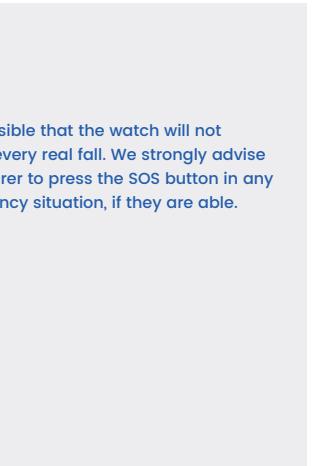

Getting a GPS fix

To get an initial GPS fix, go outdoors or near a window so the watch can get a fix on the satellites. This could take few minutes depending on your environment.



GPS functionality may be unavailable or inaccurate in some locations (for example, availability or accuracy of GPS location maybe impacted inside buildings or near tall buildings or other tall features or underground).

- The watch also provides Wi-Fi, LBS positioning, when GPS is unavailable, it will switch to Wi-Fi and LBS for locating.
- The home Wi-Fi and beacons provide at home out of home information in addition to optimizing watch battery life.
- Please note Wi-Fi and LBS tracking service is not 100% accurate, it can be used to approximate your location only.

General Specifications

- Dimension: 46.9mm*41.7mm*15.4mm/1.84in*1.64in*0.60in.
- Weight: 38g/1.34oz.
- Battery: Rechargeable, 3.8V,600mAh.
- Charging voltage: 5V/DC 1.0A.
- Waterproof: IP67.
- 4 locating technologies: GPS, Beacon, Wi-Fi, LBS.



SAFETY WATCH

FCC Warning

15.19 Labeling requirements.

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

15.21 Information to user.

Any Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

15.105 Information to the user.

Note: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that

interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:
-Reorient or relocate the receiving antenna.
-Increase the separation between the equipment and receiver.
-Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
-Consult the dealer or an experienced radio/TV technician for help.

Absorption Rate (SAR) information:

This device meets the government's requirements for exposure to radio waves. The guidelines are based on standards that were developed by independent scientific organizations through periodic and thorough evaluation of scientific studies. The standards include a substantial safety margin designed to assure the safety of all persons regardless of age or health.

FCC RF Exposure Information and Statement

The SAR limit of USA (FCC) is 1.6 W/kg averaged over one gram of tissue. This device was tested for typical body-worn operations with the back of the device kept **0mm** from the body. To maintain compliance with FCC RF exposure requirements, use accessories that maintain a appropriate separation distance between the user's body and the back of the device. The use of belt clips, holsters and similar accessories should not contain metallic components in its assembly. The use of accessories that do not satisfy these requirements may not comply with FCC RF exposure requirements, and should be avoided. Use only the supplied or an approved antenna.