
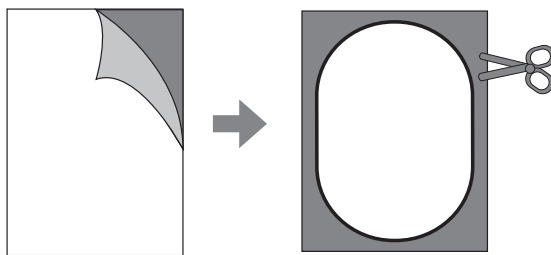


# ■ Use the magnetic papers


<AED Pads>

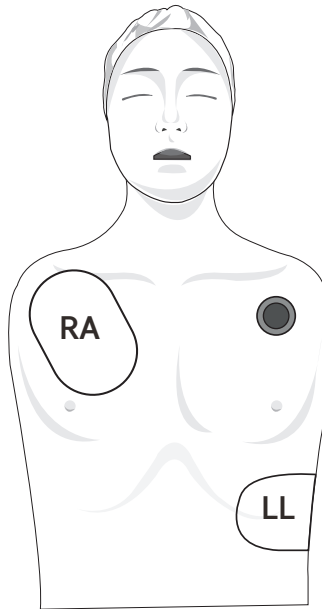

- ① Prepare the commercial adhesive AED Pads for training.
- ② Detach the white release papers from the magnetic papers.
- ③ Attach the magnetic papers onto the adhesive face of the AED pads.
- ④ Cut out the remains of magnetic papers from the AED pads.

# Basic Functions


## <Chest compression >

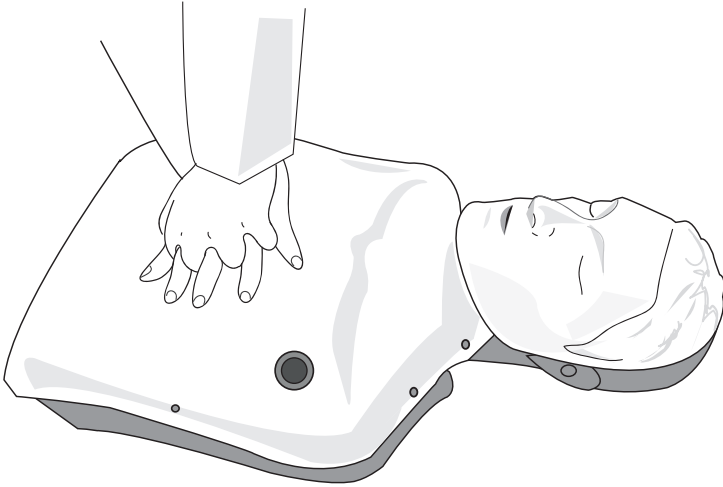


- ① Turn on the power switch after plug the power connector or install the batteries.
- ② When the display is turned on, select the display options and start the chest compression practice.
- ③ Compression depth, Compression rate, total compression counts and correct compression counts are displayed at the display panel.

# Basic Functions

## <Free Airway >

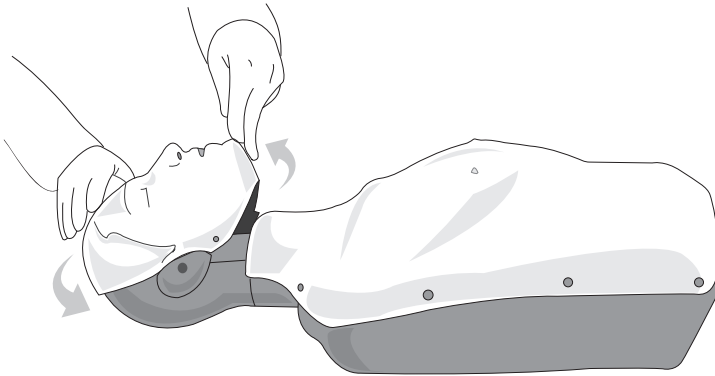


## <Artificial respiration >

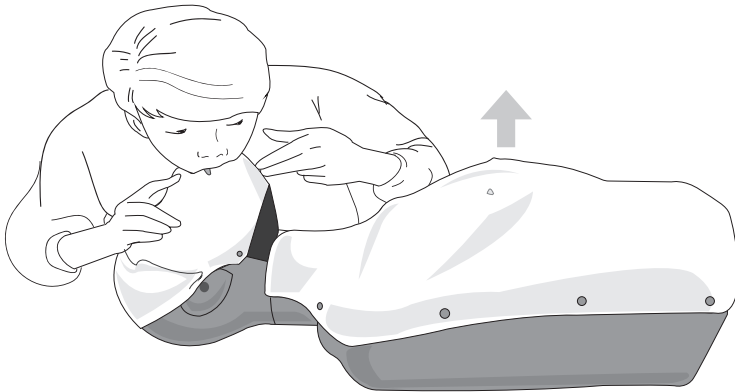


- ① Tilt the head of the manikin to open the airway.
- ② Pinch the nose of the manikin and try mouth to mouth resuscitation.  
(Pay attention to the chest inflation.)

## Basic Functions

<AED>

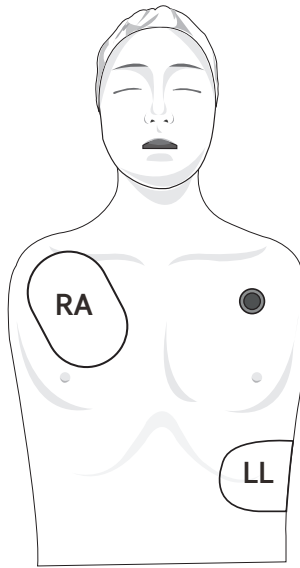



<Bleeding Control>

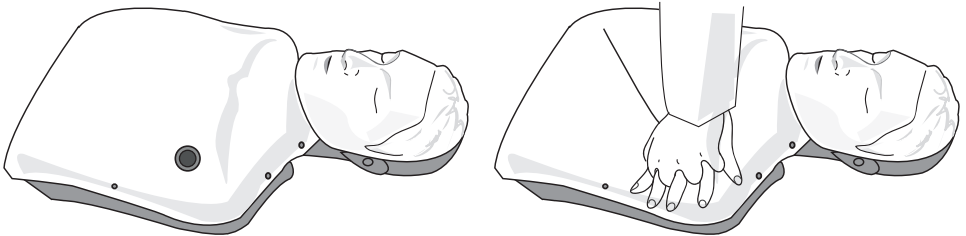

# Software

## <Bluetooth Connection>

- ① Turn on manikin and tablet PC.
- ② Select application.

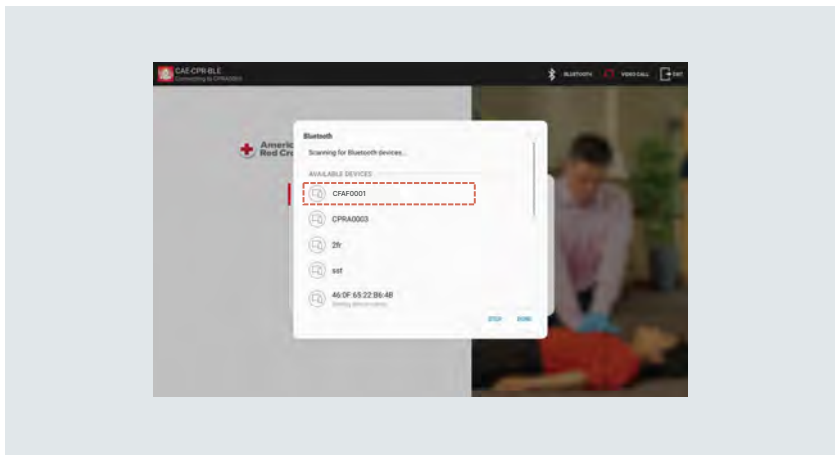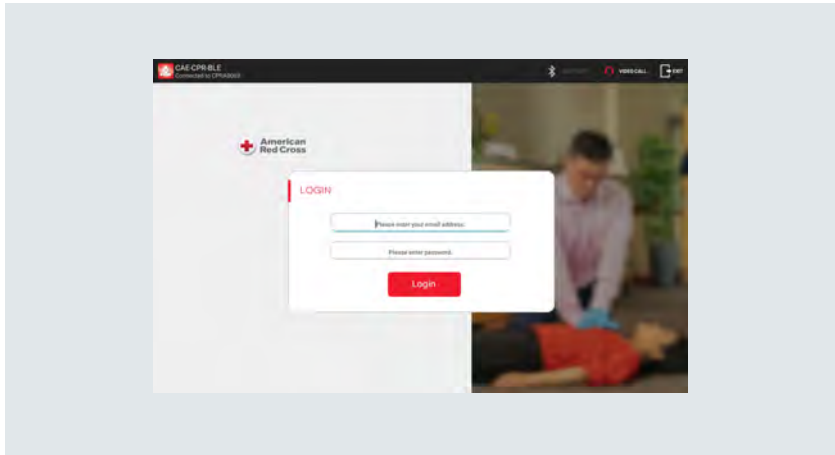


- ③ A Bluetooth pop-up screen will appear, select the Bluetooth number (Bluetooth number Sample : CFAF0001)
- ④ Once connected, it will automatically connect next time.
- ⑤ When it is now automatically connected, Pair Bluetooth by selecting Bluetooth icon.

- \* Bluetooth connection may be limited depend on the tablet.
- \* Please use a recommended tablet by BT.


# Software

## <Orientation>

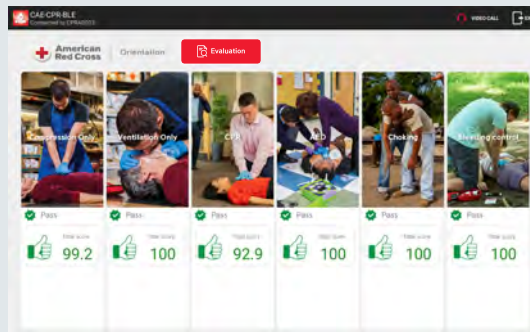

# Software


## <Training>

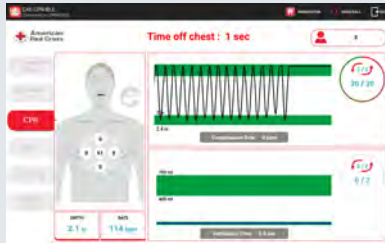

CPR



AED

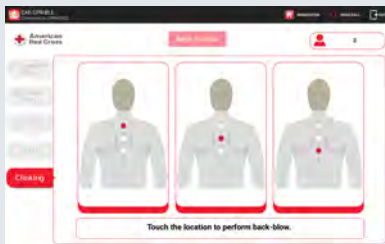

Choking



Bleeding Control

# Software

## <Evaluation>

