

WARRANTY CARD

This warranty card is used for after-sales maintenance service after purchase of eye massage instrument. Please keep it properly.

1. The after-sales service rules of eye massage instrument are as follows:
Please read the product instruction manual carefully before use and fill in the form.

2. The warranty shall be within one year with the instruction manual when the normal condition is used. The man-made fault of the warranty period shall be free of charge, but the cost of accessories and transportation shall be borne by the buyer.

Product name :

Product type :

Name of buyer:

Buyer's telephone:

Buyer's address:

Date of purchase:

Agent:

M01 EYES CARE MASSAGE

THE EYES ARE THE WINDOWS OF THE HEART ,
CARING FOR YOUR EYES



15 minutes
15 minutes to relieve
eye tiredness

10 days
10 days to lighten
black orbit

20 days
20 days to
improve vision

CARING FOR YOUR EYES

PRODUCT INTRODUCTION

The SHINING MASSAGER uses 180 ° folding design, different face can be used, using air pressure, hot compress and micro-vibration massage function, the bluetooth connection can play music at any time, adjust the volume of hearing massage, make meridian dredging, harmonize qi and blood, improve the eye blood circulation, promote metabolism, relieve eye fatigue, restore the ciliary muscle' s elasticity, make your eyes back to health.



THE BEST FOR YOU

Please love your eyes and protect your eyes

SUITABLE FOR PEOPLE

01

College students, middle school students, primary school students, myopia, hyperopia, use of the brain and excessive eye, because of long-term wear glasses caused eye fatigue;

02

Such as computer operators, accountants, designers, secretaries, teachers, drivers, people using precision instruments, etc., mental workers.

03

Crow's feet, dark circles and eye bags, people who stay up late;

04

Such as eye pain, headache, trigeminal neuralgia, neurasthenia, insomnia.



PRODUT FEATURES

Hot compress

This product USES the unique "large area temperature control" device, can feel very "balanced" heat quickly; Relax your eyes: activate eye' circulation.

Intelligent air pressure massage

Massage key point, very strong.

Massage for key points, the effect is multiplied. Effectively relieve eye strain under the hot compress

Miniature vibration massage

Effectively improve the massage effect. Quickly relax the muscles around the eyes by vibrating points

Enjoy the massage process and have a free music

Through bluetooth connection, play music! Enjoy music SPA! High-quality sound, enjoy the best sound quality effect; Mobile phone connection, want to hear what music, you decide!

Gear size adjustment

The product upgrade grade gear can be adjusted in size, and can adjust the "strong" mode or "weak" mode freely to enjoy the temperature and intensity more suitable for oneself.

Charging safety, freedom (environmental protection)

The product USES the android universal charging mode; It's as simple as charging your phone.

The product is designed with the common "Prevention and Protection" method in European and American countries, and the charging and use are absolutely separated. Please feel the humanized design with your heart.

Easy to carry

The folding design gives the owner freedom. Simple fold, lift and go

Automatic power failure in 15 minutes

When the user is asleep, the product automatically loses power within 15 minutes during use

THE BEST FOR YOU
Please love your eyes and protect your eyes



OPERATION METHOD



Hot-compress



Pressure



Vibration



Bluetooth



Work status



Charge

Mark of Legend

	Symbols are prohibited (not allowed to do)
	Symbols are mandatory (things that must be obeyed)
	The symbolic mean is: Note, please refer to the product specification)

⚠ Very dangerous

→ Please do not use it with other medical electronic devices, such as pacemaker of heart, the medical electronic equipment for artificial heart-lung. → It may cause burns in the position of a massage or damage to the massager when using high frequency surgical equipment and massages; The output of this product may be unstable if the product is used in short wave or microwave treatment equipment (1 m)	
---	--

⚠ Warn

→ The following personnel should consult the doctor before using this product, for example: Pregnant woman, Heart disease, Skin sensitive person, Blood pressure abnormality, Malignant tumor, Cerebrovascular patient, Acute venereal patient.	
→ Please do not use if the eye has damage or surgery patients. → Cannot be used in high temperature / flammable / electromagnetic radiation / wet places. → Do not use when bathing/sweating(do not in water) → In order to avoid failure or electric shock, please do not dismantle/repair and modify the massager without authorization. Please stop using and contact customer service for any failure.	

1.Press the power button (3-4 seconds), and the power will be switched on; At the same time, the hot compress function opens automatically; If you need to connect bluetooth, please open the bluetooth function in time, and automatically search [SY_BLUE_EYES]. Please operate music function on your phone and enjoy music after the connection.

2.If you need to switch the adjustment function, [double click] power button, "pneumatic massage" ~ "pneumatic massage + hot compress" ~ "hot compress"

3.Use method of gear size function: Turn on the default "hot compress" and "pneumatic massage" function for strong mode (indicator light is red), continuous [double-click] power button can be switched to "weak" mode (indicator light is blue).

4.Long press (3-4 seconds), [power supply] are closed. All functions are stopped.

5.Connect the bluetooth, the music will play at any time.
Through the simple operation of mobile phone, choose music as you wish, play the music! Enjoy the SPA of music!



Turn on the power (press the power button for 3-4 seconds); The volume icon flashes right and left.



Open the bluetooth connection equipment, scanning the bluetooth name "SHU SEN" connection (Note: The eye massager built-in bluetooth only can connect 1 devices, if change the other equipment please close connection in front of the bluetooth)



"Listen to music with your cell phone" and enjoy music. Select your favorite music and adjust the volume or close the music.

PRODUCT FEATURES

- ◎Alleviate and prevent the myopia; Relieve eye fatigue, improve oxygen supply around the eyes and improve eyesight;
- ◎Beauty the eye: Eliminate the eye bags, dark circles, wrinkles, reduce crow's feet; To erase the traces of the years and keep the secrets of age;
- ◎Prevention of eye disease for people who use mobile phones for a long time; Relieve the eye feeling of tension, effectively improve the blood circulation of eye tissues and relieve visual fatigue.
- ◎Suppress the neurasthenia and insomnia.
- ◎Prevent the eyesight decline, prevent the presbyopia, promote the blood circulation of eye ministry, accelerate the metabolism of eye skin, repair the elastic support structure.

- * The eye massager uses 180 ° folding design, use charging interface (The design concept to prevent potential safety hazards)
- * Using the plug-in charger method, the charging icon flashes (cannot be turned on)
- * The battery capacity is 600 mah;, The power is automatically cut off after 15 minutes, and the cumulative time is 60 minutes when the machine is full of electricity.

This product adopts the way of "pneumatic massage, equalizing hot compress and low-frequency vibration" to promote blood circulation in the eyes, accelerate the metabolism of eye skin, and repair the elastic support structure. Relieve the eye feeling of tension and fatigue. Thus to reduce the dark circles, pouch, wrinkles; Prevent the myopia.



MATTERS NEEDING ATTENTION

The warning signs and illustrations shown in the instructions are intended to enable you to use the product correctly and safely to prevent damage to you and your family.

The warning signs and illustrations and their meanings:

Warning sign

Meaning

Warning sign There is a risk of death or serious injury when used incorrectly

Warning There is a danger of death or serious injury when used incorrectly

Notice The danger of a person injury or damage to an item when used incorrectly

Notice

1.It should be shut down before use when the massage part is changed in the process of use, otherwise it will be stimulus strongly.

2. Do not use it for infants or the people who no ability to express themselves.



3. Please stop using and consult your doctor if you feel that your body is not in good condition .

PRODUCT SPECIFICATION

Product
standard

Name	SHINING MASSAGER
Model	M01
Power supply	1000mAh
Weight	0.55kg
Size	(L) 205 X (W) 80 X (H) 20mm

Normal
working
conditions

Humidity of environment	0℃-40℃
Relative humidity	≤ 80 %
Pressure of atmospheric	86kPa~106kPa

Environmental
performance

Compressive strength of airbag	≥ 800Pa
Output pressure of air pump	≥ 100Pa
Hot pack resistance	20Ω ± 2Ω
Temperature rise after hot compress	42℃ ± 2℃
The effective distance of Bluetooth signal	10m
Regular time	15min

FCC WARNING STATEMENT

Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment. This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- - Reorient or relocate the receiving antenna.
- - Increase the separation between the equipment and receiver.
- - Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- - Consult the dealer or an experienced radio/TV technician for help.