





USER MANUAL


Index

- 01 What's in the box
- 02 How to wear
- 03 Hearing Function
- 04 Volume Adjustment
- 05 Wireless Connection
- 06 APP Download
- 07 Charging Instruction
- 08 Specification
- 09 Warning/Caution


01 What's in the box

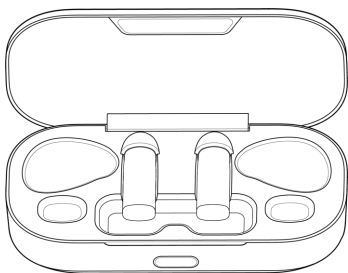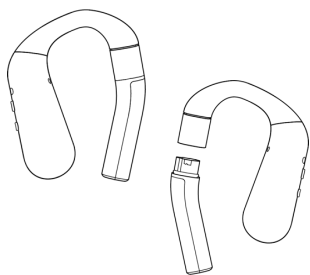

Hearing Aid(Left Ear*1, Right Ear*1)
Charging Case*1, Battery*4

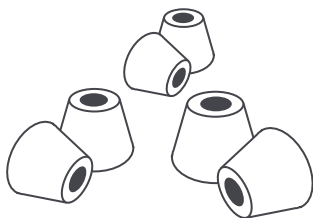

Ear Tip Set*1 (S/M/L)

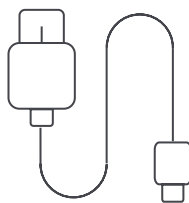

Type-C Charging Cable*1



User Manual*1

02 How to wear


● Part Introduction

Replaceable Ear-tips

- Sponge ear tip
- 3 Sizes (S/M/L)
- Choose to fit

Adjustable Ear Hook

- Made of shape memory alloys, can be bend to fit.

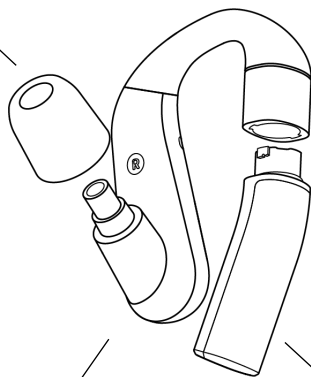

Hearing Aid


- Unique acoustic design, provide natural sound.
- Powered by RelaJet Speech Enhancement Engine

Rechargeable Battery


- Patented replaceable/rechargeable battery technic.
- Two alternate batteries for each ears.

● Install ear tip & battery


1. Choose suitable ear tips, put them on the hearing aid.
2. Power Up : Put battery into the socket, slightly rotate clockwise. Make sure to install both ears. The indicator will light up after power up.
3. Power Off : Slightly rotate the battery anti-clockwise, pull it out.

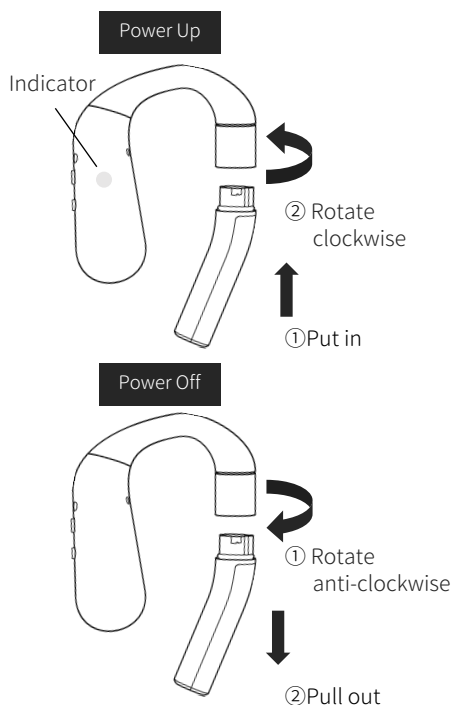

- How to wear

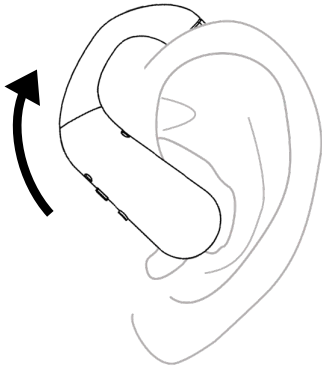


1. Put hearing aid into ear in 45°.
2. Gently insert the ear tip into ear canal.

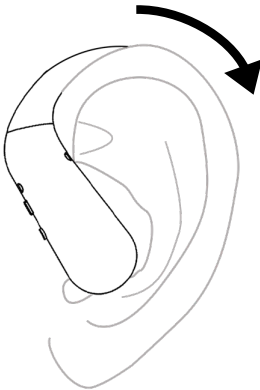

3. Rotate the hearing aid
4. Adjust the ear hook

03 Hearing Function

● Buttons Instruction

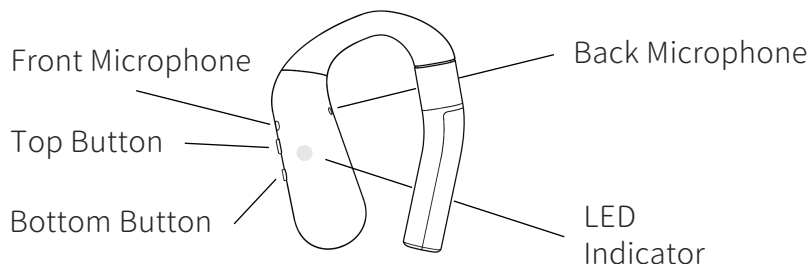


● Frequently used Hearing Function

Function	Operation	Cue Tone
Volume Up	Double click top button	2 Thump
Volume Down	Double click bottom button	2 Thump
Mode Switch	Single click bottom button, mode will change in cycle: ① Mute ② Regular mode, suitable for relatively quiet scenarios. ③ Speech enhancement mode, suitable when in need of focusing on voice. E.g., restaurant, in moving transportation, etc.	1 Thump

04 Volume Adjustment

Find a person to talk to you

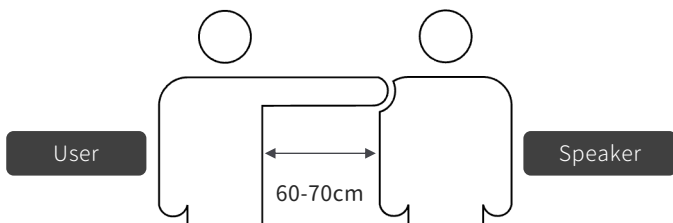

1. Switch back to regular mode.
2. The talker should stand one arm away from the user.
3. The talker keeps talking to user in regular volume.
4. The user can adjust the volume to correct level with the volume control keys in the right side, until you can hear the voice of the talker clearly.
5. If feel uncomfortable, please lower the volume.

Otoadd Classic R can work without mobile connectivity,
If you need to listen music/ make a phone call,
please follow the instruction next page and enable wireless function.

Wireless Connection

1. Pairing

Steps	Operation	Cue Tone
Enter Pairing Mode	Press top button of Otoadd Classic R for 3 seconds to enter pairing mode. LED Indicator will start blinking.	Pairing
Pair with another device	Go to Bluetooth setting of your mobile or other device, you will find Otoadd Classic R on the list.	—
Complete Pairing	Select Otoadd Classic R and pair it to your mobile or other device. LED Indicator will stop blinking.	Connected

- ◆ Otoadd Classic R can only pair with one device at the same time.
- ◆ After hang up the phone call/stop streaming audio, Otoadd Classic R will automatically switch back to hearing mode. If not, please reboot Otoadd Classic R.

2. Frequently used Function

Function	Operation	Cue Tone
Volume Up	Double click top button	2 Thump
Volume Down	Double click bottom button	2 Thump
Answer/Hang Up calls	Short press top button	—
Stop/Play Music/Video	Short press top button	—
Turn On/Off Ambient sound	Short press bottom button	—

06 APP Download


Otoadd provides App for better user experience, please scan the QR code below and download the App.



Website :

<https://www.otoadd.com/mobile-app.html>

07 Charging Instruction

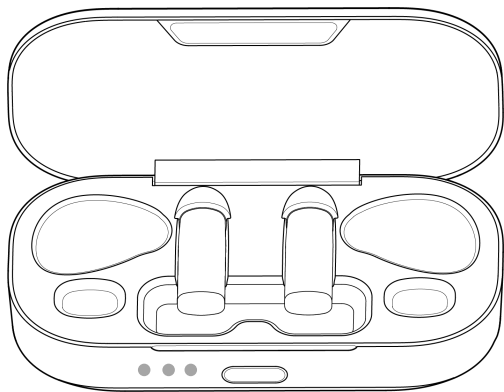

LED
Indicator(White)

Button


1. When power level is lower than 20%, hearing aid will give cue tone : battery low.
Please change battery then.
2. Before put battery into the charging case, please press the button of the charging case to check the power status of charging case.
LED Indicator status :

1 light up : Power level 0%~30%

2 light up : Power level 30%~60%

3 light up : Power level 60%~100%


3. Insert battery : put the battery into the charging case, push a little onward.
4. Take out battery : Flip the battery up, pull it out of the charging case.
5. LED indicator status :
White : Battery fully charged
Red : Battery charging

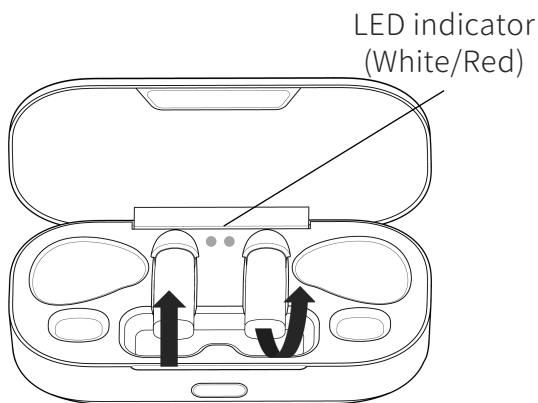

①Insert :
Push onward

②Take out :
Flip up & pull out

6. Charge the Charging Case :
Plug Type-C cable into the
Charging port

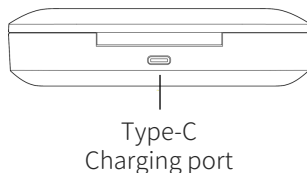

08 Specification

Model Name	HFE-001
Audio Codec	SBC , aptX
Maximum output	126±3dB SPL@2000Hz
Maximum gain	40±4dB SPL
Frequency Response	200Hz~6000Hz
Battery Capacity	Hearing Aid : 3.7V/45mAH Charging Case : 3.7V/450mAH
Weight	Hearing Aid(1 ear, include battery) : 9g Charging Case : 92g

09 Warning/Caution

Important Security Instructions

- Hearing aids are not able to cure or recover to normal hearing, nor to prevent any hearing impairment.
- Seek medical help if you experience tinnitus or any discomfort.
- Do NOT pull the ear hook roughly to avoid rupture.
- Do NOT submerge or expose the hearing aid for extended period to water, or wear while participating in water sports, e.g., swimming, waterskiing, surfing, etc.
- Do NOT expose the hearing aid to excessive heat (e.g., from storage in direct sunlight, fire or the like).
- Do NOT make unauthorized alterations to this product.
- Charge timely to achieve best battery life.
- Store properly if not use for long time.
- Contains small parts which may be a choking hazard. Not suitable for children.
- Use this product only with an agency approved power supply which meets local regulatory requirements (e.g., UL, CSA, VDE, CCC).
- Used battery should be recycled.  

Federal Communications Commission (FCC) Statement

- 15.105(b)

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation.

This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

Federal Communications Commission (FCC) Statement

- 15.19

This device complies with Part 15 of the FCC Rules.

Operation is subject to the following two conditions:

- 1) this device may not cause harmful interference and
- 2) this device must accept any interference received, including interference that may cause undesired operation.

- 15.21

You are cautioned that changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

Product Remarks

- Actual size and weight may differ due to different accessories.
- Color display on the website is only for reference. Please refer to the actual product.
- Detailed specification may differ between different models. Please refer to the actual product.
- DO NOT initiatively inform If any change of product specification, please refer to information published by manufacturer.



Product Name : Otoadd Classic R

Model Name : HFE-001

FCC ID : 2AX9Z-HFE-001

Designer Name : RelaJet Tech (Taiwan) Co., Ltd.

Designer Address :

8F., No. 150, Sec. 4, Nanjing E. Rd., Songshan Dist., Taipei City 105, Taiwan (R.O.C.)

Manufacturer Name :

DONGGUAN TYMPHANY ACOUSTIC TECHNOLOGY CO. LTD.

Manufacturer Address :

LIUWU SECT, SANHENG ROAD, XINCHENG DISTRICT, SHEKKIT TOWN, DONGGUAN CITY, GUANGDONG PROVINCE, CHINA

Website : <https://www.otoadd.com/>