
Smart ear checker N20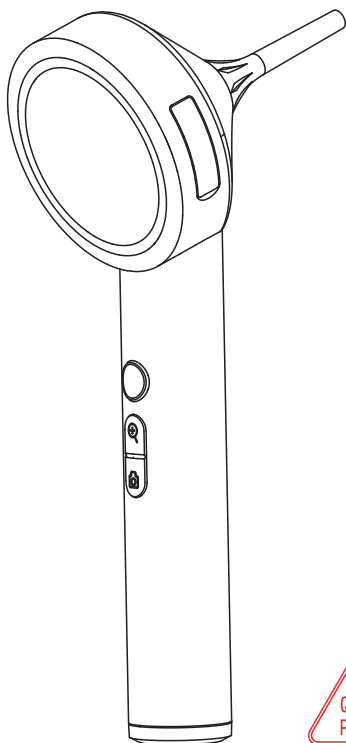

Multilingual user manual

Catalogue

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景 深：5mm-30mm

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 Please read the instruction manual carefully before use

A.What can this product do for you?

1. Check the ears

2. Check the nasal cavity


3. Check the teeth

4. Check the skin

B. Product function introduction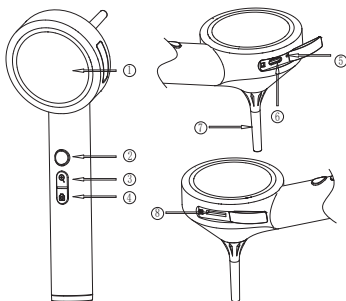

1. 1.8-inch full view display screen, resolution 360 (H) RGB x360 (V)


2. Power button, long press for 2 seconds to turn on/off, short press to adjust light brightness.

3. Zoom key, short press to enlarge the image, long press to enter the album, short press "Previous", short press to switch options on the image deletion interface.

4. Take a photo button, short press to take a photo, long press to enter/exit the recorded video, short press "Next" on the album preview interface, long press to enter the delete interface and confirm to delete the photo or exit the delete interface.

5. Reset button hole.
6. Type-C charging interface.
7. Camera module
8. T-Flash card slot

C、 Use of accessories

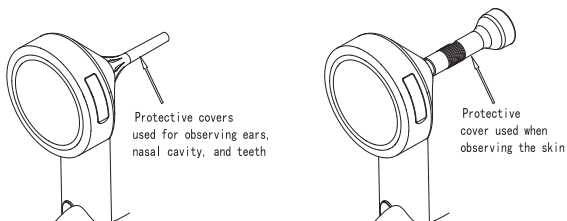1. Please choose a protective cover according to your needs.
2. When replacing and installing the protective cover, pay attention to installing it in place and aligning the bottom with the main unit.

D、 Technical Parameter

Round screen size: 1.8 inches

Camera pixel: HD 1080P

Best focal length: 15mm

Depth of field: 5mm-30mm

Lens size: 3.5mm

Camera outer diameter: 3.98mm

Battery capacity: 2000mAh

Continuous use: approximately 8.5 hours

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FCC Statement

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try correct the interference by one or more of the following measures:

- Reorient the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into and outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

Changes or modifications not expressly approved by the party responsible for compliance could void your authority to operate the equipment.

Workin voltage:4.2V

Rated power:2.5W

Input parameters:5V = 0.4A

Executrve standards:GB4943.1-2011