


Quick Guide

Charging

Use the USB-C charging cable to charge the device. The indicator light will be green when the device is fully charged.

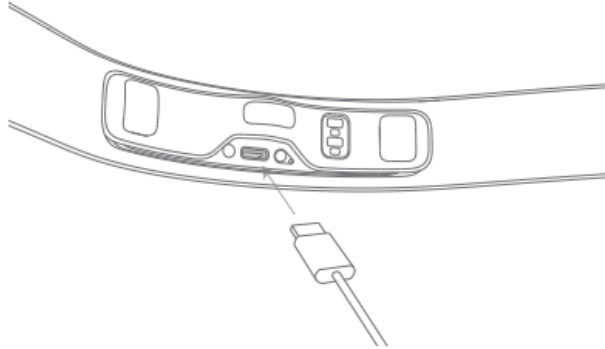

Charging precautions.

- For safe charging, please use the USB cable attached with the product and use a standard 5V charger for charging. Keep dry while charging.
- Please charge at a temperature between 0°C and 45°C.
- If the temperature of the product rises while charging, disconnect the charger immediately and contact customer service.

Description of the indicator status

01/ Bluetooth connection:

Light colour	Light status	Headband status
Blue light	Flashing	In pairing
Blue light	Always on	Connection in progress
Green light	Always on	Pairing successful

02/Charging status:


Orange light	Always on	Charging in progress
Green light	Always on	Fully charged
Orange light	Flashing (5 seconds)	Low battery protection (<3%)

03/ Other conditions:


Blue light	Flashing	OTA upgrade in progress
Red light	Flashing	Error alert

Replacing the headband

- ① Removal: Hold one of the front sides of the device and gently break the soft headband outwards by the other side. Hold the main unit module from the side and remove it.

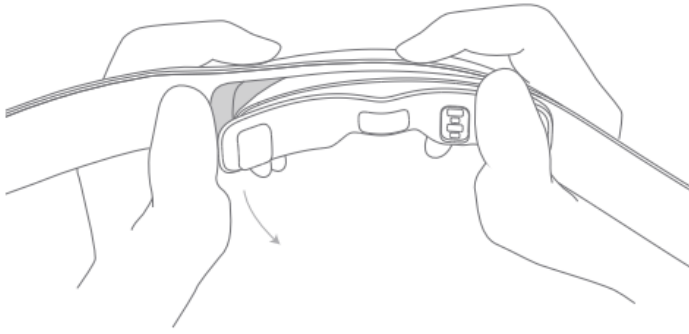


- ② Installation: Hold the main unit module squarely close to the headband cavity and automatically attach it (make sure the headband is oriented in the same direction as the main unit module).

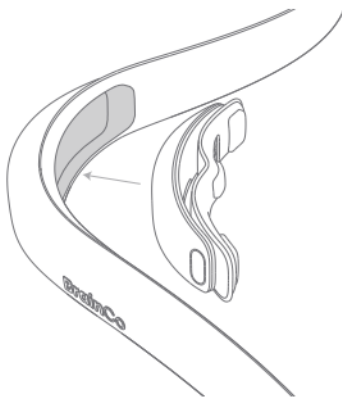

Device storage

- ① Fasten the headband with the Velcro and fold it left and right.
② After folding, place in the included storage bag and tighten the closure.

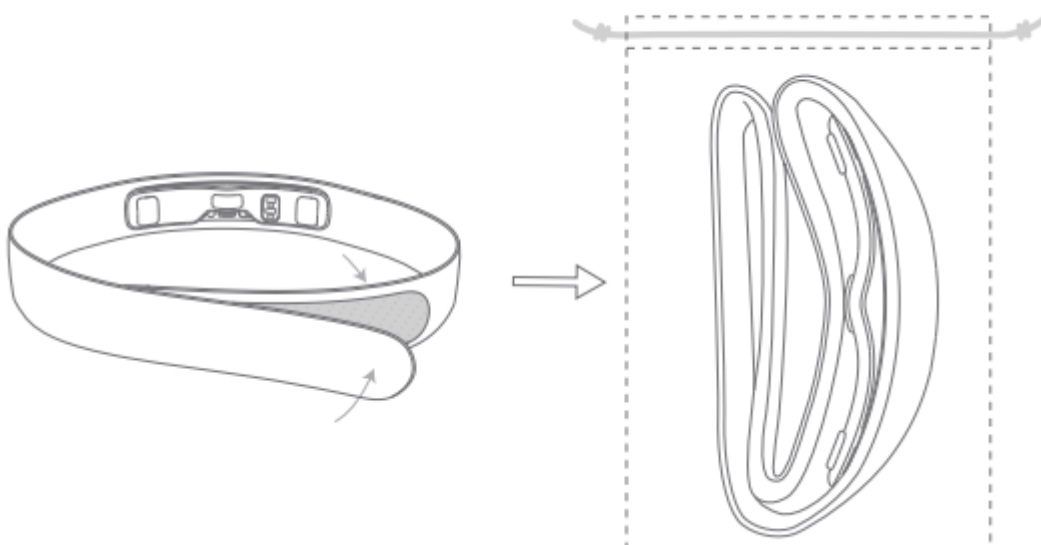

Product Specifications

Device name: Portable EEG feedback device

Model Number: ZS-11

Dimensions: 110mm*22mm*32mm



Weight: 60g

Battery capacity: 90mAh
Charging time approx.: 2 hours
Battery Operating Time: 8 hours
Connectivity: Low power consumption Bluetooth technology
Charging interface: USB-C

Packing list

EEG device (mainframe module and soft headband) *1
Quick Guide*1
Safety guide*1
Storage bag*1
USB-C charging cable*1

Product Overview

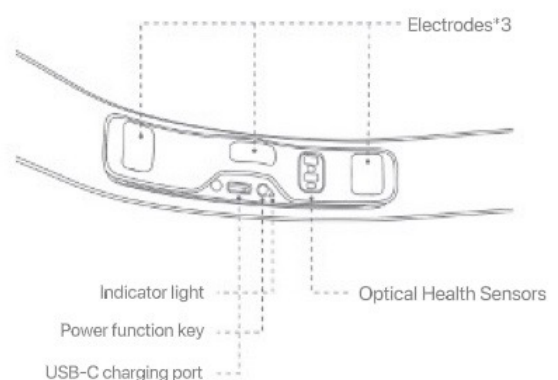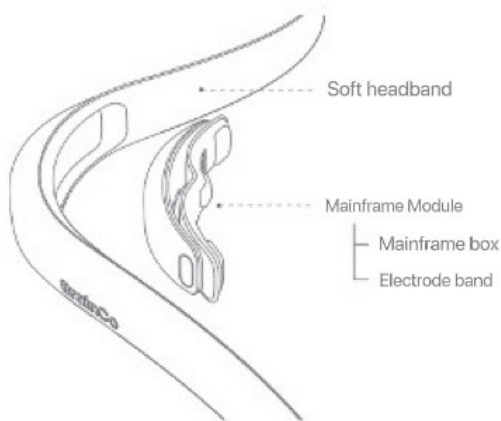

Device connection


01/Download and registration:

- ① Scan the QR code below or search "FocusZen" in the app market to download and install the app.




- ② Open the FocusZen App and complete registration or login.
 - Please ensure that your phone's Bluetooth is switched on when connecting



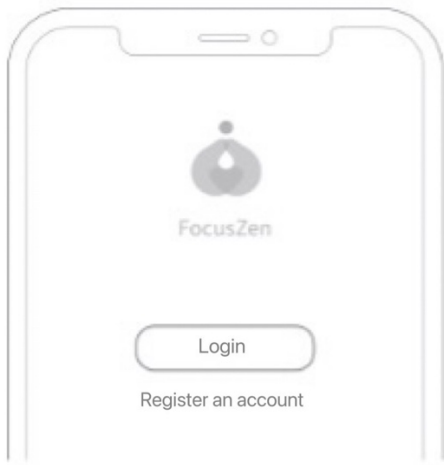


02/Pairing connections:

- ① Power on: Press the power button briefly and the blue light will come on.
 - ② Pairing: Press and hold the power button for 3 seconds, the blue light will flash, and the device will enter the pairing state.
- ❗ The first time you use the headband, it automatically enters the pairing state after switching on.。

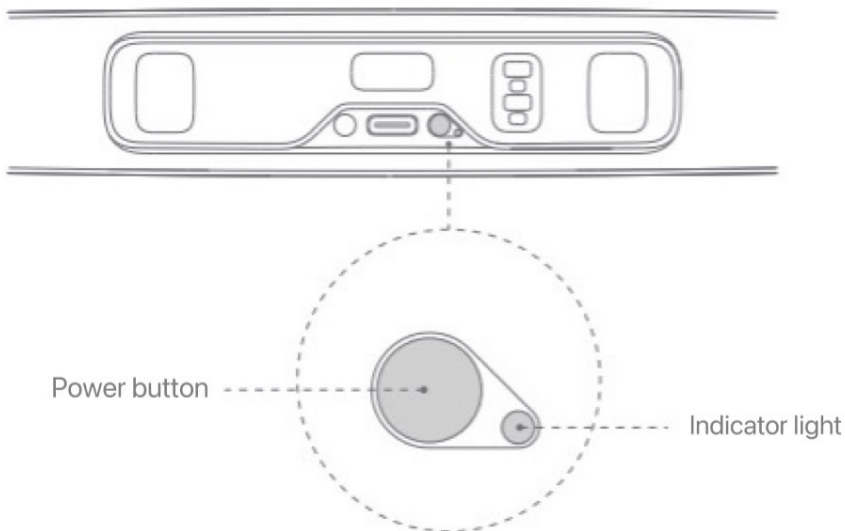


- ③ Connect: Check the Bluetooth name on your device and tap the corresponding Bluetooth name in the App interface to complete pairing.

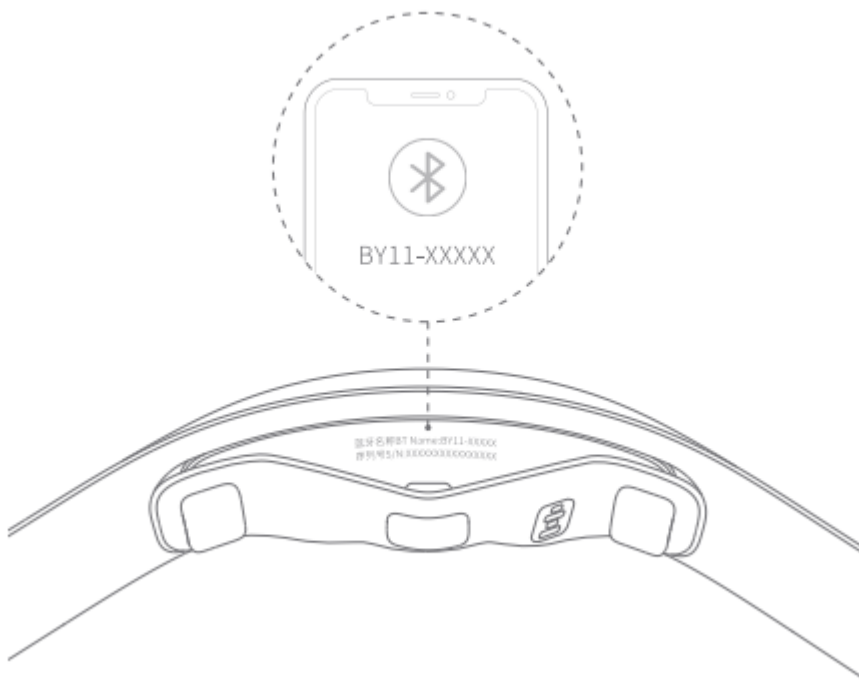

Instructions for wearing the device

① Ensure that the device is held in a positive position (with the charging port facing downwards), hold the middle of the device with both hands and place it centrally on the forehead.

❗ To ensure accurate signal measurement, wipe with a damp paper towel before wearing.

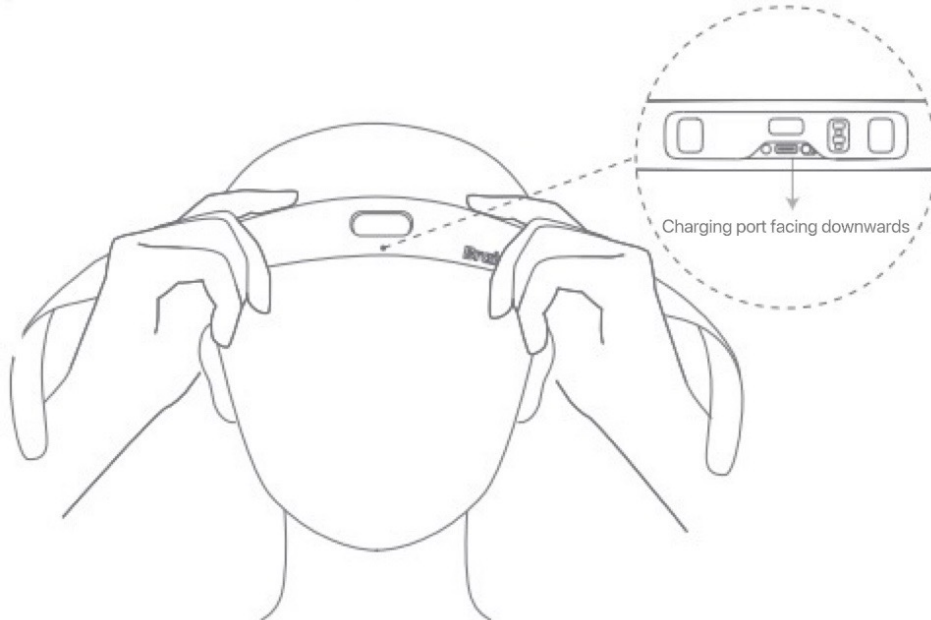

② Tighten the sides of the headband appropriately for wear and adjust the device to the appropriate position, ensuring that all electrodes and sensors are in full contact with the skin (No hair in the way).

